WHO Prequalification of In Vitro Diagnostics Programme PUBLIC ASSESSMENT REPORT

Product: Bioline Malaria Ag P.f (HRP2/pLDH)
Number: PQDx 0209-012-00

The Bioline Malaria Ag P.f (HRP2/pLDH) with product codes 05FK90, 05FK93, 05FK91 and 05FK92 manufactured by Abbott Diagnostics Korea Inc, CE marked regulatory version, was accepted for the WHO list of prequalified diagnostics and was listed on 16 October 2015.

Summary of Prequalification Assessment for the Bioline Malaria Ag P.f (HRP2/pLDH)

	Date	Outcome
Prequalification listing	16 October 2015	listed
Dossier assessment	14 August 2015	MR
Product performance evaluation	Round 3 (2011) and Round 6 (2015)	MR

MR: Meets Requirements

Report amendments and product changes

This public report has since been amended. Amendments may have arisen because of changes to the prequalified product for which the WHO has been notified and has undertaken a review. Amendments to the report are summarized in the following table, and details of each amendment are provided below.

Version	Summary of amendment and change request reference, where applicable.	Date of report amendment
1.0-3.0	Re-introduction of a stable single-use buffer vial (for product code 05FK93).	2 May 2016
4.0	Changes to the cassette labelling (for product codes 05FK90 and 05FK93).	17 November 2016
5.0	Addition of test kit configurations (05FK91, 05FK92) to include safety lancets.	22 December 2016
6.0	Changes to the manufacturer name from Standard Diagnostics, Inc. to Abbott Diagnostics Korea, Inc. and product name from SD BIOLINE Malaria Ag P.f (HRP2/pLDH) to Bioline Malaria Ag P.f (HRP2/pLDH).	9 March 2020

7.0	Updated performance evaluation results and the addition of the test's limitations detecting pLDH are in line with the Global Malaria Programme recommendations.	13 January 2024.
8.0	Accessory labelling changes according to accessory manufacturer information change (Manufacturer name change and EU representative address change) (PQC-IVD-2024-0034). Editorial and Instructional updates to IFU for Bioline Malaria Ag P.f (HRP2/pLDH) for alignment with Globally distributed product. Changes are limited to administrative changes, changes to correct errors, and clarification of existing information without changing the procedure and do not impact quality, safety or performance (PQC-IVD-2025-0044).	23 March 2026.

Intended use:

According to the claim of intended use from Abbott Diagnostics Korea Inc., *"The Bioline Malaria Ag P.f (HRP2/pLDH) test is a rapid, qualitative test for the detection of histidine-rich protein II (HRP2) antigen and lactate dehydrogenase from Malaria Plasmodium falciparum in human whole blood. Bioline Malaria Ag P.f (HRP2/pLDH) test is intended for professional use as an initial screening test only. Reactive specimens should be confirmed by a supplemental assay such as microscopic examination of thin blood smear"*.

Test kit contents:

Product code	Tests/kit	Component	Quantity
05FK90	25 T/kit	Test devise	25
		Assay diluent	1
		Disposable inverted cup	25
		Sterile lancet	25
		Alcohol swab	25
		Instruction for Use	1
05FK91	25 T/kit	Test devise	25
		Assay diluent	1
		Disposable inverted cup	25
		Safety lancet	25
		Alcohol swab	25
		Instruction for Use	1

05FK92	1T/kit x 25 each	Each kit contains a test device, an assay diluent, a disposable inverted cup, an alcohol swab, a safety lancet and a summarised IFU for a single test.	25 each of 1 test/kit
	In addition to the summarised IFU in each kit, an Instruction for Use is included in the package box.		
05FK93	1T/kit x 25 each	Each kit contains a test device, an assay diluent, a disposable inverted cup, an alcohol swab, a sterile lancet and a summarised IFU for a single test.	25 each of 1 test/kit
	In addition to the summarised IFU in each kit, an Instruction for Use is included in the package box.		

Items required but not provided :

- Micropipette
- Protective gloves
- Timer
- Biohazard container

Storage:

The test kit must be stored between 1 and 40 °C.

Shelf-life upon manufacturer¹:

24 months.

Warnings/limitations

The WHO Global Malaria Programme recommends the use of tests detecting pLDH and a panel detection of at least 75% on the panel of nonHRP2 expressing specimens in areas with high prevalence of *pfhrp2/3* gene deletions². This test does not meet these criteria and is not recommended for use in such settings.

¹ The assigned device shelf-life is based on stability data generated from the date of manufacture. The finished goods shelf-life, calculated from the date of packaging completion, may be shorter depending on the time elapsed between manufacture and final packaging of the device.

² <https://iris.who.int/bitstream/handle/10665/258972/WHO-HTM-GMP-2017.18-eng.pdf?sequence=1>

Dossier assessment

Abbott Diagnostics Inc submitted a product dossier for Bioline Malaria Ag P.f (HRP2/pLDH) as per the "Instructions for compilation of a product dossier" (PQDx_018 v1). The information submitted in the product dossier was reviewed by WHO staff and external experts (assessors) appointed by the WHO in accordance with the internal report on the screening and assessment of a product dossier (PQDx_009 v2).

Based on the product dossier screening and assessment findings, the Bioline Malaria Ag P.f (HRP2/pLDH) product dossier meets WHO prequalification requirements.

Manufacturing site inspection

The inspection of the manufacturing site(s) was conducted to assess whether the manufacturer's quality management system (QMS) and manufacturing practices are in alignment with:

- (i) applicable international standards, such as ISO 13485 (Medical devices – Quality management systems – Requirements for regulatory purposes);
- (ii) the manufacturer's own documented procedures and quality requirements; and
- (iii) other relevant international standards and guidelines applicable to in vitro diagnostic (IVD) medical devices. The WHO's Public Inspection Reports are accessible at:

<https://extranet.who.int/pgweb/vitro-diagnostics/who-public-inspection-reports>

Product performance evaluation

The product was evaluated in Round 3, Round 6 and Round 8^{3,4} of WHO/FIND product testing of RDTs for malaria antigen detection, completed in 2010, 2015 and 2018, respectively. The product was evaluated against a *Plasmodium falciparum* cultured line panel, a *P. falciparum* wild-type parasite panel, and a *P. falciparum* negative panel. In addition, in Round 8, a panel of *P. falciparum* strains and clinical specimens with *hrp2/3* deletion was used. Thermal stability was assessed after 2 months of storage at elevated temperature and humidity, and a descriptive ease of use assessment was recorded.

In Round 3, the following results were observed: *P. falciparum* panel detection score of 87.9% at 200 parasites/μL, false-positive rates of 2.0% for clean negatives, 0% for *P. vivax* at 200 parasites/μL, 0% for *P. vivax* at 2000 to 5000 parasites/μL, and an invalid rate of 0%.

In Round 6, the following results were observed: *P. falciparum* panel detection score of 88.0% at 200 parasites/μL overall; 87.0% for HRP2 and 52% for pLDH, false-positive rates of 0.0%

³ <https://iris.who.int/bitstream/handle/10665/276193/9789241514958-eng.pdf?sequence=1>

⁴ <https://www.who.int/publications/i/item/9789241514965>

for clean negatives, 0.7% for *P. vivax* at 200 parasites/ μ L, 0% for *P. vivax* at 2000 to 5000 parasites/ μ L, and an invalid rate of 0%.

In Round 8, the following results were observed: *P. falciparum* panel detection score of 90.0% at 200 parasites/ μ L overall; 88.0% for HRP2 and 71% for pLDH, false-positive rates of 0.0% for clean negatives, 0.0% for *P. vivax* at 200 parasites/ μ L, 0% for *P. vivax* at 2000 to 5000 parasites/ μ L, and invalid rate of 0.1%. In addition, in Round 8, a panel detection score of 57.5%⁵ was found on a panel of *P. falciparum* strains and clinical specimens with *hrp2/3* deletion.

Summary performance characteristics	Panel detection score		False positive rate (%)		Invalid rate (%)
	200 parasites/ μ L		200 parasites/ μ L	Clean negatives	
	<i>Pf</i> wild-type	<i>Pf</i> <i>hrp2/3</i> deletion	<i>Pv</i>		
Round 3	87.9	N/A	0.0	2.0	0.0
Round 6	88.0 (87% for HRP2, 52% for pLDH)	N/A	0.7	0.0	0.0
Round 8	90.0 (88% for HRP2, 71% for pLDH)	57.5 (see footnote 5)	0.0	0.0	0.1

Based on these results, the performance evaluation for Bioline Malaria Ag P.f (HRP2/pLDH) meets the WHO prequalification requirements. See the Warning/limitations sections regarding use in areas with *pfhrp2/3* gene deletions.

Labelling review

The labelling submitted for Bioline Malaria Ag P.f (HRP2/pLDH) was reviewed by WHO staff and external technical experts appointed by WHO. The review evaluated the labelling for clarity and consistency with the information submitted in the product dossier, alignment with international guidance and standards, and suitability for the intended users and settings in WHO Member States, including low- and middle-income countries.

The table below provides traceability of the labelling documents reviewed during the assessment, including document titles, version numbers, approval dates, and control identifiers.

⁵ Results presented in the report "Malaria Rapid Diagnostic Test Performance – Results of WHO product testing of malaria RDTs: round 8 (2016-2018)" by WHO, FIND and CDC indicate a PDS of 32.5% on the HRP2-negative *P. falciparum* panel. This is due to an error in data analysis, which was later corrected, leading to a corrected PDS of 57.5%.

Controlled Labelling References**05FK90**

Document Type	Document Title	Version / Revision	Date Approved	Controlled Document No.
Outer box artwork	Package	A0	2019-09-16	05FK90-40-A0
Pouch / Device label	Device pouch label	A0	2019-12-02	FP06-A0
Assay diluent labels	Assay diluent labels	A0	2019-09-18	FB01-A0
Inverted cup	40008361	0	2025-09-25	D10151501
Sterile lancet	P65-008	0	2023-11-10	D10151501
Sterile lancet	40006487	0	2024-10-01	D10151501
Instructions for Use (IFU)	IFU	A2	2020-09-09	05FK90-04-A2

05FK91

Document Type	Document Title	Version / Revision	Date Approved	Controlled Document No.
Outer box artwork	Package	A0	2019-09-16	05FK91-40-A0
Pouch / Device label	Device pouch label	A0	2019-12-02	FP06-A0
Assay diluent labels	Assay diluent labels	A0	2019-09-18	FB01-A0
Inverted cup	40008361	0	2025-09-25	D10151501
Sterile lancet	P65-013/000/001	0	2023-11-10	D10151501
Sterile lancet	40006494	0	2023-11-10	D10151501
IFU	IFU	A2	2020-09-09	05FK91-04-A2

05FK92

Document Type	Document Title	Version / Revision	Date Approved	Controlled Document No.
Outer box artwork	Package	A0	2019-09-16	05FK92-40-A0
Pouch / Device label	Device pouch label	A0	2019-11-01	FP14-A0
POCT Pouch	Pouch label	A0	2019-09-09	05FK92-P-A0
Inverted cup*	40005865	N/A	N/A	N/A
Sterile safety lancet*	40005867	N/A	N/A	N/A
Ampule*	P60-009/009	N/A	N/A	N/A
IFU*	Summarized IFU	A2	2020-10-30	05FK92-02-C-A2
Instructions for Use (IFU)	IFU	A2	2020-09-09	05FK92-04-A2
Reference Buffer guide	Malaria Buffer guide	A0	2022-02-07	05FK00-IC-A0

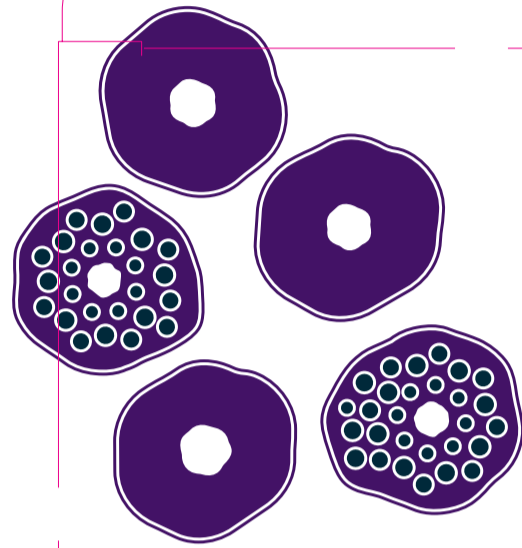
* These items are packaged together in a POCT pouch

05FK93

Document Type	Document Title	Version / Revision	Date Approved	Controlled Document No.
Outer box artwork	Package	A0	2019-09-16	05FK93-40-A0
Pouch / Device label	Device pouch label	A0	2019-11-01	FP14-A0
POCT Pouch	Pouch label	A0	2019-09-09	05FK93-P-A0
Inverted cup*	40005865	N/A	N/A	N/A
Sterile lancet*	40005864	N/A	N/A	N/A
Sterile lancet*	40009512	N/A	N/A	N/A
Ampule*	40005866	N/A	N/A	N/A
IFU*	Summarized IFU	A2	2021-03-26	05FK93-02-C-A2
IFU	IFU	A2	2020-09-09	05FK93-04-A2
Reference Buffer guide	Malaria Buffer guide	A0	2022-02-07	05FK00-IC-A0

** These items are packaged together in a POCT pouch*

Labels

Malaria Ag P.f (HRP2/pLDH)

Malaria Ag P.f (HRP2/pLDH)

EN
Rapid, Immunochromatographic test for the qualitative detection of Histidine-rich protein 2 (HRP2) antigen and Lactate dehydrogenase (pLDH) from *Malaria Plasmodium falciparum* in human whole blood

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- CONTENTS:**
- 25 Test devices with desiccant in individual foil pouch
 - Assay diluent (1 x 5 ml/vial)
 - 25 Disposable specimen applicators (5 µl)
 - 25 Safety lancets, 25 Alcohol swabs
 - 1 Instructions for use

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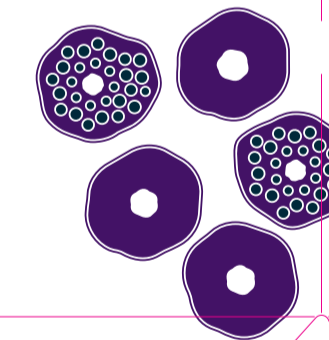
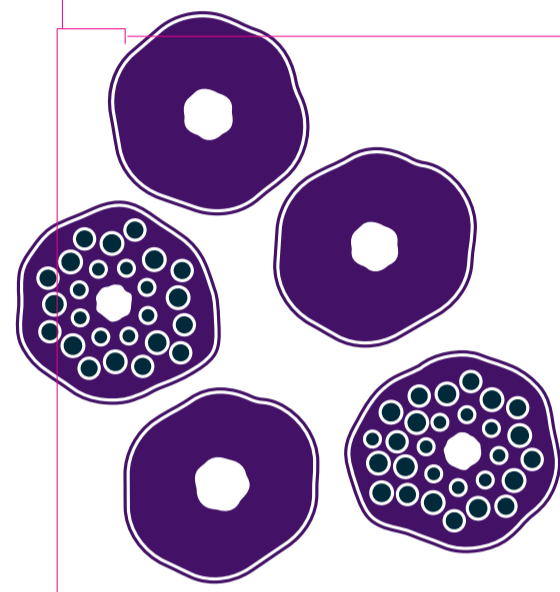
© 2019 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners.

Malaria Ag P.f (HRP2/pLDH)

Malaria Ag P.f (HRP2/pLDH)

EN
Rapid, immunochromatographic test for the qualitative detection of Histidine-rich protein 2 (HRP2) antigen and Lactate dehydrogenase (pLDH) from Malaria Plasmodium falciparum in human whole blood

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CONTENTS :
• 25 Test devices with desiccant in individual foil pouch
• 25 Assay diluents
• 25 Disposable specimen applicators (5 µl),
25 Safety lancets, 25 Alcohol swabs
• 25 Summarized instructions for use
• 1 Instructions for use

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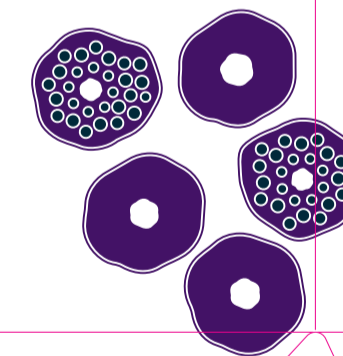
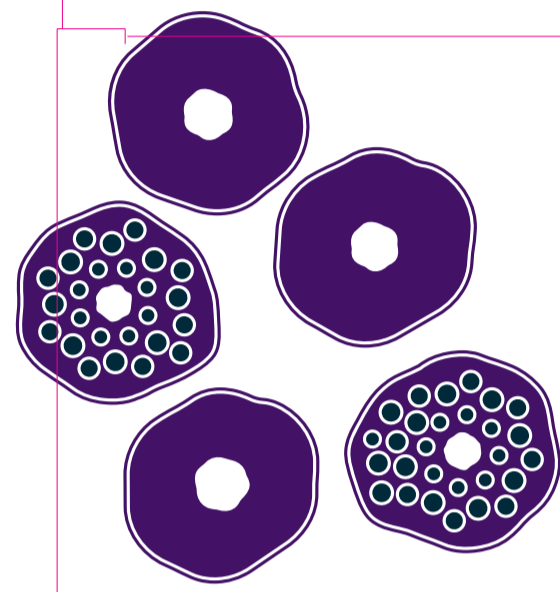
CONTENTS :
• 25 Test devices with desiccant in individual foil pouch
• 25 Assay diluents
• 25 Disposable specimen applicators (5 µl),
25 Sterile lancets, 25 Alcohol swabs
• 25 Summarized instructions for use
• 1 Instructions for use

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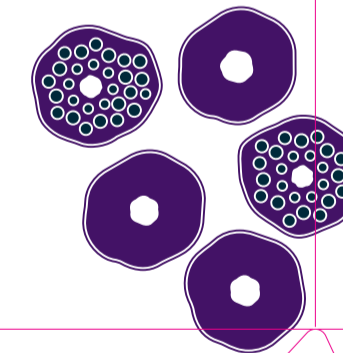
EC REP

All Diagnostics GmbH
Altenhofstrasse 80,
66386 St. Ingbert, Germany

© 2019 Abbott. All rights reserved. All trademarks
referenced are trademarks of either the Abbott group of
companies or their respective owners.

PRODUCT - Bioline™ Malaria Ag P.f
(HRP2-pLDH)

PMS 303 C
PMS 2617 C

PN. - 05FK93-40-A0
REV. DATE - 2019.09.06
CREATOR - THINKBOX

SIZE - 220 X 130 X 70MM

PRODUCT - Bioline™ Malaria Ag P.f
(HRP2/pLDH)
(Safety lancet, POCT spec) Outer pouch

SIZE - 145 x 50mm

 PMS 303 C
 PMS 354 C

CAT NO. - 05FK92-P-A0
REV. DATE - 2019. 09. 03
CREATOR - THINKBOX

Bioline™
Malaria Ag P.f
(HRP2/pLDH)

 **Abbott Diagnostics Korea Inc.**
65, Borahagal-ro, Giheung-gu, Yongin-si,
Gyeonggi-do, 17099, Republic of Korea

Malaria HRP-II (P.f) and pLDH (P.f) Antigen Rapid Test

CONTENTS : Test device, Assay diluent,
Disposable specimen applicator (5 µl), Safety lancet, Alcohol swab,
Summarized instructions for use

REF 05FK92

1°C / 34°F  40°C / 104°F

05FK92-P-A0

PRODUCT - Bioline™ Malaria Ag P.f
(HRP2/pLDH)
(POCT spec) Outer pouch

SIZE - 145 x 50mm

 PMS 303 C
 PMS 2617 C

CAT NO. - 05FK93-P-A0
REV. DATE - 2019. 09. 03
CREATOR - THINKBOX

Bioline™
Malaria Ag P.f
(HRP2/pLDH)

 **Abbott Diagnostics Korea Inc.**
65, Borahagel-ro, Giheung-gu, Yongin-si,
Gyeonggi-do, 17099, Republic of Korea

Malaria HRP-II (P.f) and pLDH (P.f) Antigen Rapid Test

CONTENTS : Test device, Assay diluent,
Disposable specimen applicator (5 µl), Sterile lancet, Alcohol swab,
Summarized instructions for use

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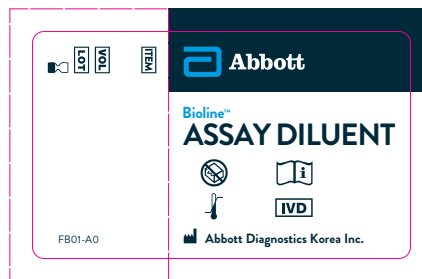
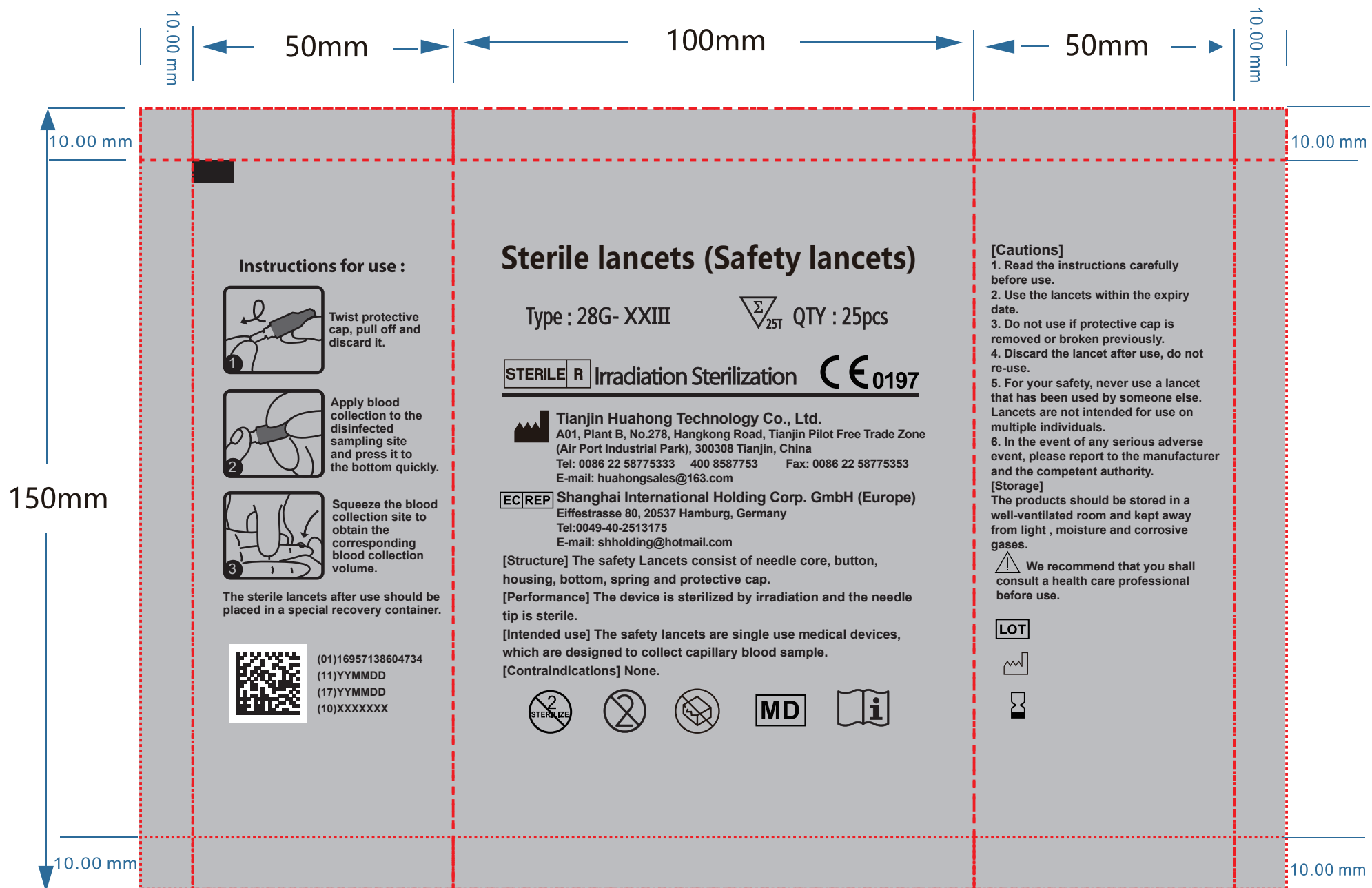
STERILE R **CE** 0123 

010004451700443617288004193285287101

EC REP Shanghai International Holding Corp. GmbH (Europe)
Eiffelstrasse 60, 20537 Hamburg, Germany


Shandong Lianfa Medical Plastic Products Co., Ltd.
No. 3 Shuangshihai Sanjian Road 250200 Zhangqiu City, Jinan, Shandong
PEOPLE'S REPUBLIC OF CHINA

Instructions for use :

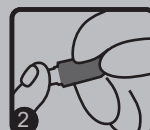Twist protective cap, pull off and discard it.

Apply blood collection to the disinfected sampling site and press it to the bottom quickly.

Squeeze the blood collection site to obtain the corresponding blood collection volume.

The sterile lancets after use should be placed in a special recovery container.

(01)16957138604734
 (11)YYMMDD
 (17)YYMMDD
 (10)XXXXXX

Sterile lancets (Safety lancets)

Type : 28G- XXIII

QTY : 25pcs

STERILE R Irradiation Sterilization CE 0197

Tianjin Huahong Technology Co., Ltd.
 A01, Plant B, No.278, Hangkong Road, Tianjin Pilot Free Trade Zone
 (Air Port Industrial Park), 300308 Tianjin, China
 Tel: 0086 22 58775333 400 8587753 Fax: 0086 22 58775353
 E-mail: huahongsales@163.com

EC/REP Shanghai International Holding Corp. GmbH (Europe)
 Eiffestrasse 80, 20537 Hamburg, Germany
 Tel:0049-40-2513175
 E-mail: shholding@hotmail.com

[Structure] The safety Lancets consist of needle core, button, housing, bottom, spring and protective cap.

[Performance] The device is sterilized by irradiation and the needle tip is sterile.

[Intended use] The safety lancets are single use medical devices, which are designed to collect capillary blood sample.

[Contraindications] None.

[Cautions]

1. Read the instructions carefully before use.
2. Use the lancets within the expiry date.
3. Do not use if protective cap is removed or broken previously.
4. Discard the lancet after use, do not re-use.
5. For your safety, never use a lancet that has been used by someone else. Lancets are not intended for use on multiple individuals.
6. In the event of any serious adverse event, please report to the manufacturer and the competent authority.

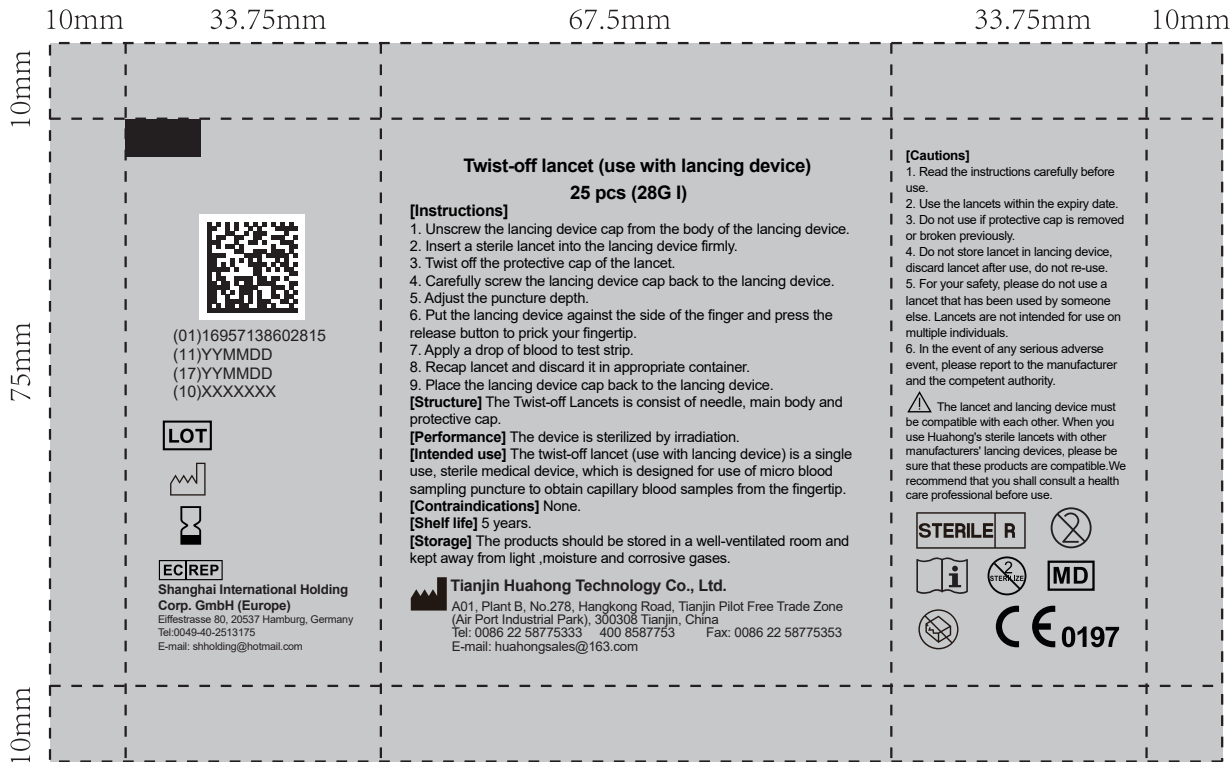
[Storage]

The products should be stored in a well-ventilated room and kept away from light , moisture and corrosive gases.

We recommend that you shall consult a health care professional before use.

LOT

(01)16957138602815
 (11)YYMMDD
 (17)YYMMDD
 (10)XXXXXXXX

LOT

EC REP

Shanghai International Holding
 Corp. GmbH (Europe)
 Eiffestrasse 80, 20537 Hamburg, Germany
 Tel: 0049-40-2513175
 E-mail: shholding@hotmail.com

**Twist-off lancet (use with lancing device)
 25 pcs (28G I)**

[Instructions]

1. Unscrew the lancing device cap from the body of the lancing device.
2. Insert a sterile lancet into the lancing device firmly.
3. Twist off the protective cap of the lancet.
4. Carefully screw the lancing device cap back to the lancing device.
5. Adjust the puncture depth.
6. Put the lancing device against the side of the finger and press the release button to prick your fingertip.
7. Apply a drop of blood to test strip.
8. Recap lancet and discard it in appropriate container.
9. Place the lancing device cap back to the lancing device.

[Structure] The Twist-off Lancets is consist of needle, main body and protective cap.

[Performance] The device is sterilized by irradiation.

[Intended use] The twist-off lancet (use with lancing device) is a single use, sterile medical device, which is designed for use of micro blood sampling puncture to obtain capillary blood samples from the fingertip.

[Contraindications] None.

[Shelf life] 5 years.

[Storage] The products should be stored in a well-ventilated room and kept away from light, moisture and corrosive gases.

Tianjin Huahong Technology Co., Ltd.

A01, Plant B, No.278, Hangkong Road, Tianjin Pilot Free Trade Zone
 (Air Port Industrial Park), 300308 Tianjin, China
 Tel: 0086 22 58775333 400 8587753 Fax: 0086 22 58775353
 E-mail: huahongsales@163.com

[Cautions]

1. Read the instructions carefully before use.
2. Use the lancets within the expiry date.
3. Do not use if protective cap is removed or broken previously.
4. Do not store lancet in lancing device, discard lancet after use, do not re-use.
5. For your safety, please do not use a lancet that has been used by someone else. Lancets are not intended for use on multiple individuals.
6. In the event of any serious adverse event, please report to the manufacturer and the competent authority.

The lancet and lancing device must be compatible with each other. When you use Huahong's sterile lancets with other manufacturers' lancing devices, please be sure that these products are compatible. We recommend that you shall consult a health care professional before use.

STERILE R

XXXXXXXXX_Inverted Cup(80mm)XXPCS_55×60

Inverted Cup 80mm (5µl)

Code No. : XXXXXXXX
LOT No. : XXXXXXXXXX
Quantity : XXPCS
MFG Date : XXXX.XX.XX

HLB Life Science Co.,Ltd
101-908, Digital EmpireII, 88, Sinwon-ro, Yeongtong-gu
Suwon-si, Gyeonggi-do, 16681, Republic Korea

(0 1) 0 8 8 0 9 8 9 9 6 2 X X X X

EC REP Cmc Medical Devices & Drugs S.L.
C/ Horacio Lengo n18 C.P 29006 Málaga-Spain

Inverted Cup 80mm (5µl)

Code No. : XXXXXXXX
LOT No. : XXXXXXXXXX
Quantity : XXPCS
MFG Date : XXXX.XX.XX

HLB Life Science Co.,Ltd
101-908, Digital EmpireII, 88, Sinwon-ro, Yeongtong-gu
Suwon-si, Gyeonggi-do, 16681, Republic Korea

(0 1) 0 8 8 0 9 8 9 9 6 2 X X X X

EC REP Cmc Medical Devices & Drugs S.L.
C/ Horacio Lengo n18 C.P 29006 Málaga-Spain

Instructions for Use⁶

⁶ English version of the IFU was the one that was assessed by WHO. It is the responsibility of the manufacturer to ensure correct translation into other languages.

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
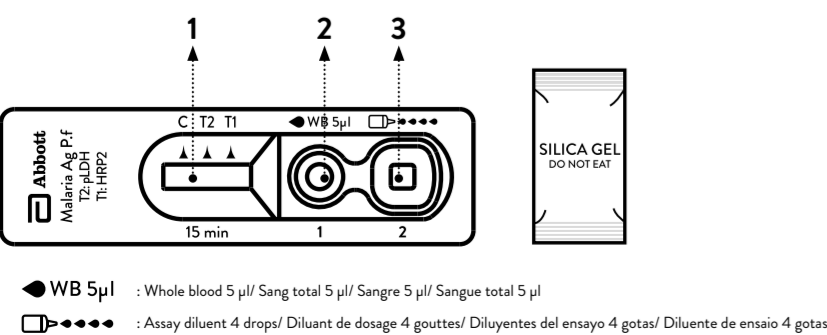
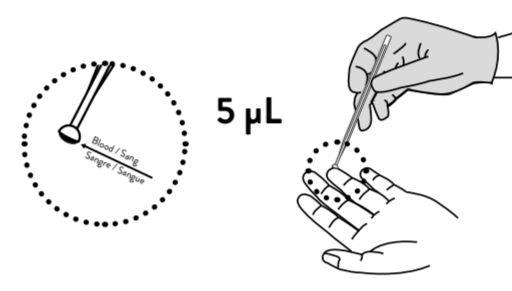
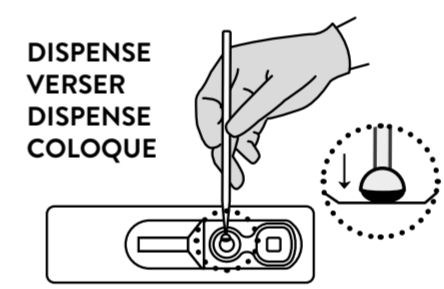
- 6** **EN** Hold assay diluent bottle vertically and dispense 4 drops of assay diluent into the assay diluent well. Exactly, 4 drops should be added. Do not let bottle tip touch device in order to avoid cross-contamination.

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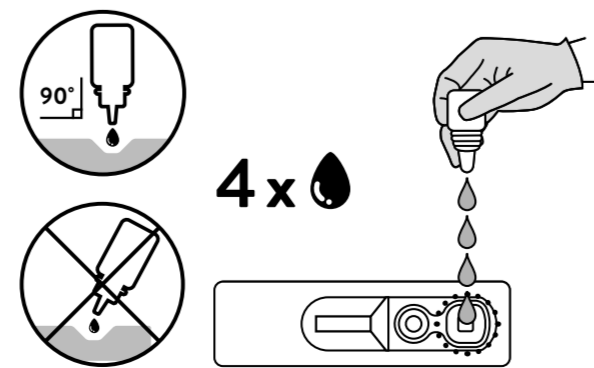- 7** **EN** Read the test result after 15 minutes.
CAUTION: Do not read test results after 30 minutes. ; reading after 30 minutes can yield erroneous results

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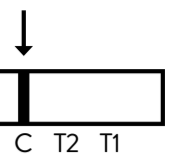
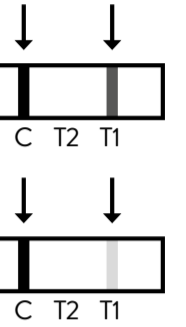
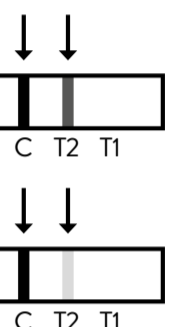
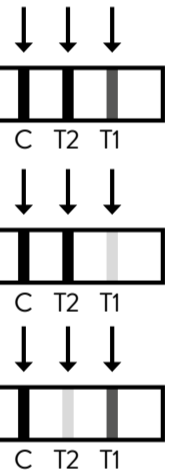
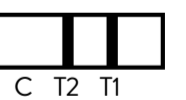
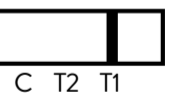
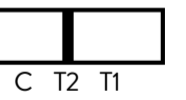
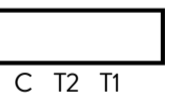
ENGLISH

About the test

Malaria is a serious and sometimes fatal parasitic disease characterized by fever, chills and anemia. The disease is caused by a parasite that is transmitted from human to another through the bite of infected Anopheles mosquitoes. There are four types of malaria that can affect humans: Plasmodium falciparum, P. vivax, P. ovale and P. malariae. In humans, the parasites (called sporozoites) migrate to the liver where they mature and then infect red blood cells. The disease now occurs in more than 90 countries worldwide and it is estimated that there are 200 million malaria cases and 5 million deaths per year. As present, malaria is diagnosed by looking for parasites in a drop of blood. Malaria is placed under a microscope slide and stained so that the parasites will be visible under a microscope.

Test Principle
Bioline® Malaria Ag P.f (HRP2/pLDH) test contains a membrane strip, which is pre-coated with mouse anti-HRP2 and anti-pLDH monoclonal antibodies specific for P.f (HRP2) and the lactate dehydrogenase (pLDH) from the test band region. This test device has a lateral "T1", "T2" (test lines) and a "C" (control line) on the surface of the device. Neither test line nor the control line is visible in the result window before applying a specimen. The control window always shows a pre-coated control and should always appear in the test procedure as performed correctly.

Test Procedure
Bioline® Malaria Ag P.f (HRP2/pLDH) test is a rapid, qualitative test for the detection of histidine-rich protein 2 (HRP2) antigen and lactate dehydrogenase from Malaria Plasmodium falciparum in human whole blood. Bioline® Malaria Ag P.f (HRP2/pLDH) test kit is intended only for use in a laboratory setting for a rapid diagnostic use. Reactive specimens should be confirmed by a supplemental assay such as microscopic examination of blood smear.

Materials provided and active ingredients of main component
1. Bioline® Malaria Ag P.f (HRP2/pLDH) test kit containing following the items to perform the assay:
- Assay diluent (5 x 5ml) and
- 25 Disposable application (S) (5 µl), 25 Safety lancets, 25 Alcohol swabs
2. Instructions for use
3. Active ingredients of main component
- 1 strip dry indicator, Gold conjugate, Mouse monoclonal antibodies specific to P.f HRP2 (0.25µl/0.05 µl), Mouse monoclonal antibodies specific to P.f pLDH (0.5µl/0.1 µg), Test line, Mouse monoclonal antibodies specific to P.f HRP2 (0.5µl/0.1 µg), Test line, Mouse monoclonal antibodies specific to P.f pLDH (0.5µl/0.1 µg), Control line, Goat anti-mouse IgG (1:100 µg), Sodium iodide (5 µg), Sodium iodate (5 µg).

Materials required but not provided
- Microspatula, Protective Goggles, Timer, Bioshield container

Kit storage and stability
1. The test should be stored at a temperature between 1°C and 40°C. Do not freeze the kit or its components. Assay diluent should be kept firmly sealed between each use.
2. **Caution:** When stored at refrigerator, all kit components must be brought to room temperature (15 - 30°C) minimum 30 mins prior to use. Do not open the pouch until you are ready to use the kit.
3. Assay diluent may be opened and refilled for each assay. It is stable until expiration date if kept at 15 - 30°C.
4. The test device is sensitive to both humidity and perform the test immediately after removing the test device from foil pouch.
5. The test kit should be kept away from expiration date.
6. The shelf life of the kit is as indicated on the outer packaging.
7. Use up the test kit of the pouch is damaged or the lot is broken.
8. During the storage, avoid storing the kit, avoid exposure to high temperature (above 45°C) for a period longer than 1 week.

Warnings
1. For in vitro diagnostic use only. Do not reuse the test device and kit components.
2. The instruments must be followed exactly to achieve accurate results. Any individual performing the test with this product must be trained in its use and must be experienced in laboratory practice.
3. Do not eat or smoke while handling specimens.
4. Wear protective gloves while handling and wash hands thoroughly afterwards.
5. Avoid splashing or aerosol formation of specimen and assay diluent.
6. Clean up spillages using an appropriate disinfectant.
7. Decontaminate and dispose of all specimens, reagents and potentially contaminated materials (e.g. lancet, specimen applicator, test device) in a biohazard container as if they were infectious waste.
8. Do not mix or interchange different specimens.
9. Do not mix reagent of different lots or for other products.
10. Do not get reagent by mouth.
11. Do not eat the desiccant in the foil pouch.
12. Do not drink assay diluent.
13. The assay diluent contains a proprietary anti-microbial agent, sodium chloride, which presents no hazard to the user if normal laboratory safety precautions are followed. If contact with assay diluent to the eyes and/or skin, wash affected area with soap and water immediately. If irritation or signs of toxicity occur, seek medical attention.

Specimen collection and storage
1. Using venipuncture, draw whole blood into the collection tube (containing anticoagulants including Heparin, EDTA and sodium citrate).
2. If the blood specimen is not immediately tested, it must be refrigerated at 2 - 8°C.
3. If stored at 2 - 8°C, the blood specimen must be tested within 3 days.
4. Do not use a blood specimen stored for more than 3 days; it can cause a nonspecific reaction.
Collection using lancet
1. Clean the area to be lanced with an alcohol swab.
2. Squeeze the tip of the fingerpad gently with a safety lancet provided.
3. Wipe away the first drop of blood with sterile gauze or cotton.
4. Using a capillary pipette (5 µl) provided, while gently squeezing the tube, immerse the open end in the blood drop and then gently release the pressure to draw blood into the capillary pipette to black line. Or, Using a disposable inverted cap (5 µl) provided, dip the circular end of inverted cap into the blood specimen.
Precaution
1. Anticoagulants including heparin, EDTA and sodium citrate do not affect the test results. Use of other anticoagulants will not be validated. Their use may affect the test results.
2. Use a new separate disposable capillary pipette or pipette tips for each specimen in order to avoid cross-contamination of specimens and produce erroneous results.
3. Discard the lancet or alcohol swab if package is pierced or damaged. The use of damaged lancet may cause any infection that is not expected and/or cause a false positive or false negative result.
4. Repeat from three-try cycle for specimen should be soaked.

Performance characteristics
1. Sensitivity of Bioline® Malaria Ag P.f (HRP2/pLDH) test
A. Internal evaluation of Bioline® Malaria Ag P.f (HRP2/pLDH) Test for the detection of P.f HRP2
B. Internal evaluation of Bioline® Malaria Ag P.f (HRP2/pLDH) Test for the detection of pLDH
C. Specificity of Bioline® Malaria Ag P.f (HRP2/pLDH) Test for the detection of P.f HRP2
D. Specificity of Bioline® Malaria Ag P.f (HRP2/pLDH) Test for the detection of pLDH
E. Following potentially interfering substances have no impact on the assay.
F. Following potentially interfering substances have no impact on the assay.
G. Following potentially interfering substances have no impact on the assay.
H. Following potentially interfering substances have no impact on the assay.
I. Following potentially interfering substances have no impact on the assay.
J. Following potentially interfering substances have no impact on the assay.
K. Following potentially interfering substances have no impact on the assay.
L. Following potentially interfering substances have no impact on the assay.
M. Following potentially interfering substances have no impact on the assay.
N. Following potentially interfering substances have no impact on the assay.
O. Following potentially interfering substances have no impact on the assay.
P. Following potentially interfering substances have no impact on the assay.
Q. Following potentially interfering substances have no impact on the assay.
R. Following potentially interfering substances have no impact on the assay.
S. Following potentially interfering substances have no impact on the assay.
T. Following potentially interfering substances have no impact on the assay.
U. Following potentially interfering substances have no impact on the assay.
V. Following potentially interfering substances have no impact on the assay.
W. Following potentially interfering substances have no impact on the assay.
X. Following potentially interfering substances have no impact on the assay.
Y. Following potentially interfering substances have no impact on the assay.
Z. Following potentially interfering substances have no impact on the assay.

Product Disclaimer:
While every precaution has been taken to ensure the diagnostic ability and accuracy of this product, the user is advised that the product is not intended for use outside of the control of the manufacturer and distributor and test results may accordingly be affected by environmental factors and/or user error. The subject of the diagnosis should consult a doctor for further confirmation of the test result.

Warning:
The manufacturer and distributors of this product shall not be liable for any direct, indirect, or consequential loss, liability, or damage arising from or related to an incorrect positive or negative diagnosis.

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- POSITIVE / POSITIF / POSTIVO**
- ⚠ **Caution:** The presence of any test line, no matter how faint, the result is considered positive.

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⚠ **Possible cases**

 - p.f. HRP2 gene deletions
 - Prozone effect (excess of HRP2 Ag)

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ES "C", "T1" y "T2" (Tres líneas)

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- INVALID / NON VALIDE / INVÁLIDO / INVÁLIDO**
- EN If the control line "C" is not visible within the result window after performing the test, the result is considered invalid. Instructions may not have been followed correctly or the test may have deteriorated beyond the expiration date. It is recommended that the specimen be retested using a new test device.

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ENGLISH

About the test
[Introduction]
 Malaria is a serious and sometimes fatal parasitic disease characterized by fever, chills and anemia. The disease is caused by a parasite that is transmitted from one human to another through the bite of infected *Anopheles* mosquitoes. There are four types of malaria that can infect humans: P. falciparum, P. vivax, P. malariae, and P. knowlesi. In humans, the parasite (female sporozoites) migrates to the liver where they mature and are released in another form: merozoites. The disease now occurs in more than 90 countries worldwide, and it is estimated that there are over 200 million clinical cases and 0.5 million malaria-associated deaths per year. As present, malaria is diagnosed by looking for parasites in a drop of blood. Blood is placed on a microscope slide and stained so that the parasites will be visible under a microscope.

[Precaution]
 Anticoagulants including heparin, EDTA and sodium citrate do not affect the test results. Use of other anticoagulants have not been validated. Their use may affect the test result. Use a new separate disposable capillary pipette or pipette tips for each specimen in order to avoid cross-contamination of specimens, which could produce erroneous results. Discard the lancet or alcohol swab if package is opened or damaged. The use of damaged lancet may cause any infection at the punctured skin due to cause to existing sterility. Repeated from-thawed cycle for specimen should be avoided.

Test procedure (Refer to figure)
 1. Bring all kit components and specimen to room temperature prior to testing.
 2. Clean the area to be lanced with an alcohol swab.
 3. Use the safety lancet to prick the safety lancet to break the seal and discard.
 4. Prick the lateral side of the finger with the safety lancet. Then, safely dispose of the lancet immediately after use.
 5. Using a capillary pipette (5 µl) provided, draw whole blood specimen to black line and then transfer draw whole blood into the round specimen well. Or, Using a disposable insulin cap (5 µl) provided, draw the whole blood from the insulin cap into the specimen well and transfer the draw whole blood into the round specimen well.
 6. Dip the lancet into the insulin cap and discard.
 7. Test and pull up to open assay window.
 8. Dispose of all the assay diluent from the diluent tube into the square well of test device.
 9. Read the test result after 15 minutes.
 10. **Caution:** Do not read test results after 30 minutes ; reading after 30 minutes can yield erroneous result.

Test interpretation (Refer to figure)
 1. **Negative result:** The presence of control line (C) within the result window indicates a negative result.
 2. **Positive result:** **Caution:** The presence of any line, no matter how faint, the result is considered positive.
 3. The presence of two colored lines (T1 and C) within the result window, regardless of which line appears first, indicate HRP2 and P.f positive result.
 4. **pLDH of P.falciparum positive:** The presence of two colored lines (T2 and C) within the result window, regardless of which line appears first, indicate pLDH of P.f positive result.
 5. **Invalid result:** If the control line (C) is not visible within the result window after 15 minutes, the test is considered invalid. The directions may not have been followed correctly or the kit may have deteriorated. It is recommended that the specimen be retested using a new test device.

Materials provided and active ingredients of main component
 1. Bioline® Malaria Ag P.f (HRP2/pLDH) individual kit contains following the items to perform the assay:
 • 25 Test strips with desiccant in individual foil pouches
 • 25 Assay diluents
 • 25 Disposable specimen applicators (5 µl), 25 Safety lancets, 25 Alcohol swabs
 • 25 Sumopaper instructions for use
 • 1 Instructions for use
 • Active ingredients of main component
 1. Test strip includes: Gold conjugate : Mouse monoclonal antibodies specific to P.f HRP2 and pLDH of P.f. Mouse monoclonal antibodies specific to P.f pLDH : gold colloid (0.25x10.05 µg), Test line 1 : Mouse monoclonal antibodies specific to P.f HRP2 (0.5x10.05 µg), Test line 2 : Mouse monoclonal antibodies specific to P.f pLDH (0.5x10.05 µg)
 2. Assay diluent includes: Bovine serum albumin (b.s.a.), 200 mM Phosphate Saline Buffer (p.s.), TRITON™ X-100 (g.s.), Sodium azide (g.s.)

Materials required but not provided
 • Micropipette, Protective gloves, Timer, Biohazard container

Kit storage and stability
 The kit should be stored at a temperature between 1°C and 40°C. Do not freeze the kit or its components.
Caution: When stored at refrigerator, kit components must be brought to room temperature before use. Do not use the kit components until they have reached room temperature.
 The test device is sensitive to both heat and humidity. Perform the test immediately after opening the sealed foil pouch.
 Do not use the test kit beyond its expiration date.
 The shelf life of the kit is as indicated on the outer package.
 Do not use the test kit if the pouch is damaged or the seal is broken.
 When transporting the kit, avoid exposure to high temperature (above 45°C) for a period longer than 1 week.

Warnings
 1. For in vitro diagnostic use only. Do not reuse the test device and kit components.
 2. The instructions must be followed exactly to achieve accurate results. The use of non-approved assay with this product must be trained in its use and must be experienced in assay procedures.
 3. Do not eat or smoke while handling specimens.
 4. Wear protective gloves while handling specimens and wash hands thoroughly afterwards.
 5. Avoid splashing or aerosol formation of specimen and assay diluent.
 6. Clean up spills thoroughly using an appropriate disinfectant.
 7. Decontaminate and dispose of all specimens, reaction kits and potentially contaminated materials in a biohazard container if you are not experienced in assay procedures.
 8. Do not mix or interchange different lot specimens.
 9. Do not mix or interchange of different lots or those for other products.
 10. Do not eat or drink after the test.
 11. Do not eat the desiccant in the foil pouch.
 12. Do not drink assay diluent.
 13. The assay diluent contains a proprietary anti-microbial agent, sodium azide, which presents no hazard to the eyes and/or skin, wash affected area with soap and water immediately. Irritation or signs of toxicity occur, seek medical attention.

Specimen collection and storage
 (Collection by venipuncture)
 1. Using venipuncture draw whole blood into the collection tube (Containing anticoagulants including heparin, EDTA and sodium citrate).
 2. If the blood specimen is not immediately tested, it must be refrigerated at 2 - 8 °C.
 3. If stored at 2 - 8 °C, the blood specimen must be tested within 1 day.
 4. Do not use a blood specimen stored for more than 3 days; it can cause a nonspecific reaction.

Performance characteristics
1. Sensitivity of Bioline® Malaria Ag P.f (HRP2/pLDH) test
 A. Internal evaluation of Bioline® Malaria Ag P.f (HRP2/pLDH) test for the detection of P.f HRP2

No. of parasitaemia of blood (x10 ⁶ parasites/ml)	Microscopy (No. of positive)	Bioline® Malaria Ag P.f (HRP2/pLDH) (No. of positive)	Sensitivity (%)
>150	15	14	93.3%
51-100	30	30	100%
101-500	83	83	100%
1001-5,000	87	87	100%
>5,000	35	35	100%
Total	312	311	99.7%

 B. Sensitivity of Bioline® Malaria Ag P.f (HRP2/pLDH) Test for the detection of pLDH specific to P.f

No. of parasitaemia of blood (x10 ⁶ parasites/ml)	Microscopy (No. of positive)	Bioline® Malaria Ag P.f (HRP2/pLDH) (No. of positive)	Sensitivity (%)
>150	15	8	53.3%
51-100	30	29	96.7%
101-500	83	83	100%
1001-5,000	87	87	100%
>5,000	35	35	100%
Total	312	311	99.7%

2. Specificity of Bioline® Malaria Ag P.f (HRP2/pLDH) test
 A. Following potentially interfering substances have no impact on the assay.

No. Potential interfering substances	No. Potential interfering substances
1. Acetaminophen	13. Acetylsalicylic acid
2. Anesthetics	14. Bilirubin (Conjugated)
3. Cephalosporins	15. Bilirubin (Unconjugated)
4. Dicyclanil	16. Caffeine
5. Erythromycin	17. Cholesterol
6. Heparin	18. Hemoglobin
7. Hydroxychloroquine sulfate	19. Insulin
8. Hydroxyurea	20. Isoniazid
9. Quinine sulfate	21. Nicotine
10. Penicillin	22. Systemic lupus erythematosus (SLE)
11. Streptomycin	23. Rheumatoid arthritis
12. Albumin	24. Sogren's syndrome

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 B. Sensitivity and specificity of Bioline® Malaria Ag P.f (HRP2/pLDH) Test for the detection of pLDH specific to P.f

No. of parasitaemia of blood (x10 ⁶ parasites/ml)	Microscopy (No. of positive)	Bioline® Malaria Ag P.f (HRP2/pLDH) (No. of positive)	Sensitivity (%)
>150	15	8	53.3%
51-100	30	29	96.7%
101-500	83	83	100%
1001-5,000	87	87	100%
>5,000	35	35	100%
Total	312	311	99.7%

4. Specificity of Bioline® Malaria Ag P.f (HRP2/pLDH) test
 A. Following potentially interfering substances have no impact on the assay.

No. Type Potential cross-reacting pathogens	No. Type Potential cross-reacting pathogens
1. Dengue	11. Borelia burgdorferi (N40 strain)
2. Yellow fever	12. Leptospira interrogans
3. Chikungunya	13. Bacetera (Cerebroamobacterias)
4. Ross River	13. Bacetera (Leptospira bilfiana/andamana)
5. Chikungunya	14. Leptospira bilfiana/andamana
6. Virus influenza B	15. Rickettsia conorii (Malish 7)
7. HIV 1 (subtype B)	16. Chente bacterium-Rickettsia (Karp)
8. Hepatitis B	17. Trypanosoma brucei gambiense
9. Hepatitis C	18. Trypanosoma brucei rhodesiense
10. Rubella	21. Plasmodium vivax
	22. Plasmodium malariae
	23. Plasmodium falciparum
	24. Sogren's syndrome

5. Specificity of Bioline® Malaria Ag P.f (HRP2/pLDH) test
 A. Following potentially interfering substances have no impact on the assay.

No. Type Potential cross-reacting pathogens	No. Type Potential cross-reacting pathogens
1. Dengue	11. Borelia burgdorferi (N40 strain)
2. Yellow fever	12. Leptospira interrogans
3. Chikungunya	13. Bacetera (Cerebroamobacterias)
4. Ross River	13. Bacetera (Leptospira bilfiana/andamana)
5. Chikungunya	14. Leptospira bilfiana/andamana
6. Virus influenza B	15. Rickettsia conorii (Malish 7)
7. HIV 1 (subtype B)	16. Chente bacterium-Rickettsia (Karp)
8. Hepatitis B	17. Trypanosoma brucei gambiense
9. Hepatitis C	18. Trypanosoma brucei rhodesiense
10. Rubella	21. Plasmodium vivax
	22. Plasmodium malariae
	23. Plasmodium falciparum
	24. Sogren's syndrome

6. Reference material
 Hepatitis A, measles, Tick borne encephalitis, Leishmania, Leptospira, M tuberculosis, Schistosoma, Toxoplasma gondii or Brucella species are not investigated for cross-reactivity. Reproducibility of Bioline® Malaria Ag P.f (HRP2/pLDH) test has been demonstrated by within-run, between-run and batch-to-batch studies using in-house reference panel acceptance criteria.
 Assay for the detection of malaria according to the level of parasite.
 Assay for the detection of malaria according to the level of parasite.
 1. Plasmodium falciparum: Sensitivity with >50 parasites/µl of blood is 100%.
 2. Plasmodium falciparum/pLDH: Sensitivity with >100 parasites/µl of blood is 100%.

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REF 05FK92

PREPARATION / PRÉPARATION / PREPARACIÓN / PREPARAÇÃO

- Open the package and look for the following:

 - Test device with desiccant in individual foil pouch
 - Assay diluent
 - Disposable specimen applicator (5 µl) (Capillary pipette or Disposable inverted cup)
 - Safety lancet
 - Alcohol swab
 - Summarized instructions for use
 - Instructions for use
- Abra el empaque y busque a continuación:

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- Next, look at the expiration date on the back of the foil pouch. If the expiration date has passed, use another lot. To avoid false results, ensure that the assay diluent used is from the same kit as the new test device.

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- Open the foil pouch and look for the following:

 - Result window
 - Specimen well
 - Assay diluent well

Then, label the device with the patient identifier.

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ENGLISH

About the test

Malaria is a serious and sometimes fatal parasitic disease characterized by fever, chills and anemia. The disease is caused by a parasite that is transmitted from one human to another through the bite of infected Anopheles mosquitoes. There are four types of malaria that can infect humans: Plasmodium falciparum, P. vivax, P. ovale and P. malariae. In humans, the parasites (called sporozoites) migrate to the liver where they and are released in another human: merozoites. The disease now occurs in more than 90 countries worldwide, and it is estimated that there are over 200 million malaria cases and deaths per year. As present malaria is diagnosed by looking for parasites in a drop of blood. Blood is placed on a microscope slide and stained so that the parasites will be visible under a microscope.

[Test principle]

Bioline[®] Malaria Ag P.f (HRP2/pLDH) test contains a membrane strip, which is pre-coated with monoclonal antibodies specific to histidine rich protein II (HRP2) and the lactate dehydrogenase of P. falciparum on the test band region. This test device has a letter of "T1", "T2" (test line) and "C" (control line) on the surface of the device. Neither the test lines nor the control line is visible until you apply a specimen to the device. The control line used for procedural control and should always appear if the test procedure is performed correctly.

[Intended Use]

Bioline[®] Malaria Ag P.f (HRP2/pLDH) test is a rapid, qualitative test for the detection of histidine-rich protein II (HRP2) antigen and lactate dehydrogenase from Malaria Plasmodium falciparum in human whole blood. Bioline[®] Malaria Ag P.f (HRP2/pLDH) test kit is intended only for professional use and for in vitro diagnostic use. Testive specimens should be confirmed by a supplemental assay such as microscopic examination of thin blood smear.

Materials provided and active ingredients of main component

- Bioline[®] Malaria Ag P.f (HRP2/pLDH) test kit contains following the items to perform the assay:
 - 25 Test device with desiccant in individual foil pouches
 - 25 Disposable specimen containers (5 µl), 25 Sterile Lancets, 25 Alcohol swabs
 - 25 Summarized instructions for use
- Active ingredients of main component
 - 1 Test strip includes: Gold conjugate, Mouse monoclonal antibodies specific to P.f HRP2 (gold conjugate) (0.10,0.10,0.02 µg), Mouse monoclonal antibodies specific to P.f pLDH (gold conjugate) (0.20,0.10,0.05 µg), Test Line 1: Mouse monoclonal antibodies specific to P.f HRP2 (0.5,0.5,0.1 µg), Test Line 2: Mouse monoclonal antibodies specific to P.f pLDH (0.5,0.1 µg), Control Line: Goat anti-mouse IgG (0.10,0.2 µg)
 - 25 Assay buffer including: 25mM ammonium (g.s.), 200 mM Phosphate Saline Buffer (g.s.), TRITON[®] X-100 (g.s.), Sodium azide (g.s.)

Materials required but not provided

- Microscope, Protective gloves, Timer, Biohazard container

Kit storage and stability

- The test kit should be stored at a temperature between 1°C and 40°C. Do not freeze the kit or its components.
- Caution:** When stored at refrigerator, all kit components must be brought to room temperature (15 - 40°C) minimum 30 min prior to the test. Do not open the pouch without any warning. If the test kit is not used immediately, it should be stored at 2 - 8°C for a period longer than 1 week.

Warnings

- For in vitro diagnostic use only. Do not reuse the test kit and its components.
- Do not use once. Once the test kit is used, it should be discarded. Any individual performing an assay with this product must be trained in its use and must be experienced in laboratory procedures.
- Do not eat or smoke while handling specimens.
- Wear protective gloves while handling specimens and wash hands thoroughly afterwards.
- Avoid splashing or aerosol formation of specimen and assay diluent.
- Clean up spill thoroughly using appropriate disinfectant.
- Decomantate and dispose of all specimens, reaction kits and potentially contaminated materials (i.e. lancet, specimen applicator, test device) in a biohazard container as they may be infectious.
- Do not mix or interchange different lots or those for other products.
- Do not pipette by mouth.
- Do not use the desiccant in the foil pouch.
- Do not drink any diluent.
- The assay diluent contains a proprietary anti-microbial agent, sodium azide, which presents a hazard to the user if normal laboratory precautions are followed. If contact with assay diluent to the eyes and/or skin, wash affected area with soap and water immediately. If irritation or signs of toxicity occur, seek medical attention.

Specimen collection and storage

- Using venipuncture, draw whole blood into the collection tube containing anticoagulants including heparin, EDTA and sodium citrate. Do not use EDTA or sodium citrate and test results may be affected or be affected by anticoagulant factors and/or user error. The subject of the diagnosis should consult a doctor for further confirmation of the test result.
- Do not use a blood specimen stored for more than 3 days; it can cause a nonspecific reaction.

- Washing: The manufacturers and distributors of this product shall not be liable for any direct, indirect, or consequential losses, liability, claims, costs or damages arising from or related to an incorrect positive or negative diagnosis using this product.

Performance characteristics

- Intervel of evaluation of Bioline[®] Malaria Ag P.f (HRP2/pLDH) test A. Internal validity of Bioline[®] Malaria Ag P.f (HRP2/pLDH) test for the detection of P.f HRP2
- | No. of parasitiferal of blood | Microscopy (No. of positive) | Bioline [®] Malaria Ag P.f (HRP2/pLDH) (No. of positive) | Sensitivity |
|-------------------------------|------------------------------|---|-------------|
| 1-50 | 15 | 14 | 93.3% |
| 51-100 | 30 | 30 | 100% |
| 101-500 | 83 | 83 | 100% |
| 501-1,000 | 87 | 87 | 100% |
| 1,001-5,000 | 62 | 62 | 100% |
| >5,000 | 35 | 35 | 100% |
| Total | 312 | 311 | 99.7% |
- Specificity of Bioline[®] Malaria Ag P.f (HRP2/pLDH) Test for the detection of pLDH specific to P.f
- | No. of parasitiferal of blood | Microscopy (No. of positive) | Bioline [®] Malaria Ag P.f (HRP2/pLDH) (No. of positive) | Sensitivity |
|-------------------------------|------------------------------|---|-------------|
| 1-50 | 15 | 8 | 53.3% |
| 51-100 | 30 | 29 | 96.7% |
| 101-500 | 83 | 83 | 100% |
| 501-1,000 | 87 | 87 | 100% |
| 1,001-5,000 | 62 | 62 | 100% |
| >5,000 | 35 | 35 | 100% |
| Total | 312 | 304 | 97.4% |
- Following potentially interfering substances have no impact on the assay.
- | No. Potential interfering substances | No. Potential interfering substances |
|--------------------------------------|---------------------------------------|
| 1 Acetaminophen | 13 Acetylsalicylic acid |
| 2 Amoxicillin | 14 Bilelirubin (Conjugate) |
| 3 Cephalosporin | 15 Bilirubin (Unconjugate) |
| 4 Dicyclanide | 16 Caffeine |
| 5 Erythromycin | 17 Cholesterol |
| 6 Heparin | 18 Hemoglobin |
| 7 Hydrochloroquine sulfate | 19 Insulin |
| 8 Ibufuprofen | 20 Tricyclenides |
| 9 Quinine sulfate | 21 Nicotine |
| 10 Penicillin | 22 Systemic lupus erythematosus (SLE) |
| 11 Streptomycin | 23 Rheumatoid arthritis |
| 12 Albumin | 24 Sjogren's syndrome |
- To evaluate the interference of Bioline[®] Malaria Ag P.f (HRP2/pLDH) test kit with known relevant interfering substances, the hemolytic specimens, untreated factors contained specimens and specific, cross-species were investigated. In these tests, those specimens did not interfere with Bioline[®] Malaria Ag P.f (HRP2/pLDH) test.
- | No. Type | Potential cross-reacting pathogens | No. Type | Potential cross-reacting pathogens |
|----------|------------------------------------|----------|-------------------------------------|
| 1 | Dengue | 11 | Borrelia burgdorferi (N40 strain) |
| 2 | Yellow fever | 12 | Leptospira interrogans |
| 3 | Chikungunya | 13 | Leptospira biflexans (farmans) |
| 4 | Chikungunya | 14 | Leptospira interrogans |
| 5 | Influenza A | 15 | Rickettsia conorii (Malish 7) |
| 6 | Influenza B | 16 | Chlamydia trachomatis (Bikolan/Kep) |
| 7 | HTLV (Type/B) | 17 | Yersinia enterocolitica |
| 8 | Hepatitis B | 18 | Yersinia enterocolitica |
| 9 | Hepatitis C | 19 | Leishmania donovani |
| 10 | Herpes | 20 | Yersinia enterocolitica |
| 20 | Proteus | 21 | Yersinia enterocolitica |
| 21 | Phaenocarpa | 22 | Phaenocarpa |
| 22 | Phaenocarpa | 23 | Phaenocarpa |
| 23 | Phaenocarpa | 24 | Phaenocarpa |
- Following potentially cross-reacting microorganism have no impact on the assay.
- | No. Type | Potential cross-reacting pathogens | No. Type | Potential cross-reacting pathogens |
|----------|------------------------------------|----------|-------------------------------------|
| 1 | Dengue | 11 | Borrelia burgdorferi (N40 strain) |
| 2 | Yellow fever | 12 | Leptospira interrogans |
| 3 | Chikungunya | 13 | Leptospira biflexans (farmans) |
| 4 | Chikungunya | 14 | Leptospira interrogans |
| 5 | Influenza A | 15 | Rickettsia conorii (Malish 7) |
| 6 | Influenza B | 16 | Chlamydia trachomatis (Bikolan/Kep) |
| 7 | HTLV (Type/B) | 17 | Yersinia enterocolitica |
| 8 | Hepatitis B | 18 | Yersinia enterocolitica |
| 9 | Hepatitis C | 19 | Leishmania donovani |
| 10 | Herpes | 20 | Yersinia enterocolitica |
| 20 | Proteus | 21 | Yersinia enterocolitica |
| 21 | Phaenocarpa | 22 | Phaenocarpa |
| 22 | Phaenocarpa | 23 | Phaenocarpa |
| 23 | Phaenocarpa | 24 | Phaenocarpa |
- Total Sensitivity and Specificity of Bioline[®] Malaria Ag P.f (HRP2/pLDH) Test
- | Reference method (Microscopic examination) | Result of Bioline [®] Malaria Ag P.f (HRP2/pLDH) | Positive | Negative | Total | Sensitivity (95% CI) |
|--|---|----------|----------|-------------------|----------------------|
| HRP2 (T1) | 2 | 298 | 300 | 99.3 (98 - 100 %) | |
| pLDH (T2) | 1 | 299 | 300 | 99.7 (98 - 100 %) | |
- Total Sensitivity and Specificity of Bioline[®] Malaria Ag P.f (HRP2/pLDH) Test
- | Reference method (Microscopic examination) | Result of Bioline [®] Malaria Ag P.f (HRP2/pLDH) | Positive | Negative | Total | Sensitivity (95% CI) |
|--|---|----------|----------|-------------------|----------------------|
| Pf Positive | 311 | 1 | 312 | 99.7 (98 - 100 %) | |
| Pf Negative | 2 | 298 | 300 | 99.3 (98 - 100 %) | |

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ENGLISH

Intended use

The Bioline™ Malaria Ag P.f (HRP2/pLDH) test is a rapid, qualitative and differential test for the detection of HRP2 (Histidinerich protein II) antigen and pLDH (Plasmodium lactate dehydrogenase) from Malaria *Plasmodium falciparum* in human blood specimen.

REF 05FK92

Materials provided

- Test device
- Assay diluent
- Inverted cup (5 µl)
- Safety lancet
- Alcohol swab

Kit storage and stability

1. The test kit should be stored at a temperature between 1 °C and 40 °C. Do not freeze the kit or its components.
Note : When stored at refrigerator, all kit components must be brought to room temperature (15 - 40 °C) minimum 30 minutes prior to the test. Do not open the pouch whilst components come to room temperature.
2. The test device is sensitive to both heat and humidity. Perform the test immediately after removing the test device from the foil pouch.
3. Do not use test kit beyond its expiration date.
4. The shelf life of the kit is as indicated on the outer package.
5. Do not use the test kit if the pouch is damaged or the seal is broken.

6. When transporting or storing the kit, avoid exposure to high temperature (above 45 °C) for a period longer than 1 week.

Specimen collection using a lancet

1. Clean the area to be lanced with an alcohol swab.
2. Squeeze the end of the fingertip and pierce with a safety lancet provided.
3. Wipe away the first drop of blood with sterile gauze or cotton.
4. Using a disposable inverted cup (5 µl) provided, dip the circular end of an inverted cup into the blood specimen.

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Authorized Representative
MT Promedt Consulting GmbH

Altenhofstrasse 80, 66386 St. Ingbert, Germany
Phone : +49 6894 581020
Fax : +49 6894 581021

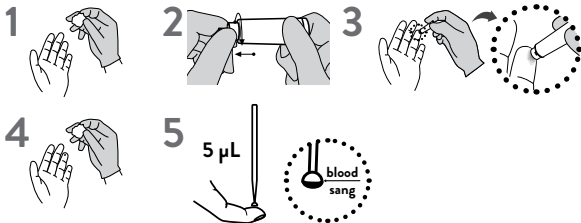
© 2021 Abbott. All rights reserved.
All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners.

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
TEST PROCEDURE / PROCÉDURE DE TEST

- EN** Dispense 5 µl of drawn blood into round specimen well.

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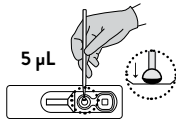
- EN** Dispense all of the assay diluent from the diluent tube into the square well of test device.

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EN **Caution:** Do not read test results after 30 minutes.; reading after 30 minutes can yield erroneous results

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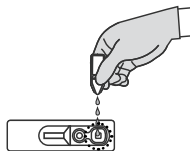- EN** "C" and "T2" (Two lines)
- *Possible cases**

- pf- HRP2 gene deletions
- Prozone effect (excess of HRP2 Ag)

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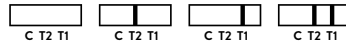
ENGLISH

Intended Use

The Bioline™ Malaria Ag P.f(HRP2/pLDH) test is a rapid, qualitative and differential test for the detection of HRP2 (Histidinerich protein II) antigen and pLDH (Plasmodium lactate dehydrogenase) from Malaria *Plasmodium falciparum* in human blood specimen.

REF 05FK93

Materials provided

- Test device
- Assay diluent
- Inverted cup (5 µl)
- Sterile lancet
- Alcohol swab

Kit storage and stability

1. The test kit should be stored at a temperature between 1 °C and 40 °C. Do not freeze the kit or its components.

Note : When stored at refrigerator, all kit components must be brought to room temperature (15 - 40 °C) minimum 30 mins prior to the test. Do not open the pouch whilst components come to room temperature.

2. The test device is sensitive to both heat and humidity. Perform the test immediately after removing the test device from the foil pouch.
3. Do not use test kit beyond its expiration date.
4. The shelf life of the kit is as indicated on the outer package.
5. Do not use the test kit if the pouch is damaged or the seal is broken.
6. When transporting or storing the kit, avoid exposure to high temperature (above 45 °C) for a period longer than 1 week.

Specimen collection using a lancet

1. Clean the area to be lanced with an alcohol swab.
2. Squeeze the end of the fingertip and pierce with a sterile lancet provided.
3. Wipe away the first drop of blood with sterile gauze or cotton.
4. With a disposable inverted cup (5 µl) provided, dip the circular end of an inverted cup into the blood specimen.

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