WHO Prequalification of In Vitro Diagnostics Programme PUBLIC ASSESSMENT REPORT

Product: Bioline Malaria Ag P.f/Pan
PQDx Reference Number: PQDx 0030-012-01

Bioline Malaria Ag P.f/Pan, with product codes 05FK60, 05FK63, 05FK61, 05FK062, and 05FK67, manufactured by Abbott Diagnostics Korea Inc., the CE-marked regulatory version was accepted for the WHO list of prequalified in vitro diagnostics and was listed on 08 July 2013.

Summary of prequalification status for the Bioline Malaria Ag P.f/Pan

	Date	Outcome
Status on PQ list	08 July 2013	listed
Dossier assessment	01 March 2011	MR
Product performance evaluation	Round 3 (2011) and Round 5(2014)	MR

MR: Meets Requirements

Report amendments and product changes

This public report has since been amended. Amendments may have arisen because of changes to the prequalified product, for which the WHO has been notified and has undertaken a review. The amendments to the report are summarized in the following table, and details of each amendment are provided below.

Version	Summary of amendment and change request reference, where applicable.	Date of report amendment
1.0-7.0	Editorial amendments and addition of a product code 05FK63	02 May 2016
8.0	Addition of product codes 05FK61 and 05FK062.	22 December 2016
9.0	Addition of product code 05FK67.	31 October 2017
10.0	Changes to the manufacturer name from Standard Diagnostics, Inc to Abbott Diagnostics Korea Inc and product name from SD BIOLINE Malaria Ag P.f/Pan and SD BIOLINE Malaria Ag P.f/Pan POCT to Bioline Malaria P.f/Pan.	20 August 2020

11.0	Addition of a manufacturing site for device assembly, pouch sealing and packaging process of Malaria Ag products.	17 May 2022
12.0	Accessory labelling changes according to accessory manufacturer information change (Manufacturer name change and EU representative address change) (PQC-IVD-2024-0034). IFU and Package version update from A1 to A2. Changes are limited to administrative changes, changes to correct errors, and clarification of existing information without changing the procedure and do not impact quality, safety or performance (PQC-IVD-2025-0046).	20 March 2026
13.0	Label update of the LATEX FREE GLOVE component used in the Bioline Malaria Ag P.f/Pan product (PQC-IVD-2025-0059).	12 May 2026

Intended use:

According to the claim of intended use from Abbott Diagnostics Korea Inc, *“Bioline Malaria Ag P.f/Pan test kit intended for the detection of Malaria infection in human blood specimen, indicating qualitative and differential diagnosis between HRP2(Histidine-rich protein II) specific to Plasmodium falciparum and pLDH (Plasmodium lactate dehydrogenase) specific to Plasmodium species (Pan) in human blood specimen. Bioline Malaria Ag P.f/Pan test is intended for professional use, only for an initial screening test and reactive specimens should be confirmed by a supplemental assay such as microscopic examination of thin blood smear”.*

Test kit contents:

	25T/kit (product code 05FK60)	1T/kit x 25 each (product code 05FK63)	25T/kit (product code 05FK61)	1T/kit x 25 each (product code 05FK62)	30T/kit (product code 05FK67)
Test cassettes individually packed in foil pouch with a desiccant	25 test devices	25x 1 test device	25 test devices	25x 1 test device	30 test devices
Assay diluent dispensed in plastic bottle	1 x 5 mL/bottle	25 x 180 μ L/vial	1 x 5 mL/bottle	25 x 180 μ L/vial	30x 180 μ L/vial
Specimen transfer devices disposable (5 μ L)	25 units of 5 μ L	25 pouches each containing one specimen transfer device (5 μ L), one lancet, one alcohol swab	25 units of 5 μ L	25 pouches each	30 units of 5 μ L
Lancets disposable, sterile	25 units		25 units (safety lancets)	containing one specimen	30 units
Alcohol swabs disposable	25 units		25 units	transfer device (5 μ L), one safety lancet, one alcohol swab	30 units
Dried swab disposable, sterile	N/A	N/A	N/A	N/A	30 units
Latex glove disposable	N/A	N/A	N/A	N/A	30x 2 packs
Instructions for use	1 copy	1 copy	1 copy	1 copy	1 copy
Summarized instructions for use	N/A	25 copies	N/A	25 copies	N/A

Storage:

The test kit must be stored between 1 and 40 °C.

Shelf-life upon manufacture¹:

24 months.

Dossier assessment

Abbott Diagnostics Korea Inc submitted a product dossier for Bioline Malaria Ag P.f/Pan as per the *Instructions for compilation of a product dossier* (PQDx_018 v1). The information submitted in the product dossier was reviewed by WHO staff and external experts (assessors) appointed by WHO in accordance with the internal report on the screening and assessment of a product dossier (PQDx_009 v2).

Based on the product dossier screening and assessment findings, the Bioline Malaria Ag P.f/Pan meets prequalification requirements.

Manufacturing site inspection

The inspection of the manufacturing site(s) was conducted to assess whether the manufacturer's quality management system (QMS) and manufacturing practices are in alignment with:

- (i) applicable international standards, such as ISO 13485 (Medical devices – Quality management systems – Requirements for regulatory purposes);
- (ii) the manufacturer's own documented procedures and quality requirements; and
- (iii) other relevant international standards and guidelines applicable to in vitro diagnostic (IVD) medical devices. The WHO's Public Inspection Reports are accessible at:

<https://extranet.who.int/pgweb/vitro-diagnostics/who-public-inspection-reports>

Product performance evaluation

Bioline Malaria Ag P.f/Pan was submitted to Round 3 and Round 5 of WHO Product Testing Programme in 2011 and 2014². The product was evaluated against a *Plasmodium falciparum* cultured line panel, *P. falciparum* wild type parasite panel, *P. vivax* wild type parasite panel and a mixed panel of *P.falciparum* and non- *P.falciparum* to determine specificity. Thermal stability was assessed after 2 months of storage at elevated temperature and humidity, and a descriptive ease of use assessment was recorded.

¹ The assigned device shelf-life is based on stability data generated from the date of manufacture. The finished goods shelf-life, calculated from the date of packaging completion, may be shorter depending on the time elapsed between manufacture and final packaging of the device.

² The same product was also submitted to Round 3 Product Testing.

In Round 3, the following results were observed: *P. falciparum* panel detection score (92.9% at 200 parasites/μL), *P. vivax* panel detection score (97.1% at 200 parasites/μL), false-positive rates (3.5% for clean negatives, 0.5 % for *P. falciparum* at 200 parasites/μL, 0 % for *P. vivax* at 200 parasites/μL, 0.5% for *P. falciparum* at 2000 or 5000 parasites/μL, 0% for *P. vivax* at 2000 or 5000 parasites/μL) and invalid rate (0.3%)

In Round 5, the following results were observed: *P. falciparum* panel detection score (94.0% at 200 parasites/μL), *P. vivax* panel detection score (91.4% at 200 parasites/μL), false-positive rates (0.0% for clean negatives, 0.8% for *P. falciparum* at 200 parasites/μL, 0.7% for *P. vivax* at 200 parasites/μL, 0.5% for *P. falciparum* at 2000 or 5000 parasites/μL, 1.4% for *P. vivax* at 2000 or 5000 parasites/μL) and invalid rate (0.0%), Bioline Malaria Ag P.f/Pan meets the current laboratory evaluation requirements for prequalification.

Summary performance characteristics	Panel detection score		False positive rate (%)			Invalid rate (%)
	200 parasites/μL		200 parasites/μL		Clean negatives	
	Pf	Pv	Pf	Pv		
Round 3	92.9%	97.1%	0.5	0	3.5	0.3
Round 5	94.0%	91.4%	0.8	0.7	0.0	0.0

Labelling review

The labelling submitted for Bioline Malaria Ag P.f/Pan was reviewed by WHO staff and external technical experts appointed by WHO. The review evaluated the labelling for clarity and consistency with the information submitted in the product dossier, alignment with international guidance and standards, and suitability for the intended users and settings in WHO Member States, including low- and middle-income countries.

The table below provides traceability of the labelling documents reviewed during the assessment, including document titles, version numbers, approval dates, and control identifiers.

Controlled Labelling References**05FK60**

Document Type	Document Title	Version / Revision	Date Approved	Controlled Document No.
Outer box artwork	Package	A1	2021-05-06	05FK60-40-A1
Pouch / Device label	Device pouch label	A0	2019-11-01	FP14-A0
Assay diluent labels	Assay diluent labels	A0	2019-10-25	FB07-A0
Inverted cup	40008361	0	2025-09-25	D10151501
Sterile lancet	P65-008	0	2023-11-10	D10151501
Sterile lancet	40006487	0	2024-10-01	D10151501
Instructions for Use (IFU)	IFU	A3	2025-09-30	05FK60-04-A3

05FK61

Document Type	Document Title	Version / Revision	Date Approved	Controlled Document No.
Outer box artwork	Package	A1	2021-05-06	05FK61-40-A1
Pouch / Device label	Device pouch label	A0	2019-11-01	FP14-A0
Assay diluent labels	Assay diluent labels	A0	2019-10-25	FB07-A0
Inverted cup	40008361	0	2025-09-25	D10151501
Sterile safety lancet	P65-013/000/001	0	2023-11-10	D10151501
IFU	IFU	A3	2025-09-30	05FK61-04-A3

05FK62

Document Type	Document Title	Version / Revision	Date Approved	Controlled Document No.
Outer box artwork	Outer box artwork	A1	2021-05-06	05FK62-40-A1
Pouch / Device label	Device pouch label	A0	2019-11-01	FP14-A0
POCT Pouch	Pouch label	A0	2019-09-09	05FK62-P-A0
Inverted cup*	40005865	N/A	N/A	N/A
Sterile safety lancet*	40005867	N/A	N/A	N/A
Ampule*	P60-009/009	N/A	N/A	N/A
IFU*	Summarized IFU	A2	2021-03-11	05FK62-02-C-A2
IFU	IFU	A3	2025-09-30	05FK62-04-A3
Reference Buffer guide	Malaria Buffer guide	A0	2022-02-07	05FK00-IC-A0

* These items are packaged together in a POCT pouch

05FK63

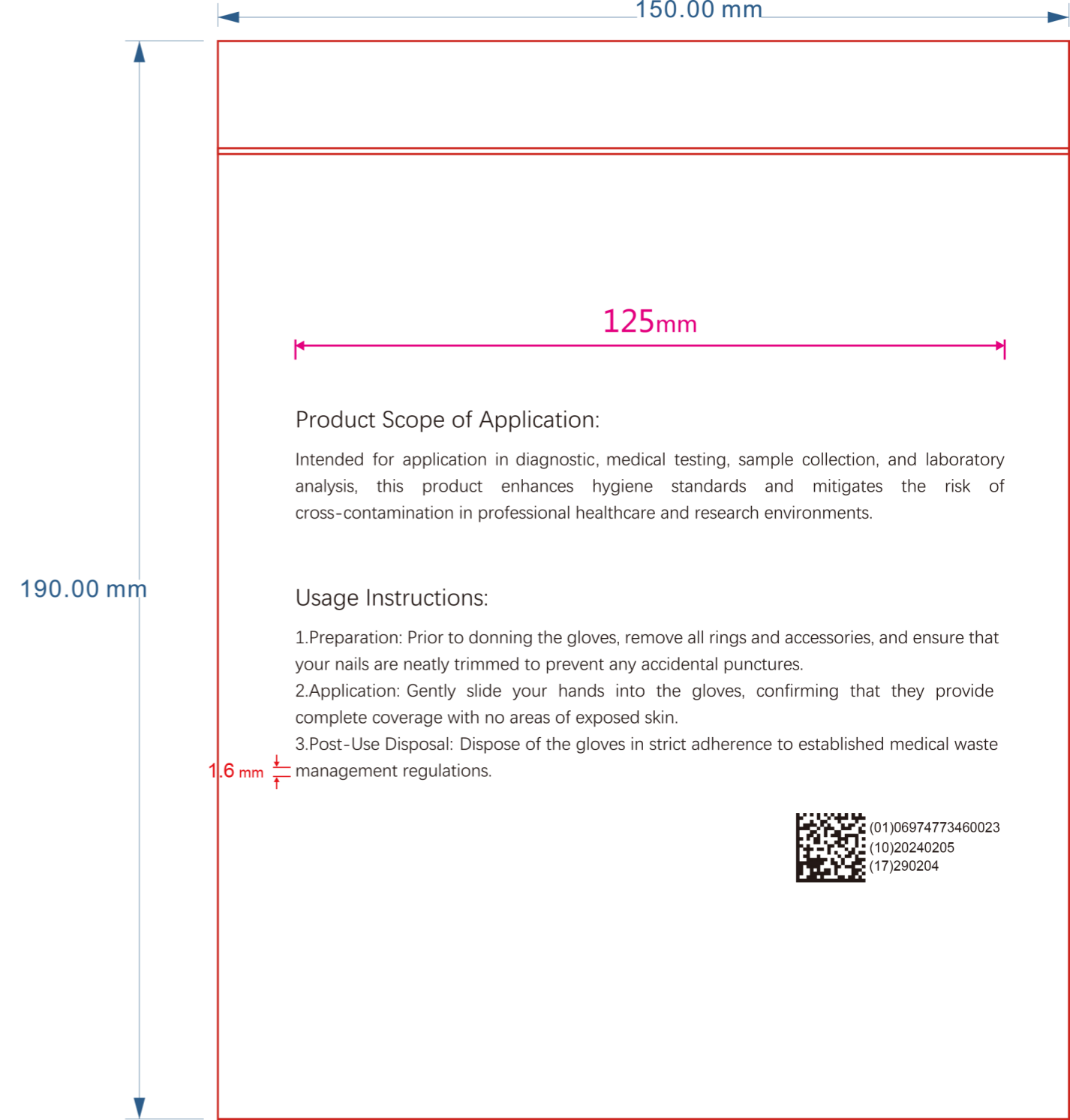
Document Type	Document Title	Version / Revision	Date Approved	Controlled Document No.
Outer box artwork	Outer box artwork	A1	2021-05-06	05FK63-40-A1
Pouch / Device label	Device pouch label	A0	2019-11-01	FP14-A0
POCT Pouch	Pouch label	A0	2019-09-09	05FK63-P-A0
Inverted cup*	40005865	N/A	N/A	N/A
Sterile lancet*	40005864	N/A	N/A	N/A
Sterile lancet*	40009512	N/A	N/A	N/A
Ampule*	P60-009/009	N/A	N/A	N/A
IFU*	Summarized IFU	A3	2025-09-30	05FK63-04-A3
IFU	IFU	A2	2021-03-11	05FK63-02-C-A2
Reference Buffer guide	Malaria Buffer guide	A0	2022-02-07	05FK00-IC-A0

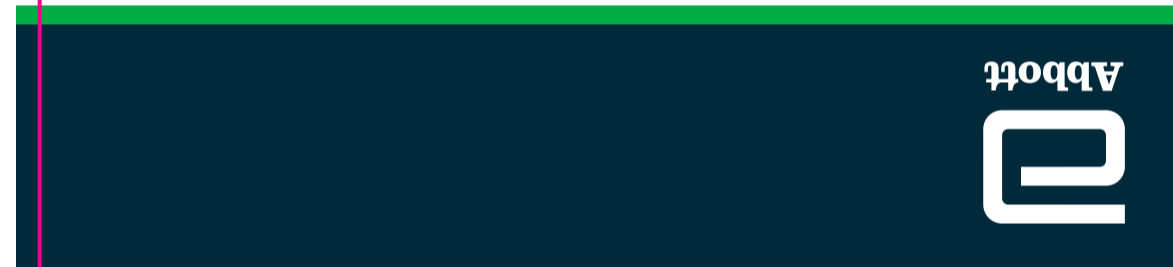
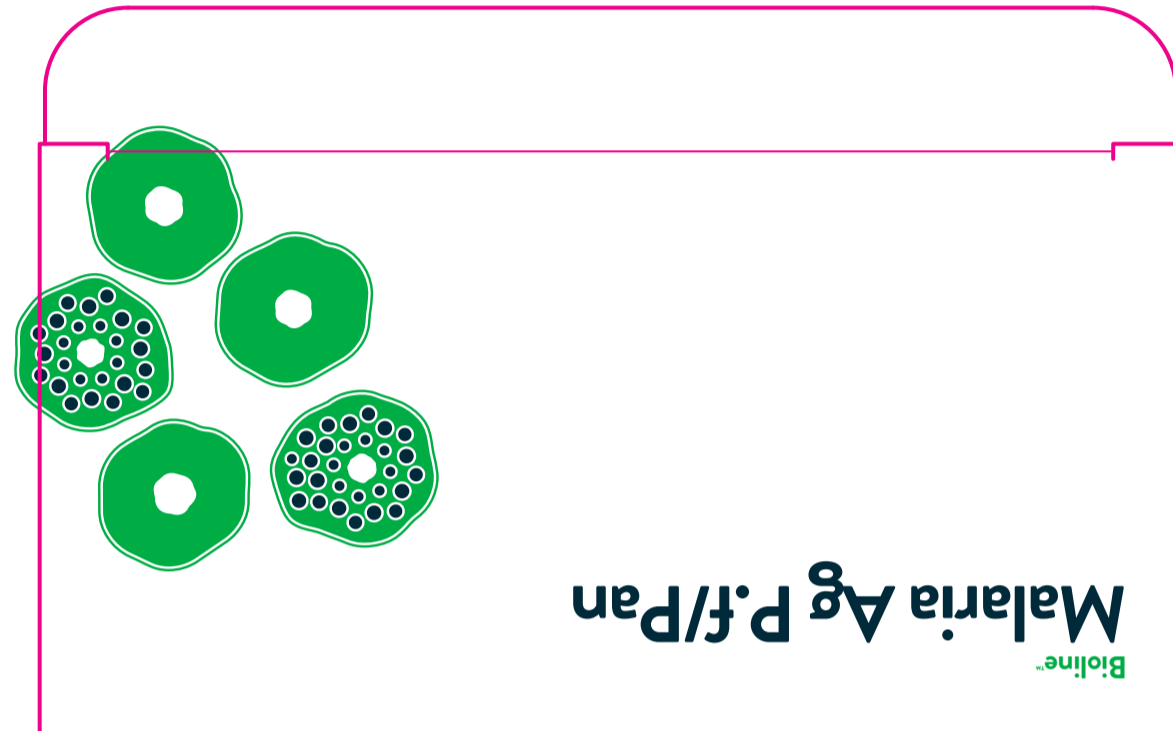
* These items are packaged together in a POCT pouch

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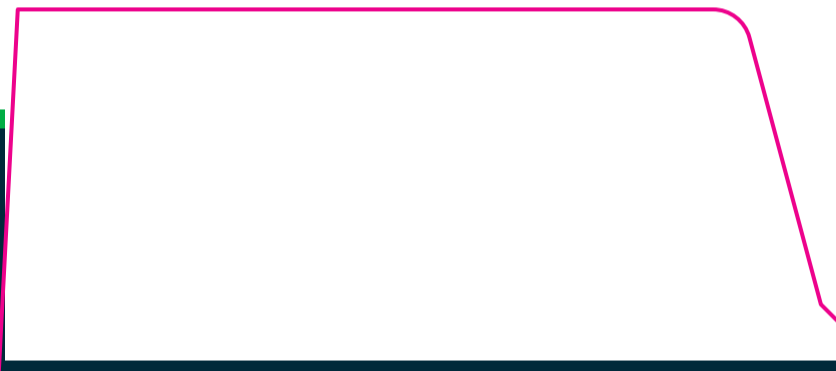
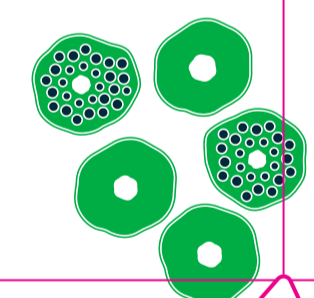
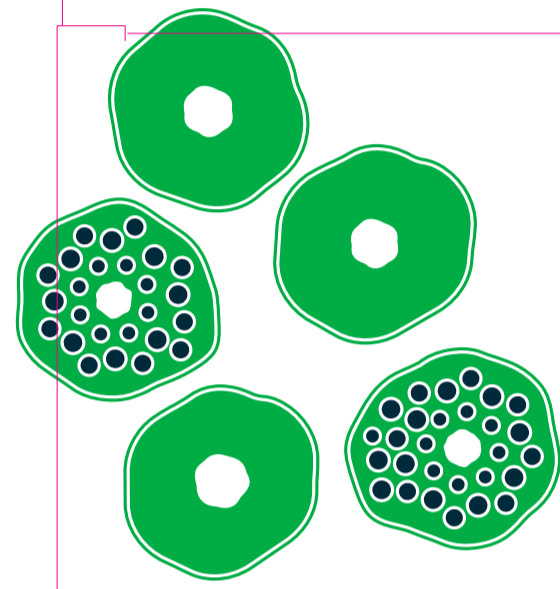
Abbott
Bioline™
Malaria Ag P.f./Pan

EC REP
MT Promed Consulting GmbH
Altenhofstrasse 80,
66386 St. Ingbert, Germany

Abbott Diagnostic Korea Inc.
65, Borahagel-ro, Gheung-gu, Yangin-si,
Gyeonggi-do, 17099, Republic of Korea
globalpointofcare@abbott

© 2021 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners.

Malaria Ag P.f/Pan

EN
Rapid test for the detection of Malaria infection in human whole blood, indicating differential diagnosis between *Plasmodium falciparum* histidine-rich protein 2 (Pf HRP2) and the other *Plasmodium* species (Pan, pLDH) (*P. vivax*, *P. malariae* or *P. ovale*)

CONTENTS :
• 25 Test devices with desiccant in individual foil pouch
• 25 Assay diluents
• 25 Disposable specimen applicators (5 µl),
• 25 Safety lancets, 25 Alcohol swabs
• 25 Summarized instructions for use
• 1 Instructions for use

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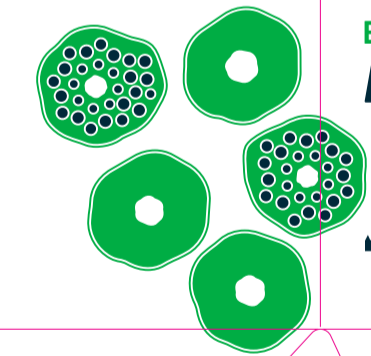
PRINT AREA

Bioline[™] Malaria Ag P.f/Pan


Barcode and regulatory symbols: 1°C 104°F, 25, 1, 2, 3, 4, 5, 6, 7, 8, 9, 10, 11, 12, 13, 14, 15, 16, 17, 18, 19, 20, 21, 22, 23, 24, 25, 26, 27, 28, 29, 30, 31, 32, 33, 34, 35, 36, 37, 38, 39, 40, 41, 42, 43, 44, 45, 46, 47, 48, 49, 50, 51, 52, 53, 54, 55, 56, 57, 58, 59, 60, 61, 62, 63, 64, 65, 66, 67, 68, 69, 70, 71, 72, 73, 74, 75, 76, 77, 78, 79, 80, 81, 82, 83, 84, 85, 86, 87, 88, 89, 90, 91, 92, 93, 94, 95, 96, 97, 98, 99, 00, REF 05FK62

Bioline[™] Malaria Ag P.f/Pan

EC REP
Abbott Diagnostic Korea Inc.
65, Borahgal-ro, Gyeongju, Yongsin-si,
Gyeonggi-do, 17079, Republic of Korea
globalpointofcare@abbott.com

PRODUCT - Bioline™ Malaria Ag Pf	<div style="display: flex; justify-content: space-between;"> <div style="width: 20px; height: 10px; background-color: #004a99;"></div> PMS 303 C <div style="width: 20px; height: 10px; background-color: #008000;"></div> PMS 354 C </div>	PN. - 05FK62-40-A1 REV. DATE - 2021.05.07 CREATOR - THINKBOX
SIZE - 220 X 130 X 70MM		

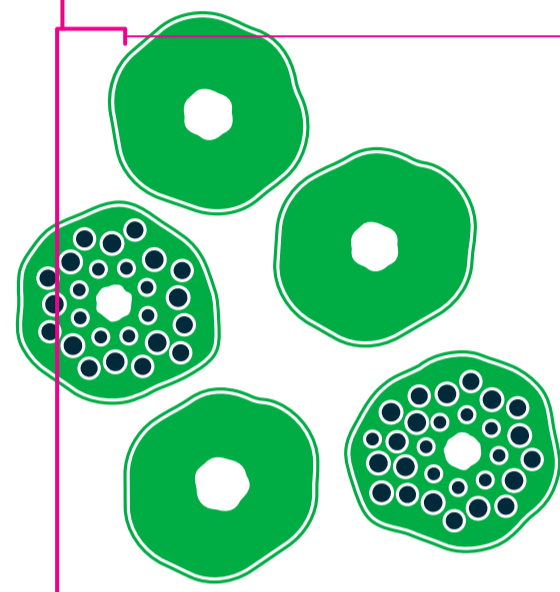Bioline
Malaria Ag P.f/Pan

Bioline
Malaria Ag P.f/Pan

Bioline
Malaria Ag P.f/Pan

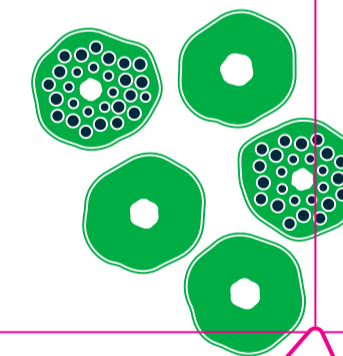
PRINT AREA

Bioline
Malaria Ag P.f/Pan

1°C 34°F 40°C 104°F 25 1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19 20 21 22 23 24 25 26 27 28 29 30 31 32 33 34 35 36 37 38 39 40 41 42 43 44 45 46 47 48 49 50 51 52 53 54 55 56 57 58 59 60 61 62 63 64 65 66 67 68 69 70 71 72 73 74 75 76 77 78 79 80 81 82 83 84 85 86 87 88 89 90 91 92 93 94 95 96 97 98 99 100
CE IVD REF 05FK63

Bioline
Malaria Ag P.f/Pan

EC REP

Abbott Diagnostic Korea Inc.
65, Borahae-ro, Gyeongju, Yongin-si,
Gyeonggi-do, 17079, Republic of Korea
globalpost@care.abbott

© 2021 Abbott. All rights reserved. All trademarks
referenced are trademarks of either the Abbott group of
companies or their respective owners.

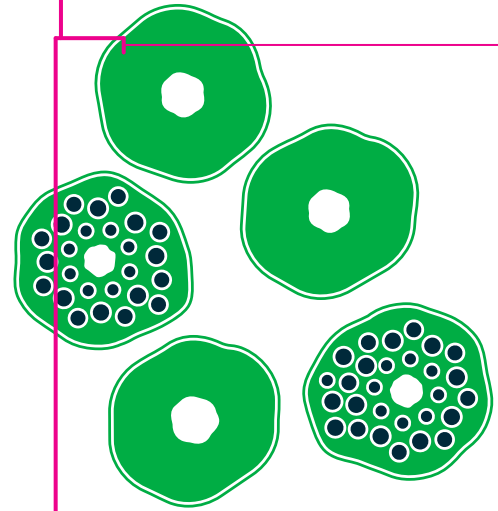
PRODUCT - Bioline™ Malaria Ag Pf

PMS 303 C
PMS 354 C

PN. - 05FK63-40-A1
REV. DATE - 2021.05.07
CREATOR - THINKBOX

SIZE - 220 X 130 X 70MM

GoR-Ministry of Health : NOT FOR SALE

Bioline[™]
Malaria Ag P.f/Pan

Bioline[™]
Malaria Ag P.f/Pan

Bioline[™]
Malaria Ag P.f/Pan

PRINT AREA

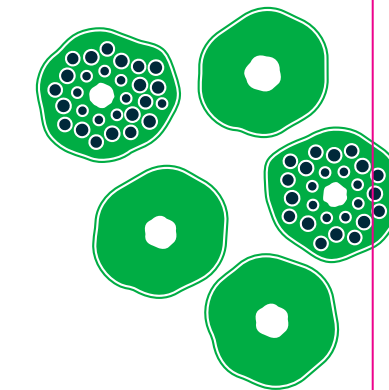
Bioline[™]
Malaria Ag P.f/Pan

1°C / 40°C
34°F / 104°F

▽₀ IVD REF 05FK67

Bioline[™]
Malaria Ag P.f/Pan

EC REP

Abbott Diagnostic Korea Inc.
65, Borhegri-ro, Gyeonggi-do, Yongin-si,
Gyeonggi-do, 17099, Republic of Korea
globalpoint@care.abbott

© 2021 Abbott. All rights reserved. All trademarks
referenced are trademarks of either the Abbott group of
companies or their respective owners.

PRODUCT - Bioline[™] Malaria Ag P.f/Pan
30T_RWANDA

PMS 303 C
 PMS 354 C
 PMS 185 C

PN. - 05FK67-40-RW-A1
REV. DATE - 2021. 05.07
CREATOR - THINKBOX

SIZE - 230 X 125 X 90MM

PRODUCT - Bioline™ AL pouch 1-40 °C

SIZE - 108 X 32 MM

PMS 303 C

PMS 2925 C

CAT NO. - FP14-A0

REV. DATE - 2019.10.28

CREATOR - SCOTT KIM

Bioline™
RAPID DIAGNOSTIC TEST

 Abbott Diagnostics Korea Inc.
65, Borahagal-ro, Giheung-gu, Yongin-si,
Gyeonggi-do, 17099, Republic of Korea

1°C - 40°C
34°F - 104°F

FP14-A0

ITEM ITEM: Bioline™

LOT LOT NO.:

MFG.DATE (YYYY.MM.DD)
EXP.DATE (YYYY.MM.DD)

11mm

PRODUCT - Bioline™ Malaria Ag P.f/Pan
(Safety lancet, POCT spec) Outer pouch

SIZE - 145 x 50mm

 PMS 303 C
 PMS 354 C

CAT NO. - 05FK62-P-A0
REV. DATE - 2019. 09. 03
CREATOR - THINKBOX

Bioline™
Malaria Ag P.f/Pan

Malaria HRP-II(P.f) and pLDH(Pan) Antigen Rapid Test

CONTENTS : Test device, Assay diluent,
Disposable specimen applicator (5 µl), Safety lancet, Alcohol swab,
Summarized instructions for use

 **Abbott Diagnostics Korea Inc.**
65, Borahagel-ro, Giheung-gu, Yongin-si,
Gyeonggi-do, 17099, Republic of Korea

1°C / 34°F  40°C / 104°F

REF 05FK62

05FK62-P-A0

PRODUCT - Bioline™ Malaria Ag P.f/Pan
(POCT spec) Outer pouch

SIZE - 145 x 50mm

 PMS 303 C
 PMS 354 C

CAT NO. - 05FK63-P-A0
REV. DATE - 2019. 09. 03
CREATOR - THINKBOX

Bioline™
Malaria Ag P.f/Pan

Malaria HRP-II(P.f) and pLDH(Pan) Antigen Rapid Test

CONTENTS : Test device, Assay diluent,
Disposable specimen applicator (5 µl), Sterile lancet, Alcohol swab,
Summarized instructions for use

 **Abbott Diagnostics Korea Inc.**
65, Borahagel-ro, Giheung-gu, Yongin-si,
Gyeonggi-do, 17099, Republic of Korea

1°C / 34°F  40°C / 104°F

REF 05FK63

05FK63-P-A0

PRODUCT - Bioline™

Assay diluent label

SIZE - 50 X 16 MM

PMS 303 C

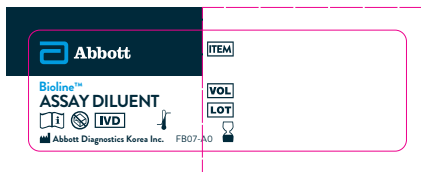
PMS 2925 C

CAT NO. - FB07-A0

REV. DATE - 2019.10.21

CREATOR - SCOTT KIM

PANTONE Reflex Blue C
Shandong
Size:50X40

Sterile Lancet for single use

LOT LOT NO./Type : 2305267303 / 28G
EXP. DATE : 2028-06-04
QTY : 25pcs

STERILE R **CE** 0123 

0100040517004436172880040103205207101

EC REP Shanghai International Holding Corp. GmbH (Europe)
Eiffelstrasse 60, 20537 Hamburg, Germany

Shandong Lianfa Medical Plastic Products Co., Ltd.
No. 3 Shuangshihai Sanjian Road 250200 Zhangqiu City, Jinan, Shandong
PEOPLE'S REPUBLIC OF CHINA

PANTONE Reflex Blue C
Shandong
Size:50X40

Sterile Lancet for single use

LOT LOT NO./Type : 2305267303 / 28G
 **EXP. DATE** : 2028-06-04
 **QTY** : 25pcs

STERILE R  **0123** 

010004051700443617288004193205207101

EC REP Shanghai International Holding Corp. GmbH (Europe)
Eiffelstrasse 60, 20537 Hamburg, Germany

 **Shandong Lianfa Medical Plastic Products Co., Ltd.**
No. 3 Shuangshihai Sanjian Road 250200 Zhangqiu City, Jinan, Shandong
PEOPLE'S REPUBLIC OF CHINA

PANTONE Reflex Blue C
Shandong
Size:50X40

Sterile Lancet for single use

LOT LOT NO./Type : 2305267305 / 28G
 **EXP. DATE** : 2028-06-08
 **QTY** : 30pcs


STERILE R  **0123** 01000405170044011728000010300207101

EC REP Shanghai International Holding Corp. GmbH (Europe)
Eiffelstrasse 60, 20537 Hamburg, Germany


 **Shandong Lianfa Medical Plastic Products Co., Ltd.**
No. 3 Shuangshihai Sanjian Road 250200 Zhangqiu City, Jinan, Shandong
PEOPLE'S REPUBLIC OF CHINA

	10mm	33.75mm	67.5mm	33.75mm	10mm
10mm	<div style="display: flex; justify-content: space-between;"> <div style="width: 20%;">   <p>(01)16957138602815 (11)YYMMDD (17)YYMMDD (10)XXXXXXXX</p> <p>LOT</p>   <p>EC REP Shanghai International Holding Corp. GmbH (Europe) Eiffestrasse 80, 20537 Hamburg, Germany Tel: 0049-40-2513175 E-mail: shholding@hotmail.com</p> </div> <div style="width: 50%; text-align: center;"> <p>Twist-off lancet (use with lancing device) 25 pcs (28G I)</p> <p>[Instructions]</p> <ol style="list-style-type: none"> 1. Unscrew the lancing device cap from the body of the lancing device. 2. Insert a sterile lancet into the lancing device firmly. 3. Twist off the protective cap of the lancet. 4. Carefully screw the lancing device cap back to the lancing device. 5. Adjust the puncture depth. 6. Put the lancing device against the side of the finger and press the release button to prick your fingertip. 7. Apply a drop of blood to test strip. 8. Recap lancet and discard it in appropriate container. 9. Place the lancing device cap back to the lancing device. <p>[Structure] The Twist-off Lancets is consist of needle, main body and protective cap.</p> <p>[Performance] The device is sterilized by irradiation.</p> <p>[Intended use] The twist-off lancet (use with lancing device) is a single use, sterile medical device, which is designed for use of micro blood sampling puncture to obtain capillary blood samples from the fingertip.</p> <p>[Contraindications] None.</p> <p>[Shelf life] 5 years.</p> <p>[Storage] The products should be stored in a well-ventilated room and kept away from light, moisture and corrosive gases.</p> <p>Tianjin Huahong Technology Co., Ltd. A01, Plant B, No.278, Hangkong Road, Tianjin Pilot Free Trade Zone (Air Port Industrial Park), 300308 Tianjin, China Tel: 0086 22 58775333 400 8587753 Fax: 0086 22 58775353 E-mail: huahongsales@163.com</p> </div> <div style="width: 20%;"> <p>[Cautions]</p> <ol style="list-style-type: none"> 1. Read the instructions carefully before use. 2. Use the lancets within the expiry date. 3. Do not use if protective cap is removed or broken previously. 4. Do not store lancet in lancing device, discard lancet after use, do not re-use. 5. For your safety, please do not use a lancet that has been used by someone else. Lancets are not intended for use on multiple individuals. 6. In the event of any serious adverse event, please report to the manufacturer and the competent authority. <p> The lancet and lancing device must be compatible with each other. When you use Huahong's sterile lancets with other manufacturers' lancing devices, please be sure that these products are compatible. We recommend that you shall consult a health care professional before use.</p> <p>STERILE R</p>    <p>  0197</p> </div> </div>				
10mm					

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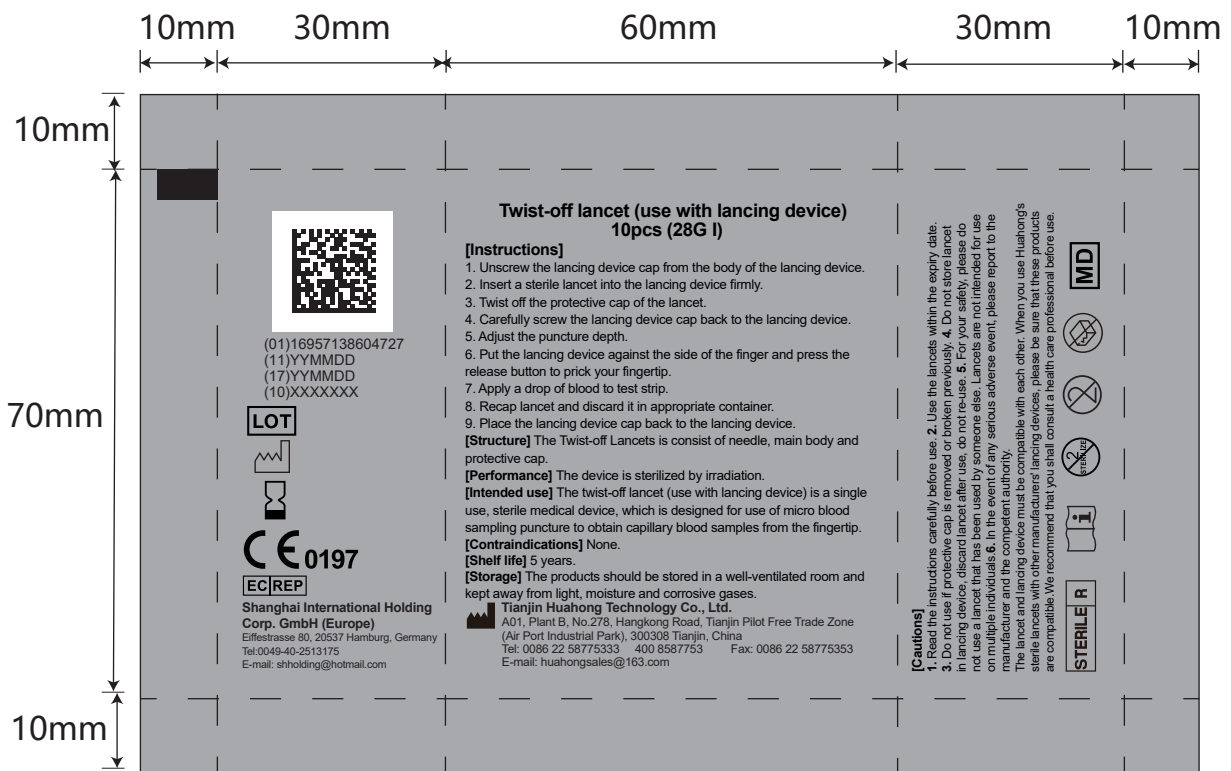
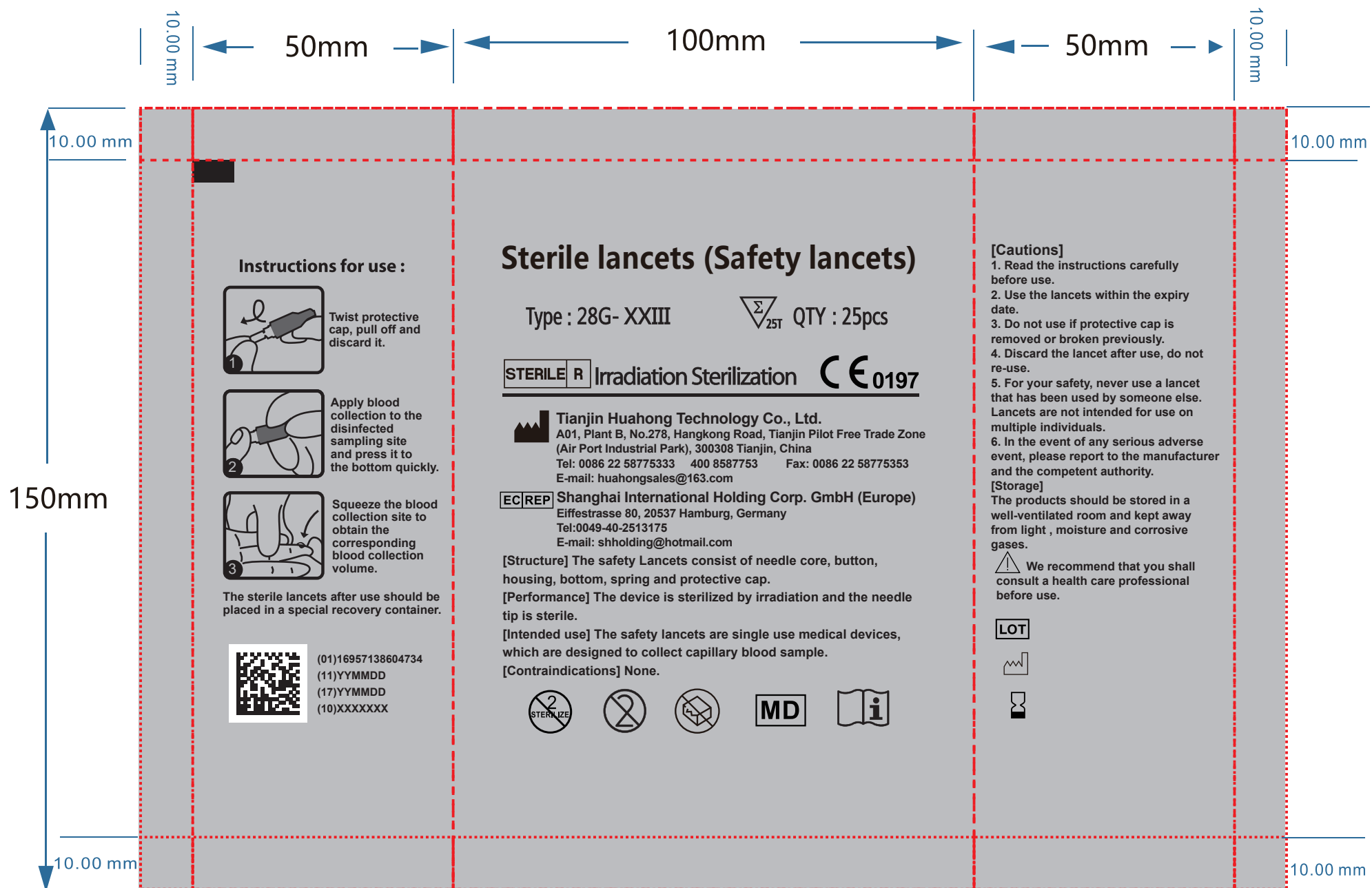
Instructions for use :

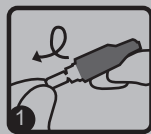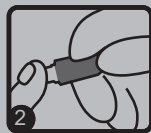Twist protective cap, pull off and discard it.

Apply blood collection to the disinfected sampling site and press it to the bottom quickly.

Squeeze the blood collection site to obtain the corresponding blood collection volume.

The sterile lancets after use should be placed in a special recovery container.

(01)16957138604734
 (11)YYMMDD
 (17)YYMMDD
 (10)XXXXXX

Sterile lancets (Safety lancets)

Type : 28G- XXIII Σ_{25T} QTY : 25pcs

STERILE R Irradiation Sterilization $\text{C} \text{E} 0197$

Tianjin Huahong Technology Co., Ltd.
 A01, Plant B, No.278, Hangkong Road, Tianjin Pilot Free Trade Zone
 (Air Port Industrial Park), 300308 Tianjin, China
 Tel: 0086 22 58775333 400 8587753 Fax: 0086 22 58775353
 E-mail: huahongsales@163.com

EC/REP Shanghai International Holding Corp. GmbH (Europe)
 Eiffestrasse 80, 20537 Hamburg, Germany
 Tel:0049-40-2513175
 E-mail: shholding@hotmail.com

[Structure] The safety Lancets consist of needle core, button, housing, bottom, spring and protective cap.

[Performance] The device is sterilized by irradiation and the needle tip is sterile.

[Intended use] The safety lancets are single use medical devices, which are designed to collect capillary blood sample.

[Contraindications] None.

[Cautions]

1. Read the instructions carefully before use.
2. Use the lancets within the expiry date.
3. Do not use if protective cap is removed or broken previously.
4. Discard the lancet after use, do not re-use.
5. For your safety, never use a lancet that has been used by someone else. Lancets are not intended for use on multiple individuals.
6. In the event of any serious adverse event, please report to the manufacturer and the competent authority.

[Storage]

The products should be stored in a well-ventilated room and kept away from light, moisture and corrosive gases.

We recommend that you shall consult a health care professional before use.

LOT

XXXXXXXXX_Inverted Cup(80mm)XXPCS_55×60

Inverted Cup 80mm (5µl)

Code No. : XXXXXXXX
LOT No. : XXXXXXXXXX
Quantity : XXPCS
MFG Date : XXXX.XX.XX

HLB Life Science Co.,Ltd
101-908, Digital EmpireII, 88, Sinwon-ro, Yeongtong-gu
Suwon-si, Gyeonggi-do, 16681, Republic Korea
(0 1) 0 8 8 0 9 8 9 9 6 2 X X X X
EC REP Cmc Medical Devices & Drugs S.L.
C/ Horacio Lengo n18 C.P 29006 Málaga-Spain

Inverted Cup 80mm (5µl)

Code No. : XXXXXXXX
LOT No. : XXXXXXXXXX
Quantity : XXPCS
MFG Date : XXXX.XX.XX

HLB Life Science Co.,Ltd
101-908, Digital EmpireII, 88, Sinwon-ro, Yeongtong-gu
Suwon-si, Gyeonggi-do, 16681, Republic Korea
(0 1) 0 8 8 0 9 8 9 9 6 2 X X X X
EC REP Cmc Medical Devices & Drugs S.L.
C/ Horacio Lengo n18 C.P 29006 Málaga-Spain

XXXXXXXXX_Inverted Cup(80mm)XXPCS_55×60

Inverted Cup 80mm (5µl)

Code No. : XXXXXXXX
LOT No. : XXXXXXXXXX
Quantity : XXPCS
MFG Date : XXXX.XX.XX

HLB Life Science Co.,Ltd
101-908, Digital EmpireII, 88, Sinwon-ro, Yeongtong-gu
Suwon-si, Gyeonggi-do, 16681, Republic Korea

(0 1) 0 8 8 0 9 8 9 9 6 2 X X X X

EC REP Cmc Medical Devices & Drugs S.L.
C/ Horacio Lengo n18 C.P 29006 Málaga-Spain

Inverted Cup 80mm (5µl)

Code No. : XXXXXXXX
LOT No. : XXXXXXXXXX
Quantity : XXPCS
MFG Date : XXXX.XX.XX

HLB Life Science Co.,Ltd
101-908, Digital EmpireII, 88, Sinwon-ro, Yeongtong-gu
Suwon-si, Gyeonggi-do, 16681, Republic Korea

(0 1) 0 8 8 0 9 8 9 9 6 2 X X X X

EC REP Cmc Medical Devices & Drugs S.L.
C/ Horacio Lengo n18 C.P 29006 Málaga-Spain

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Instructions for use³

³ English version of the IFU was the one that was assessed by WHO. It is the responsibility of the manufacturer to ensure correct translation into other languages.

Abbott

Bioline

Malaria Ag P.f/Pan

Malaria HRP2(P.f) e pLDH(Pan) Antigen Rapid Test Teste de detecção rápida des antígenos HRP2(P.f) e pLDH (Pan) du paludisme Prueba Rápida para la detección de antígenos de Malaria HRP2(P.f) y pLDH(Pan) Teste rápido para deteção dos antígenos HRP2 (P.f) e pLDH (Pan) da Malária

ENGLISH

About the test

Malaria is a serious and sometimes fatal parasitic disease characterized by fever, chills and anemia. The disease is caused by a parasite that is transmitted from one human to another through the bite of infected *Anopheles* mosquitoes. There are four types of malaria that can infect humans: Plasmodium falciparum, Plasmodium vivax, Plasmodium malariae, and Plasmodium (P. falciparum) sporozites) migrate to the liver where they mature and are released in another form: merozoites. The disease now occurs in more than 90 countries worldwide, and it is estimated that there are over 200 million clinical cases and 0.5 million malaria-related deaths per year. At present, malaria is diagnosed by looking for parasites in a drop of blood. Blood is placed on a microscope slide and stained so that the parasites will be visible under a microscope.

[Tested U]nited

Bioline® Malaria Ag P.f/Pan test kit intended for the detection of Malaria infection in human blood. This test kit includes:
• 25 Disposable specimen applicators (5 µl), 25 Sterile Lancets, 25 Alcohol swabs
• 1 Instructions folder
• 2 Active ingredients in main component
• 1 Easy-to-use individual kit containing:
• 1 Malaria monoclonal antibodies specific to PF HRP2 - gold colloid (0.180±0.036 µg)
• 1 Malaria monoclonal antibodies specific to pLDH - gold colloid (0.060±0.012 µg)
• 1 Test line P.f: Malaria monoclonal antibodies specific to PF HRP2 (0.320±0.064 µg)
• 1 Test line Pan: Malaria monoclonal antibodies pan-specific to pLDH (0.640±0.128 µg)
• 1 Assay diluent, Bovine serum albumin (s.a.), 200 mM Phosphate Saline Buffer (pH 7.4), TRITON™ X-100 (s.a.), Sodium azide (s.a.)

Materials provided and active ingredients of main component
• 25 Disposable specimen applicators (5 µl), 25 Sterile Lancets, 25 Alcohol swabs
• 1 Instructions folder
• 2 Active ingredients in main component
• 1 Easy-to-use individual kit containing:
• 1 Malaria monoclonal antibodies specific to PF HRP2 - gold colloid (0.180±0.036 µg)
• 1 Malaria monoclonal antibodies specific to pLDH - gold colloid (0.060±0.012 µg)
• 1 Test line P.f: Malaria monoclonal antibodies specific to PF HRP2 (0.320±0.064 µg)
• 1 Test line Pan: Malaria monoclonal antibodies pan-specific to pLDH (0.640±0.128 µg)
• 1 Assay diluent, Bovine serum albumin (s.a.), 200 mM Phosphate Saline Buffer (pH 7.4), TRITON™ X-100 (s.a.), Sodium azide (s.a.)

Materials required but not provided
• Micropipette, Protective Gloves, Timer, Biohazard container

Kit storage and stability
• The test kit should be stored at a temperature between 1° C and 40° C. Do not freeze the kit or its components. Assay diluent can be kept firmly sealed between each use.
• **Caution:** When stored at refrigerator, test kit components must be brought to room temperature before use. Do not use the test kit after the expiration date. Do not open the pouch until pouch components come to room temperature.
• Assay diluent may be opened and resealed for each use. It is stable until expiration date if kept at room temperature.
• The test device is sensitive to both heat and humidity. Perform the test immediately after removing the test device from foil pouch.
• Do not use the test kit beyond its expiration date.
• The shelf life of the kit is indicated on the outer package.
• Do not use the test kit if the package is damaged or the seal is broken.
• When transporting or storing the kit, avoid exposure to high temperature (above 45° C) for a period longer than 1 week.

Warnings

- For **in vitro** diagnostic use only. Do not reuse the test device and kit components.
- The instructions must be followed exactly to achieve accurate results. Any individual performing an assay with this product must be trained in its use and must be experienced in laboratory procedures.
- Do not eat or smoke while handling specimens.
- Wear protective gloves while handling specimens and wash hands thoroughly afterwards.
- Avoid splashing or aerosol formation of specimen and assay diluent.
- Clean up spills thoroughly using an approved disinfectant.
- Decomantate and dispose of all specimens, reaction kits and potentially contaminated materials (i.e. lancet, specimen applicator, test device) in a biohazard container as if they were infectious material.
- Do not mix or interchange different lot specimens.
- Do not mix reagent of different lots or those for other products.
- Do not pipette by mouth.
- Do not eat the desiccant in the foil pouch.
- Do not drink assay diluent.
- The assay diluent contains a proprietary anti-microbial agent, sodium azide, which presents no hazard to the user if normal laboratory safety precautions are followed. If contact with assay diluent to the eyes and/or skin, wash affected area with soap and water immediately. Irritation or signs of toxicity occur, seek medical attention.

Specimen collection and storage
[Collection by venipuncture]

- Using venipuncture draw whole blood into the collection tube containing anticoagulants including heparin, EDTA and sodium citrate.
- If the blood specimen is not immediately tested, it must be refrigerated at 2 - 8° C.
- If stored at 2 - 8° C, the blood specimen must be tested within 3 days.
- Do not use a blood specimen stored for more than 3 days if it can cause a nonspecific reaction. [Collection using a lancet]
 - Clean the area to be lanced with an alcohol swab.
 - Squeeze the end of the fingertip and pierce with a sterile lancet provided.
 - Wipe away the first drop of blood with sterile gauze pad.
 - Using a capillary pipette (5 µl) volume, while gently squeezing the tube, immerse the open end of the blood cup into the gently release the pressure to draw blood into the capillary pipette to black line. Or, Using a disposable microfuge cup (5 µl) provided, dip the circular end of the inverted cup and then specimen.

Performance characteristics
A. Internal evaluation of Bioline® Malaria Ag P.f/Pan test
1. Bioline® Malaria Ag P.f/Pan test for the detection of *P. falciparum* malaria according to the level of parasite. (Table 1)
B. Bioline® Malaria Ag P.f/Pan test for the detection of non-*P. falciparum* malaria according to the level of parasite. (Table 2)
C. Sensitivity and specificity of Bioline® Malaria Ag P.f/Pan test. (Table 3)
Reproducibility of Bioline® Malaria Ag P.f/Pan test has been demonstrated by within-run, between-run and batch-to-batch studies using in-house reference panels. All values were identical to reference panel acceptance criteria.

- To evaluate the inter-assay of Bioline® Malaria Ag P.f/Pan test kits known relevant interfering specimen, the haemolytic specimens, rheumatoid factors contained specimens and lipemic, icteric specimens were investigated. In these studies, those specimens did not interfere with the Bioline® Malaria Ag P.f/Pan test.
- Assay reproducibility of Bioline® Malaria Ag P.f/Pan test.
- Assay for the detection of Malaria according to the level of parasite.
 - Plasmodium falciparum : Sensitivity with >50 parasitofol of blood is 100%.
 - Non-Plasmodium falciparum : Sensitivity with >50 parasitofol of blood is greater than 98%.

Nº de parasitas /µl de blood	Microscopy (No. of positive)	Bioline® Malaria Ag P.f/Pan (No. of positive)	Sensitivity (%)
1-50	16	15	93.8%
51-100	35	35	100%
101-500	96	96	100%
501-1,000	81	81	100%
1,001-5,000	79	79	100%
>5,000	61	61	100%
Total	368	367	99.7%

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- Histidine-Rich Protein Is a Novel Approach to Malaria Drug Sensitivity Testing ANTIMICROBIAL AGENTS AND CHEMOTHERAPY, June 2002, p. 1658 -1664 Vol. 46, No. 6

Natural Antibody Responses against the Non-Repeat-Sequence-Based B-Cell Epitopes of the Plasmodium falciparum Circumsporozoite Protein Vol. 61, No. 6 INFECTION AND IMMUNITY, June 1993, p. 2425-2433

Plasmodium falciparum Histidine-Rich Protein 2-Band Immunocapture Diagnostic Assay for Malaria: Cross-Reactivity with Rheumatoid Factors. JOURNAL OF CLINICAL MICROBIOLOGY, March 2000, p. 1184-1186 Vol. 38, No. 3

Transfection of Plasmodium falciparum with human red blood cells. Proc. Natl. Acad. Sci. USA, Vol. 92, pp. 973-977, February 1995

Secretion of a Malarial Histidine-rich Protein (PfHRP II) from Plasmodium f

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- EN** Next, look at the expiration date on the back of the foil pouch. If the expiration date has passed, use another lot. To avoid false results, ensure that the assay diluent used is from the same kit as the new test device.

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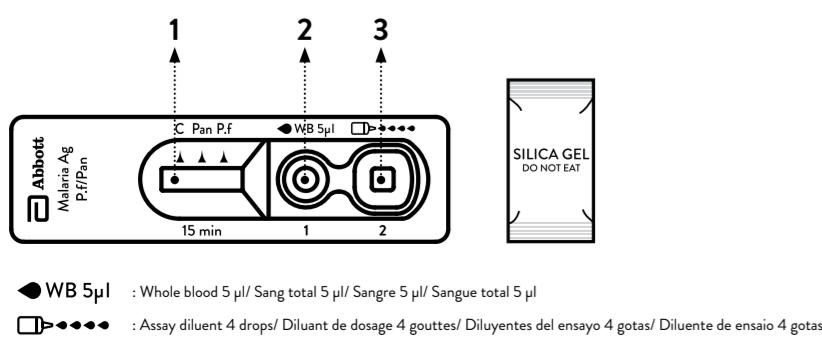
- EN** Clean the area to be lanced with an alcohol swab.

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
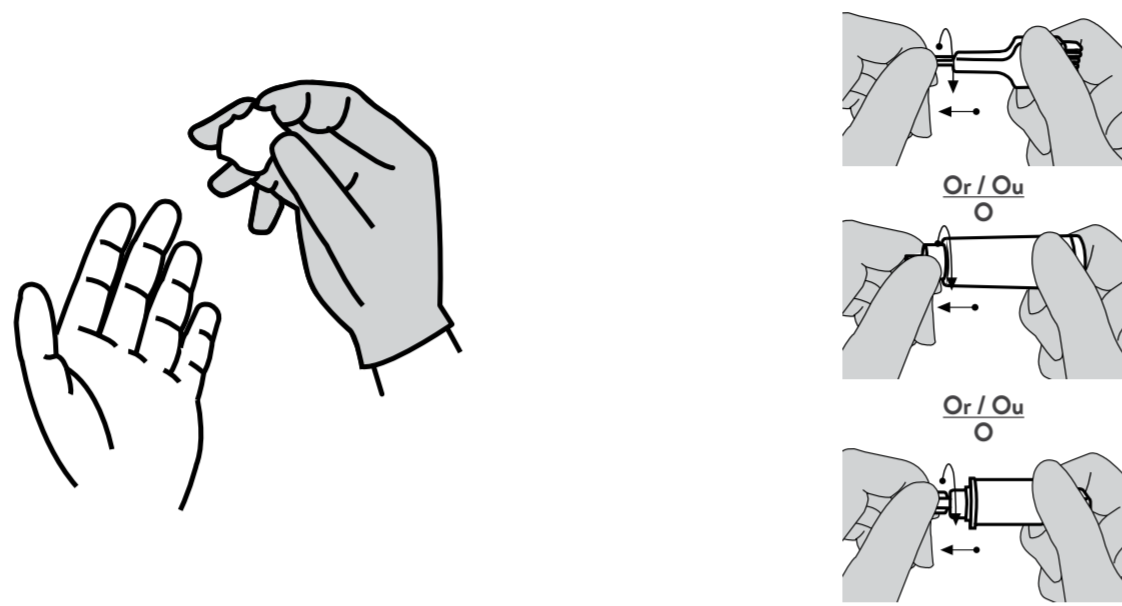
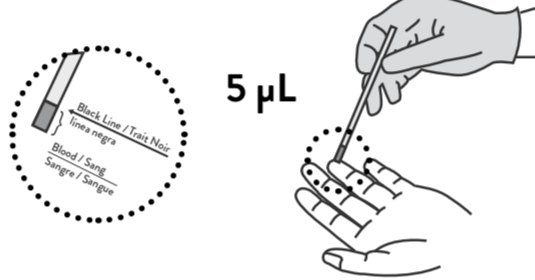
- EN** Dispense 5 µl of drawn blood into round specimen well.

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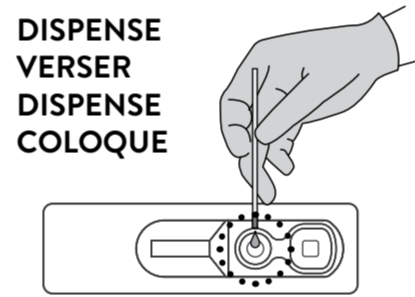
- EN** Hold assay diluent bottle vertically and dispense 4 drops of assay diluent into the assay diluent well. Exactly, 4 drops should be added. Do not let bottle tip touch device in order to avoid cross-contamination.

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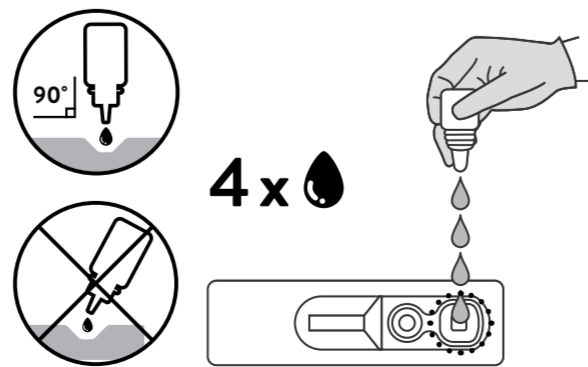- EN** Prick the lateral side of the finger with the safety lancet provided. Then, safely dispose of the lancet immediately after.

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
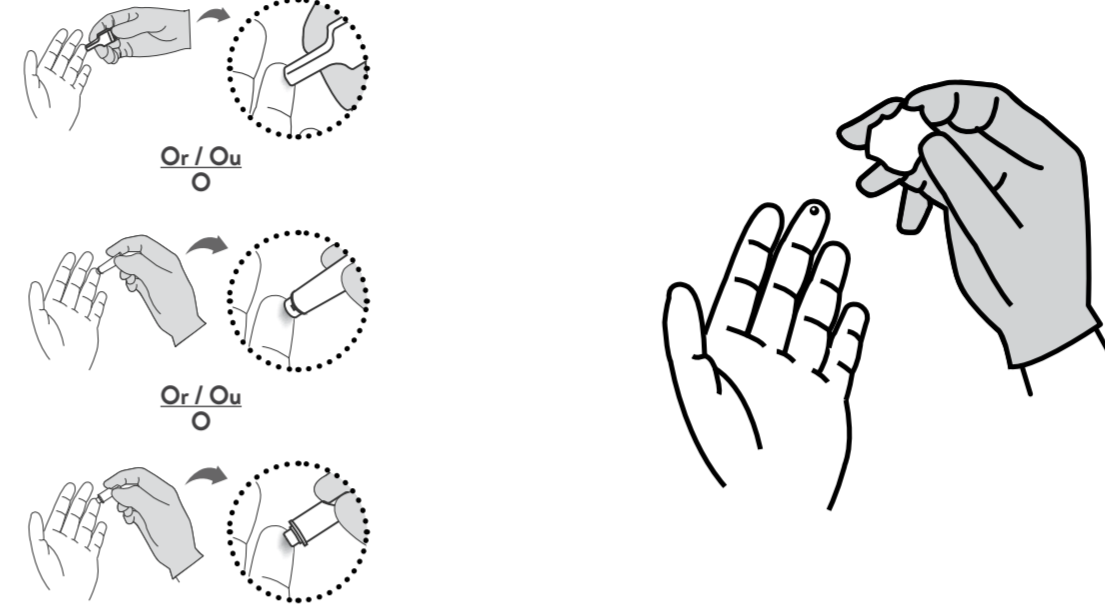
- EN** Using a disposable inverted cup (5 µl) provided, dip the circular end of a inverted cup into the blood specimen.

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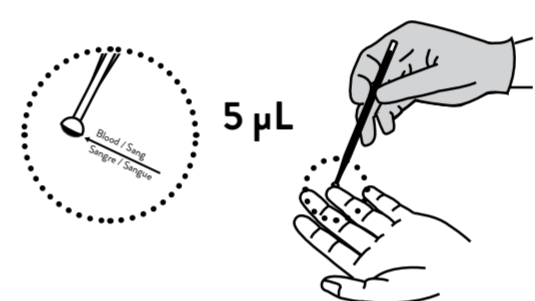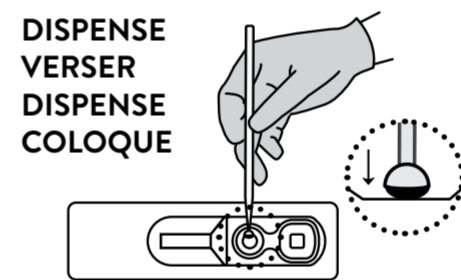
- EN** Use Inverted cup : Let the circular end of the inverted cup touch the pad, then press down lightly.

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- EN** Read the test result after 15 minutes.

⚠ **Caution:** Do not read test results after 30 minutes. ; reading after 30 minutes can yield erroneous results

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PREPARATION / PRÉPARATION / PREPARACIÓN / PREPARAÇÃO

- 1** **EN** Open the package and look for the following:
1. Test device with desiccant in individual foil pouch
 2. Assay diluent
 3. Disposable specimen applicator (5 µl) (Capillary pipette or Disposable inverted cup)
 4. Sterile lancet
 5. Alcohol swab
 6. Summarized instructions for use
 7. Instructions for use

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- 3** **EN** Next, look at the expiration date on the back of the foil pouch. If the expiration date has passed, use another lot. To avoid false results, ensure that the assay diluent used is from the same kit as the new test device.

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- 4** **EN** Open the foil pouch and look for the following:
1. Result window
 2. Specimen well
 3. Assay diluent well
- Then, label the device with the patient identifier.

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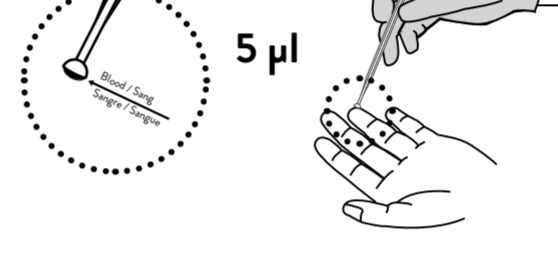
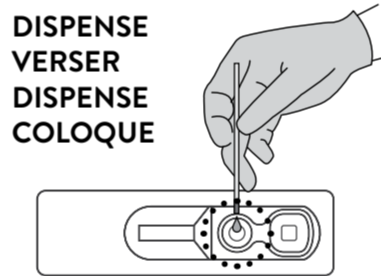
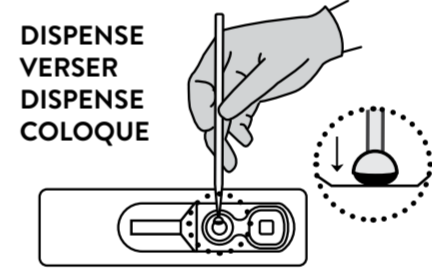
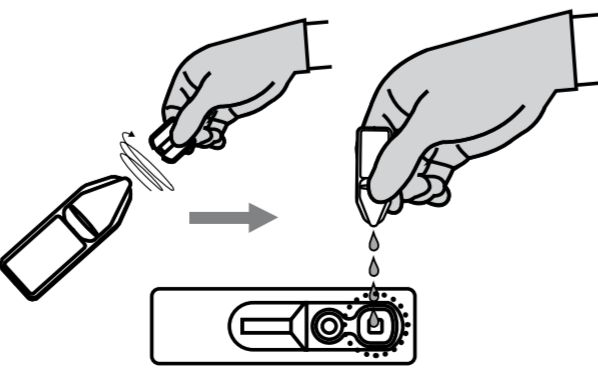
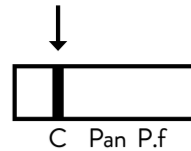
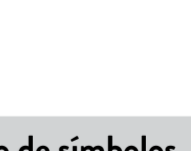
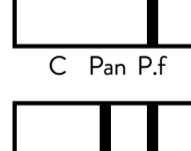
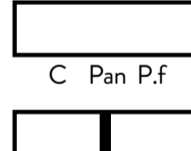
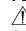
Test interpretation (Refer to figure)

- Negative result :**
The presence of control line ("C") within the result window indicates a negative result.
- Positive result :**
 **Caution** : The presence of any test line, no matter how faint, the result is considered positive.
 - P. falciparum positive** : The presence of two colored lines (test line "P.F" and control line "C") or three colored lines (test lines "P.F", "Pan" and control line "C") within the result window, regardless of which line appears first, indicate *P.f* positive result.
 - Other plasmodium species (P.v or P.m or P.o) positive** : The presence of two colored lines (test line "Pan" and control line "C") within the result window, regardless of which line appears first, indicate Pan (*P.v* or *P.m* or *P.o*) positive result.
 - P.f positive or mixed infection of P.f and P.v or P.m or P.o.** : The presence of three colored lines (test lines "P.F", "Pan" and control line "C") within the result window, regardless of which line appears first, indicate *P.f* positive or mixed infection of *P.f* and *P.v* or *P.m* or *P.o*.
- Invalid result :**
If the control line ("C") is not visible within the result window after performing the test, the result is considered invalid. Instructions may not have been followed correctly or the test may have deteriorated beyond the expiration date. It is recommended that the specimen be retested using a new test device.

* Note : pLDH is Pan specific to the lactate dehydrogenase of Plasmodium species (*P.f*, *P.v*, *P.o*, *P.m*).

Limitations and Interferences

- The test procedure, precautions and interpretation of results for this test must be followed when testing.
- The test is limited to the detection of antigen to *Malaria Plasmodium sp.* Although the test is very accurate in detecting HRP2 specific to *P. falciparum* or pLDH specific to plasmodium species (*P. falciparum*, *vivax*, *malariae*, *ovale*) a low incidence of false results can occur. Other clinically available tests are required if questionable results are obtained. As with all diagnostic tests, a definitive clinical diagnosis should not be based on the results of a single test, but should only be made by the physician after all clinical and laboratory findings have been evaluated.

Internal quality control

Bioline™ Malaria Ag P.f/Pan test device has test lines ("Pan", "P.F") and control line ("C") on the surface of the device. Neither the test lines nor the control line is visible in the result window before applying a specimen. The control line is used for procedural control and shows only that the diluent has been applied successfully and that the active ingredients of the main components on the strip are functional, but is not a guarantee that the specimen has been properly applied and does not represent a positive specimen control.

Expected values

Bioline™ Malaria Ag P.f/Pan test has been compared with microscopic examination. The overall accuracy is greater or equal to 95 %.

Performance characteristics

- Internal evaluation of Bioline™ Malaria Ag P.f/Pan test

A. Bioline™ Malaria Ag P.f/Pan test for the detection of *P. falciparum* malaria according to the level of parasite. (Table-1)
Table-1

No. of parasites / μ L of blood	Microscopy (No. of positive)	Bioline™ Malaria Ag P.f/Pan (No. of positive)	Sensitivity (%)
1-50	16	15	93.8 %
51-100	35	35	100 %
101-500	96	96	100 %
501-1,000	81	81	100 %
1,001-5,000	79	79	100 %
>5,000	61	61	100 %
Total	368	367	99.7 %

B. Bioline™ Malaria Ag P.f/Pan test for the detection of non-*P. falciparum* malaria according to the level of parasite. (Table-2)
Table-2

No. of parasites / μ L of blood	Microscopy (No. of positive)	Bioline™ Malaria Ag P.f/Pan (No. of positive)	Sensitivity (%)
1-50	6	3	50 %
51-100	10	9	90 %
101-500	25	24	96 %
501-1,000	18	18	100 %
1,001-5,000	29	29	100 %
>5,000	23	23	100 %
Total	111	106	95.5 %

C. Sensitivity and specificity of Bioline™ Malaria Ag P.f/Pan test. (Table-3)

Table-3

Reference method (Microscopic examination)	Bioline™ Malaria Ag P.f/Pan		
	Positive	Negative	Total
P.f Positive	367	1	368
Non-P.f Positive	106	5	111
Negative	1	199	200
P.f Sensitivity (95 % CI)	99.7 % (98.5 - 100 %)		
Non-P.f Sensitivity (95 % CI)	95.5 % (89.9 - 98.1 %)		
Specificity (95 % CI)	99.5 % (97.2 - 99.9 %)		

- Reproducibility of Bioline™ Malaria Ag P.f/Pan has been demonstrated by within-run, between-run and batch-to-batch studies using in-house reference panels. All values were identical to reference panel acceptance criteria.
- To evaluate the interference of Bioline™ Malaria Ag P.f/Pan test kit with known relevant interfering specimens, the haemolytic specimens, rheumatoid factors-contained specimens and lipaemic, icteric specimens were investigated. In these studies, those specimens did not interfere with Bioline™ Malaria Ag P.f/Pan test.
- Assay for the detection of Malaria according to the level of parasite.
 - Plasmodium falciparum* : Sensitivity with > 50 parasite/ μ l of blood is 100 %.
 - Non-Plasmodium falciparum* : Sensitivity with > 50 parasite/ μ l of blood is greater than 98 %.

Bibliography of suggested reading

- Histidine-Rich Protein II: a Novel Approach to Malaria Drug Sensitivity Testing ANTIMICROBIAL AGENTS AND CHEMOTHERAPY, June 2002, p. 1658 -1664 Vol. 46, No. 6
- Natural Antibody Responses against the Non-Repeat-Sequence-Based B-Cell Epitopes of the Plasmodium falciparum Circumsporozoite Protein Vol. 61, No. 6 INFECTION AND IMMUNITY, June 1993, p. 2425-2433
- Plasmodium falciparum Histidine-Rich Protein 2-Based Immunocapture Diagnostic Assay for Malaria: Cross-Reactivity with Rheumatoid Factors. JOURNAL OF CLINICAL MICROBIOLOGY, Mar. 2000, p. 1184 -1186 Vol. 38, No. 3
- Transfection of Plasmodium falciparum within human red blood cells. Proc. Natl. Acad. Sci. USA. Vol. 92, pp. 973-977, February 1995
- Secretion of a Malarial Histidine-rich Protein (Pf HRP II) from Plasmodium falciparum-infected Erythrocytes. The Journal of Cell Biology, Volume 103, October 1986 1269-1277 1269-1277
- Bell D, Peeling RW: Evaluation of rapid diagnostic tests: malaria. Nat Rev Microbiol. 2006, 4: S34-S38. 10.1038/nrmicro1524

Product Disclaimer:

While every precaution has been taken to ensure the diagnostic ability and accuracy of this product, the product is used outside of the control of the manufacturer and distributor and test results may accordingly be affected by environmental factors and/or user error. The subject of the diagnosis should consult a doctor for further confirmation of the test result.

Warning:

The manufacturers and distributors of this product shall not be liable for any direct, indirect, or consequential losses, liability, claims, costs or damages arising from or related to an incorrect positive or negative diagnosis using this product.

Date issued : 2025.09
05FK67-01-EN-A3

Bioline™

Malaria Ag P.f/Pan

Malaria HRP2(P.f) and pLDH(Pan) Antigen Rapid Test
About the test

[Introduction]

Malaria is a serious and sometimes fatal parasitic disease characterized by fever, chills and anemia. The disease is caused by a parasite that is transmitted from one human to another through the bite of infected Anopheles mosquitoes. There are four types of malaria that can infect humans: *Plasmodium falciparum*, *P. vivax*, *P. ovale* and *P. malariae*. In humans, the parasites (called sporozoites) migrate to the liver where they mature and are released in another form: merozoites. The disease now occurs in more than 90 countries worldwide, and it is estimated that there are over 200 million clinical cases and 0.5 million malaria-caused deaths per year. At present, malaria is diagnosed by looking for parasites in a drop of blood. Blood is placed on a microscope slide and stained so that the parasites will be visible under a microscope.

[Test principle]

Bioline™ Malaria Ag P.f/Pan test device contains a membrane strip, which is precoated with mouse monoclonal antibodies specific to HRP2 of *P. falciparum* on test line P.f region and with mouse monoclonal antibodies specific to lactate dehydrogenase of Plasmodium species Pan (*P. falciparum*, *P. vivax*, *P. malariae* and *P. ovale*) on test line Pan region respectively. The mixture of mouse monoclonal antibodies specific to HRP2 of *P.f* and mouse monoclonal antibodies specific to pLDH of pan - colloid gold conjugate reacts with the malaria antigen in the specimen. They move along the membrane chromatographically to the test region (*P.f* and Pan) and form a visible line as the antibody-antigen-antibody gold particle complex with high degree of sensitivity and specificity.

[Intended Use]

Bioline™ Malaria Ag P.f/Pan test kit intended for the detection of Malaria infection in human blood specimen, indicating qualitative and differential diagnosis between HRP2(Histidine-rich protein II) specific to *Plasmodium falciparum* and pLDH (Plasmodium lactate dehydrogenase) specific to Plasmodium species (Pan) in human blood specimen. Bioline™ Malaria Ag P.f/Pan test kit is intended only for professional use and for *in vitro* diagnostic use. Reactive specimens should be confirmed by a supplemental assay such as microscopic examination of thin blood smear.

Materials provided and active ingredients of main components

- Bioline™ Malaria Ag P.f/Pan test kit contains the following items to perform the assay:
 - 30 Test devices with desiccant in individual foil pouches
 - 30 Assay diluents
 - 30 Disposable specimen applicators (5 μ l), 30 Sterile lancets, 30 Alcohol swabs, 30 Dried swabs
 - 60 Latex gloves
 - 1 Instructions for use
- Active ingredients of main component
 - 1 test strip includes : Gold conjugate : Mouse monoclonal antibodies specific to *P.f* HRP2 - gold colloid (0.180 \pm 0.036 μ g), Mouse monoclonal antibodies pan specific to pLDH - gold colloid (0.060 \pm 0.012 μ g), Test line P.f : Mouse monoclonal antibodies specific to P.f HRP2 (0.320 \pm 0.064 μ g), Test line Pan : Mouse monoclonal antibodies pan specific to pLDH (0.80 \pm 0.16 μ g), Control line : Recombinant pPM-LDH antigen (0.640 \pm 0.128 μ g)
 - Assay diluent includes; Bovine serum albumin (q.s.), 200 mM Phosphate Saline Buffer (q.s.), TRITON™ X-100 (q.s.), Sodium azide (q.s.)

Materials required but not provided

- Micropipette, Timer, Biohazard container

Kit storage and stability

- The test kit must be stored at a temperature between 1 °C and 40 °C. Do not freeze the kit or its components.
 -  **Caution** : When stored at refrigerator, all kit components must be brought to room temperature (15 - 40 °C) minimum 30 minutes prior to the test.
 - Do not open the pouch whilst components come to room temperature.
- The test device is sensitive to both heat and humidity. Perform the test immediately after removing the test device from the foil pouch, to avoid exposure to moisture.
- Do not use the test kit beyond its expiration date.
- The shelf life of the kit is as indicated on the outer package.
- Do not use the test kit if the pouch is damaged or the seal is broken.
- When transporting or storing the kit, avoid exposure to high temperature (above 45 °C) for a period longer than 1 week.

Warnings

- For *in vitro* diagnostic use only. Do not reuse the test device and kit components.
- The instructions must be followed exactly to achieve accurate results. Any individual performing an assay with this product must be trained in its use and must be experienced in laboratory procedures.
- Do not eat or smoke while handling specimens.
- Wear protective gloves while handling specimens and wash hands thoroughly afterwards.
- Avoid splashing or aerosol formation of specimen and assay diluent.
- Clean up spills thoroughly using an appropriate disinfectant.
- Decontaminate and dispose of all specimens, reaction kits and potentially contaminated materials (i.e. lancet, specimen applicator, test device, latex glove) in a biohazard container as if they were infectious waste.
- Do not mix or interchange different specimens.
- Do not mix reagent of different lots or those for other products.
- Do not pipette by mouth.
- Do not eat the desiccant in the foil pouch.
- Do not drink assay diluent.
- The assay diluent contains a proprietary anti-microbial agent, sodium azide, which presents no hazard to the user if normal laboratory safety precautions are followed. If contact with assay diluent to the eyes and/or skin, wash affected area with soap and water immediately. If irritation or signs of toxicity occur, seek medical attention.

Specimen collection and storage

[Collection by venipuncture]

- Using venipuncture, collect whole blood into the collection tube (containing anticoagulants including heparin, EDTA and sodium citrate).
- If the blood specimen is not immediately tested, it must be refrigerated at 2 - 8 °C.
- If stored at 2 - 8 °C, the blood specimen must be tested within 3 days.
- Do not use a blood specimen stored for more than 3 days; it can cause a nonspecific reaction.

[Collection using a lancet]

- Clean the area to be lanced with an alcohol swab.
- Squeeze the end of the fingertip and pierce with a sterile lancet provided.
- Wipe away the first drop of blood with a dried swab.
- Using a disposable inverted cup (5 μ l) provided, dip the circular end of inverted cup into the blood specimen.

[Precaution]

- Anticoagulants including heparin, EDTA and sodium citrate do not affect the test results. Use of other anticoagulants have not been validated. Their use may affect the test result.
- Use a new disposable inverted cup for each specimen in order to avoid cross-contamination of specimens, which could produce erroneous results.
- Discard the lancet or alcohol swab if package is pierced or damaged. The use of damaged lancet may cause any infection at the punctured skin due to cease to existing its sterility.
- Repeated frozen-thawed cycle for specimen should be avoided.

Test procedure (Refer to figure)

- Bring all kit components and specimen to room temperature (15 - 40 °C) prior to testing.
- Remove the test device from foil pouch and place it on a flat, dry surface. Label the test device with a patient identifier.
- Clean the area to be lanced with an alcohol swab.
- Prick the lateral side of the finger with the sterile lancet provided. Then, safely dispose of the lancet immediately after.
- Wipe away the first drop of blood with a dried swab.
- Using a disposable inverted cup (5 μ l) provided, dip the circular end of inverted cup into the blood specimen and carefully place the circular end of the inverted cup into the round specimen well.
- Twist and pull cap to open assay diluent.
- Dispense all of the assay diluent from the diluent tube into the square well of test device.
- Read the test result after 15 minutes.

Abbott Diagnostics Korea Inc.

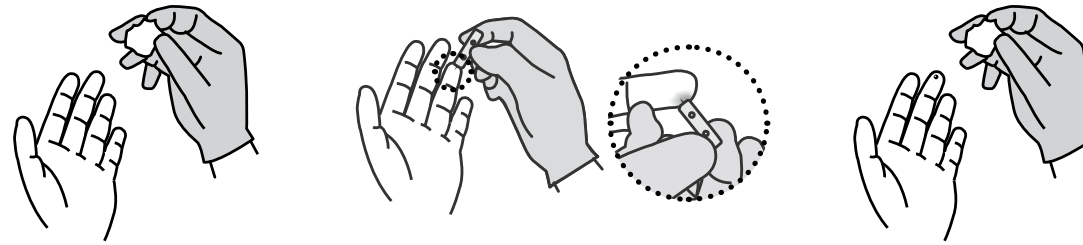
65, Borahagal-ro, Giheung-gu, Yongin-si,
Gyeonggi-do, 17099, Republic of Korea
Tel : 82-31-899-2800
Fax : 82-31-899-2840
abbott.com/poct


Authorized Representative
MT Promedt Consulting GmbH
Altenhofstrasse 80, 66386 St. Ingbert, Germany
Phone : +49 6894 581020
Fax : +49 6894 581021

TEST PROCEDURE

- Clean the area to be lanced with an alcohol swab.
- Prick the lateral side of the finger with the sterile lancet provided. Then, safely dispose of the lancet immediately after.
- Wipe away the first drop of blood with a dried swab.

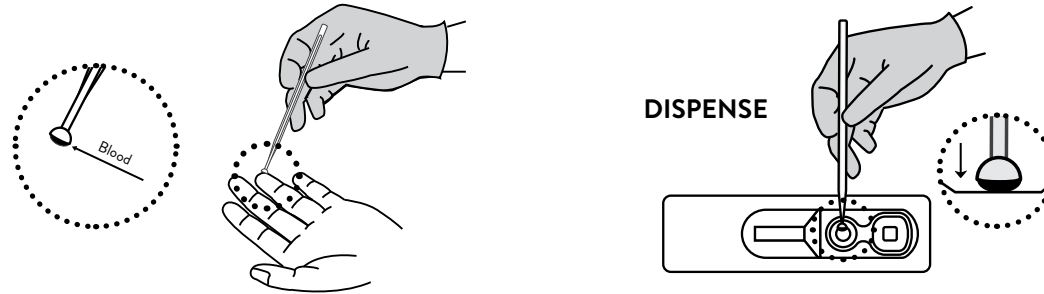
- Using a disposable inverted cup (5 µl) provided, dip the circular end of an inverted cup into the blood specimen.
- Dispense 5 µl of whole blood into round specimen well touching pad.

Use inverted cup : Let the circular end of the inverted cup touch the pad, then press down lightly.

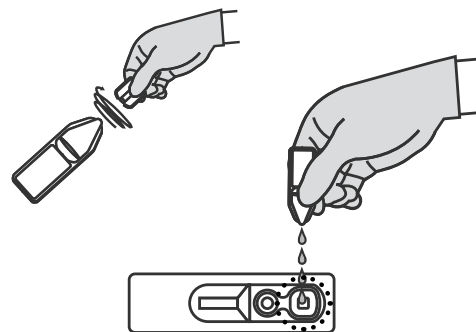
- Twist and pull cap to open assay diluent.
- Dispense all of the assay diluent from the diluent tube into the square well of test device.
- Read the test result after 15 minutes. **Caution** : Do not read test results after 30 minutes ; reading after 30 minutes can yield erroneous results

INTERPRETATION

Negative

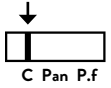The presence of control line ("C") within the result window indicates a negative result.


Positive

Caution : The presence of any test line, no matter how faint, the result is considered positive.


P.f Positive

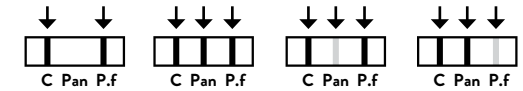The presence of two colored lines (test line "P.f" and control line "C") or three colored lines (test lines "P.f", "Pan" and control line "C") within the result window, regardless of which line appears first, indicate P.f positive result.


Other Plasmodium species (P.v, P.m, P.o)

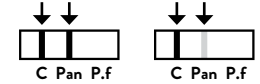The presence of two colored lines ("Pan" and "C") within the result window, regardless of which line appears first, indicate Pan (P.v or P.m or P.o) positive result.

Mixed infection : P.f and P.v (or P.m, P.o)

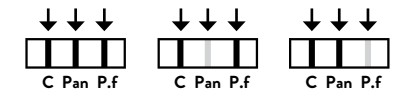The presence of three colored lines ("P.f", "Pan" and "C") within the result window, regardless of which line appears first, indicate P.f positive or mixed infection of P.f and P.v or P.m or P.o.

Invalid

If the control line ("C") is not visible within the result window after performing the test, the result is considered invalid. Instructions may not have been followed correctly or the test may have deteriorated beyond the expiration date. It is recommended that the specimen be retested using a new test device.

Bioline- Malaria Ag P.f/Pan

Malaria HRP2(P.f) and pLDH(Pan) Antigen Rapid Test

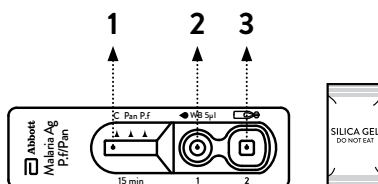
PREPARATION

- Open the package and look for the following:
 - Test device with desiccant in individual foil pouch
 - Assay diluent
 - Disposable specimen applicator (5 µl) (Disposable inverted cup)
 - Sterile lancet
 - Alcohol swab
 - Dried swab
 - Latex glove
 - Instructions for use

- First, carefully read the instructions for using the Bioline™ Malaria Ag P.f/Pan test kit.

- Next, look at the expiration date on the back of the foil pouch. If the expiration date has passed, use another kit. To avoid false results, ensure that the assay diluent used is from the same kit as the new test device.

- Open the foil pouch and look for the following:
 - Result window
 - Specimen well
 - Assay diluent well
 Then, label the device with the patient identifier.

: Whole blood 5 µl
 : Assay diluent

GLOSSARY OF SYMBOLS

Temperature limitation	LOT Lot Number	Manufacturer
IVD For in vitro diagnostic use only	REF Catalog Number	Date of manufacture
Do not reuse	EC REP Authorized Representative	CE CE marking according to IVD Medical Devices Directive 98/79/EC
Use By YYYYMMDD	Consult instructions for use	Keep away from sunlight
Contains sufficient for X tests	Keep dry	Caution
Do not use if package is damaged	Biological Risks	

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ENGLISH

Intended Use

The Bioline™ Malaria Ag P.f/Pan test is a rapid, qualitative and differential test for the detection of HRP2 (Histidine-rich protein II) specific to *Plasmodium falciparum* and pLDH (Plasmodium lactate dehydrogenase) pan specific to *Plasmodium* species in human whole blood.

REF 05FK62

Materials provided

- Test device
- Assay diluent
- Inverted cup (5 µl)
- Safety lancet
- Alcohol swab

Kit storage and stability

1. The test kit should be stored at a temperature between 1 °C and 40 °C. Do not freeze the kit or its components.

Note : When stored at refrigerator, all kit components must be brought to room temperature (15 - 40 °C) minimum 30 mins prior to the test. Do not open the pouch whilst components come to room temperature.

2. The test device is sensitive to both heat and humidity. Perform the test immediately after removing the test device from the foil pouch.
3. Do not use test kit beyond its expiration date.
4. The shelf life of the kit is as indicated on the outer package.
5. Do not use the test kit if the pouch is damaged or the seal is broken.
6. When transporting or storing the kit, avoid exposure to high temperature (above 45 °C) for a period longer than 1 week.

Specimen collection using a lancet

1. Clean the area to be lanced with an alcohol swab.
2. Squeeze the end of the fingertip and pierce with a safety lancet provided.
3. Wipe away the first drop of blood with sterile gauze or cotton.
4. With a disposable inverted cup (5 µl) provided, dip the circular end of an inverted cup into the blood specimen and carefully place the circular end of the inverted cup into the round specimen well.

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Authorized Representative
MT Promedt Consulting GmbH

Altenhofstrasse 80, 66386 St. Ingbert, Germany
Phone : +49 6894 581020
Fax : +49 6894 581021

© 2021 Abbott. All rights reserved.
All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners.

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
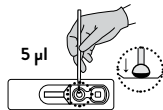
- EN** Dispense all of the assay diluent from the diluent tube into the square well of test device.

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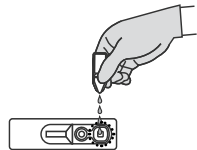EN **Caution:** Do not read test results after 30 minutes. ; reading after 30 minutes can yield erroneous results.

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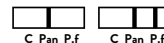
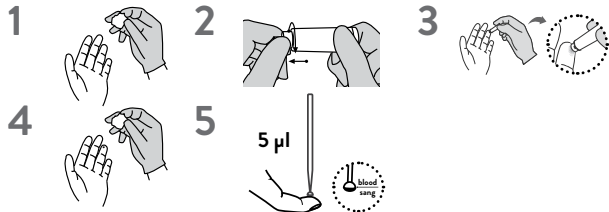
ENGLISH

Intended use

The Bioline™ Malaria Ag P.f/Pan test is a rapid, qualitative and differential test for the detection of HRP2 (Histidine-rich protein II) specific to

REF 05FK63

Plasmodium falciparum and pLDH (*Plasmodium* lactate dehydrogenase) pan specific to *Plasmodium* species in human whole blood.

Materials Provided

- Test device
- Assay diluent
- Inverted cup (5 µl)
- Sterile lancet
- Alcohol swab

Kit storage and stability

1. The test kit should be stored at a temperature between 1 °C and 40 °C. Do not freeze the kit or its components.
Note : When stored at refrigerator, all kit components must be brought to room temperature (15 - 40 °C) minimum 30 mins prior to the test. Do not open the pouch whilst components come to room temperature.
2. The test device is sensitive to both heat and humidity. Perform the test immediately after removing the test device from the foil pouch.
3. Do not use test kit beyond its expiration date.
4. The shelf life of the kit is as indicated on the outer package.
5. Do not use the test kit if the pouch is damaged or the seal is broken.

6. When transporting or storing the kit, avoid exposure to high temperature (above 45 °C) for a period longer than 1 week.

Specimen collection using a lancet

1. Clean the area to be lanced with an alcohol swab.
2. Squeeze the end of the fingertip and pierce with a sterile lancet provided.
3. Wipe away the first drop of blood with alcohol swab.
4. With a disposable inverted cup (5 µl) provided, dip the circular end of an inverted cup into the blood specimen.

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