WHO Prequalification of In Vitro Diagnostics Programme PUBLIC REPORT

Product: *CareStart*[™] Malaria Pf (HRP2/pLDH) Ag RDT
Number: PQDx 0188-049-00

Abstract

CareStart[™] Malaria HRP2/pLDH (Pf) labelled as *CareStart*[™] Malaria Pf (HRP2/pLDH) Ag RDT from October 2015 with product codes RMPM-02571 and RMPM-05071 (old product code G0181),¹ manufactured by Access Bio, Inc., at Somerset NJ, USA, rest-of-world regulatory version, was accepted for the WHO list of prequalified in vitro diagnostics and was listed on 28 May 2015.

This public report was amended on 28 June 2018 to reflect the addition of a new product configuration, updates of product codes and inclusion of labelling. The product now has two configurations: **multi kit** (product codes: **RMPM-02571, RMPM-02571CB, RMPM-02571RB, RMPM-02571RI, RMPM-05071, RMPM-05071CB, RMPM-05071RB and RMPM-05071RI**) and **single kit** (product codes: **RMPU-02571, RMPU-02571CB, RMPU-02571RB, RMPU-02571RI, RMPU-05071, RMPU-05071CB, RMPU-05071RB and RMPU-05071RI**). Each multi kit and single kit has 8 different packaging configurations. For further details, please refer to the test kit content table.

Summary of prequalification status for *CareStart*[™] Malaria Pf (HRP2/pLDH) Ag RDT

	Date	Outcome
PQ public report amended	28 June 2018 Inclusion of a new configuration (addition of single-use buffer vials and an 'Instruction Card'), updates of product codes and inclusion of labelling.	listed
Status on PQ list	28 May 2015	listed
Dossier assessment	15 April 2015	MR
Inspection status	8-10 February 2017	MR
Laboratory evaluation	2009	MR

MR: Meets Requirements

CareStart[™] Malaria Pf (HRP2/pLDH) Ag RDT was accepted for the WHO list of prequalified in vitro diagnostics on the basis of data submitted and publicly available information.

¹ The old product codes RMPM-02571, RMPM-05071, and G0181 are obsolete.

CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT is used for the rapid qualitative detection of malaria HRP2 (histidine-rich protein 2) and pLDH (plasmodium lactate dehydrogenase) in human blood as an aid in the diagnosis of malaria infection of *P. falciparum*.

The test is designed for the diagnosis of *P.falciparum* infection. The device contains a membrane strip, which is pre-coated with two monoclonal antibodies as a single line across the strip. Both monoclonal antibodies are *P.falciparum* specific to pLDH and HRP2. The conjugate pad is dispensed with antibodies absorbed on gold particles, which are specific to pLDH and HRP2 of *P.falciparum*.

The presence of two color-lines (one line band in the screen next to “C” and another line band in the screen next to “T”) indicates a positive result for *P.falciparum*. The presence of a line next to “C” indicates a negative result.

Other clinically available tests are required if the obtained results are questionable. A definitive clinical diagnosis should not be made based on the result of this test, but should only be made by a qualified physician after all clinical and laboratory findings have been evaluated.

Storage:

The test kit should be stored at 1 to 40 °C.

Shelf-life:

24 months.

Reading time:

20 minutes.

Consumables:

Configuration	Packaging	Item(s)
Multi kit	25 tests/kit	1 × Assay buffer bottle (4 mL)
		25 × Specimen transfer device
		25 × Lancet
		25 × Alcohol swab
		1 × Instructions for Use
	50 tests/kit	1 × Assay buffer bottle (6mL)
		50 × Specimen transfer device
		50 × Lancet
		50 × Alcohol swab
		1 × Instructions for Use
Single kit	25 single kit bags/kit	Each single kit bag contains one of each of the following items:
		- Single use buffer vial
		- Specimen transfer device

		- Lancet
		- Alcohol swab
		- Instruction card
		1 × Instructions for Use
	40 single kit bags /kit	Each single kit bag contains one of each of the following items:
		- Single use buffer vial
		- Specimen transfer device
		- Lancet
		- Alcohol swab
		- Instruction card
1 × Instructions for use		

Materials needed but not provided:

1. Pair of disposable gloves
2. Timer
3. Sharp box
4. Pencil or pen
5. Sterile gauze or cotton

The test kit contains:

	Packaging	Product code	Item	Instructions for use	Instruction card	RDT Box	Single kit bag (Lab in a pack)	Cassette packaging
Multi kit	25 tests/kit	RMPM-02571	<ul style="list-style-type: none"> Lancets (Capped) Specimen transfer devices (Inverted cup) 	IFU-RMPM71-EFSP	n/a	LB-KMP2571CIA	n/a	PROL6140
		RMPM-02571CB	<ul style="list-style-type: none"> Lancets (Capped) Specimen transfer devices (Bulb) 	IFU-RMPM71CB-EFSP		LB-KMP2571CBA		
		RMPM-02571RB	<ul style="list-style-type: none"> Lancets (Retractable) Specimen transfer devices (Bulb) 	IFU-RMPM71RB-EFSP		LB-KMP2571RBA		
		RMPM-02571RI	<ul style="list-style-type: none"> Lancets (Retractable) Specimen transfer devices (Inverted cup) 	IFU-RMPM71RI-EFSP		LB-KMP2571RIA		
	50 tests/kit	RMPM-05071	<ul style="list-style-type: none"> Lancets (Capped) Specimen transfer devices (Inverted cup) 	IFU-RMPM71-EFSP		LB-KMP5071CIA		
		RMPM-05071CB	<ul style="list-style-type: none"> Lancets (Capped) Specimen transfer devices (Bulb) 	IFU-RMPM71CB-EFSP		LB-KMP5071CBA		
		RMPM-05071RB	<ul style="list-style-type: none"> Lancets (Retractable) Specimen transfer devices (Bulb) 	IFU-RMPM71RB-EFSP		LB-KMP5071RBA		
		RMPM-05071RI	<ul style="list-style-type: none"> Lancets (Retractable) Specimen transfer devices (Inverted cup) 	IFU-RMPM71RI-EFSP		LB-KMP5071RIA		
Single kit	25 tests/kit	RMPU-02571	<ul style="list-style-type: none"> Lancets (Capped) Specimen transfer devices (Inverted cup) 	IFU-RMPU71-EFSP	IFU-RMPU71-E	LB-KUP2571CIA	PABO140	
		RMPU-02571CB	<ul style="list-style-type: none"> Lancets (Capped) Specimen transfer devices (Bulb) 	IFU-RMPU71CB-EFSP	IFU-RMPU71CB-E	LB-KUP2571CBA		
		RMPU-02571RB	<ul style="list-style-type: none"> Lancets (Retractable) Specimen transfer devices (Bulb) 	IFU-RMPU71RB-EFSP	IFU-RMPU71RB-E	LB-KUP2571RBA		
		RMPU-02571RI	<ul style="list-style-type: none"> Lancets (Retractable) Specimen transfer devices (Inverted cup) 	IFU-RMPU71RI-EFSP	IFU-RMPU71RI-E	LB-KUP2571RIA		
	40 tests/kit	RMPU-04071	<ul style="list-style-type: none"> Lancets (Capped) Specimen transfer devices (Inverted cup) 	IFU-RMPU71-EFSP	IFU-RMPU71-E	LB-KUP4071CIA		
		RMPU-04071CB	<ul style="list-style-type: none"> Lancets (Capped) Specimen transfer devices (Bulb) 	IFU-RMPU71CB-EFSP	IFU-RMPU71CB-E	LB-KUP4071CBA		
		RMPU-04071RB	<ul style="list-style-type: none"> Lancets (Retractable) Specimen transfer devices (Bulb) 	IFU-RMPU71RB-EFSP	IFU-RMPU71RB-E	LB-KUP4071RBA		
		RMPU-04071RI	<ul style="list-style-type: none"> Lancets (Retractable) Specimen transfer devices (Inverted cup) 	IFU-RMPU71RI-EFSP	IFU-RMPU71RI-E	LB-KUP4071RIA		

Background information

Access Bio, Inc. submitted an application for prequalification of *CareStart*[™] Malaria Pf (HRP2/pLDH) Ag RDT. Based on the results of the WHO product testing of malaria RDTs Round 5, *CareStart*[™] Malaria Pf (HRP2/pLDH) Ag RDT was given priority for prequalification.

Product dossier assessment

Access Bio, Inc. submitted a product dossier for *CareStart*[™] Malaria Pf (HRP2/pLDH) Ag RDT as per the “Instructions for compilation of a product dossier” (PQDx_018 v1). The information submitted in the product dossier was reviewed by WHO staff and external experts (assessors) appointed by WHO in accordance with the internal report on the screening and assessment of a product dossier (PQDx_009 v2). Based on the product dossier screening and assessment findings, a recommendation was made to accept the product dossier for *CareStart*[™] Malaria Pf (HRP2/pLDH) Ag RDT for prequalification.

Manufacturing site inspection

A comprehensive inspection was performed at the site of manufacture (65 Clyde Road Suite A, Somerset NJ, USA) of *CareStart*[™] Malaria Pf (HRP2/pLDH) Ag RDT in February 2017 as per the “Information for manufacturers on prequalification inspection procedures for the sites of manufacture of diagnostics” (PQDx_014 v1). The inspection found that the manufacturer had an acceptable quality management system and good manufacturing practices in place that ensured the consistent manufacture of a product of good quality. The manufacturer's responses to the nonconformities found at the time of the inspection were accepted 23 April 2018.

Laboratory evaluation

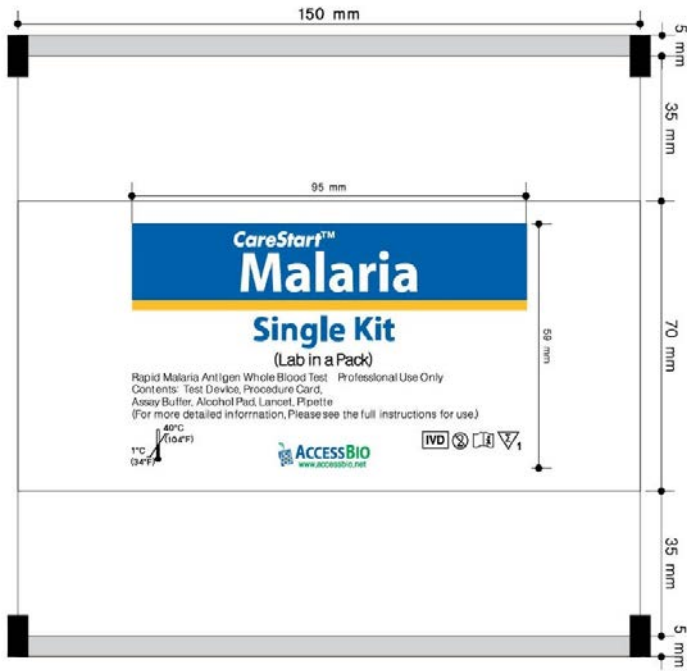
The second round of WHO product testing of RDTs for malaria antigen detection was completed in 2009. The product was evaluated against a *P. falciparum* parasites cultured line panel, *P. falciparum* wild type parasite panel and a *P. falciparum* parasites negative panel. Thermal stability was assessed after 2 months of storage at elevated temperature and humidity, and a descriptive ease of use assessment was recorded.

Based on the demonstrated *P. falciparum* panel detection score (98.0% at 200 parasites/μl), false-positive rate (3.0% for clean negatives and 0.6% for *P. vivax* at 200 1.3% at 2000 or 5000 μl/ml) and invalid rate (0%), *CareStart*[™] Malaria Pf (HRP2/pLDH) Ag RDT meets the current laboratory evaluation requirements for prequalification.

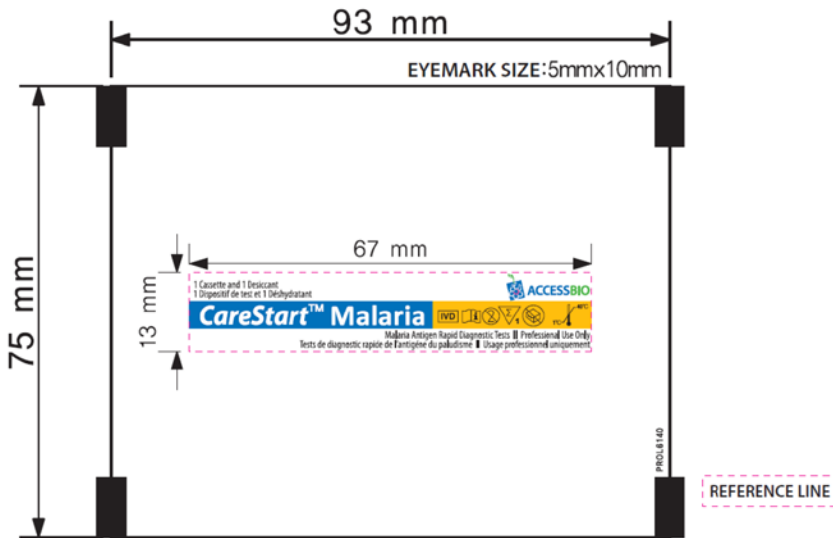
Labelling

- 1. Labels**
- 2. Instructions for use**
- 3. Instruction Cards**
- 4. RDT box designs**

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LIMITATIONS AND INTERFERENCES:

- The following anticoagulants have been validated for use with this test: heparin, EDTA and citrate.
- This test is designed to detect HRP2 and pLDH antigens of *P. falciparum*. Other clinically available tests are required if the obtained results are questionable. A definitive clinical diagnosis should not be made solely based on the result of this test, but should only be made by a qualified physician after all clinical and laboratory findings have been evaluated.
- A faint test line should be read as positive. A false-negative is possible due to a low parasite density. This test may still produce a positive result after successful anti-malarial treatment. Therefore, its use is not recommended for monitoring a response to anti-malarial treatment.³
- If specimens cannot be tested immediately, they should be refrigerated at 2 - 8°C for up to 3 days. The test may produce a false positive result for a patient with acute schistosomiasis, a high level of rheumatoid factor or presence of human anti-mouse IgG antibody.^{6,7}
- No cross reactivity has been observed when the samples from a patient of chagas, dengue and leishmaniasis.
- Results are stable for 30 minutes after assay time of 20 minutes. The prozone effect may cause a false-negative result.

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REFERENCES / RÉFÉRENCES / REFERENCIAS / REFERÊNCIAS

- Valecha N., Eapen A., Usha Devi C., Ravindran J., Aggarwal A., and Subbarao SK. (2002). Field Evaluation of the ICT Malaria P.f./P.v. Immunochromatographic Test in India. *Annals of Tropical Medicine & Parasitology*. 96, 333-336.
- Iqbal J., Hira PR., Sher A., and AL-Enezi AA. (2001). Diagnosis of Imported Malaria by Plasmodium Lactate Dehydrogenase (pLDH) and Histidine-Rich Protein 2 (PfHRP-2)-based Immunocapture Assays. *American Journal of Tropical Medicine and Hygiene*. 64, 20-23.
- Tijtra E., Suprianto S., Dyer ME., Currie BJ. and Anstey NM. (2001). Detection of Histidine-Rich Protein 2 and Panmalarial ICT Malaria P.f./P.v. Test Antigens After Chloroquine Treatment of Uncomplicated falciparum Malaria Does Not Reliably Predict Outcome in Eastern Indonesia. *American Journal of Tropical Medicine and Hygiene*. 65, 593-598.
- Panton LJ., McPhie P., Malloy WL., Welles TE., Taylor DW. and Howard RJ. (1989). Purification and Partial Characterization of an Unusual Protein of Plasmodium falciparum: Histidine-Rich Protein II. *Molecular and Biochemical Parasitology*. 35,149-160.
- Leonard K. Basco, Frederique Marquet, Michael M. Makler, and Jacques Le Bras. (1995). Plasmodium falciparum and Plasmodium vivax: Lactate Dehydrogenase Activity and its Application for in vitro Drug Susceptibility Assay. *Experimental Parasitology*. 80, 260-271.
- Eyal L., Nathan K., Daphna G., Tamar G., Michal S., Esther M. and Eli S. (2011). False-Positive Plasmodium falciparum Histidine-Rich Protein 2 Immunocapture Assay Results for Acute Schistosomiasis Caused by Schistosoma mekongi. *Journal of Clinical Microbiology*. 49, 2331-2332.
- Malaria Rapid Diagnostic Test Performance. Results of WHO product testing of malaria RDTs: round 7 (2015-2016). Table A4.7: Phase 2 false-positive rate for *P. falciparum* in samples containing specific non-malarial infectious pathogens. 124.

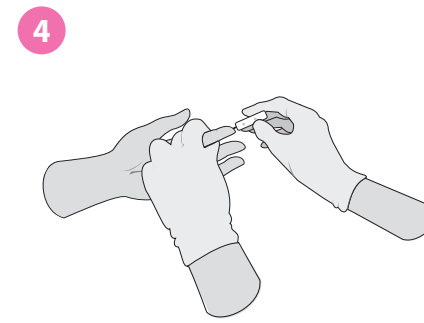
65 Clyde Road, Suite A Tel: 1-732-873-4040 E-mail: info@accessbio.net
Somerset, NJ 08873 U.S.A. Fax: 1-732-873-4043 Web: www.accessbio.net

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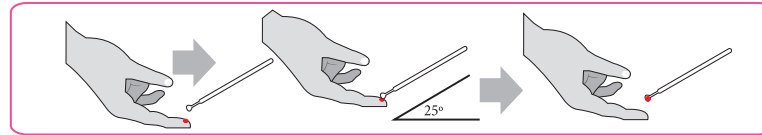
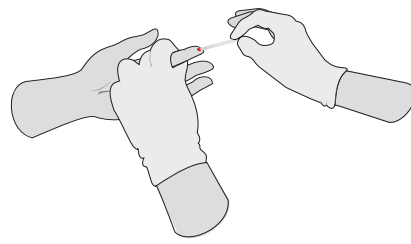
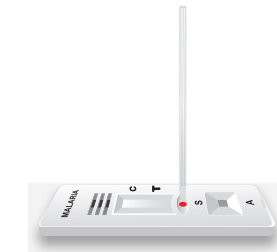
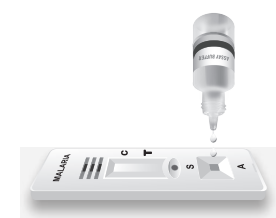
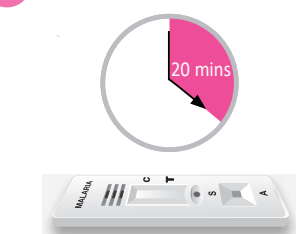
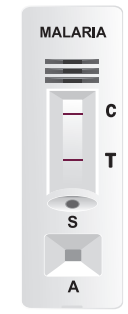
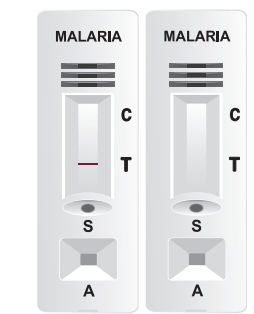
LIMITATIONS AND INTERFERENCES:

- The following anticoagulants have been validated for use with this test: heparin, EDTA and citrate.
- This test is designed to detect HRP2 and pLDH antigens of *P. falciparum*. Other clinically available tests are required if the obtained results are questionable. A definitive clinical diagnosis should not be made solely based on the result of this test, but should only be made by a qualified physician after all clinical and laboratory findings have been evaluated.
- A faint test line should be read as positive. A false-negative is possible due to a low parasite density.
- This test may still produce a positive result after successful anti-malarial treatment. Therefore, its use is not recommended for monitoring a response to anti-malarial treatment.³⁾
- If specimens cannot be tested immediately, they should be refrigerated at 2 - 8°C for up to 3 days.
- The test may produce a false positive result for a patient with acute schistosomiasis, a high level of rheumatoid factor or presence of human anti-mouse IgG antibody.^{6),7)}
- No cross reactivity has been observed when the samples from a patient of chagas, dengue and leishmaniasis.
- Results are stable for 30 minutes after assay time of 20 minutes. The prozone effect may cause a false-negative result.

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REFERENCES / RÉFÉRENCES / REFERENCIAS / REFERÊNCIAS

- Valecha N, Eapen A, Usha Devi C, Ravindran J, Aggarwal A, and Subbarao SK. (2002). Field Evaluation of the ICT Malaria P.f./P.v. Immunochromatographic Test in India. *Annals of Tropical Medicine & Parasitology*. 96, 333-336.
- Iqbal J, Hira PR, Sher A, and Al-Enezi AA. (2001). Diagnosis of Imported Malaria by Plasmodium Lactate Dehydrogenase (pLDH) and Histidine-Rich Protein 2 (PfHRP-2)-based Immunocapture Assays. *American Journal of Tropical Medicine and Hygiene*. 64, 20-23.
- Tijtra E, Suprianto S, Dyer ME, Currie BJ, and Anstey NM. (2001). Detection of Histidine-Rich Protein 2 and Panmalarial ICT Malaria P.f./P.v. Test Antigens After Chloroquine Treatment of Uncomplicated falciparum Malaria Does Not Reliably Predict Outcome in Eastern Indonesia. *American Journal of Tropical Medicine and Hygiene*. 65, 593-598.
- Panton LJ, McPhie P, Malloy WL, Welles TE, Taylor DW, and Howard RJ. (1989). Purification and Partial Characterization of an Unusual Protein of Plasmodium falciparum: Histidine-Rich Protein II. *Molecular and Biochemical Parasitology*. 35, 149-160.
- Leonard K, Basco, Frederique Marquet, Michael M. Makler, and Jacques Le Bras. (1995). Plasmodium falciparum and Plasmodium vivax: Lactate Dehydrogenase Activity and its Application for in vitro Drug Susceptibility Assay. *Experimental Parasitology*. 80, 260-271.
- Eyal L, Nathan K, Daphna G, Tamar G, Michal S, Esther M, and Eli S. (2011). False-Positive Plasmodium falciparum Histidine-Rich Protein 2 Immunocapture Assay Results for Acute Schistosomiasis Caused by Schistosoma mekongi. *Journal of Clinical Microbiology*. 49, 2331-2332.
- Malaria Rapid Diagnostic Test Performance. Results of WHO product testing of malaria RDTs: round 7 (2015-2016). Table A4.7: Phase 2 false-positive rate for *P. falciparum* in samples containing specific non-malarial infectious pathogens. 124.

65 Clyde Road, Suite A Tel: 1-732-873-4040 E-mail: info@accessbio.net
Somerset, NJ 08873 U.S.A. Fax: 1-732-873-4043 Web: www.accessbio.net

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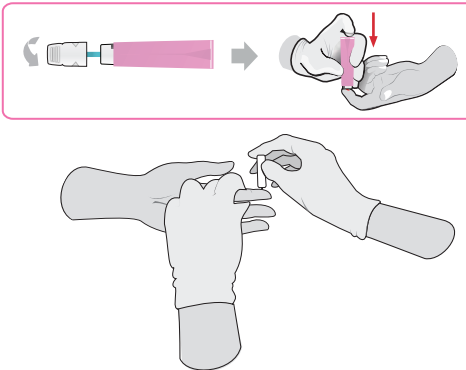
EN Squeeze the end of a fingertip and pierce the cleaned area of the fingertip using a lancet provided. Discard the lancet in the sharps box.

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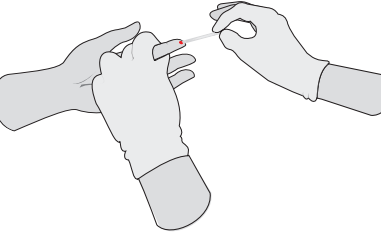
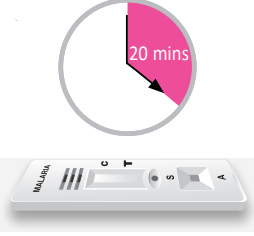
EN Add 3 drops (60µl) of buffer solution into "A" Well. Start a timer.

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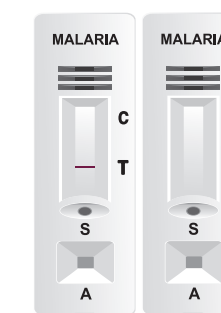
LIMITATIONS AND INTERFERENCES:

- The following anticoagulants have been validated for use with this test: heparin, EDTA and citrate.
- This test is designed to detect HRP2 and pLDH antigens of *P. falciparum*. Other clinically available tests are required if the obtained results are questionable. A definitive clinical diagnosis should not be made solely based on the result of this test, but should only be made by a qualified physician after all clinical and laboratory findings have been evaluated.
- A faint test line should be read as positive. A false-negative is possible due to a low parasite density.
- This test may still produce a positive result after successful anti-malarial treatment. Therefore, its use is not recommended for monitoring a response to anti-malarial treatment.³⁾
- If specimens cannot be tested immediately, they should be refrigerated at 2 - 8°C for up to 3 days.
- The test may produce a false positive result for a patient with acute schistosomiasis, a high level of rheumatoid factor or presence of human anti-mouse IgG antibody.^{6),7)}
- No cross reactivity has been observed when the samples from a patient of chagas, dengue and leishmaniasis.
- Results are stable for 30 minutes after assay time of 20 minutes.
- The prozone effect may cause a false-negative result.

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REFERENCES / RÉFÉRENCES / REFERENCIAS / REFERÊNCIAS

- Valecha N., Eapen A., Usha Devi C., Ravindran J., Aggarwal A., and Subbarao SK. (2002). Field Evaluation of the ICT Malaria P.f./P.v. Immunochromatographic Test in India. *Annals of Tropical Medicine & Parasitology*. 96, 333-336.
- Iqbal J., Hira PR., Sher A., and AL-Enezi AA. (2001). Diagnosis of Imported Malaria by Plasmodium Lactate Dehydrogenase (pLDH) and Histidine-Rich Protein 2 (PfHRP-2)-based Immunocapture Assays. *American Journal of Tropical Medicine and Hygiene*. 64, 20-23.
- Tijtra E., Suprianto S., Dyer ME., Currie BJ. and Anstey NM. (2001). Detection of Histidine-Rich Protein 2 and Panmalarial ICT Malaria P.f./P.v. Test Antigens After Chloroquine Treatment of Uncomplicated falciparum Malaria Does Not Reliably Predict Outcome in Eastern Indonesia. *American Journal of Tropical Medicine and Hygiene*. 65, 593-598.
- Panton LJ., McPhie P., Malloy WL., Welles TE., Taylor DW. and Howard RJ. (1989). Purification and Partial Characterization of an Unusual Protein of Plasmodium falciparum: Histidine-Rich Protein II. *Molecular and Biochemical Parasitology*. 35, 149-160.
- Leonard K. Basco, Frederique Marquet, Michael M. Makler, and Jacques Le Bras. (1995). Plasmodium falciparum and Plasmodium vivax: Lactate Dehydrogenase Activity and its Application for in vitro Drug Susceptibility Assay. *Experimental Parasitology*. 80, 260-271.
- Eyal L., Nathan K., Daphna G., Tamar G., Michal S., Esther M. and Eli S. (2011). False-Positive Plasmodium falciparum Histidine-Rich Protein 2 Immunocapture Assay Results for Acute Schistosomiasis Caused by Schistosoma mekongi. *Journal of Clinical Microbiology*. 49, 2331-2332.
- Malaria Rapid Diagnostic Test Performance. Results of WHO product testing of malaria RDTs: round 7 (2015-2016). Table A4.7: Phase 2 false-positive rate for *P. falciparum* in samples containing specific non-malarial infectious pathogens. 124.

65 Clyde Road, Suite A Tel: 1-732-873-4040 E-mail: info@accessbio.net
Somerset, NJ 08873 U.S.A. Fax: 1-732-873-4043 Web: www.accessbio.net

Instruction cards

CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

REF RMPU-02561CB / RMPU-04061CB / RMPU-02571CB / RMPU-04071CB

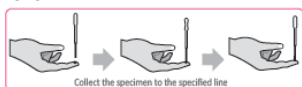
TEST PROCEDURE

1. Put on a new pair of gloves.
2. Write the patient's name on the cassette.
3. Clean the area to be pierced using an alcohol swab. Let the alcohol dry completely before proceeding to the next step.

4.  Squeeze the end of a fingertip and pierce the cleaned area of the fingertip using a lancet provided. Discard the lancet in the sharps box.

5. Wipe out the first drop of blood with sterile gauze or cotton.

6.  Collect the blood sample (5µl) using a provided specimen transfer device or a micropipette.

7.  Add 5µl of whole blood into "S" well.

8.  Add 3 drops (60µl) of buffer solution into "A" Well. Start a timer.

9. Read result at 20 minutes.

INTERPRETATION OF THE TEST RESULT

Negative

The presence of a line next to "C" indicates a negative result.

Positive

The presence of two lines (one line in the result window next to "C" and another in the result window next to "T") indicates a positive result for *P. falciparum*.

Invalid

The test is invalid when a line does not appear next to "C". If this occurs, the test should be repeated using a new cassette.

For detailed information about the product and procedure, refer to the full version of instructions for use (Doc. No. IFU-RMPU71CB-EFSP) included in the RDT box.

TEST PROCEDURE

- Put on a new pair of gloves.
- Write the patient's name on the cassette.
- Clean the area to be pierced using an alcohol swab. Let the alcohol dry completely before proceeding to the next step.

- 

Squeeze the end of a fingertip and pierce the cleaned area of the fingertip using a lancet provided. Discard the lancet in the sharps box.

- Wipe out the first drop of blood with sterile gauze or cotton.

- 

Collect the blood sample (5µl) using a provided specimen transfer device or a micropipette.

- 

Add 5µl of whole blood into "S" well.

- 

Add 3 drops (60µl) of buffer solution into "A" Well. Start a timer.

- Read result at 20 minutes.

INTERPRETATION OF THE TEST RESULT

Negative

The presence of a line next to "C" indicates a negative result.

Positive

The presence of two lines (one line in the result window next to "C" and another in the result window next to "T") indicates a positive result for *P. falciparum*.

Invalid

The test is invalid when a line does not appear next to "C". If this occurs, the test should be repeated using a new cassette.

For detailed information about the product and procedure, refer to the full version of instructions for use (Doc. No. IFU-RMPU71-EFSP) included in the RDT box.

TEST PROCEDURE

REF RMPU-02561RB / RMPU-04061RB / RMPU-02571RB / RMPU-04071RB

INTERPRETATION OF THE TEST RESULT

- Put on a new pair of gloves.
- Write the patient's name on the cassette.
- Clean the area to be pierced using an alcohol swab. Let the alcohol dry completely before proceeding to the next step.

Negative

The presence of a line next to "C" indicates a negative result.

Positive

The presence of two lines (one line in the result window next to "C" and another in the result window next to "T") indicates a positive result for *P. falciparum*.

Invalid

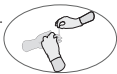
The test is invalid when a line does not appear next to "C". If this occurs, the test should be repeated using a new cassette.

- 

Squeeze the end of a fingertip and pierce the cleaned area of the fingertip using a lancet provided. Discard the lancet in the sharps box.

- Wipe out the first drop of blood with sterile gauze or cotton.

Collect the blood sample (5µl) using a provided specimen transfer device or a micropipette.

- 

Add 5µl of whole blood into "S" well.

- 

Add 3 drops (60µl) of buffer solution into "A" Well. Start a timer.

- Read result at 20 minutes.

CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

TEST PROCEDURE

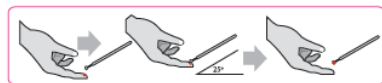
1. Put on a new pair of gloves.
2. Write the patient's name on the cassette.
3. Clean the area to be pierced using an alcohol swab. Let the alcohol dry completely before proceeding to the next step.

4.  Squeeze the end of a fingertip and pierce the cleaned area of the fingertip using a lancet provided. Discard the lancet in the sharps box.

5. Wipe out the first drop of blood with sterile gauze or cotton.

6.  Collect the blood sample (5µl) using a provided specimen transfer device or a micropipette.

7.  Add 5µl of whole blood into "S" well.

8.  Add 3 drops (60µl) of buffer solution into "A" Well. Start a timer.

9. Read result at 20 minutes.

REF RMPU-02561RI / RMPU-04061RI / RMPU-02571RI / RMPU-04071RI

INTERPRETATION OF THE TEST RESULT

Negative

The presence of a line next to "C" indicates a negative result.

Positive

The presence of two lines (one line in the result window next to "C" and another in the result window next to "T") indicates a positive result for *P. falciparum*.

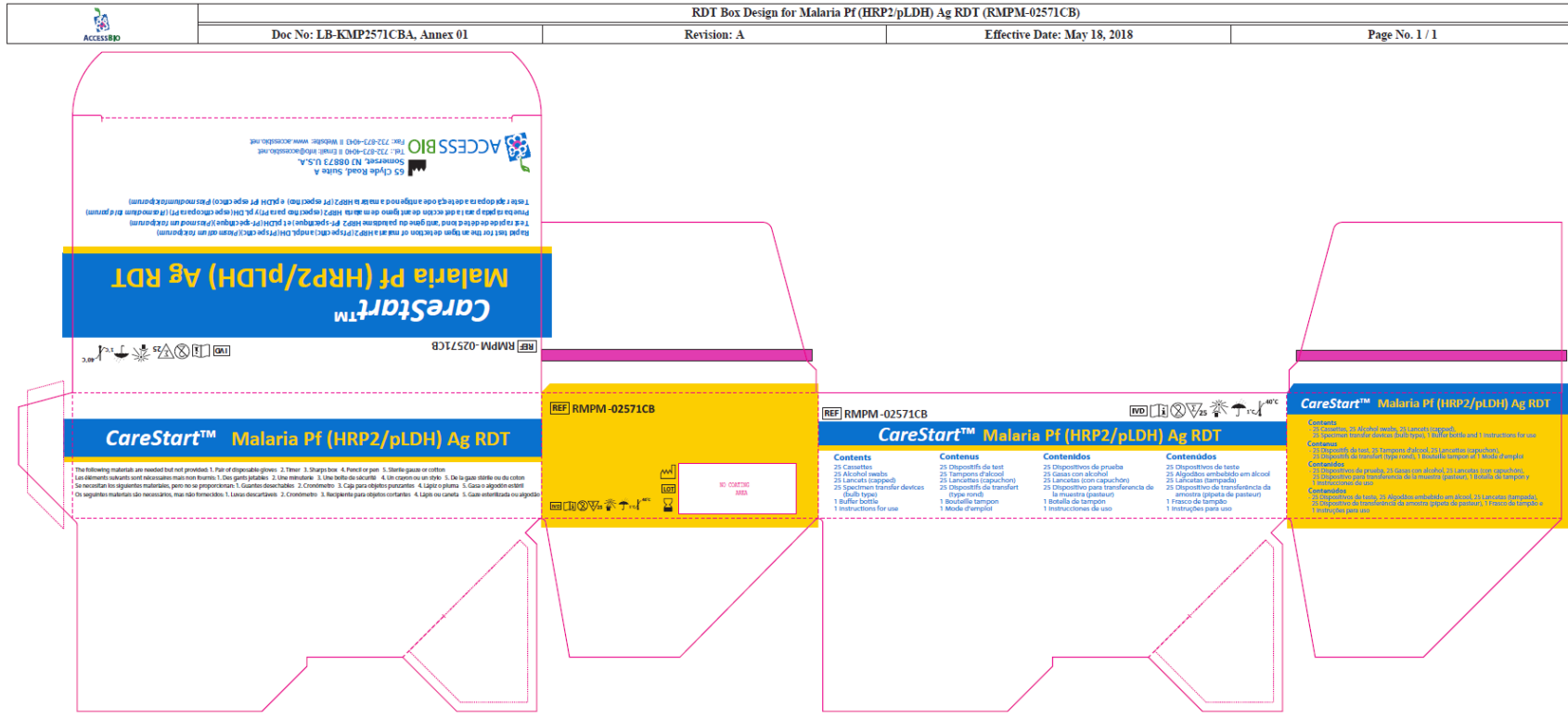
Invalid

The test is invalid when a line does not appear next to "C". If this occurs, the test should be repeated using a new cassette.

For detailed information about the product and procedure, refer to the full version of instructions for use (Doc. No. IFU-RMPU71RI-EFSP) included in the RDT box.

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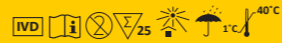
CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents

- 25 Cassettes
- 25 Alcohol swabs
- 25 Lancets (capped)
- 25 Specimen transfer devices (inverted cup type)
- 1 Buffer bottle
- 1 Instructions for use

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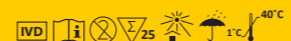
CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents

- 25 Cassettes
- 25 Alcohol swabs
- 25 Lancets (retractable)
- 25 Specimen transfer devices (inverted cup type)
- 1 Buffer bottle
- 1 Instructions for use

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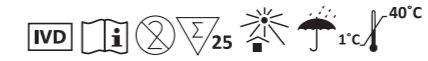
CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents
 - 25 Cassettes, 25 Alcohol swabs, 25 Lancets (retractable), 25 Specimen transfer devices (bulb type), 1 Buffer bottle and 1 Instructions for use

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents

- 50 Cassettes
- 50 Alcohol swabs
- 50 Lancets (capped)
- 50 Specimen transfer devices (bulb type)
- 1 Buffer bottle
- 1 Instructions for use

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents

- 50 Cassettes, 50 Alcohol swabs, 50 Lancets (capped), 50 Specimen transfer devices (bulb type), 1 Buffer bottle and 1 Instructions for use

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents

- 50 Cassettes
- 50 Alcohol swabs
- 50 Lancets (capped)
- 50 Specimen transfer devices (inverted cup type)
- 1 Buffer bottle
- 1 Instructions for use

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents

- 50 Cassettes, 50 Alcohol swabs, 50 Lancets (capped), 50 Specimen transfer devices (inverted cup type), 1 Buffer bottle and 1 Instructions for use

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents

- 50 Cassettes
- 50 Alcohol swabs
- 50 Lancets (retractable)
- 50 Specimen transfer devices (bulb type)
- 1 Buffer bottle
- 1 Instructions for use

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents

- 50 Cassettes, 50 Alcohol swabs, 50 Lancets (retractable), 50 Specimen transfer devices (bulb type), 1 Buffer bottle and 1 Instructions for use

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents

- 50 Cassettes
- 50 Alcohol swabs
- 50 Lancets (retractable)
- 50 Specimen transfer devices (inverted cup type)
- 1 Buffer bottle
- 1 Instructions for use

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents

- 50 Cassettes, 50 Alcohol swabs, 50 Lancets (retractable), 50 Specimen transfer devices (inverted cup type), 1 Buffer bottle and 1 Instructions for use

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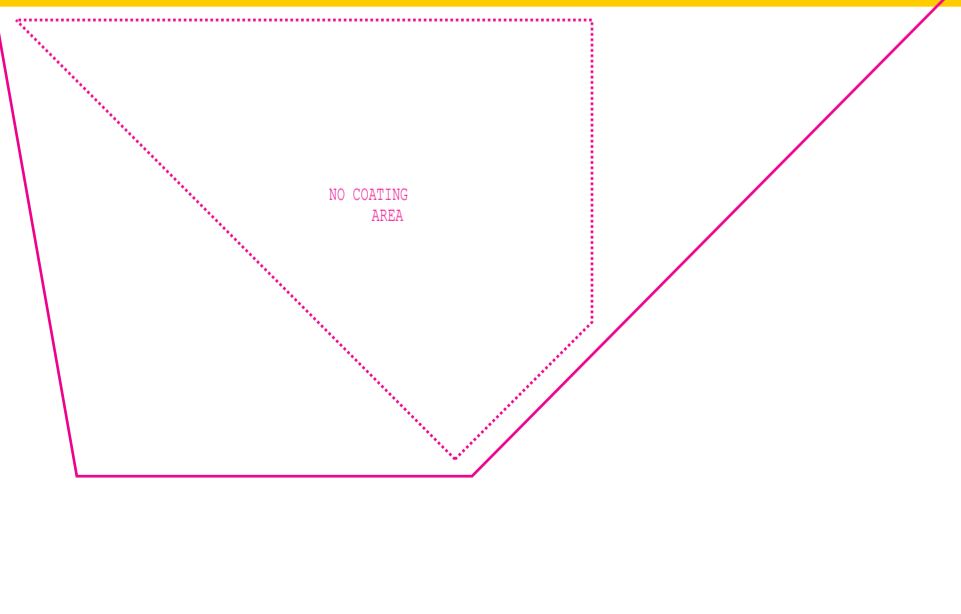
CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

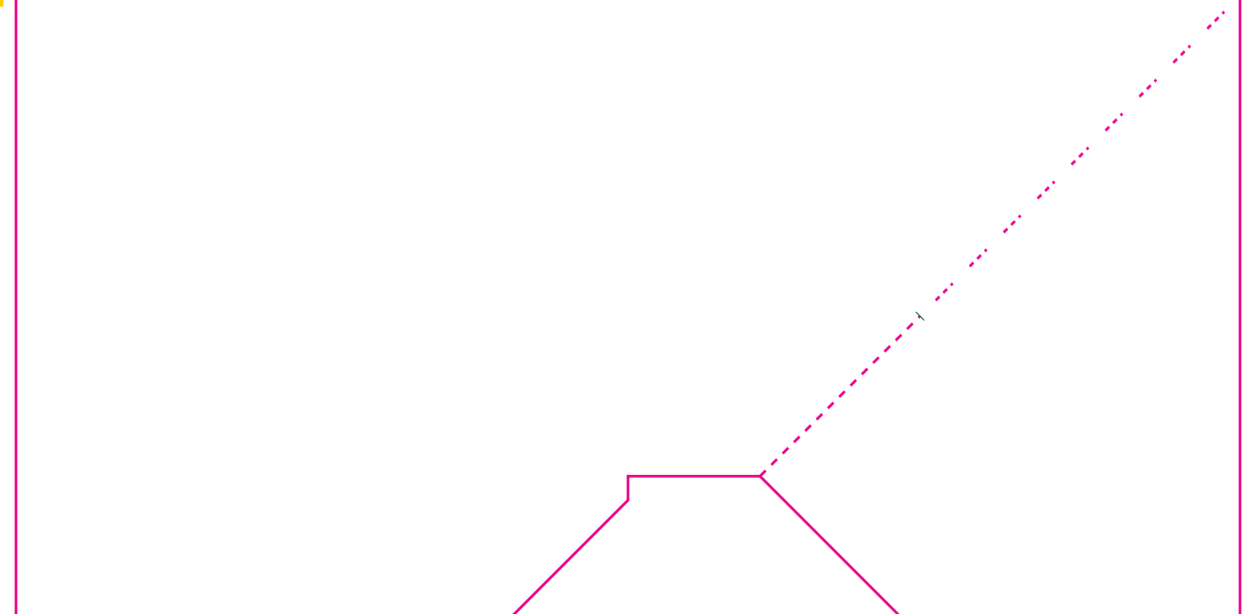
Single Kit

Contents
 1 Instructions for Use and 25 Single kit bags - Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (capped), 1 specimen transfer device (bulb type), 1 buffer vial and 1 instruction card

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

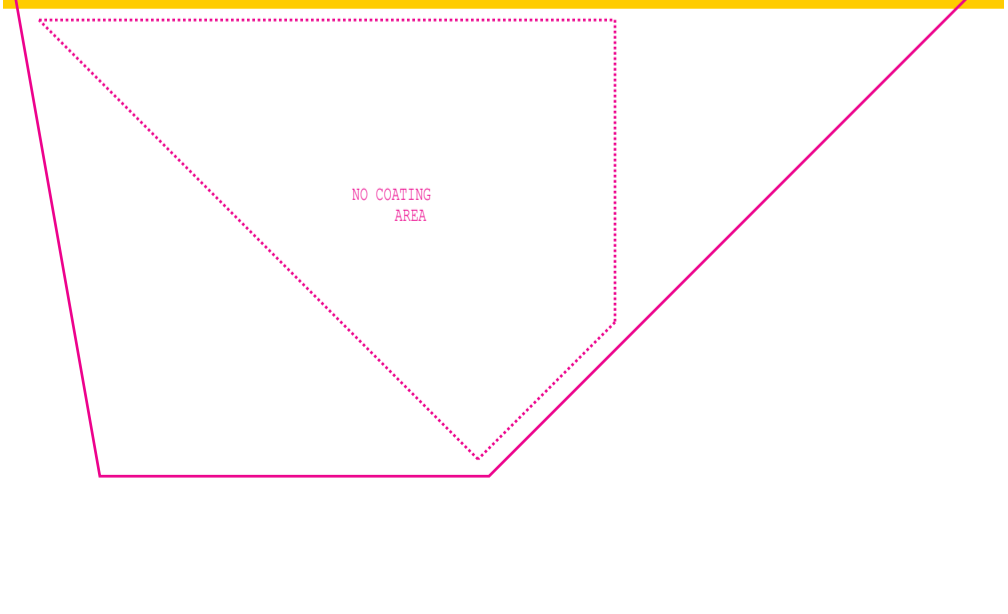
Single Kit

Contents - 1 Instructions for Use and 25 Single kit bags
 Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (capped), 1 specimen transfer device (bulb type), 1 buffer vial and 1 instruction card

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Single Kit

Contents
 1 Instructions for Use and 25 Single kit bags - Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (capped), 1 specimen transfer device (inverted cup type), 1 buffer vial and 1 instruction card

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

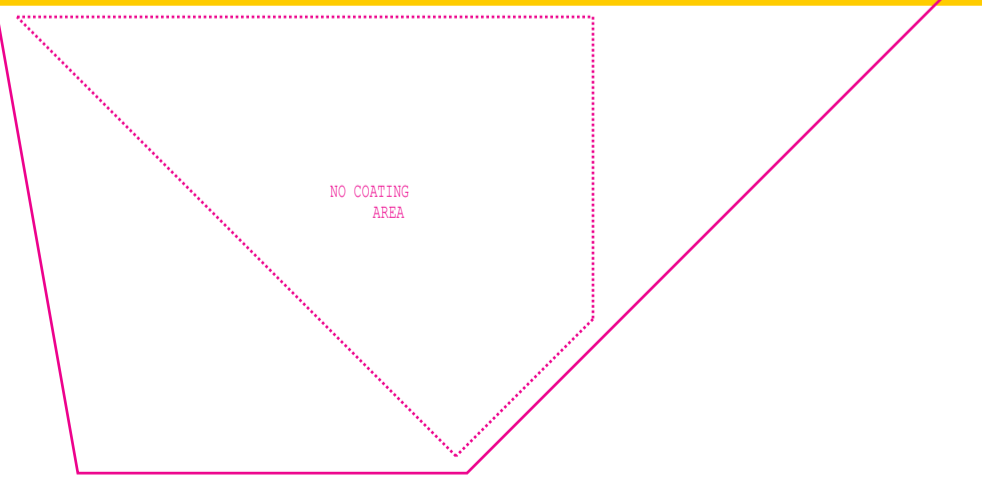
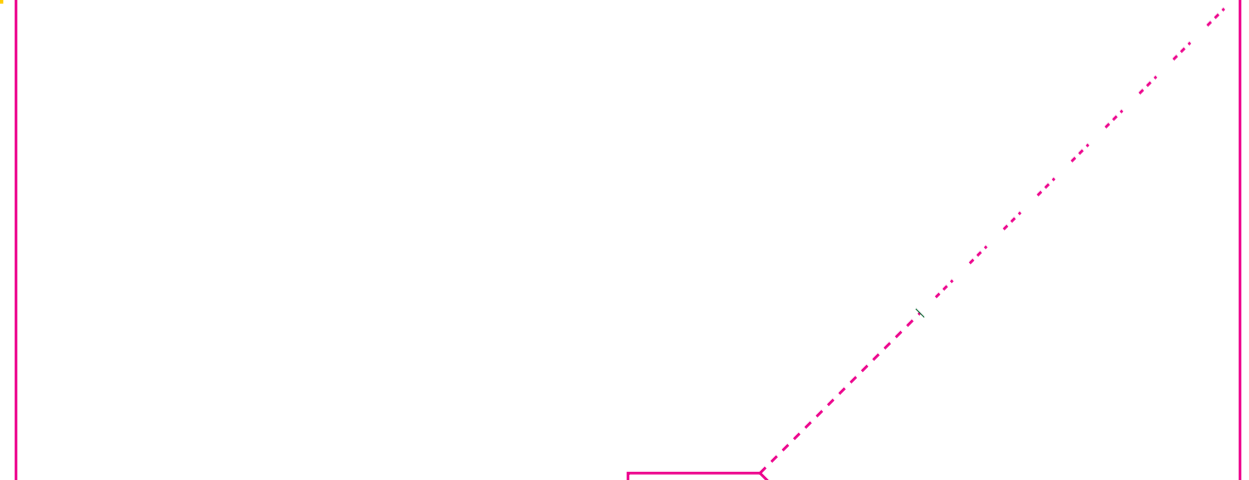
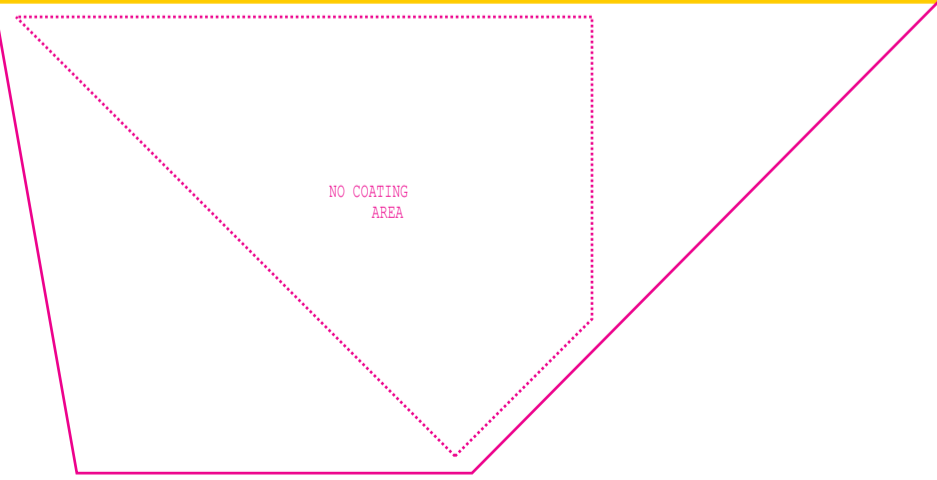
Single Kit

Contents - 1 Instructions for Use and 25 Single kit bags
 Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (capped), 1 specimen transfer device (inverted cup type), 1 buffer vial and 1 instruction card

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Single Kit

Contents
 1 Instructions for Use and 25 Single kit bags - Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (retractable), 1 specimen transfer device (bulb type), 1 buffer vial and 1 instruction card

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Single Kit

Contents - 1 Instructions for Use and 25 Single kit bags
 Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (retractable), 1 specimen transfer device (bulb type), 1 buffer vial and 1 instruction card

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Single Kit

Contents
 1 Instructions for Use and 25 Single kit bags - Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (retractable), 1 specimen transfer device (inverted cup type), 1 buffer vial and 1 instruction card

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Single Kit

Contents - 1 Instructions for Use and 25 Single kit bags
 Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (retractable), 1 specimen transfer device (inverted cup type), 1 buffer vial and 1 instruction card

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 Tel.: 732-873-4040 | Email: info@accessbio.net

 Fax: 732-873-4043 | Website: www.accessbio.net

Rapid test for the antigen detection of malaria HRP2 (Pf specific) and pLDH (Pf specific) (*Plasmodium falciparum*)

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Single Kit

The following materials are needed but not provided: 1. Pair of disposable gloves 2. Timer 3. Sharps box 4. Pencil or pen 5. Sterile gauze or cotton

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Single Kit

Contents - 1 Instructions for Use and 40 Single kit bags
 Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (capped), 1 specimen transfer device (bulb type), 1 buffer vial and 1 instruction card

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Single Kit

Contents - 1 Instructions for Use and 40 Single kit bags
 Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (retractable), 1 specimen transfer device (bulb type), 1 buffer vial and 1 instruction card

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Contents

1 Instructions for Use and 25 Single kit bags - Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (retractable), 1 specimen transfer device (inverted cup type), 1 buffer vial and 1 instruction card

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CareStart™ Malaria Pf (HRP2/pLDH) Ag RDT

Single Kit

Contents - 1 Instructions for Use and 40 Single kit bags
 Each bag contains 1 cassette, 1 alcohol swab, 1 lancet (retractable), 1 specimen transfer device (inverted cup type), 1 buffer vial and 1 instruction card

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