

**WHO Prequalification of In Vitro Diagnostics
PUBLIC REPORT**

**Product: cobas HIV-1/HIV-2 Qualitative nucleic acid test for use on the cobas 6800/8800 and cobas HIV-1/HIV-2 Qualitative Nucleic acid test for use on the cobas 5800/6800/8800 Systems
WHO reference number: PQDx 0528-046-00**

cobas HIV-1/HIV-2 Qualitative nucleic acid test for use on the cobas 6800/8800 and cobas HIV-1/HIV-2 Qualitative Nucleic acid test for use on the cobas 5800/6800/8800 Systems with product codes 07862113190 and 09040528190, manufactured by Roche Molecular Systems, Inc., CE-marked regulatory version, was accepted for the WHO list of prequalified in vitro diagnostics and was listed on 10 November 2023.

Summary of WHO Prequalification Assessment for cobas HIV-1/HIV-2 Qualitative nucleic acid test for use on the cobas 6800/8800 and cobas HIV-1/HIV-2 Qualitative Nucleic acid test for use on the cobas 5800/6800/8800 Systems

	Date	Outcome
Prequalification listing	10 November 2023	Listed
Dossier review	Abridged assessment	NA
Site inspection(s) of the quality management system	13 November 2020	MR
Product performance evaluation	3 rd and 4 th quarters of 2022	MR

MR: Meet Requirements, NA: Not Applicable.

Intended use:

According to the claim of intended use from Roche Molecular Systems, Inc., “cobas HIV-1/HIV-2 Qualitative nucleic acid test for use on the cobas 6800/8800 and cobas HIV-1/HIV-2 Qualitative Nucleic acid test for use on the cobas 5800/6800/8800 Systems is an in vitro nucleic acid amplification test for the qualitative detection and differentiation of human immunodeficiency virus (HIV) type 1 (HIV-1) and type 2 (HIV-2) in human serum, plasma, and dried blood spots (DBS).

The test is intended to be used as an aid in diagnosis of HIV-1/HIV-2. Detection of HIV-1 or HIV-2 nucleic acid is indicative of HIV-1 or HIV-2 infection, respectively. The presence of HIV-1 or HIV-2 nucleic acid in the plasma or serum of individuals without antibodies to HIV-1 or HIV-2 is indicative of acute or primary infection. In infants born to HIV-infected mothers and who have maternal antibodies to HIV-1 or HIV-2, the presence of HIV nucleic acid is indicative

of active infection. cobas HIV-1/HIV-2 Qualitative may also be used to confirm HIV-1 or HIV-2 infection in an individual with specimens reactive for HIV-1 or HIV-2 antibodies or antigens.”

Assay description:

According to the claim of assay description from Roche Molecular Systems, Inc, “*cobas HIV-1/HIV-2 Qualitative is based on fully automated sample preparation (nucleic acid extraction and purification) followed by PCR amplification and detection. The cobas 5800 System is designed as one integrated instrument. The cobas 6800/8800 Systems consist of the sample supply module, the transfer module, the processing module, and the analytic module. Automated data management is performed by the cobas 5800 or cobas 6800/8800 Systems software which assigns test results for all tests as non-reactive, reactive, or invalid. Results can be reviewed directly on the system screen, exported, or printed as a PDF report.*

Nucleic acid from patient samples and added armored RNA internal control (IC) molecules (which serve as the sample preparation and amplification/detection process control) is simultaneously extracted. In addition, the test utilizes three external controls: two positive and a negative control. In summary, viral nucleic acid is released by addition of proteinase and lysis reagent to the sample. The released nucleic acid binds to the silica surface of the added magnetic glass particles. Unbound substances and impurities, such as denatured protein, cellular debris and potential PCR inhibitors are removed with subsequent wash steps and purified nucleic acid is eluted from the magnetic glass particles with elution buffer at elevated temperature.

Selective amplification of target nucleic acid from the sample is achieved by the use of target virus-specific forward and reverse primers which are selected from highly conserved regions of the HIV-1 and HIV-2 genomes. The HIV-1 gag gene, the HIV-1 LTR region (dual target for HIV-1) and the HIV-2 LTR region are amplified by cobas HIV-1/HIV-2 Qualitative.

Selective amplification of IC is achieved by the use of sequence-specific forward and reverse primers which are selected to have no homology with the HIV-1 or HIV-2 genomes. A thermostable DNA polymerase enzyme is used for both reverse-transcription and PCR amplification. The target and IC sequences are amplified simultaneously utilizing a universal PCR amplification profile with predefined temperature steps and number of cycles. The master mix includes deoxyuridine triphosphate (dUTP), instead of deoxythymidine triphosphate (dTTP), which is incorporated into the newly synthesized DNA (amplicon). Any contaminating amplicons from previous PCR runs are eliminated by the AmpErase enzyme, which is included in the PCR master mix, during the first thermal cycling step. However, newly formed amplicons are not eliminated since the AmpErase enzyme is inactivated once exposed to temperatures above 55°C.

cobas HIV-1/HIV-2 Qualitative master mix contains two detection probes specific for the HIV-1 target sequences, one for HIV-2 target sequences and one for the IC. The probes are labeled with target specific fluorescent reporter dyes allowing simultaneous detection of HIV-1 target, HIV-2 target and IC in three different target channels. When not bound to the target sequence, the fluorescent signal of the intact probes is suppressed by a quencher dye. During the PCR amplification step, hybridization of the probes to the specific single-stranded DNA template results in cleavage of the probe by the 5' to 3' exonuclease activity of the DNA polymerase resulting in separation of the reporter and quencher dyes and the generation of

a fluorescent signal. With each PCR cycle, increasing amounts of cleaved probes are generated and the cumulative signal of the reporter dye increases concomitantly. Real-time detection and discrimination of PCR products is accomplished by measuring the fluorescence of the released reporter dyes for the viral targets and IC, respectively.”

Test kit contents:

Component	Product code and number of tests/kit (T/k)
cobas HIV-1/HIV-2 Qualitative	07862113190 (96 T/kit) (for use on the cobas 6800/8800 Systems)
cobas HIV-1/HIV-2 Qualitative	09040528190 (192 T/kit) (for use on the cobas 5800/6800/8800 Systems)

Items required but not provided:

Item	Description (product code and number of tests/kit (T/kit))
cobas HIV-1/HIV-2 Qualitative Control Kit	07862091190 (for use on the cobas 6800/8800 Systems)
cobas HIV-1/HIV-2 Qualitative Control Kit	09040536190 (for use on the cobas 5800/6800/8800 Systems)
cobas NHP Negative Control Kit	07002220190 (for use on the cobas 6800/8800 Systems)
cobas NHP Negative Control Kit	09051554190 (for use on the cobas 5800/6800/8800 Systems)
cobas Specimen Pre-Extraction Reagent	08064695190
cobas omni reagents for sample preparation	
cobas omni MGP Reagent (MGP)	06997546190, 480 T/kit
cobas omni Specimen Diluent (SPEC DIL)	06997511190, 4 x 875 mL
cobas omni Lysis Reagent (LYS)	06997538190, 4 x 875 mL
cobas omni Wash Reagent (WASH)	Product code 06997503190, 4.2 L

Material and consumables for use on cobas 5800 System

Material	P/N
cobas omni Processing Plate 24	08413975001
cobas omni Amplification Plate 24	08499853001
cobas omni Liquid Waste Plate 24	08413983001
Tip CORE TIPS with Filter, 1mL	04639642001
Tip CORE TIPS with Filter, 300uL	07345607001
cobas omni Liquid Waste Container	07094388001
cobas omni Lysis Reagent	06997538190
cobas omni MGP Reagent	06997546190
cobas omni Specimen Diluent	06997511190
cobas omni Wash Reagent	06997503190
Solid Waste Bag or Solid Waste Bag With Insert	07435967001 or 08030073001

Materials and consumables for use on cobas 6800/8800 Systems

Material	P/N
cobas omni Processing Plate	05534917001
cobas omni Amplification Plate	05534941001
cobas omni Pipette Tips	05534925001
cobas omni Liquid Waste Container	07094388001
cobas omni Lysis Reagent	06997538190
cobas omni MGP Reagent	06997546190
cobas omni Specimen Diluent	06997511190
cobas omni Wash Reagent	06997503190
Solid Waste Bag and Solid Waste Container or Solid Waste Bag With Insert and Kit Drawer	07435967001 and 07094361001 or 08030073001 and 08387281001

Other materials and consumables required for DBS application only

Materials
Whatman 903 filter card, Munktell Specimen Collection card TFN or equivalent (12-13 mm spot diameter)
Tubes, 5 mL, internal thread, 12.5 mm diameter, polypropylene (i.e., Cryo.s) with caps
Eppendorf Thermomixer (e.g., model R 5355 or equivalent) with Thermoblock for 24 cryotubes.
Sterile or disposable forceps or tweezers
Resealable bags and desiccant sachets (for DBS storage)

Instrumentation and software required:

The cobas 5800 software and cobas HIV-1/HIV-2 Qualitative analysis package for the cobas 5800 System, SW cobas 5800 HIV-1/2 Qual-Serum/Plasma ASAP and/or SW cobas 5800 HIV-1/2 Qual-DBS ASAP, shall be installed on the cobas 5800 instrument. The Data Manager software and PC for the cobas 5800 System will be provided with the system.

The cobas 6800/8800 software and cobas HIV-1/HIV-2 Qualitative analysis package(s) for the cobas 6800/8800 Systems, SW cobas HIV-1/2 Qual-Serum/Plasma ASAP and/or SW cobas HIV-1/2 Qual-DBS ASAP, shall be installed on the instrument(s). The Instrument Gateway (IG) server will be provided with the system.

Equipment	P/N
cobas 5800 System	08707464001
cobas 6800 System (Option Moveable)	05524245001 and 06379672001
cobas 6800 System (Fix)	05524245001 and 06379664001
Cobas 8800 System	05412722001
Sample Supply Module	06301037001

Storage (when a reagent is not on the system):

Reagent	Storage temperature
cobas HIV-1/HIV-2 Qualitative	2–8°C
cobas HIV-1/HIV-2 Qualitative Control Kit	2–8°C
cobas NHP Negative Control Kit	2–8°C
cobas omni Lysis Reagent	2–8°C
cobas omni MGP Reagent	2–8°C

cobas omni Specimen Diluent	2–8°C
cobas omni Wash Reagent	15–30°

Shelf-life upon manufacture:

24 months.

Warnings/limitations:

Please refer to the instructions for use attached to this public report.

Prioritization for prequalification

Based on the established eligibility criteria, the cobas HIV-1/HIV-2 Qualitative nucleic acid test for use on the cobas 6800/8800 and cobas HIV-1/HIV-2 Qualitative Nucleic acid test for use on the cobas 5800/6800/8800 Systems was given priority for the WHO prequalification assessment.

Product dossier assessment

In accordance with the WHO procedure for abridged prequalification assessment, Roche Molecular Systems, Inc. was not required to submit a product dossier for cobas HIV-1/HIV-2 Qualitative as per the “*Instructions for compilation of a product dossier*” (PQDx_018 version 3). Notwithstanding, certain aspects of the product dossier previously submitted for stringent regulatory review were reviewed by an assessor during the site inspection.

Manufacturing site inspection

A desk assessment of Roche Molecular Systems, Inc., located at 1080 US Hwy 202 South Branchburg Township, 8879, United States of America, was completed on 13 November 2020. At the time of considering the product application for Prequalification, the Manufacturer of the product had a well-established quality management system and manufacturing practices in place that would support the manufacture of a product of consistent quality.

Routine inspections of the Manufacturing site will be conducted with copies of the WHO Public Inspection Report (WHOPIR) published on the WHO Prequalification web page as per Resolution WHA57.14 of the World Health Assembly. Note that a WHOPIR reflects the information on the most current inspection performed at a manufacturing site for *in vitro* diagnostic products and gives a summary of the inspection findings.

Information on the most current inspection can be found at:

<https://extranet.who.int/pqweb/inspection-services/prequalification-reports/whopirs-vitro-diagnostics>

All published WHOPIRs are with the agreement of the manufacturer.

Product performance evaluation

cobas HIV-1/HIV-2 Qualitative nucleic acid test for use on the cobas 6800/8800 and cobas HIV-1/HIV-2 Qualitative Nucleic acid test for use on the cobas 5800/6800/8800 Systems (Roche Molecular Systems, Inc.) was evaluated at NHLS (South Africa) in the 3rd and 4th quarters of 2022, at RETRO CI (Côte d'Ivoire) in the 3rd quarter of 2022, and the CDC (United States) in the 2nd and 3rd quarters of 2021, on behalf of WHO in the according to protocol PQDx_199, version 3.0.

cobas HIV-1/HIV-2 Qualitative is intended for use in serum, plasma and dried blood spots. However, serum and plasma were not evaluated because WHO indicates the preferred use of whole blood for infant diagnosis due to high rates of maternal treatment and/or infant prophylaxis that can lead to false negative infant test results when only serum or plasma is used (only detecting circulating RNA). Whole blood allows for the additional detection of intracellular RNA and DNA to increase the sensitivity (or likelihood of detection).

Clinical performance evaluation

In this limited laboratory-based evaluation of clinical performance characteristics, a panel of 276 dried blood spot (DBS) specimens were used at the NHLS site, and 344 DBS were used at the CDC RETRO-CI site. The specimens were characterized using the following reference assay: COBAS AmpliPrep/COBAS TaqMan HIV-1 Qualitative Test, version 2.0 (Roche Diagnostics).

Clinical performance characteristics for the detection of HIV-1		
	Infants < 18 months	Adults
Sensitivity, % (95% CI)	100.0% (97.4-100) (N=141)	97.1% (91.9-99.4) (N=105)
Specificity, % (95% CI)	100.0% (98.2-100) (N=204)	100% (97.0-100) (N=123)
Invalid/error rate	1.3% (N= 620)	

In the absence of an adequate reference nucleic acid test for the detection of HIV-2, the performance of cobas HIV-1/HIV-2 Qualitative for the detection of HIV-2 was not assessed. However, a descriptive analysis was conducted on a panel of 47 HIV-2-positive specimens characterized using an ELISA-based algorithm followed by INNO-LIA (Fujirebio) for the differentiation of HIV-1 and HIV-2. Of 38 specimens classified as HIV-2 only by serology, 20 were detected as HIV-2 by cobas HIV-1/HIV-2 Qualitative and 18 were not detected. Of the 9 specimens classified as HIV-1/HIV-2 by serology, 4 were detected as HIV-1/2, 3 as HIV-1, 1 as HIV-2 and 1 was not detected by cobas HIV-1/HIV-2 Qualitative.

Analytical performance evaluation

Analytical performance characteristics	
Limit of detection (LoD) using the 4 th WHO International Standard for HIV-1 RNA –DBS specimens	The LoD for HIV-1 was estimated at 805.8 IU/mL (95% CI: 75-4335) or 474 cp/mL (95% CI: 162-2550) using the conversion factor provided by the manufacturer. The LoD claimed by the manufacturer (255 cp/mL) was verified.
Limit of detection (LoD) using the 2 nd WHO International Standard for HIV-2 RNA – DBS specimens	The LoD for HIV-2 was estimated at 227 IU/mL (95% CI: 137-612) or 1136 cp/mL (95% CI: 683-3062) using the conversion factor provided by the manufacturer. The LoD claimed by the manufacturer (984 cp/mL) was verified.
Within-laboratory precision (reproducibility)	The hit rate for HIV-1 subtype B, at 765 cp/mL, was 100% (40/40). The hit rate for HIV-1 subtype C, at 765 cp/mL, was 100% (39/39). The hit rate for HIV-2, at 3112 cp/mL, was 100% (40/40).
Subtype detection	HIV-1 subtypes D, A and AG were detected at 5000 cp/mL.
Cross-contamination/carry-over	No cross-contamination was observed when high positive and negative specimens were tested alternatively.

Operational characteristics and ease of use

This assay requires laboratory equipment and cannot be performed in laboratories with limited facilities or non-laboratory settings. The instrument requires a stable source of electricity and significant physical space. Furthermore, training and implementation of good laboratory practice are essential to obtaining accurate results. Adequate technical support from the manufacturer or representative is critical.

The assay was found easy to use by the operators performing the evaluation, who had received a 5-day training by the manufacturer prior to the evaluation.

Key operational characteristics	
Number of steps for one specimen*	9 steps in total for DBS 1 step with precision pipetting <i>Note: The number of steps may change depending on the workflow of the testing laboratory.</i>
Number of steps for instrument management**	3 steps per day
Time to result for one run	210 minutes

Operator hands-on time for one run	65 minutes
Level of automation	Fully automated
Quality controls	The manufacturer provides QC, and should be purchased separately. Three controls (negative, HIV-1M/HIV-2 (+)C and HIV-1O (+)C) are processed with each batch.
Operating temperature	15- 28°C
Result display and connectivity	Results are displayed on the instrument. They may be printed using a standard printer. The results can be exported to the laboratory information system and other health information systems.
Power sources	Main power The use of a UPS is strongly recommended, as stable electricity is required.
Biosafety (<i>outside of infectious specimen handling</i>)	Reagents and waste need to be handled according to the manufacturer's instructions. If not, they may cause irritations and corrosion or may be carcinogenic. Sodium azide, sensitizers, guanidine, and other hazardous chemicals should be handled according to the manufacturer's instructions.
Waste	The volume of liquid waste is approx. 2.2 L per run of 96 tests. The weight of solid waste is approx. 2 kg per run of 96 tests. Waste disposal requires specific measures in addition to usual laboratory biohazard waste disposal procedures. Liquid waste contains guanidinium thiocyanate (GTC), an environmentally hazardous chemical that cannot be mixed with bleach and must be disposed of according to local guidance.
Calibration	No calibration is needed.
Maintenance	Daily and weekly maintenance is required.
Other specific requirements	Floor stability must be formally assessed before Installation.

Based on these results, the performance evaluation for cobas HIV-1/HIV-2 Qualitative nucleic acid test for use on the cobas 6800/8800 and cobas HIV-1/HIV-2 Qualitative Nucleic acid test for use on the cobas 5800/6800/8800 Systems meets the WHO prequalification requirements.

Labelling

- 1. Labels**
- 2. Instructions for use**

1. Labels

1.1 Labels for cobas HIV-1/HIV-2 Qualitative for use on the cobas 6800/8800 Systems (product code 07862113190 (96 T/kit))

cobas

Rx Only
96
3

IVD

20°C
8°C

Nucleic acid test for use on the **cobas® 6800/8800 Systems**

cobas® HIV-1/HIV-2 Qualitative

REF P/N: 07862113190

Nucleic acid test for use on the
cobas® 6800/8800 Systems

Nukleinsäuretest zur Verwendung auf den
cobas® 6800/8800 Systemen

Test de l'acide nucléique à utiliser avec les systèmes
cobas® 6800/8800

Test degli acidi nucleici per l'utilizzo su sistemi
cobas® 6800/8800

Prueba para ácidos nucleicos para uso en sistemas
cobas® 6800/8800

Teste de ácidos nucleicos para utilização nos sistemas
cobas® 6800/8800

Nukleinsyretest til brug på
cobas® 6800/8800-systemerne

Nukleinsyretest för användning på
cobas® 6800/8800-systemen

Test kwasu nukleinowego do stosowania w
systemach **cobas® 6800/8800**

Test lyselny nukleovej na použítie v systémoch
cobas® 6800/8800

CONTENT

HIV-1/HIV-2 Cassette	x 1
MMX-R1	5.5 mL
HIV-1/HIV-2 MMX-R2	6 mL
PASE	13 mL
EB	13 mL
IC	13 mL

For USA*

HIV-1/HIV-2 Cassette	x 1
MMX-R1	5.5 mL
Manganese acetate, Potassium hydroxide, < 0.1% Sodium azide	
HIV-1/HIV-2 MMX-R2	6 mL

Tricine buffer, Potassium acetate, 18% Dimethyl sulfoxide, Glycerol, Tween 20, EDTA, < 0.06% dATP, dCTP, dGTP, < 0.14% dUTP, < 0.01% Upstream and downstream HIV-1, HIV-2 and internal control primers, < 0.01% Fluorescent-labeled HIV-1, HIV-2 probes, < 0.01% Fluorescent-labeled internal control probe, < 0.01% Oligonucleotide aptamer, < 0.01% Z05D DNA polymerase, < 0.01% AmpErase (uracil-N- glycosylase) enzyme (microbial), < 0.1% Sodium azide

PASE

13 mL
Tris buffer, < 0.05% EDTA, Calcium chloride, Calcium acetate, 8% (w/v) Proteinase, Glycerol

EB

13 mL
Tris buffer, 0.2% Methyl-4 hydroxybenzoate

IC

13 mL
Tris buffer, < 0.05% EDTA, < 0.001% Internal control armored RNA construct (non-infectious RNA encapsulated in MS2 bacteriophage), < 0.002% Poly rA RNA (synthetic), < 0.1% Sodium azide

*Composition information required for US only.

For USA:

cobas® HIV-1/HIV-2 Qualitative nucleic acid test for use on the **cobas® 6800/8800 Systems** is an *in vitro* nucleic acid amplification test for the qualitative detection and differentiation of human immunodeficiency virus (HIV) type 1 (HIV-1) and type 2 (HIV-2) in human serum and plasma. **cobas® HIV-1/HIV-2** is not intended for use as a screening test for the presence of HIV-1 or HIV-2 in donated blood or plasma.

Refer to the product specific Instructions For Use for detailed information regarding hazards.

HIV-1/HIV-2

Roche Molecular Systems, Inc.
1080 US Highway 202 South
Branchburg, NJ 08876 USA

Distributed by/ Vertriebt/ Distributeur/ Distribuido da/ Distribuido por/ Distribueret af/ Distribueras av/ Dystrybuca/ Distributor

Roche Diagnostics
9115 Hague Road
Indianapolis, IN 46250-0457 USA
(For Technical Assistance call the Roche Response Center toll-free: 1-800-526-1247)

Roche Diagnostics GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany

EC REP

Roche Diagnostics GmbH
Sandhofer Str. 116
68305 Mannheim
Germany

CE 2797

COBAS is a trademark of Roche. ©2020 Roche Molecular Systems, Inc.

Made in USA

Roche

cobas® HIV-1/HIV-2 Qualitative

Nucleic acid test for use on the **cobas® 6800/8800 Systems**

cobas

HIV-1/HIV-2

UDI label here

EUH208 Contains Subtilisin. May produce an allergic reaction/ Enthält Subtilisin. Kann allergische Reaktionen hervorrufen/ Contient de la subtilisine. Peut produire une réaction allergique/ Contiene subtilisina. Pudo provocar una reaccion alergica/ Contiene subtilisina. Puede provocar una reaccion alergica/ Contien subtilisina. Pode provocar uma reacção alergica/ Indeholder subtilisin. Kan udløse allergisk reaktion/ Innehåller subtilisin. Kan orsaka en allergisk reaktion/ Zawiera subtilizynę. Może powodować wystąpienie reakcji alergicznej/ Obsahuje subtilizin. Může vyvolat alergickou reakci.

EUH210 Safety data sheet available on request/ Sicherheitsdatenblatt auf Anfrage erhältlich/ Fiche de données de sécurité disponible sur demande/ Scheda dati di sicurezza disponibile su richiesta/ Puede solicitarse la ficha de datos de seguridad/ Folha de segurança fornecida a pedido/ Sikkerhedsattest kan på anmodning rekvireres/ Sakerhetsattestblad finns att rekvirera/ Karta charakterystyki dostępna na zapytanie/ Na požádanie možno poskytnúť kartu bezpečnostných údajov.

08020582001-05


cobas[®] HIV-1/HIV-2

07862113190



IVD

Roche Molecular Systems, Inc.
Branchburg, NJ 08876 USA

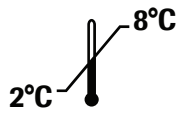

08020566001-02

↑
32 mm x 12.7 mm
unvarnished
area

**1.2 cobas HIV-1/HIV-2 Qualitative for use on the cobas 5800/6800/8800
Systems (product code 09040528190 (192 T/kit))**

cobas



IVD

Rx Only
192

cobas® HIV-1/HIV-2 Qualitative

REF 09040528190

CONTENT

HIV-1/HIV-2 Cassette	x 1
PASE	22.3 mL
IC	21.2 mL
EB	21.2 mL
MMX-R1	7.5 mL
HIV-1/HIV-2 MMX-R2	9.7 mL

For USA*

HIV-1/HIV-2 Cassette	x 1
PASE	22.3 mL
(Proteinase Solution)	
Tris buffer, < 0.05% EDTA, Calcium chloride, Calcium acetate, 8% (w/v) Proteinase	
IC	21.2 mL
(Internal Control)	
Tris buffer, < 0.05% EDTA, < 0.001% Internal control armored RNA construct (non-infectious RNA encapsulated in MS2 bacteriophage), 0.002% Poly rA RNA (synthetic), < 0.1% Sodium azide	
EB	21.2 mL
(Elution Buffer)	
Tris buffer, 0.2% Methyl-4 hydroxybenzoate	

MMX-R1	7.5 mL
(Master Mix Reagent 1)	
Manganese acetate, Potassium hydroxide, < 0.1% Sodium azide	
HIV-1/HIV-2 MMX-R2	9.7 mL
(HIV-1/HIV-2 Master Mix Reagent 2)	
Tricine buffer, Potassium acetate, 18% Dimethyl sulfoxide, Glycerol, Tween 20, EDTA, < 0.06% dATP, dCTP, dGTP, < 0.14% dUTP, < 0.01% Upstream and downstream HIV-1, HIV-2 and internal control primers, < 0.01% Fluorescent-labeled HIV-1 and HIV-2 probes, < 0.01% Fluorescent-labeled internal control probe, < 0.01% Oligonucleotide aptamer, < 0.01% ZO5D DNA polymerase, < 0.01% AmpErase (uracil-N-glycosylase) enzyme (microbial), < 0.1% Sodium azide	

*Composition information required for US only.

For USA:
cobas® HIV-1/HIV-2 Qualitative nucleic acid test for use on the cobas® 5800/6800/8800 Systems is an in vitro nucleic acid amplification test for the qualitative detection and differentiation of human immunodeficiency virus (HIV) type 1 (HIV-1) and type 2 (HIV-2) in human serum and plasma. cobas® HIV-1/HIV-2 is not intended for use as a screening test for the presence of HIV-1 or HIV-2 in donated blood or plasma.
Refer to the product specific Instructions For Use for detailed information regarding hazards.

HIV-1/HIV-2



Roche Molecular Systems, Inc.
1080 US Highway 202 South
Branchburg, NJ 08876 USA



Roche Diagnostics GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany

EC Importer¹

Distributed by
Roche Diagnostics
9115 Hague Road
Indianapolis, IN 46250-0457 USA
(For Technical Assistance call the
Roche Response Center
toll-free: 1-800-526-1247)²

¹ Symbol text required by USA only.
² For USA only.

COBAS and AMPERASE are trademarks of Roche. ©2022 Roche Molecular Systems, Inc.

Made in USA



cobas® HIV-1/HIV-2 Qualitative

cobas

HIV-1/HIV-2

EC REP

Roche Diagnostics GmbH
Sandhofer Str. 116
68305 Mannheim
Germany

CE
2797

UDI label here

EU/210
Safety data sheet available on request.
Sicherheitsdatenblatt auf Anfrage erhältlich.
Fiche de données de sécurité disponible sur demande.
Scheda dati di sicurezza disponibile su richiesta.
Puede solicitarse la ficha de datos de seguridad.
Ficha de segurança fornecida a pedido.
Sikkerhedsdatablad kan på forespørgsel leveres.
Sikkerhedsdatablad finns att beställa.
Kada ocharakteristični listički su završeni.
Na pozivanie možno poskytnúť kartu bezpečnostných údajov.

EU/218
Contains Subtilisin. May produce an allergic reaction.
Enthält Subtilisin. Kann allergische Reaktionen hervorrufen.
Contient de la subtilisine. Peut produire une réaction allergique.
Contiene Subtilisina. Può provocare una reazione allergica.
Contiene Subtilisina. Puede provocar una reacción alérgica.
Contém subtilisina. Pode provocar uma reação alérgica.
Innehåller subtilisin. Kan utlösa allergisk reaktion.
Zawiera substyzyne. Może powodować wysłajpnie reakcji alergicznej.
Összehajp subtilizin. Måske vrykker allergisk reaktion.

09112367001-04

cobas[®] HIV-1/HIV-2

09040528190



IVD

Roche Molecular Systems, Inc.
Branchburg, NJ 08876 USA



09112359001-01

32 mm x 12.7 mm
unvarnished
area

2. Instructions for use¹

¹ English version of the IFU was the one that was assessed by WHO. It is the responsibility of the manufacturer to ensure correct translation into other languages.

**2.1 IFU for cobas HIV-1/HIV-2 Qualitative for use on the cobas 6800/8800
Systems (product code 07862113190 (96 T/kit))**

cobas[®] HIV-1/HIV-2 Qualitative

Nucleic acid test for use on the cobas[®] 6800/8800 Systems

For in vitro diagnostic use

cobas[®] HIV-1/HIV-2 Qualitative	P/N: 07862113190
cobas[®] HIV-1/HIV-2 Qualitative Control Kit	P/N: 07862091190
cobas[®] NHP Negative Control Kit	P/N: 07002220190
cobas[®] Specimen Pre-Extraction Reagent	P/N: 08064695190

Table of contents

Intended use	4
Summary and explanation of the test	4
Reagents and materials	7
cobas® HIV-1/HIV-2 Qualitative reagents and controls	7
cobas omni reagents for sample preparation	10
cobas® Specimen Pre-Extraction Reagent	11
Reagent storage and handling requirements	12
Additional materials required.....	13
Instrumentation and software required.....	14
Precautions and handling requirements	14
Warnings and precautions	14
Reagent handling	15
Good laboratory practice	15
Sample collection, transport, and storage	16
Samples	16
EDTA plasma and serum samples	16
Dried blood spots.....	17
Instructions for use.....	18
Procedural notes.....	18
Dried blood spot sample preparation.....	18
Running cobas® HIV-1/HIV-2 Qualitative.....	19
Results	20
Quality control and validity of results.....	20
Interpretation of results	21
Procedural limitations.....	22

Non-clinical performance evaluation.....	23
Key performance characteristics	23
Limit of Detection (LoD).....	23
Precision – within laboratory.....	27
Group/subtype verification and inclusivity.....	29
Specificity.....	32
Seroconversion panels.....	32
Analytical specificity.....	34
Analytical specificity – interfering substances	35
Correlation	36
Whole system failure.....	37
Cross contamination.....	37
Additional information.....	38
Key test features	38
Symbols.....	39
Manufacturer and distributors	41
Trademarks and patents.....	41
Copyright.....	41
References.....	42
Document revision	43

Intended use

cobas® HIV-1/HIV-2 Qualitative nucleic acid test for use on the cobas® 6800/8800 Systems is an *in vitro* nucleic acid amplification test for the qualitative detection and differentiation of human immunodeficiency virus (HIV) type 1 (HIV-1) and type 2 (HIV-2) in human serum, plasma, and dried blood spots (DBS).

The test is intended to be used as an aid in diagnosis of HIV-1/HIV-2. Detection of HIV-1 or HIV-2 nucleic acid is indicative of HIV-1 or HIV-2 infection, respectively. The presence of HIV-1 or HIV-2 nucleic acid in the plasma or serum of individuals without antibodies to HIV-1 or HIV-2 is indicative of acute or primary infection. In infants born to HIV-infected mothers and who have maternal antibodies to HIV-1 or HIV-2, the presence of HIV nucleic acid is indicative of active infection. cobas® HIV-1/HIV-2 Qualitative may also be used to confirm HIV-1 or HIV-2 infection in an individual with specimens reactive for HIV-1 or HIV-2 antibodies or antigens.

Summary and explanation of the test

Background

Human immunodeficiency virus (HIV) is the etiologic agent of acquired immunodeficiency syndrome (AIDS).¹ HIV-1 is the predominant cause of AIDS worldwide, with over 35 million people infected.² After infection, infected individuals typically enter a clinically stable, relatively asymptomatic phase that can last for years. Without antiretroviral treatment, individuals typically progress to AIDS, which is marked by immune system depletion of CD4+ cells, susceptibility to opportunistic infections, and eventual death.³ HIV-2, mainly found in West Africa, can also cause AIDS. Between 1 and 2 million people are thought to be infected with HIV-2 worldwide.

The distinction between HIV-1 and HIV-2 is important for several reasons: (1) HIV-2 appears less virulent than HIV-1, with lower viral loads, a slower rate of CD4+ cell loss, and a slower progression to opportunistic infections; (2) HIV-2 viral loads may be incorrectly quantified by HIV-1 viral load tests; and (3) some HIV-1 medications, particularly non-nucleoside reverse transcriptase inhibitors, are not effective against HIV-2.⁴ Co-infection with both HIV-1 and HIV-2 is also possible. Co-infection has no obvious effect on the rate of individuals' progression to AIDS, but does complicate viral load monitoring and antiretroviral treatment.⁴ Due to the importance of distinguishing between HIV-1 and HIV-2 infection, national and international guidelines have included the diagnosis and differentiation of HIV-1 and HIV-2 as a requirement for the proper diagnosis of HIV infection.^{5,6}

Rationale for PCR testing

Historically, HIV testing has been based on the antibody response that patients make to the virus. Although these antibodies are ineffective at combating the virus, they are found in almost all chronically infected patients. The major limitation of antibody testing is the several week "window period" during acute infection before the onset of a detectable antibody response. This window period has been decreased by "fourth generation" HIV immunoassay tests, which detect HIV p24 antigen as well as antibody.⁷ However, nucleic acid amplification tests have the potential to reduce the window period of fourth generation immunoassay tests even further, because of the sensitivity of PCR methods over protein methods.⁷

Depending on the risk of HIV infection in the population being tested, the reduction in window period from nucleic acid testing can be important for both the individual and the community.⁸ For an individual, diagnosis of HIV during acute infection offers the opportunity for immediate treatment, which may potentially delay disease progression by preventing immune system damage and by preserving anti-HIV cellular immune responses. Early treatment may also limit the size and

genetic diversity of the viral reservoir which is established, making it easier to achieve a functional cure in patients treated during acute infection. For the community, acutely infected patients play a major role in HIV transmission, because these patients typically have very high viral loads and are unaware of their infected status. Identifying and treating these patients may play a critical role in stopping the spread of HIV epidemics.^{9,10} PCR is already the standard of care for the diagnosis of HIV in infants, and the high sensitivity and specificity of PCR would not only allow for the detection of acute infection in individuals of all ages, but also the confirmation of HIV diagnosis in seropositive or serology-indeterminate individuals.^{11,12}

Explanation of the test

cobas® HIV-1/HIV-2 Qualitative is a qualitative test performed on the **cobas® 6800 System** and **cobas® 8800 System**. **cobas® HIV-1/HIV-2 Qualitative** enables the simultaneous detection and discrimination of HIV-1 and HIV-2 nucleic acid in EDTA plasma, serum and DBS of infected patients. Two probes are used to detect HIV-1, but not to discriminate group M subtypes of HIV-1 and of HIV-1 group O and group N. A third probe is used to detect HIV-2, but not to discriminate HIV-2 group A and group B.

Principles of the procedure

cobas® HIV-1/HIV-2 Qualitative is based on fully automated sample preparation (nucleic acid extraction and purification) followed by PCR amplification and detection. The **cobas® 6800/8800 Systems** consist of the sample supply module, the transfer module, the processing module, and the analytic module. Automated data management is performed by the **cobas® 6800/8800 software** which assigns test results for all tests as non-reactive, reactive, or invalid. Results can be reviewed directly on the system screen, exported, or printed as a report.

Nucleic acid from patient samples and added armored RNA internal control (IC) molecules (which serve as the sample preparation and amplification/detection process control) is simultaneously extracted. In addition the test utilizes three external controls: two positive and a negative control. In summary, viral nucleic acid is released by addition of proteinase and lysis reagent to the sample. The released nucleic acid binds to the silica surface of the added magnetic glass particles. Unbound substances and impurities, such as denatured protein, cellular debris and potential PCR inhibitors are removed with subsequent wash reagent steps and purified nucleic acid is eluted from the magnetic glass particles with elution buffer at elevated temperature.

Selective amplification of target nucleic acid from the sample is achieved by the use of target virus-specific forward and reverse primers which are selected from highly conserved regions of the HIV-1 and HIV-2 genomes. The HIV-1 gag gene, the HIV-1 LTR region (dual target for HIV-1) and the HIV-2 LTR region are amplified by **cobas® HIV-1/HIV-2**. Selective amplification of IC is achieved by the use of sequence-specific forward and reverse primers which are selected to have no homology with the HIV-1 or HIV-2 genomes. A thermostable DNA polymerase enzyme is used for both reverse-transcription and PCR amplification. The target and IC sequences are amplified simultaneously utilizing a universal PCR amplification profile with predefined temperature steps and number of cycles. The master mix includes deoxyuridine triphosphate (dUTP), instead of deoxythymidine triphosphate (dTTP), which is incorporated into the newly synthesized DNA (amplicon).¹³⁻¹⁵ Any contaminating amplicons from previous PCR runs are eliminated by the AmpErase enzyme, which is included in the PCR master mix, during the first thermal cycling step. However, newly formed amplicons are not eliminated since the AmpErase enzyme is inactivated once exposed to temperatures above 55°C.

cobas® HIV-1/HIV-2 Qualitative master mix contains two detection probes specific for the HIV-1 target sequences, one for HIV-2 target sequences and one for the IC. The probes are labeled with target specific fluorescent reporter dyes allowing simultaneous detection of HIV-1 target, HIV-2 target and IC in three different target channels.^{16,17}

When not bound to the target sequence, the fluorescent signal of the intact probes is suppressed by a quencher dye. During the PCR amplification step, hybridization of the probes to the specific single-stranded DNA template results in cleavage of the probe by the 5' to 3' exonuclease activity of the DNA polymerase resulting in separation of the reporter and quencher dyes and the generation of a fluorescent signal. With each PCR cycle, increasing amounts of cleaved probes are generated and the cumulative signal of the reporter dye increases concomitantly. Real-time detection and discrimination of PCR products is accomplished by measuring the fluorescence of the released reporter dyes for the viral targets and IC, respectively.

Reagents and materials





cobas® HIV-1/HIV-2 Qualitative reagents and controls

All unopened reagents and controls shall be stored as recommended in Table 1 to Table 5.

Table 1 cobas® HIV-1/HIV-2 Qualitative

cobas® HIV-1/HIV-2 Qualitative		
Store at 2-8°C		
96 test cassette (P/N 07862113190)		
Kit components	Reagent ingredients	Quantity per kit 96 tests
Proteinase Solution (PASE)	Tris buffer, < 0.05% EDTA, calcium chloride, calcium acetate, 8% (w/v) proteinase EUH210: Safety data sheet available on request. EUH208: Contains Subtilisin. May produce an allergic reaction.	13 mL
Internal Control (IC)	Tris buffer, < 0.05% EDTA, <0.001% internal control armored RNA construct (non-infectious RNA encapsulated in MS2 bacteriophage), < 0.002% Poly rA RNA (synthetic), < 0.1% sodium azide	13 mL
Elution Buffer (EB)	Tris buffer, 0.2% methyl-4 hydroxybenzoate	13 mL
Master Mix Reagent 1 (MMX-R1)	Manganese acetate, potassium hydroxide, < 0.1% sodium azide	5.5 mL
HIV-1/HIV-2 Master Mix Reagent 2 (HIV-1/HIV-2 MMX-R2)	Tricine buffer, potassium acetate, 18% dimethyl sulfoxide, glycerol, Tween 20, EDTA, < 0.06% dATP, dCTP, dGTP, < 0.14% dUTP, < 0.01% upstream and downstream HIV-1, HIV-2 and internal control primers, < 0.01% fluorescent-labeled HIV-1 and HIV-2 probes, < 0.01% fluorescent-labeled internal control probe, < 0.01% oligonucleotide aptamer, < 0.01% Z05D DNA polymerase, < 0.01% AmpErase (uracil-N-glycosylase) enzyme (microbial), < 0.1% sodium azide	6 mL


Table 2 cobas® HIV-1/HIV-2 Qualitative Control Kit

cobas® HIV-1/HIV-2 Qualitative Control Kit			
Store at 2–8°C (P/N 07862091190)			
Kit components	Reagent ingredients	Quantity per kit	Safety symbol and warning*
HIV-1M/HIV-2 Positive Control (HIV-1M/HIV-2 (+)C)	< 0.001% Synthetic (armored) HIV-1 Group M RNA encapsulated in MS2 bacteriophage coat protein, < 0.001% Synthetic (armored) HIV-2 RNA encapsulated in MS2 bacteriophage coat protein, Normal human plasma, non-reactive by licensed tests for antibody to HIV-1/2; HIV-1 RNA and HIV-2 RNA not detectable by PCR methods. 0.1% ProClin® 300 preservative**	5.2 mL (8 x 0.65 mL)	  WARNING H317: May cause an allergic skin reaction. P261: Avoid breathing dust/fume/gas/mist/vapours/spray. P272: Contaminated work clothing should not be allowed out of the workplace. P280: Wear protective gloves. P333 + P313: If skin irritation or rash occurs: Get medical advice/attention. P362 + P364: Take off contaminated clothing and wash it before reuse. P501: Dispose of contents/ container to an approved waste disposal plant. 55965-84-9 reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H -isothiazol-3-one [EC no. 220-239-6] (3:1)
HIV-1O Positive Control (HIV-1O (+)C)	< 0.001% Synthetic (armored) HIV-1 Group O RNA encapsulated in MS2 bacteriophage coat protein, Normal human plasma, non-reactive by licensed tests for antibody to HIV-1/2; HIV-1 RNA and HIV-2 RNA not detectable by PCR methods. 0.1% ProClin® 300 preservative**	5.2 mL (8 x 0.65 mL)	  WARNING H317: May cause an allergic skin reaction. P261: Avoid breathing dust/fume/gas/mist/vapours/spray. P272: Contaminated work clothing should not be allowed out of the workplace. P280: Wear protective gloves. P333 + P313: If skin irritation or rash occurs: Get medical advice/attention. P362 + P364: Take off contaminated clothing and wash it before reuse. P501: Dispose of contents/ container to an approved waste disposal plant. 55965-84-9 reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H -isothiazol-3-one [EC no. 220-239-6] (3:1)

* Product safety labeling primarily follows EU GHS guidance

** Hazardous substance

Table 3 cobas® NHP Negative Control Kit**cobas® NHP Negative Control Kit**Store at 2-8°C
(P/N 07002220190)


Kit components	Reagent ingredients	Quantity per kit	Safety symbol and warning*
Normal Human Plasma Negative Control (NHP-NC)	Normal human plasma, non-reactive by licensed tests for antibody to HIV-1/2; HIV-1 RNA and HIV-2 RNA not detectable by PCR methods. < 0.1% ProClin® 300 preservative**	16 mL (16 x 1 mL)	 <p>WARNING</p> <p>H317: May cause an allergic skin reaction.</p> <p>P261: Avoid breathing dust/fume/gas/mist/vapours/spray.</p> <p>P272: Contaminated work clothing should not be allowed out of the workplace.</p> <p>P280: Wear protective gloves.</p> <p>P333 + P313: If skin irritation or rash occurs: Get medical advice/attention.</p> <p>P362 + P364: Take off contaminated clothing and wash it before reuse.</p> <p>P501: Dispose of contents/ container to an approved waste disposal plant.</p> <p>55965-84-9 Mixture of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazol-3-one [EC no. 220-239-6] (3:1)</p>

* Product safety labeling primarily follows EU GHS guidance

** Hazardous substance

cobas omni reagents for sample preparation

Table 4 cobas omni reagents for sample preparation*

Reagents	Reagent ingredients	Quantity per kit	Safety symbol and warning**
cobas omni MGP Reagent (MGP) Store at 2–8°C (P/N 06997546190)	Magnetic glass particles, Tris buffer, 0.1% methyl-4 hydroxybenzoate, < 0.1% sodium azide	480 tests	Not applicable
cobas omni Specimen Diluent (SPEC DIL) Store at 2–8°C (P/N 06997511190)	Tris buffer, 0.1% methyl-4 hydroxybenzoate, < 0.1% sodium azide	4 x 875 mL	Not applicable
cobas omni Lysis Reagent (LYS) Store at 2–8°C (P/N 06997538190)	43% (w/w) guanidine thiocyanate***, 5% (w/v) polydocanol***, 2% (w/v) dithiothreitol***, dihydro sodium citrate	4 x 875 mL	 <p>DANGER</p> <p>H302 + H332: Harmful if swallowed or if inhaled. H314: Causes severe skin burns and eye damage. H412: Harmful to aquatic life with long lasting effects. EUH032: Contact with acids liberates very toxic gas. P261: Avoid breathing dust/ fume/ gas/ mist/ vapours/ spray. P273: Avoid release to the environment. P280: Wear protective gloves/protective clothing/eye protection/face protection. P303 + P361 + P353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P304 + P340 + P310: IF INHALED: Remove person to fresh air and keep comfortable for breathing. Immediately call a POISON CENTER/doctor. P305 + P351 + P338 + P310: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor.</p> <p>593-84-0 Guanidinium thiocyanate 9002-92-0 Polidocanol 3483-12-3 (R*,R*)-1,4-dimercaptobutane-2,3-diol</p>
cobas omni Wash Reagent (WASH) Store at 15–30°C (P/N 06997503190)	Sodium citrate dihydrate, 0.1% methyl-4 hydroxybenzoate	4.2 L	Not applicable


* These reagents are not included in the cobas® HIV-1/HIV-2 Qualitative kit. See listing of additional materials required (Table 10 and Table 11).

** Product safety labeling primarily follows EU GHS guidance

***Hazardous substance

cobas® Specimen Pre-Extraction Reagent

Table 5 cobas® Specimen Pre-Extraction Reagent*

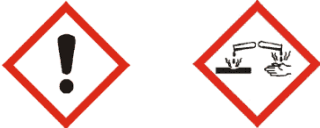cobas® Specimen Pre-Extraction Reagent			
Store at 2-8°C (P/N 08064695190)			
Reagent	Reagent ingredients	Quantity per kit	Safety symbol and warning**
cobas® Specimen Pre-Extraction Reagent (SPER)	28% (w/w) guanidine thiocyanate***, 6% (w/v) polydocanol***, 1% (w/v) dithiothreitol, dihydro sodium citrate	600 mL (15 x 40 mL)	 <p>DANGER</p> <p>H302: Harmful if swallowed. H314: Causes severe skin burns and eye damage. H412: Harmful to aquatic life with long lasting effects.</p> <p>EUH032: Contact with acids liberates very toxic gas. P273: Avoid release to the environment.</p> <p>P280: Wear protective gloves/ protective clothing/ eye protection/ face protection. P301 + P330 + P331: IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. P303 + P361 + P353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P304 + P340 + P310 IF INHALED: Remove person to fresh air and keep comfortable for breathing. Immediately call a POISON CENTER/doctor. P305 + P351 + P338 + P310: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. 593-84-0 guanidinium thiocyanate 9002-92-0 Polidocanol</p>

* This reagent is not included in the cobas® HIV-1/HIV-2 Qualitative kit. See listing of additional materials required (Table 10 and Table 11).

** Product safety labeling primarily follows EU GHS guidance

*** Hazardous substance

Reagent storage and handling requirements

Reagents shall be stored and will be handled as specified in Table 6 and Table 7. **cobas**® Specimen Pre-Extraction Reagent (SPER), used in DBS workflow, shall be stored and handled as specified in Table 8 and Table 9.

When reagents are not loaded on the **cobas**® 6800/8800 Systems, store them at the corresponding temperature specified in Table 6.

Table 6 Reagent storage (when reagent is not on the system)

Reagent	Storage temperature
cobas ® HIV-1/HIV-2 Qualitative	2–8°C
cobas ® HIV-1/HIV-2 Qualitative Control Kit	2–8°C
cobas ® NHP Negative Control Kit	2–8°C
cobas omni Lysis Reagent	2–8°C
cobas omni MGP Reagent	2–8°C
cobas omni Specimen Diluent	2–8°C
cobas omni Wash Reagent	15–30°C

Reagents loaded onto the **cobas**® 6800/8800 Systems are stored at appropriate temperatures and their expiration is monitored by the system. The **cobas**® 6800/8800 Systems allow reagents to be used only if all of the conditions shown in Table 7 are met. The system automatically prevents use of expired reagents. Table 7 allows the user to understand the reagent handling conditions enforced by the **cobas**® 6800/8800 Systems.

Table 7 Reagent expiry conditions enforced by the **cobas**® 6800/8800 Systems

Reagent	Kit expiration date	Open-kit stability	Number of runs for which this kit can be used	On-board stability (cumulative time on board outside refrigerator)
cobas ® HIV-1/HIV-2 Qualitative	Date not passed	30 days from first usage	Max 10 runs	Max 8 hours
cobas ® HIV-1/HIV-2 Qualitative Control Kit	Date not passed	Not applicable	Not applicable	Max 8 hours
cobas ® NHP Negative Control Kit	Date not passed	Not applicable	Not applicable	Max 10 hours
cobas omni Lysis Reagent	Date not passed	30 days from loading*	Not applicable	Not applicable
cobas omni MGP Reagent	Date not passed	30 days from loading*	Not applicable	Not applicable
cobas omni Specimen Diluent	Date not passed	30 days from loading*	Not applicable	Not applicable
cobas omni Wash Reagent	Date not passed	30 days from loading*	Not applicable	Not applicable

* Time is measured from the first time that reagent is loaded onto the **cobas**® 6800/8800 Systems.

Store cobas® Specimen Pre-Extraction Reagent (used in DBS workflow) at the corresponding temperature specified in Table 8.

Table 8 cobas® Specimen Pre-Extraction Reagent storage

Reagent	Storage temperature
cobas® Specimen Pre-Extraction Reagent	2–8°C

cobas® Specimen Pre-Extraction Reagent is stable until the expiration date indicated. Once opened, this reagent is stable for 30 days when stored at 2–8°C including cumulative 13 hours at 30°C or until expiration date, whichever comes first as specified in Table 9.

Table 9 cobas® Specimen Pre-Extraction Reagent expiry conditions

Reagent	Kit expiration date	Open-kit stability	Number of runs for which this kit can be used	Stability at 30°C outside refrigerator (cumulative time)
cobas® Specimen Pre-Extraction Reagent	Date not passed	30 days from first usage	Not applicable	Max 13 hours

Additional materials required

Table 10 Materials and consumables for use on cobas® 6800/8800 Systems

Material	P/N
cobas omni Processing Plate	05534917001
cobas omni Amplification Plate	05534941001
cobas omni Pipette Tips	05534925001
cobas omni Liquid Waste Container	07094388001
cobas omni Lysis Reagent	06997538190
cobas omni MGP Reagent	06997546190
cobas omni Specimen Diluent	06997511190
cobas omni Wash Reagent	06997503190
Solid Waste Bag	07435967001
Solid Waste Container	07094361001

Table 11 Other materials and consumables required for DBS application only

Materials
Whatman 903® filter card, Munktell Specimen Collection card TFN or equivalent (12–13 mm spot diameter)
Tubes, 5 mL, internal thread, 12.5 mm diameter, polypropylene (i.e., Cryo.s™) with caps
Eppendorf Thermomixer (e.g., model R 5355 or equivalent) with Thermoblock for 24 cryo tubes
Sterile or disposable forceps or tweezers
Resealable bags and desiccant sachets (for DBS storage)

Instrumentation and software required

The cobas® 6800/8800 software and cobas® HIV-1/HIV-2 Qualitative analysis package(s), cobas® HIV-1/2 Qual-Serum/Plasma ASAP and/or cobas® HIV-1/2 Qual-DBS ASAP, shall be installed on the instrument(s). The Instrument Gateway (IG) server will be provided with the system.

Table 12 Instrumentation

Equipment	P/N
cobas® 6800 System (Option Moveable)	05524245001 and 06379672001
cobas® 6800 System (Fix)	05524245001 and 06379664001
cobas® 8800 System	05412722001
Sample Supply Module	06301037001

Refer to the cobas® 6800/8800 Systems Operator's Manual for additional information for primary and secondary sample tubes accepted on the instruments.

Note: Contact your local Roche representative for a detailed order list for sample racks, racks for clotted tips and rack trays accepted on the instruments.

Precautions and handling requirements

Warnings and precautions

As with any test procedure, good laboratory practice is essential to the proper performance of this assay. Due to the high sensitivity of this test, care should be taken to keep reagents and amplification mixtures free of contamination.

- For *in vitro* diagnostic use only.
- cobas® HIV-1/HIV-2 Qualitative has not been evaluated for use as a screening test for the presence of HIV-1/HIV-2 in blood or blood products.
- All patient samples should be handled as if infectious, using good laboratory procedures as outlined in Biosafety in Microbiological and Biomedical Laboratories and in the CLSI Document M29-A4.^{18,19} Only personnel proficient in handling infectious materials and the use of cobas® HIV-1/HIV-2 Qualitative and cobas® 6800/8800 Systems should perform this procedure.
- All human-sourced materials should be considered potentially infectious and should be handled with universal precautions. If spillage occurs, immediately disinfect following appropriate site procedures.
 - If spillage of DBS samples in cobas® Specimen Pre-Extraction Reagent (which contain guanidine thiocyanate) occurs, do not allow it to come in contact with sodium hypochlorite containing disinfectants such as bleach. This mixture can produce a highly toxic gas.
- cobas® HIV-1/HIV-2 Qualitative Control Kit and cobas® NHP Negative Control Kit contain plasma derived from human blood. The source material has been tested by licensed antibody tests and found non-reactive for the presence of antibody to HIV-1/2. Testing of normal human plasma by PCR methods also showed no detectable HIV-1 (Groups M and O) RNA and HIV-2 RNA. No known test method can offer complete assurance that products derived from human blood will not transmit infectious agents.
- **Do not freeze whole blood or any samples stored in primary tubes.**
- cobas® Specimen Pre-Extraction Reagent is light sensitive and shipped in light protective bottles.
- Use only supplied or specified required consumables to ensure optimal test performance.

- Safety Data Sheets (SDS) are available on request from your local Roche representative.
- Closely follow procedures and guidelines provided to ensure that the test is performed correctly. Any deviation from the procedures and guidelines may affect optimal test performance.
- False positive results may occur if carryover of samples is not adequately controlled during sample handling and processing.

Reagent handling

- Handle all reagents, controls, and samples according to good laboratory practice in order to prevent carryover of samples or controls.
- Before use, visually inspect each reagent cassette, diluent, lysis reagent, wash reagent, and **cobas**® Specimen Pre-Extraction Reagent (required for DBS application only) to ensure that there are no signs of leakage. If there is any evidence of leakage, do not use that material for testing.
- **cobas omni** Lysis Reagent and **cobas**® Specimen Pre-Extraction Reagent contain guanidine thiocyanate, a potentially hazardous chemical. Avoid contact of reagents with the skin, eyes, or mucous membranes. If contact does occur, immediately wash with generous amounts of water; otherwise, burns can occur.
- **cobas**® HIV-1/HIV-2 Qualitative kits, **cobas omni** MGP Reagent, and **cobas omni** Specimen Diluent contain sodium azide as a preservative. Avoid contact of reagents with the skin, eyes, or mucous membranes. If contact does occur, immediately wash with generous amounts of water; otherwise, burns can occur. If these reagents are spilled, dilute with water before wiping dry.
- Do not allow **cobas omni** Lysis Reagent or **cobas**® Specimen Pre-Extraction Reagent, which contain guanidine thiocyanate, to contact sodium hypochlorite (bleach) solution. This mixture can produce a highly toxic gas.
- Dispose of all materials that have come in contact with samples and reagents in accordance with country, state, and local regulations.

Good laboratory practice

- Do not pipette by mouth.
- Do not eat, drink, or smoke in designated work areas.
- Wear laboratory gloves, laboratory coats, and eye protection when handling samples and reagents. Gloves must be changed between handling samples and **cobas**® HIV-1/HIV-2 Qualitative kits and **cobas omni** reagents to prevent contamination. Avoid contaminating gloves when handling samples and controls.
- Wash hands thoroughly after handling samples and kit reagents, and after removing the gloves.
- Thoroughly clean and disinfect all laboratory work surfaces with a freshly prepared solution of 0.5% sodium hypochlorite in distilled or deionized water (dilute household bleach 1:10). Follow by wiping the surface with 70% ethanol.
- If spills occur on the **cobas**® 6800/8800 instrument, follow the instructions in the **cobas**® 6800/8800 Systems Operator's Manual to properly clean and decontaminate the surface of the instrument(s).

Sample collection, transport, and storage

Note: Handle all samples and controls as if they are capable of transmitting infectious agents.

Store all samples at specified temperatures.


Sample stability is affected by elevated temperatures.

If using frozen samples in secondary tubes, place the samples at room temperature (15-30°C) until completely thawed and then briefly mix (e.g. vortex for 3-5 seconds) and centrifuge to collect all sample volume at the bottom of the tube.

Samples

EDTA plasma and serum samples

- Whole blood should be collected in SST™ Serum Separation Tubes, BD Vacutainer® PPT™ Plasma Preparation Tubes for Molecular Diagnostic Test Methods or in sterile tubes using EDTA as the anticoagulant. Follow the sample collection tube manufacturer instructions.
- Whole blood collected in SST™ Serum Separation Tubes, BD Vacutainer® PPT™ Plasma Preparation Tubes for Molecular Diagnostic Test Methods or in sterile tubes using EDTA as the anticoagulant may be stored and/or transported for up to 24 hours at 2°C to 25°C prior to plasma or serum preparation. Centrifugation should be performed according to manufacturer instructions.
- Upon separation EDTA plasma or serum samples may be stored in secondary tubes for up to 24 hours at 30°C followed by up to 5 days at 2°C to 8°C or up to 6 weeks at $\leq -20^{\circ}\text{C}$. For long-term storage, temperatures at $\leq -60^{\circ}\text{C}$ are recommended.
- Plasma samples are stable for up to three freeze/thaw cycles when frozen at $\leq -20^{\circ}\text{C}$.
- Refer to Figure 1 for sample storage conditions.

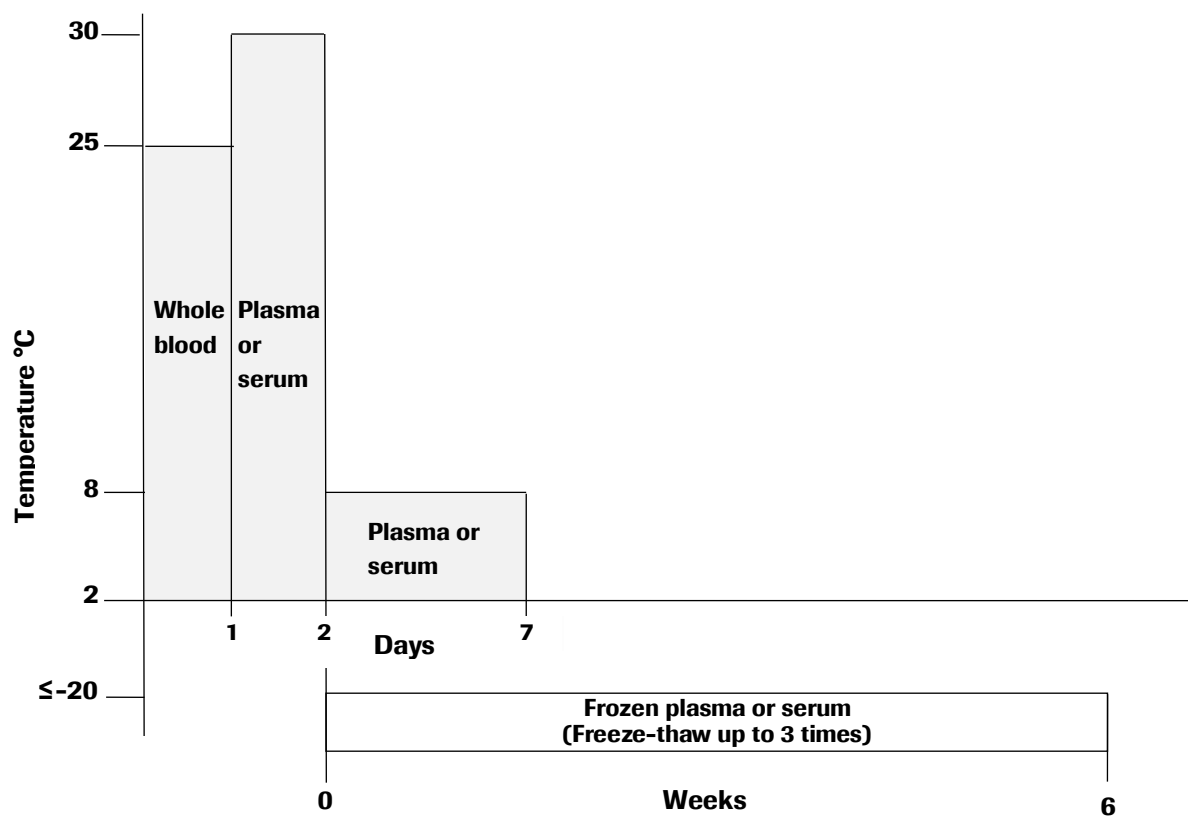Figure 1 Whole blood, plasma and serum sample storage conditions

- If samples are to be shipped, they should be packaged and labeled in compliance with applicable country and/or international regulations covering the transport of samples and etiologic agents.

Dried blood spots

- Collect DBS samples using appropriate clinical procedures.
- It is recommended to apply a minimum of 70 µL of capillary blood inside each delineated circle on the DBS card.
- Ensure that BOTH sides of the paper are saturated and completely fill the delineated circle.
- Allow DBS to dry at room temperature (18-25°C) for at least 3 hours, protecting the DBS card from direct sunlight.
- For further details consult package insert of filter cards used.
- It is recommended to prepare at least 3 paper disks per patient sample.
- Store DBS in individual resealable bags with a desiccant sachet in each bag.
- DBS may be transported or stored at 15-30°C for up to three months.
- If samples are to be shipped, they should be packaged and labeled in compliance with applicable country and/or international regulations covering the transport of samples and etiologic agents.

Instructions for use

Procedural notes

- Do not use **cobas**® HIV-1/HIV-2 Qualitative reagents, **cobas**® HIV-1/HIV-2 Qualitative Control Kit, **cobas**® NHP Negative Control Kit, **cobas** **omni** reagents, or **cobas**® Specimen Pre-Extraction Reagent after their expiry dates.
- Do not reuse consumables. They are for one-time use only.
- Refer to the **cobas**® 6800/8800 Systems Operator's Manual for proper maintenance of instruments.

Dried blood spot sample preparation


- Allow **cobas**® Specimen Pre-Extraction Reagent (SPER) to equilibrate to ambient temperature before use.
- Excise one DBS from the DBS card.
- Transfer the spot into a tube (5 mL, internal thread, 12.5 mm diameter, polypropylene [i.e., Cryo.s™]) using sterile or disposable forceps or tweezers.
- Ensure the DBS is located at the bottom of the tube as shown in Figure 2.
- Pipette 1150 µL of SPER into the tube containing the DBS and cap the tube.
- Ensure the DBS is completely covered with SPER.
- Place tubes in each of the positions 1 - 24 on a preheated Eppendorf Thermomixer (e.g., model R 5355 or equivalent) with Thermoblock for 24 cryo tubes and incubate for 10 minutes, at 56°C and 1000 rpm to extract the virus from the dried whole blood.
- Uncap the tubes and ensure the DBS is attached to the tube wall (Figure 3) to avoid sample clots.
- Eliminate any potential liquid film located above the liquid level (Figure 4) using a sterile pipette tip (to avoid early level detection).
- Transfer the tubes to the **cobas**® 6800/8800 Systems.

Figure 2 DBS in tube



Valid

Figure 3 DBS location

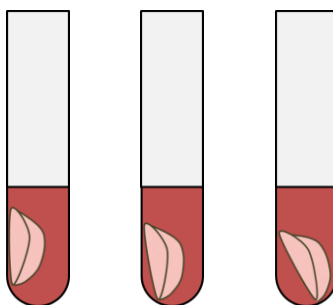


Valid

Valid

Potential
risk of clot

Figure 4 Liquid film

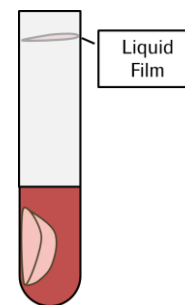


Potential risk of
early level detection

Running cobas® HIV-1/HIV-2 Qualitative

cobas® HIV-1/HIV-2 Qualitative can be run with a minimum required sample volume of 650 µL (for the 500 µL plasma or serum sample workflow) or 1150 µL cobas® Specimen Pre-Extraction Reagent (for the 850 µL DBS sample workflow). Please note, DBS samples cannot be run in mixed batch mode with plasma or serum samples. The test procedure is described in detail in the cobas® 6800/8800 Systems Operator's Manual. Figure 5 below summarizes the procedure.

Figure 5 cobas® HIV-1/HIV-2 Qualitative test procedure

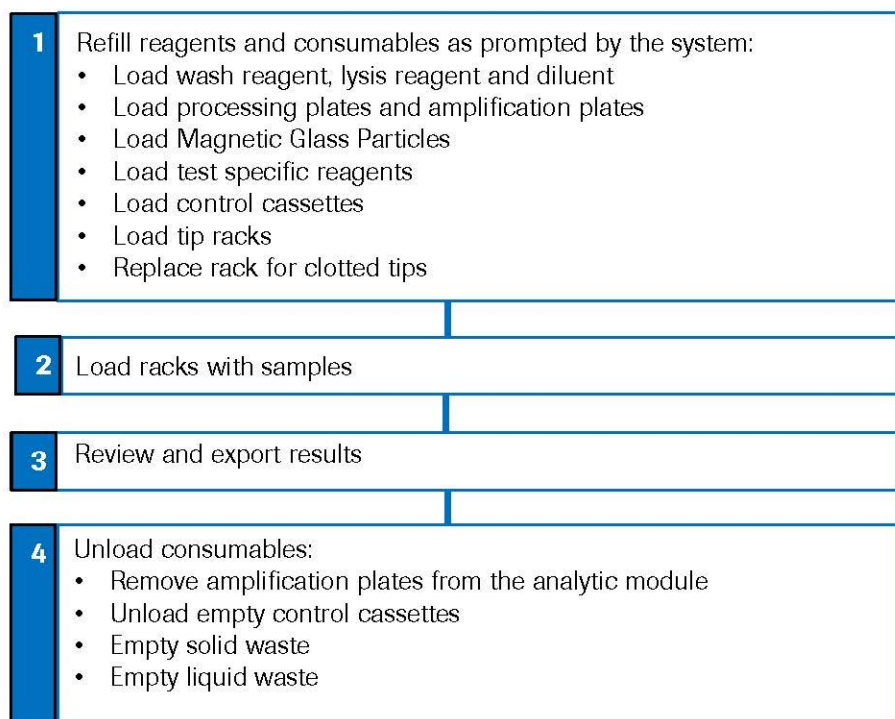

Results

The cobas® 6800/8800 Systems automatically detect and discriminate HIV-1 and HIV-2 simultaneously for the samples and controls, displaying test validity, overall results, as well as individual target results.

Quality control and validity of results

- One Normal Human Plasma Negative Control [(-) C] and two positive controls [HIV-1M/HIV-2 (+)C and HIV-1O (+)C] are processed with each batch.
- In the cobas® 6800/8800 software and/or report, check for flags and their associated results to ensure the batch validity.
- The batch is valid if no flags appear for all three controls.

Invalidation of results is performed automatically by the cobas® 6800/8800 software based on negative and positive control failures.

Control flags

Table 13 Control flags for negative and positive controls

Negative Control	Flag	Result	Interpretation
(-) C	Q02 (Control batch failed)	Invalid	The entire batch is assigned invalid if the result for the (-) C is invalid.
Positive Control	Flag	Result	Interpretation
HIV-1M/HIV-2 (+)C	Q02 (Control batch failed)	Invalid	The entire batch is assigned invalid if the result for the HIV-1M/HIV-2 (+)C is invalid.
HIV-1O (+)C	Q02 (Control batch failed)	Invalid	The entire batch is assigned invalid if the result for the HIV-1O (+)C is invalid.

If the batch is invalid, repeat testing of the entire batch including samples and controls.

HIV-1M/HIV-2 (+) C stands for cobas® HIV-1M/HIV-2 positive control and HIV-1O (+) C stands for cobas® HIV-1O positive control in the cobas® 6800/8800 software.

Interpretation of results

For a valid batch, check each individual sample for flags in the **cobas**® 6800/8800 software and/or report. The result interpretation should be as follows:

- A valid batch may include both valid and invalid sample results.
- Samples are marked with “Yes” in the column ‘Valid’ if all requested Target Results reported valid results. Samples marked with “No” in the column ‘Valid’ may require additional interpretation and action.
- The values in “Overall Result” column for individual samples should be interpreted as follows:
 - Reactive – All requested results are reactive or one of the requested results is reactive and the other non-reactive
 - Non-Reactive – All requested results are non-reactive
 - Invalid – At least one requested result is invalid
- Reported target results for individual samples are valid unless indicated otherwise.

Results and their corresponding interpretation for detecting HIV-1 and HIV-2 are shown below in Table 14.

Table 14 Target results for individual target result interpretation

Valid	Overall Result	Target 1	Target 2	Interpretation
Yes	Reactive	HIV-1 Reactive	HIV-2 Reactive	All requested results were valid. Target signal detected for HIV-1 and HIV-2.
Yes	Reactive	HIV-1 Reactive	HIV-2 Non-Reactive	All requested results were valid. Target signal detected for HIV-1. No target signal detected for HIV-2.
Yes	Reactive	HIV-1 Non-Reactive	HIV-2 Reactive	All requested results were valid. No target signal detected for HIV-1. Target signal detected for HIV-2.
Yes	Non-Reactive	HIV-1 Non-Reactive	HIV-2 Non-Reactive	All requested results were valid. No target signal detected for HIV-1 or HIV-2.
No	Invalid	Invalid	Invalid	Both HIV-1 and HIV-2 results are invalid. Original specimen should be re-tested to obtain valid HIV-1 and HIV-2 results. If the results are still invalid, a new specimen should be obtained.

Procedural limitations

- **cobas® HIV-1/HIV-2 Qualitative** has been evaluated only for use in combination with the **cobas® HIV-1/HIV-2 Qualitative Control Kit**, **cobas® NHP Negative Control Kit**, **cobas omni MGP Reagent**, **cobas omni Lysis Reagent**, **cobas omni Specimen Diluent**, **cobas omni Wash Reagent**, and **cobas® Specimen Pre-Extraction Reagent** (used in dried blood spot workflow) for use on the **cobas® 6800/8800 Systems**.
- Reliable results depend on proper sample collection, storage and handling procedures.
- Detection of HIV-1 and HIV-2 nucleic acid is dependent on the number of virus particles present in the sample and may be affected by sample collection, storage and handling, patient factors (i.e., age, presence of symptoms), and/or stage of infection.
- Though rare, mutations within the highly conserved regions of a viral genome covered by **cobas® HIV-1/HIV-2 Qualitative** may affect primers and/or probe binding resulting in the failure to detect the presence of virus.
- Due to inherent differences between technologies, it is recommended that, prior to switching from one technology to the next, users perform method correlation studies in their laboratory to qualify technology differences. Users should follow their own specific policies/procedures.
- **cobas® HIV-1/HIV-2 Qualitative** is not intended for use as a screening test for the presence of HIV-1/HIV-2 in blood or blood products.

Non-clinical performance evaluation

Key performance characteristics

Limit of Detection (LoD)

WHO International Standards/Roche Primary Standards

The limit of detection of the cobas® HIV-1/HIV-2 Qualitative was determined by using the following standards:

- WHO 3rd International Standard for HIV-1 group M RNA (NIBSC code 10/152) for EDTA plasma, serum and DBS samples
- WHO International Standard for HIV-2 RNA (NIBSC code 08/150) for plasma and serum samples
- Roche Primary Standards for HIV-2 RNA for DBS samples
- Roche Primary Standards for HIV-1 group O RNA for EDTA plasma and serum samples

No international standard is currently available for HIV-1 group O RNA. The Roche HIV-1 group O RNA Standard is traceable to the CBER HIV-1 Subtype RNA Reference Panel #1 Lot 01. The Roche Primary Standards for HIV-1 group O RNA are derived from commercially available cultured virus stocks, P/N 2420 (Cat. No. 500493, SeraCare Life Sciences). The Roche HIV-2 RNA Standard is traceable to the WHO International Standard for HIV-2 RNA (NIBSC code 08/150). The Roche Primary Standards for HIV-2 RNA are derived from commercially available cultured virus stocks, P/N HIV-2 NIH-Z (Cat. No. 10-27-000, Applied Biotechnologies, Inc.). One copy of HIV-1 RNA is equivalent to 1.7 International Unit (IU) and one copy of HIV-2 RNA is equivalent to 0.2 IU.

Serial dilutions of the standards in HIV-negative human EDTA plasma, serum or whole blood for DBS were prepared. Panels of five or six concentration levels plus a negative were tested over three lots of cobas® HIV-1/HIV-2 Qualitative reagents, multiple runs, days, operators, and instruments.

For each virus, 95% PROBIT analysis on the data combined across dilution series and reagent lots was used to estimate the LoD, along with the lower and upper limit of the 95% confidence interval (Table 15). The reactivity rates observed in the LoD studies for each virus are summarized in Table 16 to Table 18.

Table 15 Results of 95% PROBIT analysis on LoD data collected with viral standards in EDTA plasma, serum and DBS

Matrices	Analyte	Measuring units	LoD	Lower 95% confidence limit	Upper 95% confidence limit
EDTA plasma	HIV-1 group M	copies/mL	12.6	10.9	15.2
	HIV-1 group O	copies/mL	14.8	12.8	17.7
	HIV-2	copies/mL	27.9	22.9	36.6
Serum	HIV-1 group M	copies/mL	12.1	10.5	14.5
	HIV-1 group O	copies/mL	12.6	10.9	15.2
	HIV-2	copies/mL	23.4	19.6	29.7
DBS	HIV-1 group M	copies/mL	255	224	299
	HIV-2	copies/mL	984	856	1169

Table 16 Reactivity rates summary for HIV-1 group M in EDTA plasma, serum and DBS

Matrices	HIV-1 group M RNA concentration (cp/mL)	Number of reactive	Number of valid replicates	% Reactive
EDTA plasma	40	189	189	100%
	30	189	189	100%
	20	187	189	99%
	10	174	189	92%
	5	124	189	66%
	2.5	91	189	48%
	0	0	189	0%
Serum	40	189	189	100%
	30	189	189	100%
	20	187	189	99%
	10	176	189	93%
	5	126	189	67%
	2.5	86	189	46%
	0	0	189	0%
DBS	750	252	252	100%
	600	252	252	100%
	360	246	250	98%
	180	220	249	88%
	90	163	252	65%
	45	109	250	44%
	0	0	107	0%

Table 17 Reactivity rates summary for HIV-1 group O in EDTA plasma and serum

Matrices	HIV-1 group O RNA concentration (cp/mL)	Number of reactive	Number of valid replicates	% Reactive
EDTA plasma	40	189	189	100%
	30	189	189	100%
	20	185	188	98%
	10	163	189	86%
	5	117	189	62%
	2.5	78	189	41%
	0	0	189	0%
Serum	40	189	189	100%
	30	189	189	100%
	20	186	189	98%
	10	173	189	92%
	5	132	189	70%
	2.5	91	189	48%
	0	0	189	0%

Table 18 Reactivity rates summary for HIV-2 in EDTA plasma, serum and DBS

Matrices	HIV-2 RNA concentration (cp/mL)	Number of reactive	Number of valid replicates	% Reactive
EDTA plasma	80	126	126	100%
	40	124	126	98%
	20	115	126	91%
	10	81	126	64%
	5	61	126	48%
	0	0	189	0%
Serum	80	126	126	100%
	40	125	126	99%
	20	114	126	90%
	10	96	126	76%
	5	49	126	39%
	0	0	189	0%
DBS	3000	252	252	100%
	1450	241	247	98%
	725	226	246	92%
	362	167	248	67%
	181	103	250	41%
	0	0	108	0%

Precision – within laboratory

The precision of cobas® HIV-1/HIV-2 Qualitative was determined using the following standards:

- Roche Secondary Standard for HIV-1 group M
- Roche Primary Standard for HIV-2

Two panels of individually formulated HIV-1 group M and HIV-2 target, each comprising 3 panel members at concentrations of approximately 0.6 x, 1 x and 3 x of the LoD of cobas® HIV-1/HIV-2 Qualitative were tested in this study. Testing was performed for the following variability components:

- day-to-day variability over 4 days
- lot-to-lot variability using 3 different reagent lots of cobas® HIV-1/HIV-2 Qualitative
- instrument-to-instrument variability using 3 different cobas® 6800/8800 Systems

Approximately 84 replicates were tested with each of the 3 panel members for each reagent lot for a total of 252 replicates over all reagent lots per target. Precision results were evaluated by calculating the percentage of reactive test results at each concentration level for each of the variability components analyzed.

The limits of two-sided 95% confidence intervals for each reactive rate were calculated for each of the three levels of HIV-1 group M and HIV-2 tested across 4 days, 3 reagent lots, and 3 cobas® 6800/8800 Systems. cobas® HIV-1/HIV-2 Qualitative is reproducible over multiple days, reagent lots and multiple instruments. The results from reagent lot-to-lot variability are summarized in Table 19 and Table 20.

Table 19 cobas® HIV-1/HIV-2 Qualitative reagent lot-to-lot precision summary (EDTA plasma)

Analyte	Concentration	Reagent lot	% Reactive (reactive/valid replicates)	Lower limit of 95% confidence interval	Upper limit of 95% confidence interval
HIV-1 group M	~3 x LoD	1	100% (84/84)	95.7%	100%
		2	100% (84/84)	95.7%	100%
		3	100% (84/84)	95.7%	100%
	~1 x LoD	1	98.8% (83/84)	93.5%	100%
		2	98.8% (83/84)	93.5%	100%
		3	100% (84/84)	95.7%	100%
	~0.6 x LoD	1	77.4% (65/84)	67.0%	85.8%
		2	76.2% (64/84)	65.7%	84.8%
		3	82.1% (69/84)	72.3%	89.6%
HIV-2	~3 x LoD	1	98.8% (83/84)	93.5%	100%
		2	96.4% (81/84)	89.9%	99.3%
		3	100% (84/84)	95.7%	100%
	~1 x LoD	1	98.8% (83/84)	93.5%	100%
		2	98.8% (83/84)	93.5%	100%
		3	97.6% (82/84)	91.7%	99.7%
	~0.6 x LoD	1	66.7% (56/84)	55.5%	76.6%
		2	69.0% (58/84)	58.0%	78.7%
		3	69.0% (58/84)	58.0%	78.7%

Table 20 cobas® HIV-1/HIV-2 Qualitative reagent lot-to-lot precision summary (DBS)

Analyte	Concentration	Reagent lot	% Reactive (reactive/valid replicates)	Lower limit of 95% confidence interval	Upper limit of 95% confidence interval
HIV-1 group M	~3 x LoD	1	100% (84/84)	95.7%	100%
		2	100% (83/83)	95.7%	100%
		3	97.6% (82/84)	91.7%	99.7%
	~1 x LoD	1	96.4% (81/84)	89.9%	99.3%
		2	96.4% (81/84)	89.9%	99.3%
		3	95.2% (80/84)	88.3%	98.7%
	~0.6 x LoD	1	88.0% (73/83)	79.0%	94.1%
		2	83.3% (70/84)	73.6%	90.6%
		3	88.1% (74/84)	79.2%	94.1%
HIV-2	~3 x LoD	1	100% (83/83)	95.7%	100%
		2	100% (84/84)	95.7%	100%
		3	100% (83/83)	95.7%	100%
	~1 x LoD	1	97.6% (82/84)	91.7%	99.7%
		2	97.6% (82/84)	91.7%	99.7%
		3	98.8% (83/84)	93.5%	100%
	~0.6 x LoD	1	88.1% (74/84)	79.2%	94.1%
		2	91.7% (77/84)	83.6%	96.6%
		3	85.7% (72/84)	76.4%	92.4%

Group/subtype verification and inclusivity

The performance of cobas® HIV-1/HIV-2 Qualitative on HIV-1 group M subtypes, group O, group N and HIV-2 group B was evaluated by:

- Verification of the limit of detection for HIV-1 group M subtypes, group O (verified by dilution in whole blood for DBS), group N and HIV-2 group B
- Verification of the inclusivity for HIV-1 group M subtypes, group O, group N and HIV-2 group A and group B

Verification of limit of detection for HIV-1 group M subtypes, group O, group N and HIV-2 group B

Clinical or cultured HIV samples for HIV-1 group M (A, C, D, F, G, H) and circulating recombinant forms (CRF01_AE, CRF02_AG), HIV-1 group N and HIV-2 group B were diluted in EDTA plasma, serum or whole blood for DBS, and additionally HIV-1 group O in whole blood for DBS, to the LoD concentration of the predominant group/subtype (HIV-1 group M subtype B or HIV-2 group A) based on the LoD determined with 95% PROBIT analysis over all lots combined. The reactive rate determination was performed with 42 replicates. Testing was conducted with 1 lot of cobas® HIV-1/HIV-2 Qualitative reagents. The results from HIV-1 are shown in Table 21 and the results from HIV-2 are shown in Table 22. These results verify that cobas® HIV-1/HIV-2 Qualitative detected HIV for HIV-1 group M (A, C, D, F, G, H, CRF01_AE,

CRF02_AG), HIV-1 group O, HIV-1 group N and HIV-2 group B at the claimed concentration for each matrix or below with an upper 95% confidence interval being equal to or greater to the expected reactivity rate of 95%.

Table 21 LoD verification of HIV-1 group M subtypes, group O, and group N in EDTA plasma, serum or whole blood for DBS

Group	Subtype	Plasma: 12.6 cp/mL			Serum: 12.1 cp/mL			DBS: 255 cp/mL		
		Number of valid replicates	Number of reactive	% Reactive (95% CI*)	Number of valid replicates	Number of reactive	% Reactive (95% CI*)	Number of valid replicates	Number of reactive	% Reactive (95% CI*)
M	A	42	40	95% (99%)	42	40	95% (99%)	41	37	90% (97%)
	C	42	41	98% (100%)	42	42	100% (99.4%)	42	42	100% (100%)
	D	42	37	88% (96%)	42	37	88% (96%)	42	39	93% (99%)
	F	42	38	90% (97%)	42	38	90% (97%)	42	40	95% (99%)
	G	42	40	95% (99%)	42	39	93% (99%)	42	42	100% (100%)
	H	42	38	90% (97%)	42	41	98% (100%)	42	41	98% (100%)
	CRF01_AE	42	38	90% (97%)	42	38	90% (97%)	42	41	98% (100%)
	CRF02_AG	42	36	86% (95%)	42	39	93% (99%)	42	42	100% (100%)
O		N/A	N/A	N/A	N/A	N/A	N/A	41	39	95% (99%)
N		42	39	93% (99%)	42	37	88% (96%)	41	40	98% (100%)

* Upper 95% confidence interval

Table 22 LoD verification of HIV-2 group B in EDTA plasma, serum or whole blood for DBS

Group	Plasma: 27.9 cp/mL			Serum: 23.4 cp/mL			DBS: 984 cp/mL		
	Number of valid replicates	Number of positives	% Reactive (95% CI*)	Number of valid replicates	Number of positives	% Reactive (95% CI*)	Number of valid replicates	Number of positives	% Reactive (95% CI*)
B	42	42	100% (100%)	42	42	100% (100%)	42	42	100% (100%)

* Upper 95% confidence interval

Verification of inclusivity for HIV-1 group M subtypes, group O, group N and HIV-2 group B

The performance of cobas® HIV-1/HIV-2 Qualitative to detect subtypes of HIV-1 group M (A, C, D, F, G, H, J, K) and circulating recombinant forms (CRF01_AE, CRF02_AG, CRF12_BF, CRF14_BG), HIV-1 group O, HIV-1 group N, HIV-2 group A and HIV-2 group B was determined by testing unique clinical samples and/or culture isolated for each group or subtype in EDTA plasma or serum.

HIV-1 group M

A total of 105 unique HIV-1 group M clinical samples with known HIV-1 subtype were tested neat (undiluted) and after dilution to $\sim 5 \times \text{LoD}$ of cobas® HIV-1/HIV-2 Qualitative. All 105 clinical samples with known subtypes were detected neat and at $\sim 5 \times \text{LoD}$ (Table 23).

In addition, four HIV-1 group M subtype CRF12_BF and one HIV-1 group M subtype CRF14_BG clinical sample was tested after dilution series were prepared. One replicate of each of the neat samples and one of each dilution from 1:1.0E+01 to 1:5.0E+02 (2-4 dilutions per sample) for HIV-1 group M subtype CRF12_BF and from 1:2.0E+01 to 1:1.2E+02 (4 dilutions) for HIV-1 group M subtype CRF14_BG was tested, all yielding in reactive results. All of the tested clinical samples were detected at $\leq 5 \times \text{LoD}$.

Table 23 HIV-1 group M clinical samples

Subtype / circulating recombinant forms	% Reactive (reactive/samples tested) neat	% Reactive (reactive/samples tested) diluted to $\sim 5 \times \text{LoD}$
A	100% (10/10)	100% (10/10)
C	100% (10/10)	100% (10/10)
D	100% (10/10)	100% (10/10)
F	100% (10/10)	100% (10/10)
G	100% (10/10)	100% (10/10)
H	100% (10/10)	100% (10/10)
J	100% (5/5)	100% (5/5)
K	100% (9/9)	100% (9/9)
CRF01_AE	100% (10/10)	100% (10/10)
CRF02_AG	100% (10/10)	100% (10/10)
CRF12_BF	100% (2/2)	100% (2/2)
CRF14_BG	100% (9/9)	100% (9/9)

HIV-1 group O and HIV-1 group N

A total of 10 HIV-1 group O and one HIV-1 group N clinical or cultured sample was tested after dilution series were prepared. Two replicates of each of the neat samples and four of each dilution from 1:1.0E+01 to 1:4.8E+05 (3-5 dilutions per sample) for HIV-1 group O were tested, all yielding in reactive results. Two replicates of neat sample and four of each dilution from 1:1.0E+04 to 1:1.4E+05 (5 dilutions) for HIV-1 group N were tested. The neat sample and the dilutions from 1:1.0E+04 to 1:4.5E+04 yielded in 100% reactive results, while dilution 1:1.4E+05 yielded in a 50% reactive result. All of the tested samples were detected at $\leq 3 \times \text{LoD}$.

HIV-2

A total of 16 unique HIV-2 group A and group B clinical or cultured samples were tested neat (undiluted) and after dilution to ~5 x LoD of cobas® HIV-1/HIV-2 Qualitative. All 16 HIV-2 samples were detected neat and at ~5 x LoD (Table 24).

In addition, six HIV-2 group A and four HIV-2 group B clinical samples were tested after dilution series were prepared. One replicate of each of the neat samples and one of each dilution from 1:1.0E+01 to 1:9.0E+02 (2-5 dilutions per sample) for HIV-2 group A and from 1:2.0E+01 to 1:6.0E+01 (2-4 dilutions) for HIV-2 group B was tested, all yielding in reactive results. All of the tested clinical samples were detected at ≤ 3 x LoD.

Table 24 HIV-2 clinical or cultured samples

Subtype	% Reactive (reactive/samples tested) neat	% Reactive (reactive/samples tested) diluted to ~5 x LoD
A	100% (4/4)	100% (4/4)
B	100% (6/6)	100% (6/6)

Specificity

The specificity of cobas® HIV-1/HIV-2 Qualitative was determined by analyzing HIV negative EDTA plasma, HIV negative serum and HIV negative DBS samples from individual blood donors. A total of 613 individual EDTA plasma and 607 individual serum samples were tested with two lots of cobas® HIV-1/HIV-2 Qualitative reagents. In addition, 604 individual DBS samples were tested with three lots of cobas® HIV-1/HIV-2 Qualitative reagents. All samples tested were found non-reactive for HIV-1 and HIV-2. In each of the EDTA plasma, serum and DBS samples, the specificity of cobas® HIV-1/HIV-2 Qualitative was 100% (95% confidence limit: $\geq 99.5\%$).

Seroconversion panels

The performance of cobas® HIV-1/HIV-2 Qualitative was evaluated using commercially available seroconversion panels for HIV-1 group M.

HIV-1 group M Seroconversion panels

Twenty five commercially available seroconversion panels were used. Each panel member was tested neat with cobas® HIV-1/HIV-2 Qualitative and the results were compared to the results obtained with an FDA licensed 4th generation HIV Ag/Ab serology test and an FDA licensed comparator nucleic acid test tested neat. The overall performance results are shown in Table 25.

Table 25 Performance of cobas® HIV-1/HIV-2 Qualitative on HIV Seroconversion panels

HIV Seroconversion panel	Number of Panel Members tested	Number of Panel Members with reactive result			Days to first reactive result			Days earlier detection with cobas® HIV-1/HIV-2 Qualitative	
		cobas® HIV-1/HIV-2 Qualitative	Comparator NAT	HIV Ag/Ab Assay	cobas® HIV-1/HIV-2 Qualitative	Comparator NAT	HIV Ag/Ab Assay	Comparator NAT	HIV Ag/Ab Assay
HIV6243	10	6	6	4	18	18	25	0	7
HIV9011	11	3	3	2	30	30	38	0	8
HIV9012	8	5	5	3	9	7	16	-2	7
HIV9013	7	3	2	2	18	23	23	5	5
HIV9018	10	5	5	3	21	21	28	0	7
HIV9020	21	5	5	3	83	83	90	0	7
HIV9022	9	3	4	2	23	17	25	-6	2
HIV9030	16	6	5	3	40	40	47	0	7
HIV9031	19	8	6	4	120	131	146	11	26
HIV9034	13	4	4	3	41	41	46	0	5
HIV9076	9	3	3	3	66	66	66	0	0
HIV9089	6	5	5	3	7	7	16	0	9
HIV12008	13	7	7	5	21	21	28	0	7
PRB954	7	5	5	2	7	7	17	0	10
PRB956	5	4	4	2	40	40	47	0	7
PRB958	6	6	6	4	0	0	7	0	7
PRB961	9	4	4	2	19	19	27	0	8
PRB962	6	4	4	2	7	7	14	0	7
PRB963	7	4	5	2	9	7	17	-2	8
PRB967	6	5	5	3	3	3	17	0	14
PRB968	10	6	6	4	15	15	26	0	11
PRB969	10	7	6	3	53	53	70	0	17
PRB973	4	4	4	2	0	0	7	0	7
PRB976	4	4	4	2	0	0	7	0	7
PRB977	4	4	2	2	0	-*	13	-*	13
Total	230	120	115	70					

* Invalid result for the comparator NAT for the first panel member of panel PRB977, where a reactive NAT result was expected. The panel (PRB977) was therefore not included in the evaluation of the comparator NAT results.

Analytical specificity

The analytical specificity of cobas® HIV-1/HIV-2 Qualitative was evaluated for cross-reactivity with a panel of microorganisms at 10⁵ or 10⁶ particles, copies, or PFU/mL, for viral isolates and bacterial strains/yeast isolates, respectively (Table 26). The microorganisms were added to HIV negative human EDTA plasma and tested with and without HIV-1 and HIV-2 virus added to a concentration of approximately 3 x LoD of cobas® HIV-1/HIV-2 Qualitative for each virus. Non-reactive results were obtained with cobas® HIV-1/HIV-2 Qualitative for all microorganism samples without HIV-1 and HIV-2 target and reactive results were obtained for all of the microorganism samples with HIV-1 and HIV-2 targets. The tested microorganisms do not cross-react or interfere with cobas® HIV-1/HIV-2 Qualitative.

Table 26 Microorganisms tested for cross-reactivity

Viruses		Bacteria	Yeast
Adenovirus type 5	Varicella-Zoster Virus	Propionibacterium acnes	Candida albicans
Cytomegalovirus	West Nile Virus	Staphylococcus aureus	
Epstein-Barr Virus	St. Louis encephalitis Virus		
Hepatitis A Virus	Murray Valley encephalitis Virus		
Hepatitis B Virus	Dengue virus types 1, 2, 3, and 4		
Hepatitis C Virus	TBE Virus (strain HYPR)		
Hepatitis D Virus	Influenza A Virus		
Human T-Cell Lymphotropic Virus types 1 and 2	Zika Virus		
Human Herpes Virus Type-6	Human Papillomavirus		
Herpes Simplex Virus Type 1 and 2	Yellow Fever Virus		

EDTA plasma samples from each of the disease states (one from Adenovirus type 5 and ten from each of the other disease states) listed in Table 27 were tested with and without HIV-1 and HIV-2 added to a concentration of approximately 3 x LoD of cobas® HIV-1/HIV-2 Qualitative for each virus. These disease states do not cross-react or interfere with cobas® HIV-1/HIV-2 Qualitative.

Table 27 Disease states samples tested for analytical specificity

Disease state		
Adenovirus type 5	Hepatitis B Virus	Herpes Simplex Virus type 2
Cytomegalovirus	Hepatitis C Virus	Human T-cell lymphotropic Virus type I
Dengue Virus	Hepatitis E Virus	Human T-cell lymphotropic Virus type II
Epstein-Barr Virus	Herpes Simplex Virus type1	West Nile Virus

Analytical specificity – interfering substances

Elevated levels of triglycerides (33 g/L), conjugated bilirubin (0.2 g/L), unconjugated bilirubin (0.2 g/L), albumin (60 g/L), hemoglobin (2 g/L) and human DNA (2 mg/L) in samples as well as the presence of autoimmune diseases such as systemic lupus erythematosus (SLE), rheumatoid arthritis (RA), and antinuclear antibody (ANA) have been tested in the presence and absence of HIV-1 and HIV-2 RNA.

In addition, drug compounds listed in Table 28 were tested at three times the C_{max} in the presence and absence of HIV-1 and HIV-2 RNA.

All potentially interfering substances show no interference with the test performance with the exception of triglycerides and human DNA. At concentrations higher than 25 g/L triglycerides and 1.5 mg/L human DNA invalid results could occur due to interference. Non-reactive results were obtained with cobas® HIV-1/HIV-2 Qualitative for all samples without HIV target and reactive results were obtained on all of the samples with HIV-1 and HIV-2 targets.

Table 28 Drug compounds tested for interference with the cobas® HIV-1/HIV-2 Qualitative

Class of drug	Generic drug name	
Immune Modulators	Peginterferon α -2a	Ribavirin
	Peginterferon α -2b	
HCV Inhibitors	Simeprevir	Sofosbuvir
Reverse Transcriptase or DNA Polymerase Inhibitors	Emtricitabine	Tenofovir
	Entecavir	Adefovir dipivoxil
	Foscarnet	Telbivudine
	Cidofovir	Aciclovir
	Lamivudine	Valganciclovir
	Ganciclovir	
Compounds for Treatment of Opportunistic Infections	Azithromycin	Pyrazinamide
	Clarithromycin	Rifabutin
	Ethambutol	Rifampicin
	Fluconazole	Sulfamethoxazole
	Isoniazid	Trimethoprim
Statin	Atorvastatin	
Selective Serotonin Reuptake Inhibitor	Fluoxetine	Paroxetine
	Sertraline	
Antihistamine	Loratadine	
Beta-blocker	Nadolol	
Decongestant	Phenylephrine HCl	
Nonsteroidal Anti-inflammatory drug	Naproxen	Ibuprofen
Pain reliever	Acetaminophen	Acetylsalicylic Acid
Vitamins	Ascorbic Acid	

Correlation

EDTA plasma and serum

The performance of cobas® HIV-1/HIV-2 Qualitative was compared to the COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0 (HIV-1 EDTA plasma samples) and to a CE-marked HIV-1/HIV-2 antibody differentiation test (HIV-1 and HIV-2 EDTA plasma and serum samples).

For the correlation to COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0, clinical EDTA plasma samples were analyzed at one external site. For HIV-1 positive and HIV-1 negative clinical EDTA plasma samples, a total percent agreement of 100% between the two tests was shown, demonstrating that the performance of cobas® HIV-1/HIV-2 Qualitative and COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0 is equivalent (Table 29).

Table 29 Summary of results for method correlation for HIV-1 EDTA plasma samples

Method Correlation HIV-1 EDTA plasma samples		COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0	
		Reactive	Non-reactive
cobas® HIV-1/HIV-2 Qualitative	Reactive	68	0
	Non-reactive	0	80

For the comparison to a CE-marked HIV-1/HIV-2 antibody differentiation test, clinical EDTA plasma or serum samples were analyzed at one external site. For HIV-1 positive and HIV-1 negative EDTA plasma or serum samples, a total percent agreement of 100% between the two tests was shown. For HIV-2 positive and HIV-2 negative EDTA plasma or serum samples, a total percent agreement of 99.7% between the two tests was shown. This demonstrates that the performance of cobas® HIV-1/HIV-2 Qualitative and a CE-marked HIV-1/HIV-2 antibody differentiation test is equivalent (Table 30 and Table 31).

Table 30 Summary of results for method correlation for HIV-1 EDTA plasma and serum samples

Method Correlation HIV-1 EDTA plasma and serum samples		CE-marked HIV-1/HIV-2 antibody differentiation test		
		Reactive	Non-reactive	Indeterminate
cobas® HIV-1/HIV-2 Qualitative	Reactive	138	0	0
	Non-reactive	0	164	1*

* The sample which showed a non-reactive result with cobas® HIV-1/HIV-2 Qualitative and an indeterminate result with the CE-marked HIV-1/HIV-2 antibody differentiation test was confirmed to be negative with an alternative CE-marked 4th generation HIV Ag/Ab serological test.

Table 31 Summary of results for Method Correlation for HIV-2 EDTA plasma and serum samples

Method Correlation HIV-2 EDTA plasma and serum samples		CE-marked HIV-1/HIV-2 antibody differentiation test		
		Reactive	Non-reactive	Indeterminate
cobas® HIV-1/HIV-2 Qualitative	Reactive	14	0	0
	Non-reactive	1	287	1*

* The sample which showed a non-reactive result with cobas® HIV-1/HIV-2 Qualitative and an indeterminate result with the CE-marked HIV-1/HIV-2 antibody differentiation test was confirmed to be negative with an alternative CE-marked 4th generation HIV Ag/Ab serological test.

DBS

The performance of cobas® HIV-1/HIV-2 Qualitative was compared to the COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0 by analysis of clinical early infant DBS samples at one external site. For HIV-1 positive and HIV-1 negative DBS samples, a total percent agreement of 99.6% between the two tests was shown, demonstrating that the performance of cobas® HIV-1/HIV-2 Qualitative and COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0 is equivalent (Table 32).

Table 32 Summary of results for Method Correlation for HIV-1 DBS samples

Method Correlation HIV-1 DBS samples		COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0	
		Reactive	Non-reactive
cobas® HIV-1/HIV-2 Qualitative	Reactive	127	1*
	Non-reactive	0	151

* The sample which showed a reactive result with cobas® HIV-1/HIV-2 Qualitative and a non-reactive result with the COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0 was confirmed to be HIV-1 positive with nested PCR.

Whole system failure

The Whole System Failure rate for cobas® HIV-1/HIV-2 Qualitative was determined by testing 100 replicates of EDTA plasma and 100 replicates of whole blood for DBS spiked with both HIV-1 group M subtype B and HIV-2. These samples were tested at a target concentration of approximately 3 x LoD.

The results of this study determined that all replicates were valid and positive for the HIV-1 and HIV-2 targets, resulting in a Whole System Failure rate of 0% for EDTA plasma and DBS. The two-sided 95% exact confidence interval was 0% for the lower bound and 3.6% for the upper bound for each target and matrix.

Cross contamination

The cross-contamination rate for cobas® HIV-1/HIV-2 Qualitative was determined by testing 240 replicates of an HIV negative DBS sample and 225 replicates of a high titer HIV-1 DBS sample at 2.0E+07 cp/mL. The study was performed following the DBS sample preparation workflow. In total, five runs were performed with positive and negative samples in a checkerboard configuration.

All 240 replicates of the negative sample were non-reactive, resulting in a cross-contamination rate of 0%. The 95% confidence interval was 0% for the lower bound and 1.5% for the upper bound.

Additional information





























Key test features

Sample type	EDTA plasma, serum and dried blood spot (DBS)	
Minimum amount of sample required	650 µL for EDTA plasma and serum samples or one DBS sample (70 µL dried blood per spot) in 1150 µL cobas ® Specimen Pre-Extraction Reagent (SPER)	
Sample process volume	500 µL for EDTA plasma and serum samples or 850 µL for DBS samples	
Analytical sensitivity	<u>HIV-1M</u>	<u>HIV-2</u>
	EDTA plasma	12.6 cp/mL
	Serum	27.9 cp/mL
	DBS	12.1 cp/mL
		23.4 cp/mL
		255 cp/mL
		984 cp/mL
Specificity	100% (one-sided 95% confidence interval: 99.5%) (EDTA plasma/serum)	
	100% (one-sided 95% confidence interval: 99.5%) (DBS)	
Groups/subtypes – inclusivity	HIV-1M (A–D, F–H, J, K, CRF01_AE, CRF02_AG, CRF12_BF, CRF14_BG), HIV-1O, HIV-1N, HIV-2 (A and B)	

Symbols

The following symbols are used in labeling for Roche PCR diagnostic products.

Table 33 Symbols used in labeling for Roche PCR diagnostics products

 Age or Date of Birth	 Date of manufacture
 Ancillary Software	 Distributed by
 Assigned Range (copies/mL)	 Do not re-use
 Assigned Range (IU/mL)	 Female
 Authorized representative in the European Community	 For IVD performance evaluation only
 Barcode Data Sheet	 Global Trade Item Number
 Batch code	 <i>In vitro</i> diagnostic medical device
 Biological risks	 Lower Limit of Assigned Range
 Catalogue number	 Male
 Collect date	 Manufacturer
 Consult instructions for use	 Negative control
 Contains sufficient for <n> tests	 Non sterile
 Contents of kit	 Patient number
 Control	 Patient Name



Peel here




This way up



Positive control




Unique Device Identification

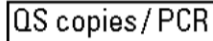

QS copies per PCR reaction, use the QS copies per PCR reaction in calculation of the results.



Ultrasensitive Procedure

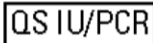

QS IU per PCR reaction, use the QS International Units (IU) per PCR reaction in calculation of the results.



Upper Limit of Assigned Range



Urine Fill Line



Serial number

Rx Only

US Only: Federal law restricts this device to sale by or on the order of a physician.



Site



Use-by date



Standard Procedure



Device for near-patient testing



Sterilized using ethylene oxide



Device Not for Near Patient Testing



Store in the dark



Device for self-testing



Temperature limit



Device not for self-testing



Test Definition File



CE marking of conformity; this device is in conformity with the applicable requirements for CE marking of an *in vitro* diagnostic medical device

US Customer Technical Support 1-800-526-1247

Manufacturer and distributors

Table 34 Manufacturer and distributors



Roche Molecular Systems, Inc.
1080 US Highway 202 South
Branchburg, NJ 08876 USA
www.roche.com

Made in USA

Distributed by

Roche Diagnostics
9115 Hague Road
Indianapolis, IN 46250-0457 USA
(For Technical Assistance call the
Roche Response Center
toll-free: 1-800-526-1247)

Roche Diagnostics GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany

Trademarks and patents

This product is covered by one or more of US Patent Nos. 8962293, 9102924, 8609340, 9234250, 8097717, 8192958, 10059993, 10358675, 8129118, and 6727067, and foreign equivalent patents of each.

COBAS, COBAS OMNI, and AMPERASE are trademarks of Roche.

The trademark “Armored RNA®” is owned by Asuragen, Inc. and Cenetron Diagnostics, Ltd.

ProClin® is a registered trademark of Rohm and Haas Company.

Vacutainer® is a registered trademark of Becton Dickinson & Company.

All other product names and trademarks are the property of their respective owners.

Carryover prevention technology in the AmpErase® enzyme is covered by U.S. Patent 7,687,247 owned by Life Technologies and licensed to Roche Molecular Systems, Inc.

Certain components of this product are covered by one or more US Patents and their foreign equivalents issued to Novartis Vaccines and Diagnostics, Inc. and licensed to Roche Molecular Systems, Inc. and F. Hoffman-La Roche Ltd.

See <http://www.roche-diagnostics.us/patents>

Copyright

©2021 Roche Molecular Systems, Inc.



Roche Diagnostics GmbH
Sandhofer Str. 116
68305 Mannheim
Germany



References

1. AAHVM. Fundamentals of HIV Medicine. Washington, DC;2007.
2. UNAIDS. Global AIDS Update. Geneva;2016.
3. Hoover DR, Saah AJ, Bacellar H, et al. Clinical manifestations of AIDS in the era of pneumocystis prophylaxis. *N Engl J Med.* 1993;329(69):1922-6.
4. Campbell-Yesufu OT, Gandhi RT. Update on human immunodeficiency virus (HIV)-2 infection. *Clin Infect Dis.* 2011;52(6):780-7.
5. CDC. Laboratory Testing for the Diagnosis of HIV Infection. Atlanta, GA;2014.
6. Gökengin D, Geretti AM, Begovac J, et al. 2014 European Guideline on HIV testing. *Int J STD AIDS.* 2014;25(10):695-704.
7. Masciotra S, McDougal JS, Feldman J, Sprinkle P, Wesolowski L, Owen SM. Evaluation of an alternative HIV diagnostic algorithm using specimens from seroconversion panels and persons with established HIV infections. *J Clin Virol.* 2011;52 Suppl 1:S17-22.
8. O'Brien M, Markowitz M. Should we treat acute HIV infection? *Current HIV/AIDS Reports.* 2012;9(2):101-10.
9. Branson BM, Mermin J. Establishing the diagnosis of HIV infection: new tests and a new algorithm for the United States. *J Clin Virol.* 2011;52 Suppl 1:S3-4.
10. Cohen MS, Shaw GM, McMichael AJ, Haynes BF. Acute HIV-1 Infection. *N Engl J Med.* 2011;364(20):1943-54.
11. World Health Organization. Diagnosis of HIV infection in infants and children. Geneva, 2010.
12. Wittek M, Stürmer M, Doerr HW, Berger A. Molecular assays for monitoring HIV infection and antiretroviral therapy. *Expert Rev Mol Diagn.* 2007;7(3):237-46.
13. Longo MC, Berninger MS, Hartley JL. Use of uracil DNA glycosylase to control carry-over contamination in polymerase chain reactions. *Gene.* 1990;93:125-8.
14. Savva R, McAuley-Hecht K, Brown T, Pearl L. The structural basis of specific base-excision repair by uracil-DNA glycosylase. *Nature.* 1995;373:487-93.
15. Mol CD, Arvai AS, Slupphaug G, et al. Crystal structure and mutational analysis of human uracil-DNA glycosylase: structural basis for specificity and catalysis. *Cell.* 1995;80:869-78.
16. Higuchi R, Dollinger G, Walsh PS, Griffith R. Simultaneous amplification and detection of specific DNA sequences. *Biotechnology (NY).* 1992;10:413-7.
17. Heid CA, Stevens J, Livak JK, Williams PM. Real time quantitative PCR. *Genome Res.* 1996;6:986-94.
18. Center for Disease Control and Prevention. Biosafety in microbiological and biomedical laboratories, 5th ed. U.S. Department of Health and Human Services, Public Health Service, Centers for Disease Control and Prevention, National Institutes of Health HHS Publication No. (CDC) 21-1112, revised December 2009.
19. Clinical and Laboratory Standards Institute (CLSI). Protection of laboratory workers from occupationally acquired infections. Approved Guideline-Fourth Edition. CLSI Document M29-A4:Wayne, PA;CLSI, 2014.

Document revision

Document Revision Information	
Doc Rev. 6.0 03/2021	Updated hazard warnings. Updated Trademarks and patents section. Inserted Rx Only symbol on first page. Added Made in statement. Updated the harmonized symbol page. Please contact your local Roche Representative if you have any questions.

**2.2 1 IFU for cobas HIV-1/HIV-2 Qualitative for use on the cobas
5800/6800/8800 Systems (product code 09040528190 (192 T/kit))**



Rx Only

cobas[®] HIV-1/HIV-2 Qualitative

Nucleic acid test for use on the cobas[®] 5800/6800/8800 Systems

For in vitro diagnostic use

cobas[®] HIV-1/HIV-2 Qualitative P/N: 09040528190

For use on the cobas[®] 5800 System

cobas[®] HIV-1/HIV-2 Qualitative Control Kit P/N: 09040536190

cobas[®] NHP Negative Control Kit P/N: 09051554190

For use on the cobas[®] 6800/8800 Systems

cobas[®] HIV-1/HIV-2 Qualitative Control Kit P/N: 07862091190 or

P/N: 09040536190

cobas[®] NHP Negative Control Kit P/N: 07002220190 or

P/N: 09051554190

cobas[®] Specimen Pre-Extraction Reagent P/N: 08064695190

Table of contents

Intended use	5
Summary and explanation of the test.....	5
Reagents and materials.....	8
cobas® HIV-1/HIV-2 Qualitative reagents and controls	8
cobas omni reagents for sample preparation.....	11
cobas® Specimen Pre-Extraction Reagent.....	12
Reagent storage requirements.....	13
Additional materials required for cobas® 5800 System.....	15
Additional materials required for cobas® 6800/8800 Systems.....	16
Instrumentation and software required.....	17
Precautions and handling requirements	18
Warnings and precautions	18
Reagent handling	18
Good laboratory practice.....	19
Sample collection, transport, and storage.....	20
Samples	20
EDTA plasma and serum samples.....	20
Dried blood spots.....	21
Instructions for use.....	22
Procedural notes	22
Dried blood spot sample preparation.....	22
Running cobas® HIV-1/HIV-2 Qualitative on cobas® 5800 System.....	23
Running cobas® HIV-1/HIV-2 Qualitative on cobas® 6800/8800 Systems.....	24
Results.....	25
Quality control and validity of results on the cobas® 5800 System	25
Control results on cobas® 5800 System	25

Quality control and validity of results on the cobas ® 6800/8800 Systems.....	25
Control flags on cobas ® 6800/8800 Systems.....	26
Interpretation of results on the cobas ® 5800 System	27
Interpretation of results on the cobas ® 6800/8800 Systems.....	28
Procedural limitations.....	29
Non-clinical performance evaluation	30
Key performance characteristics performed on the cobas ® 6800/8800 Systems	30
Limit of Detection (LoD)	30
Precision – within laboratory.....	34
Group/subtype verification and inclusivity.....	35
Specificity.....	38
Seroconversion panels	38
Analytical specificity.....	40
Analytical specificity – interfering substances.....	41
Method Correlation.....	42
Whole system failure	43
Cross contamination	43
Clinical performance evaluation	44
Reproducibility	44
Clinical method comparison	45
HIV-1 and HIV-2 clinical sensitivity	45
Testing of specimens from HIV-1 infected individuals	45
Testing of specimens from HIV-2 infected individuals	46
System equivalency / system comparison	48
Additional information	48
Key test features.....	48
Symbols.....	49
Technical Support	50
Manufacturer and importer	50
Trademarks and patents	50

Copyright..... 50
References..... 51
Document revision..... 53

Intended use

cobas® HIV-1/HIV-2 Qualitative nucleic acid test for use on the **cobas**® 5800/6800/8800 Systems is an in vitro nucleic acid amplification test for the qualitative detection and differentiation of human immunodeficiency virus (HIV) type 1 (HIV-1) and type 2 (HIV-2) in human serum, plasma, and dried blood spots (DBS).

The test is intended to be used as an aid in diagnosis of HIV-1/HIV-2. Detection of HIV-1 or HIV-2 nucleic acid is indicative of HIV-1 or HIV-2 infection, respectively. The presence of HIV-1 or HIV-2 nucleic acid in the plasma or serum of individuals without antibodies to HIV-1 or HIV-2 is indicative of acute or primary infection. In infants born to HIV-infected mothers and who have maternal antibodies to HIV-1 or HIV-2, the presence of HIV nucleic acid is indicative of active infection. **cobas**® HIV-1/HIV-2 Qualitative may also be used to confirm HIV-1 or HIV-2 infection in an individual with specimens reactive for HIV-1 or HIV-2 antibodies or antigens.

Summary and explanation of the test

Background

Human immunodeficiency virus (HIV) is the etiologic agent of acquired immunodeficiency syndrome (AIDS).¹ HIV-1 is the predominant cause of AIDS worldwide, with over 35 million people infected.² After infection, infected individuals typically enter a clinically stable, relatively asymptomatic phase that can last for years. Without antiretroviral treatment, individuals typically progress to AIDS, which is marked by immune system depletion of CD4+ cells, susceptibility to opportunistic infections, and eventual death.³ HIV-2, mainly found in West Africa, can also cause AIDS. Between 1 and 2 million people are thought to be infected with HIV-2 worldwide.

The distinction between HIV-1 and HIV-2 is important for several reasons: (1) HIV-2 appears less virulent than HIV-1, with lower viral loads, a slower rate of CD4+ cell loss, and a slower progression to opportunistic infections; (2) HIV-2 viral loads may be incorrectly quantified by HIV-1 viral load tests; and (3) some HIV-1 medications, particularly non-nucleoside reverse transcriptase inhibitors, are not effective against HIV-2.⁴ Co-infection with both HIV-1 and HIV-2 is also possible. Co-infection has no obvious effect on the rate of individuals' progression to AIDS, but does complicate viral load monitoring and antiretroviral treatment.⁴ Due to the importance of distinguishing between HIV-1 and HIV-2 infection, national and international guidelines have included the diagnosis and differentiation of HIV-1 and HIV-2 as a requirement for the proper diagnosis of HIV infection.^{5,6}

Rationale for PCR testing

Historically, HIV testing has been based on the antibody response that patients make to the virus. Although these antibodies are ineffective at combating the virus, they are found in almost all chronically infected patients. The major limitation of antibody testing is the several week "window period" during acute infection before the onset of a detectable antibody response. This window period has been decreased by "fourth generation" HIV immunoassay tests, which detect HIV p24 antigen as well as antibody.⁷ However, nucleic acid amplification tests have the potential to reduce the window period of fourth generation immunoassay tests, in detecting HIV-infection even further, because of the sensitivity of PCR methods over protein methods.⁷

Depending on the risk of HIV infection in the population being tested, the reduction in window period from nucleic acid testing can be important for both the individual and the community.⁸ For an individual, diagnosis of HIV during acute infection offers the opportunity for immediate treatment, which may potentially delay disease progression by preventing

immune system damage and by preserving anti-HIV cellular immune responses. Early treatment may also limit the size and genetic diversity of the viral reservoir which is established, making it easier to achieve a functional cure in patients treated during acute infection. For the community, acutely infected patients play a major role in HIV transmission, because these patients typically have very high viral loads and are unaware of their infection status. Identifying and treating these patients may play a critical role in stopping the spread of HIV epidemics.^{9,10} PCR is already the standard of care for the diagnosis of HIV in infants, and the high sensitivity and specificity of PCR would not only allow for the detection of acute infection in individuals of all ages, but also the confirmation of HIV diagnosis in seropositive or serology-indeterminate individuals.^{11,12}

Explanation of the test

cobas® HIV-1/HIV-2 Qualitative is a qualitative test performed on the **cobas**® 5800 System, **cobas**® 6800 System or **cobas**® 8800 System. **cobas**® HIV-1/HIV-2 Qualitative enables the simultaneous detection and differentiation of HIV-1 and HIV-2 nucleic acid in ethylenediaminetetraacetic acid (EDTA) plasma, serum and DBS of infected patients. Two probes are used to detect HIV-1, but not to discriminate HIV-1 group M subtypes and HIV-1 group O and group N. A third probe is used to detect HIV-2, but not to discriminate HIV-2 group A and group B.

Principles of the procedure

cobas® HIV-1/HIV-2 Qualitative is based on fully automated sample preparation (nucleic acid extraction and purification) followed by PCR amplification and detection. The **cobas**® 5800 System is designed as one integrated instrument. The **cobas**® 6800/8800 Systems consist of the sample supply module, the transfer module, the processing module, and the analytic module. Automated data management is performed by the **cobas**® 5800 or **cobas**® 6800/8800 Systems software which assigns test results for all tests as non-reactive, reactive, or invalid. Results can be reviewed directly on the system screen, exported, or printed as a PDF report.

Nucleic acid from patient samples and added armored RNA internal control (IC) molecules (which serve as the sample preparation and amplification/detection process control) is simultaneously extracted. In addition, the test utilizes three external controls: two positive and a negative control. In summary, viral nucleic acid is released by addition of proteinase and lysis reagent to the sample. The released nucleic acid binds to the silica surface of the added magnetic glass particles. Unbound substances and impurities, such as denatured protein, cellular debris and potential PCR inhibitors are removed with subsequent wash steps and purified nucleic acid is eluted from the magnetic glass particles with elution buffer at elevated temperature.

Selective amplification of target nucleic acid from the sample is achieved by the use of target virus-specific forward and reverse primers which are selected from highly conserved regions of the HIV-1 and HIV-2 genomes. The HIV-1 gag gene, the HIV-1 LTR region (dual target for HIV-1) and the HIV-2 LTR region are amplified by **cobas**® HIV-1/HIV-2 Qualitative.

Selective amplification of IC is achieved by the use of sequence-specific forward and reverse primers which are selected to have no homology with the HIV-1 or HIV-2 genomes. A thermostable DNA polymerase enzyme is used for both reverse-transcription and PCR amplification. The target and IC sequences are amplified simultaneously utilizing a universal PCR amplification profile with predefined temperature steps and number of cycles. The master mix includes deoxyuridine triphosphate (dUTP), instead of deoxythymidine triphosphate (dTTP), which is incorporated into the newly synthesized DNA (amplicon).¹³⁻¹⁵ Any contaminating amplicons from previous PCR runs are eliminated by the AmpErase enzyme, which is included in the PCR master mix, during the first thermal cycling step. However, newly formed amplicons are not eliminated since the AmpErase enzyme is inactivated once exposed to temperatures above 55°C.

cobas[®] HIV-1/HIV-2 Qualitative master mix contains two detection probes specific for the HIV-1 target sequences, one for HIV-2 target sequences and one for the IC. The probes are labeled with target specific fluorescent reporter dyes allowing simultaneous detection of HIV-1 target, HIV-2 target and IC in three different target channels.^{16,17} When not bound to the target sequence, the fluorescent signal of the intact probes is suppressed by a quencher dye. During the PCR amplification step, hybridization of the probes to the specific single-stranded DNA template results in cleavage of the probe by the 5' to 3' exonuclease activity of the DNA polymerase resulting in separation of the reporter and quencher dyes and the generation of a fluorescent signal. With each PCR cycle, increasing amounts of cleaved probes are generated and the cumulative signal of the reporter dye increases concomitantly. Real-time detection and discrimination of PCR products is accomplished by measuring the fluorescence of the released reporter dyes for the viral targets and IC, respectively.

Reagents and materials





cobas® HIV-1/HIV-2 Qualitative reagents and controls

All unopened reagents and controls shall be stored as recommended in Table 1 to Table 5.

Table 1 cobas® HIV-1/HIV-2 Qualitative

cobas® HIV-1/HIV-2 Qualitative Store at 2-8°C 192 test cassette (P/N 09040528190)		
Kit components	Reagent ingredients	Quantity per kit 192 tests
Proteinase Solution (PASE)	Tris buffer, < 0.05% EDTA, calcium chloride, calcium acetate, 8% (w/v) proteinase, Glycerol EUH210: Safety data sheet available on request. EUH208: Contains Subtilisin. May produce an allergic reaction.	22.3 mL
Internal Control (IC)	Tris buffer, < 0.05% EDTA, < 0.001% internal control armored RNA construct (non-infectious RNA encapsulated in MS2 bacteriophage), < 0.002% Poly rA RNA (synthetic), < 0.1% sodium azide	21.2 mL
Elution Buffer (EB)	Tris buffer, 0.2% methyl-4 hydroxybenzoate	21.2 mL
Master Mix Reagent 1 (MMX-R1)	Manganese acetate, potassium hydroxide, < 0.1% sodium azide	7.5 mL
HIV-1/HIV-2 Master Mix Reagent 2 (HIV-1/HIV-2 MMX-R2)	Tricine buffer, potassium acetate, 18% dimethyl sulfoxide, glycerol, Tween 20, EDTA, < 0.06% dATP, dCTP, dGTP, < 0.14% dUTP, < 0.01% upstream and downstream HIV-1, HIV-2 and internal control primers, < 0.01% fluorescent-labeled HIV-1 and HIV-2 probes, < 0.01% fluorescent-labeled internal control probe, < 0.01% oligonucleotide aptamer, < 0.01% Z05D DNA polymerase, < 0.01% AmpErase (uracil-N-glycosylase) enzyme (microbial), < 0.1% sodium azide	9.7 mL

Table 2 cobas® HIV-1/HIV-2 Qualitative Control Kit

cobas® HIV-1/HIV-2 Qualitative Control Kit			
Store at 2–8°C			
For use on the cobas® 5800 System (P/N: 09040536190)			
For use on the cobas® 6800/8800 Systems (P/N 07862091190 and P/N: 09040536190)			
Kit components	Reagent ingredients	Quantity per kit	Safety symbol and warning*
HIV-1M/HIV-2 Positive Control (HIV-1M/HIV-2 (+)C)	< 0.001% Synthetic (armored) HIV-1 Group M RNA encapsulated in MS2 bacteriophage coat protein, < 0.001% Synthetic (armored) HIV-2 RNA encapsulated in MS2 bacteriophage coat protein, Normal human plasma, non-reactive by licensed tests for antibody to HIV-1/2; HIV-1 RNA and HIV-2 RNA not detectable by PCR methods. 0.1% ProClin® 300 preservative**	5.2 mL (8 x 0.65 mL)	  <p>WARNING</p> <p>H317: May cause an allergic skin reaction. H412: Harmful to aquatic life with long lasting effects. P261: Avoid breathing dust/fume/gas/mist/vapours/spray. P273: Avoid release to the environment. P280: Wear protective gloves. P333 + P313: If skin irritation or rash occurs: Get medical advice/attention. P362 + P364: Take off contaminated clothing and wash it before reuse. P501: Dispose of contents/ container to an approved waste disposal plant. 55965-84-9 reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7]and 2-methyl-2H -isothiazol-3- one [EC no. 220-239-6] (3:1)</p>
HIV-1O Positive Control (HIV-1O (+)C)	< 0.001% Synthetic (armored) HIV-1 Group O RNA encapsulated in MS2 bacteriophage coat protein, Normal human plasma, non-reactive by licensed tests for antibody to HIV-1/2; HIV-1 RNA and HIV-2 RNA not detectable by PCR methods. 0.1% ProClin® 300 preservative**	5.2 mL (8 x 0.65 mL)	  <p>WARNING</p> <p>H317: May cause an allergic skin reaction. H412: Harmful to aquatic life with long lasting effects. P261: Avoid breathing dust/ fume/ gas/ mist/ vapours/ spray. P273: Avoid release to the environment. P280: Wear protective gloves. P333 + P313: If skin irritation or rash occurs: Get medical advice/attention. P362 + P364: Take off contaminated clothing and wash it before reuse. P501: Dispose of contents/ container to an approved waste disposal plant. 55965-84-9 reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7]and 2-methyl-2H -isothiazol-3- one [EC no. 220-239-6] (3:1)</p>



* Product safety labeling primarily follows EU GHS guidance

** Hazardous substance

09198814001-03EN

Doc Rev. 3.0

Table 3 cobas® NHP Negative Control Kit

cobas® NHP Negative Control Kit Store at 2-8°C For use on cobas® 5800 System (P/N 09051554190) For use on cobas® 6800/8800 Systems (P/N 07002220190 and P/N 09051554190)			
Kit components	Reagent ingredients	Quantity per kit	Safety symbol and warning*
Normal Human Plasma Negative Control (NHP-NC)	Normal human plasma, non-reactive by licensed tests for antibody to HIV-1/2; HIV-1 RNA and HIV-2 RNA not detectable by PCR methods. < 0.1% ProClin® 300 preservative**	16 mL (16 x 1 mL)	  <p>WARNING</p> <p>H317: May cause an allergic skin reaction.</p> <p>P261: Avoid breathing dust/ fume/ gas/ mist/ vapours/ spray.</p> <p>P272: Contaminated work clothing should not be allowed out of the workplace.</p> <p>P280: Wear protective gloves.</p> <p>P333 + P313: If skin irritation or rash occurs: Get medical advice/attention.</p> <p>P362 + P364: Take off contaminated clothing and wash it before reuse.</p> <p>P501: Dispose of contents/ container to an approved waste disposal plant.</p> <p>55965-84-9 reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H -isothiazol-3- one [EC no. 220-239-6] (3:1)</p>

* Product safety labeling primarily follows EU GHS guidance

** Hazardous substance

cobas omni reagents for sample preparation

Table 4 cobas omni reagents for sample preparation*

Reagents	Reagent ingredients	Quantity per kit	Safety symbol and warning**
cobas omni MGP Reagent (MGP) Store at 2–8°C (P/N 06997546190)	Magnetic glass particles, Tris buffer, 0.1% methyl-4 hydroxybenzoate, < 0.1% sodium azide	480 tests	Not applicable
cobas omni Specimen Diluent (SPEC DIL) Store at 2–8°C (P/N 06997511190)	Tris buffer, 0.1% methyl-4 hydroxybenzoate, < 0.1% sodium azide	4 x 875 mL	Not applicable
cobas omni Lysis Reagent (LYS) Store at 2–8°C (P/N 06997538190)	43% (w/w) guanidine thiocyanate***, 5% (w/v) polydocanol***, 2% (w/v) dithiothreitol, dihydro sodium citrate	4 x 875 mL	<p>DANGER</p> <p>H302 + H332: Harmful if swallowed or if inhaled. H314: Causes severe skin burns and eye damage. H412: Harmful to aquatic life with long lasting effects. EUH032: Contact with acids liberates very toxic gas. P261: Avoid breathing dust/ fume/ gas/ mist/ vapours/ spray. P273: Avoid release to the environment. P280: Wear eye protection/ face protection. P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P304 + P340 + P310 IF INHALED: Remove person to fresh air and keep comfortable for breathing. Call a POISON CENTER/doctor if you feel unwell. P305 + P351 + P338 + P310 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. 593-84-0 Guanidinium thiocyanate 9002-92-0 Polidocanol 3483-12-3 (R*,R*)-1,4-dimercaptobutane-2,3-diol</p>
cobas omni Wash Reagent (WASH) Store at 15–30°C (P/N 06997503190)	Sodium citrate dihydrate, 0.1% methyl-4 hydroxybenzoate	4.2 L	Not applicable


* These reagents are not included in the cobas® HIV-1/HIV-2 Qualitative kit. See listing of additional materials required (Table 11 and Table 12).

** Product safety labeling primarily follows EU GHS guidance

***Hazardous substance

cobas® Specimen Pre-Extraction Reagent

Table 5 cobas® Specimen Pre-Extraction Reagent*

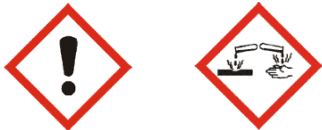cobas® Specimen Pre-Extraction Reagent Store at 2-8°C (P/N 08064695190)			
Reagent	Reagent ingredients	Quantity per kit	Safety symbol and warning**
cobas® Specimen Pre-Extraction Reagent (SPER)	28% (w/w) guanidine thiocyanate, 6% (w/v) polydocanol, 1% (w/v) dithiothreitol, dihydro sodium citrate	600 mL (15 x 40 mL)	 <p>DANGER</p> <p>H302: Harmful if swallowed. H314 Causes severe skin burns and eye damage. H412: Harmful to aquatic life with long lasting effects. EUH032: Contact with acids liberates very toxic gas. P273: Avoid release to the environment. P280: Wear eye protection/face protection. P301 + P330 + P331 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P304 + P340 + P310 IF INHALED: Remove person to fresh air and keep comfortable for breathing. Immediately call a POISON CENTER/doctor. P305 + P351 + P338 + P310 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor.</p> <p>593-84-0 Guanidinium thiocyanate 9002-92-0 Polidocanol</p>

* This reagent is not included in the cobas® HIV-1/HIV-2 Qualitative kit. See listing of additional materials required (Table 11 and Table 12).

** Product safety labeling primarily follows EU GHS guidance

Reagent storage requirements

Reagents shall be stored and will be handled as specified in Table 6, Table 7, and Table 8. cobas® Specimen Pre-Extraction Reagent (SPER), used in DBS workflow, shall be stored and handled as specified in Table 9 and Table 10.

When reagents are not loaded on the cobas® 5800 System or cobas® 6800/8800 Systems, store them at the corresponding temperature specified in Table 6.

Table 6 Reagent storage (when reagent is not on the system)

Reagent	Storage temperature
cobas® HIV-1/HIV-2 Qualitative	2–8°C
cobas® HIV-1/HIV-2 Qualitative Control	2–8°C
cobas® NHP Negative Control Kit	2–8°C
cobas omni Lysis Reagent	2–8°C
cobas omni MGP Reagent	2–8°C
cobas omni Specimen Diluent	2–8°C
cobas omni Wash Reagent	15–30°C

Reagent handling requirements for cobas® 5800 System

Reagents loaded onto the cobas® 5800 System are stored at appropriate temperatures and their expiration is monitored by the system. The system allows reagents to be used only if all of the conditions shown in Table 7 are met. The system automatically prevents use of expired reagents. Table 7 allows the user to understand the reagent handling conditions enforced by the cobas® 5800 System.

Table 7 Reagent expiry conditions enforced by the cobas® 5800 System

Reagent	Kit expiration date	Open-kit stability	Number of runs for which this kit can be used	On-board stability
cobas® HIV-1/HIV-2 Qualitative	Date not passed	90 days from first usage	Max 40 runs	Max 36 days*
cobas® HIV-1/HIV-2 Qualitative Control Kit	Date not passed	Not applicable**	Not applicable	Max 36 days*
cobas® NHP Negative Control Kit	Date not passed	Not applicable**	Not applicable	Max 36 days*
cobas omni Lysis Reagent	Date not passed	30 days from loading*	Not applicable	Not applicable
cobas omni MGP Reagent	Date not passed	30 days from loading*	Not applicable	Not applicable
cobas omni Specimen Diluent	Date not passed	30 days from loading*	Not applicable	Not applicable
cobas omni Wash Reagent	Date not passed	30 days from loading*	Not applicable	Not applicable

* Time is measured from the first time that reagent is loaded onto the cobas® 5800 System.

** Single use reagent

Reagent handling requirements for cobas® 6800/8800 Systems

Reagents loaded onto the cobas® 6800/8800 Systems are stored at appropriate temperatures and their expiration is monitored by the system. The cobas® 6800/8800 Systems allow reagents to be used only if all of the conditions shown in Table 8 are met. The system automatically prevents use of expired reagents. Table 8 allows the user to understand the reagent handling conditions enforced by the cobas® 6800/8800 Systems.

Table 8 Reagent expiry conditions enforced by the cobas® 6800/8800 Systems

Reagent	Kit expiration date	Open-kit stability	Number of runs for which this kit can be used	On-board stability (cumulative time on board outside refrigerator)
cobas® HIV-1/HIV-2 Qualitative	Date not passed	90 days from first usage	Max 40 runs	Max 40 hours
cobas® HIV-1/HIV-2 Qualitative Control Kit	Date not passed	Not applicable ^a	Not applicable	Max 8 hours
cobas® NHP Negative Control Kit	Date not passed	Not applicable ^a	Not applicable	Max 10 hours
cobas omni Lysis Reagent	Date not passed	30 days from loading*	Not applicable	Not applicable
cobas omni MGP Reagent	Date not passed	30 days from loading*	Not applicable	Not applicable
cobas omni Specimen Diluent	Date not passed	30 days from loading*	Not applicable	Not applicable
cobas omni Wash Reagent	Date not passed	30 days from loading*	Not applicable	Not applicable

^a Single use reagent

* Time is measured from the first time that reagent is loaded onto the cobas® 6800/8800 Systems.

Store cobas® Specimen Pre-Extraction Reagent (used in DBS workflow) at the corresponding temperature specified in Table 9.

Table 9 cobas® Specimen Pre-Extraction Reagent storage

Reagent	Storage temperature
cobas® Specimen Pre-Extraction Reagent	2–8°C

cobas® Specimen Pre-Extraction Reagent is stable until the expiration date indicated. Once opened, this reagent is stable for 30 days when stored at 2–8°C including cumulative 13 hours at 30°C or until expiration date, whichever comes first as specified in Table 10.

Table 10 cobas® Specimen Pre-Extraction Reagent expiry conditions

Reagent	Kit expiration date	Open-kit stability	Number of runs for which this kit can be used	Stability at 30°C outside refrigerator (cumulative time)
cobas® Specimen Pre-Extraction Reagent	Date not passed	30 days from first usage	Not applicable	Max 13 hours

Additional materials required for cobas® 5800 System

Table 11 Material and consumables for use on cobas® 5800 System

Material	P/N
cobas omni Processing Plate 24	08413975001
cobas omni Amplification Plate 24	08499853001
cobas omni Liquid Waste Plate 24	08413983001
Tip CORE TIPS with Filter, 1mL	04639642001
Tip CORE TIPS with Filter, 300uL	07345607001
cobas omni Liquid Waste Container	07094388001
cobas omni Lysis Reagent	06997538190
cobas omni MGP Reagent	06997546190
cobas omni Specimen Diluent	06997511190
cobas omni Wash Reagent	06997503190
Solid Waste Bag	07435967001
or	or
Solid Waste Bag With Insert	08030073001

Additional materials required for cobas® 6800/8800 Systems

Table 12 Materials and consumables for use on cobas® 6800/8800 Systems

Material	P/N
cobas omni Processing Plate	05534917001
cobas omni Amplification Plate	05534941001
cobas omni Pipette Tips	05534925001
cobas omni Liquid Waste Container	07094388001
cobas omni Lysis Reagent	06997538190
cobas omni MGP Reagent	06997546190
cobas omni Specimen Diluent	06997511190
cobas omni Wash Reagent	06997503190
Solid Waste Bag and Solid Waste Container or Solid Waste Bag With Insert and Kit Drawer	07435967001 and 07094361001 or 08030073001 and 08387281001

Table 13 Other materials and consumables required for DBS application only

Materials
Whatman 903® filter card, Munktell Specimen Collection card TFN or equivalent (12-13 mm spot diameter)
Tubes, 5 mL, internal thread, 12.5 mm diameter, polypropylene (i.e., Cryo.s™) with caps
Eppendorf Thermomixer (e.g., model R 5355 or equivalent) with Thermoblock for 24 cryo tubes
Sterile or disposable forceps or tweezers
Resealable bags and desiccant sachets (for DBS storage)

Instrumentation and software required

The **cobas**® 5800 software and **cobas**® HIV-1/HIV-2 Qualitative analysis package for the **cobas**® 5800 System, SW **cobas**® 5800 HIV-1/2 Qual-Serum/Plasma ASAP and/or SW **cobas**® 5800 HIV-1/2 Qual-DBS ASAP, shall be installed on the **cobas**® 5800 instrument. The Data Manager software and PC for the **cobas**® 5800 System will be provided with the system.

The **cobas**® 6800/8800 software and **cobas**® HIV-1/HIV-2 Qualitative analysis package(s) for the **cobas**® 6800/8800 Systems, SW **cobas**® HIV-1/2 Qual-Serum/Plasma ASAP and/or SW **cobas**® HIV-1/2 Qual-DBS ASAP, shall be installed on the instrument(s). The Instrument Gateway (IG) server will be provided with the system.

Table 14 Instrumentation

Equipment	P/N
cobas ® 5800 System	08707464001
cobas ® 6800 System (Option Moveable)	05524245001 and 06379672001
cobas ® 6800 System (Fix)	05524245001 and 06379664001
cobas ® 8800 System	05412722001
Sample Supply Module	06301037001

Refer to the **cobas**® 5800 System or **cobas**® 6800/8800 Systems User Assistance and/or User Guides for additional information

Note: Contact your local Roche representative for a detailed order list for sample racks, racks for clotted tips and rack trays accepted on the instruments.

Precautions and handling requirements

Warnings and precautions

As with any test procedure, good laboratory practice is essential to the proper performance of this assay. Due to the high sensitivity of this test, care should be taken to keep reagents and amplification mixtures free of contamination.

- For in vitro diagnostic use only.
- This test is not intended for use in screening blood or plasma donors.
- All patient samples should be handled as if infectious, using good laboratory procedures as outlined in Biosafety in Microbiological and Biomedical Laboratories and in the CLSI Document M29-A4.^{18,19} Only personnel proficient in handling infectious materials and the use of cobas® HIV-1/HIV-2 Qualitative and cobas® 5800/6800/8800 Systems should perform this procedure.
- All human-sourced materials should be considered potentially infectious and should be handled with universal precautions. If spillage occurs, immediately disinfect following appropriate site procedures.
 - If spillage of DBS samples in cobas® Specimen Pre-Extraction Reagent (which contain guanidine thiocyanate) occurs, do not allow it to come in contact with sodium hypochlorite containing disinfectants such as bleach. This mixture can produce a highly toxic gas.
- cobas® HIV-1/HIV-2 Qualitative Control Kit and cobas® NHP Negative Control Kit contain plasma derived from human blood. The source material has been tested by licensed antibody tests and found non-reactive for the presence of antibody to HIV-1/2. Testing of normal human plasma by PCR methods also showed no detectable HIV-1 (Groups M and O) RNA and HIV-2 RNA. No known test method can offer complete assurance that products derived from human blood will not transmit infectious agents.
- **Do not freeze whole blood or any samples stored in primary tubes.**
- cobas® Specimen Pre-Extraction Reagent is light sensitive and shipped in light protective bottles.
- Use only supplied or specified required consumables to ensure optimal test performance.
- Safety Data Sheets (SDS) are available on request from your local Roche representative.
- Closely follow procedures and guidelines provided to ensure that the test is performed correctly. Any deviation from the procedures and guidelines may affect optimal test performance.
- False positive results may occur if carryover of samples is not adequately controlled during sample handling and processing.
- Inform your local competent authority about any serious incidents which may occur when using this assay.

Reagent handling

- Handle all reagents, controls, and samples according to good laboratory practice in order to prevent carryover of samples or controls.
- Before use, visually inspect each reagent cassette, diluent, lysis reagent, wash reagent, and cobas® Specimen Pre-Extraction Reagent (required for DBS application only) to ensure that there are no signs of leakage. If there is any evidence of leakage, do not use that material for testing.
- **cobas omni** Lysis Reagent and cobas® Specimen Pre-Extraction Reagent contain guanidine thiocyanate, a potentially hazardous chemical. Avoid contact of reagents with the skin, eyes, or mucous membranes. If contact does occur, immediately wash with generous amounts of water; otherwise, burns can occur.

- **cobas**® HIV-1/HIV-2 Qualitative kits, **cobas omni** MGP Reagent, and **cobas omni** Specimen Diluent contain sodium azide as a preservative. Avoid contact of these reagents with the skin, eyes, or mucous membranes. If contact does occur, immediately wash with generous amounts of water; otherwise, burns can occur. If these reagents are spilled, dilute with water before wiping dry.
- Do not allow **cobas omni** Lysis Reagent or **cobas**® Specimen Pre-Extraction Reagent, which contain guanidine thiocyanate, to contact sodium hypochlorite (bleach) solution. This mixture can produce a highly toxic gas.
- Dispose of all materials that have come in contact with samples and reagents in accordance with country, state, and local regulations.

Good laboratory practice

- Do not pipette by mouth.
- Do not eat, drink, or smoke in designated work areas.
- Wear laboratory gloves, laboratory coats, and eye protection when handling samples and reagents. Gloves must be changed between handling samples and **cobas**® HIV-1/HIV-2 Qualitative kits and **cobas omni** reagents to prevent contamination. Avoid contaminating gloves when handling samples and controls.
- Wash hands thoroughly after handling samples and kit reagents, and after removing the gloves.
- Thoroughly clean and disinfect all laboratory work surfaces with a freshly prepared solution of 0.5% sodium hypochlorite in distilled or deionized water (dilute household bleach 1:10). Follow by wiping the surface with 70% ethanol.
- If spills occur on the **cobas**® 5800 instrument, follow the instructions in the **cobas**® 5800 Systems User Assistance and/or User Guide to properly clean and decontaminate the surface of instrument(s).
- If spills occur on the **cobas**® 6800/8800 instrument, follow the instructions in the **cobas**® 6800/8800 Systems User Assistance and/or User Guide to properly clean and decontaminate the surface of the instrument(s).

Sample collection, transport, and storage

Note: Handle all samples and controls as if they are capable of transmitting infectious agents.

Store all samples at specified temperatures.


Sample stability is affected by elevated temperatures.

If using frozen samples in secondary tubes, place the samples at room temperature (15-30°C) until completely thawed and then briefly mix (e.g. vortex for 3-5 seconds) and centrifuge to collect all sample volume at the bottom of the tube.

Samples

EDTA plasma and serum samples

- Whole blood should be collected in SST™ Serum Separation Tubes, BD Vacutainer® PPT™ Plasma Preparation Tubes for Molecular Diagnostic Test Methods or in sterile tubes using EDTA as the anticoagulant. Follow the sample collection tube manufacturer instructions.
- Whole blood collected in SST™ Serum Separation Tubes, BD Vacutainer® PPT™ Plasma Preparation Tubes for Molecular Diagnostic Test Methods or in sterile tubes using EDTA as the anticoagulant may be stored and/or transported for up to 24 hours at 2°C to 25°C prior to plasma or serum preparation. Centrifugation should be performed according to manufacturer's instructions.
- Upon separation EDTA plasma or serum samples may be stored in secondary tubes for up to 24 hours at 30°C followed by up to 5 days at 2°C to 8°C or up to 6 weeks at $\leq -20^{\circ}\text{C}$. For long-term storage, temperatures at $\leq -60^{\circ}\text{C}$ are recommended.
- Plasma and serum samples are stable for up to three freeze/thaw cycles when frozen at $\leq -20^{\circ}\text{C}$.
- Refer to Figure 1 for sample storage conditions.

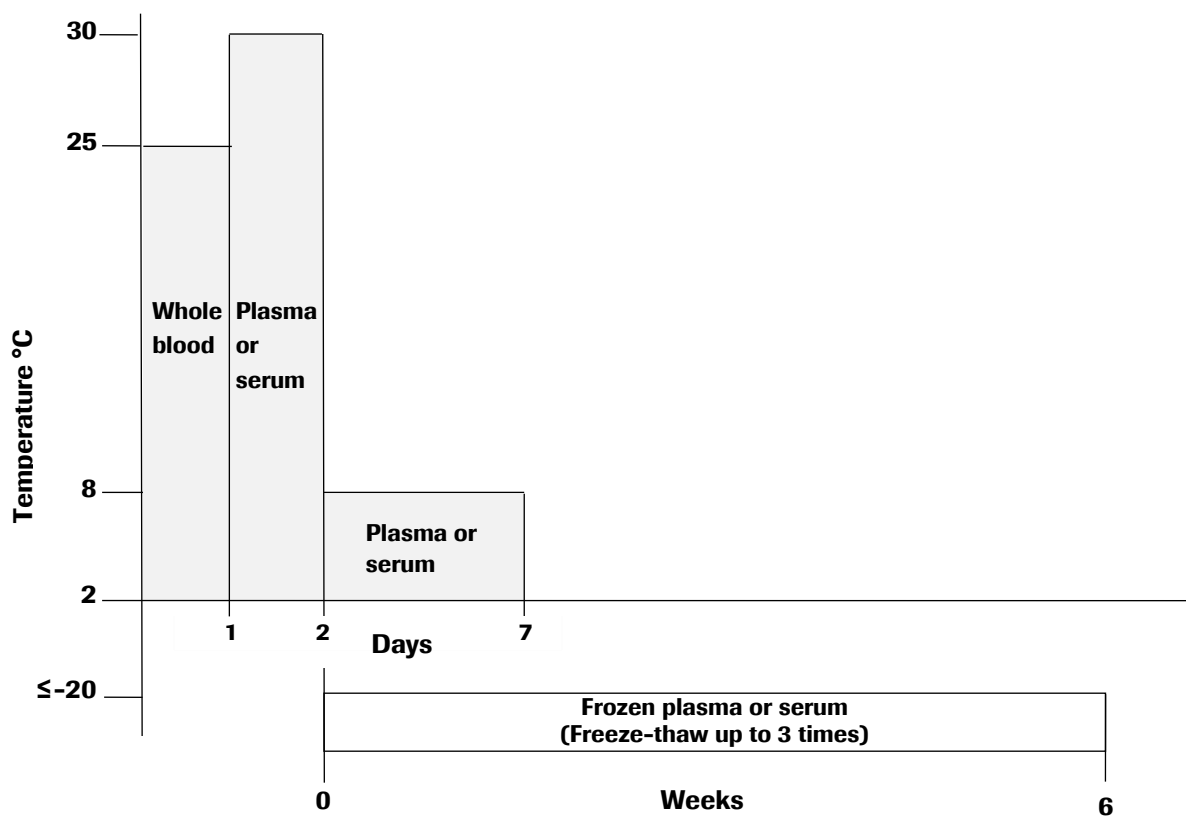Figure 1 Whole blood, plasma and serum sample storage conditions

- If samples are to be shipped, they should be packaged and labeled in compliance with applicable country and/or international regulations covering the transport of samples and etiologic agents.

Dried blood spots

- Collect DBS samples using appropriate clinical procedures.
- It is recommended to apply a minimum of 70 µL of capillary blood inside each delineated circle on the DBS card.
- Ensure that BOTH sides of the paper are saturated and completely fill the delineated circle.
- Allow DBS to dry at room temperature (18-25°C) for at least 3 hours, protecting the DBS card from direct sunlight.
- For further details consult package insert of filter cards used.
- It is recommended to prepare at least 3 paper disks per patient sample.
- Store DBS in individual resealable bags with a desiccant sachet in each bag.
- DBS may be transported or stored at 15-30°C for up to three months.
- If samples are to be shipped, they should be packaged and labeled in compliance with applicable country and/or international regulations covering the transport of samples and etiologic agents.

Instructions for use

Procedural notes

- Do not use **cobas**® HIV-1/HIV-2 Qualitative reagents, **cobas**® HIV-1/HIV-2 Qualitative Control Kit, **cobas**® NHP Negative Control Kit, **cobas** **omni** reagents, or **cobas**® Specimen Pre-Extraction Reagent after their expiry dates.
- Do not reuse consumables. They are for one-time use only.
- Refer to the **cobas**® 5800 System or **cobas**® 6800/8800 Systems User Assistance and/or User Guide for proper maintenance of instruments.

Dried blood spot sample preparation


- Allow **cobas**® Specimen Pre-Extraction Reagent (SPER) to equilibrate to ambient temperature before use.
- Excise one DBS from the DBS card.
- Transfer the spot into a tube (5 mL, internal thread, 12.5 mm diameter, polypropylene [i.e., Cryo.s™]) using sterile or disposable forceps or tweezers.
- Ensure the DBS is located at the bottom of the tube as shown in Figure 2.
- Pipette 1150 µL of SPER into the tube containing the DBS and cap the tube.
- Ensure the DBS is completely covered with SPER.
- Place tubes in each of the positions 1 - 24 on a preheated Eppendorf Thermomixer (e.g., model R 5355 or equivalent) with Thermoblock for 24 cryo tubes and incubate for 10 minutes, at 56°C and 1000 rpm to extract the virus from the dried whole blood.
- Uncap the tubes and ensure the DBS is attached to the tube wall (Figure 3) to avoid sample clots.
- Eliminate any potential liquid film located above the liquid level (Figure 4) using a sterile pipette tip (to avoid early level detection).
- Transfer the tubes to the **cobas**® 5800 System or **cobas**® 6800/8800 Systems.

Figure 2 DBS in tube



Valid

Figure 3 DBS location

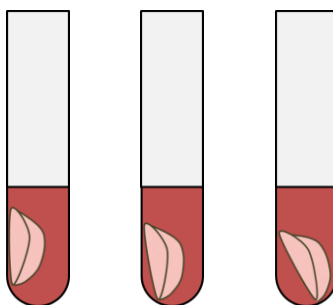


Valid

Valid

Potential
risk of clot

Figure 4 Liquid film

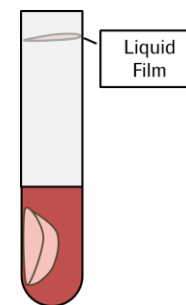

Potential risk of
early level detection

Running cobas® HIV-1/HIV-2 Qualitative on cobas® 5800 System

cobas® HIV-1/HIV-2 Qualitative can be run with required sample volumes of 650 µL (the 500 µL plasma/serum sample workflow) or 1150 µL cobas® Specimen Pre-Extraction Reagent (for the 850 µL DBS sample workflow). Please note, DBS samples can be run in mixed batch mode with plasma, serum, or the cobas® HIV-1 PSC workflows. The test procedure is described in detail in the cobas® 5800 System User Assistance and/or User Guide. Figure 5 below summarizes the procedure.

Figure 5 cobas® HIV-1/HIV-2 Qualitative test procedure on cobas® 5800 System

1	Log onto the system
2	Loading samples onto the system <ul style="list-style-type: none"> • Load sample racks onto the system • The system prepares automatically • Order tests
3	Refill reagents and consumables as prompted by the system <ul style="list-style-type: none"> • Load test specific reagent cassette(s) • Load control mini racks • Load processing tips • Load elution tips • Load processing plates • Load liquid waste plates • Load amplification plates • Load MGP cassette • Refill specimen diluent • Refill lysis reagent • Refill wash reagent
4	Start the run by choosing the Start processing button on the user interface, all subsequent runs will start automatically if not manually postponed
5	Review and export results
6	Remove and cap any sample tubes meeting the minimum volume requirements if needed for future use Clean up the instrument <ul style="list-style-type: none"> • Unload empty control mini racks • Unload empty test specific reagent cassette(s) • Empty amplification plate drawer • Empty liquid waste • Empty solid waste

Running cobas® HIV-1/HIV-2 Qualitative on cobas® 6800/8800 Systems

cobas® HIV-1/HIV-2 Qualitative can be run with a minimum required sample volume of 650 µL (for the 500 µL plasma or serum sample workflow) or 1150 µL cobas® Specimen Pre-Extraction Reagent (for the 850 µL DBS sample workflow). Please note, cobas® HIV-1/ HIV-2 Qualitative Dried Blood Spot workflow has been evaluated in mixed batch functionality only with the cobas® HIV-1 PSC workflow. The test procedure is described in detail in the cobas® 6800/8800 Systems User Assistance and/or User Guide. Figure 6 below summarizes the procedure.

Figure 6 cobas® HIV-1/HIV-2 Qualitative test procedure on cobas® 6800/8800 Systems

1	Log onto the system Press Start to prepare the system Order tests
2	Refill reagents and consumables as prompted by the system <ul style="list-style-type: none"> • Load test specific reagent cassette • Load control cassettes • Load pipette tips • Load processing plates • Load MGP reagent • Load amplification plates • Refill specimen diluent • Refill lysis reagent • Refill wash reagent
3	Loading samples onto the system <ul style="list-style-type: none"> • Load sample racks and clotted tip racks onto the sample supply module • Confirm samples have been accepted into the transfer module
4	Start the run by choosing the Start manually button on the user interface or have it start automatically after 120 minutes or if the batch is full
5	Review and export results
6	Remove and cap any sample tubes meeting the minimum volume requirements if needed for future use Clean up the instrument <ul style="list-style-type: none"> • Unload empty control cassettes • Empty amplification plate drawer • Empty liquid waste • Empty solid waste

Results

The **cobas**® 5800 System and **cobas**® 6800/8800 Systems automatically detect and discriminate HIV-1 and HIV-2 simultaneously in samples and controls, displaying test validity, overall results, as well as individual target results.

Quality control and validity of results on the **cobas**® 5800 System

- One Normal Human Plasma Negative Control [(-) C] and two positive controls [HIV-1 M/HIV-2 (+)C and HIV-1 O (+)C] are processed at least every 72 hours or with every new kit lot. Positive and/or negative controls can be scheduled more frequently based on laboratory procedures and/or local regulations.
- In the **cobas**® 5800 System software and/or report, check for flags and their associated results to ensure the batch validity.

Invalidation of results is performed automatically by the **cobas**® 5800 software based on negative or positive control failures.

NOTE: The **cobas**® 5800 System will be delivered with the standard setting of running a set of controls (positive and negative) with every run, but can be configured to a less frequent scheduling up to every 72 hours based on laboratory procedures and/or local regulations. Please contact your Roche service engineer and/or Roche customer technical support for more information.

Control results on **cobas**® 5800 System

The results of the controls are shown in the **cobas**® 5800 software in the “Controls” app.

- Controls are marked with “Valid” in the column “Control result” if all Targets of the control are reported valid. Controls are marked with ‘Invalid’ in the column “Control result” if all or one Target of the control are reported invalid.
- Controls marked with ‘Invalid’ show a flag in the “Flags” column. More information on why the control is reported invalid including flag information is shown in the detail view.
- If one of the positive controls is invalid, repeat testing of the all positive controls and all associated samples. If the negative control is invalid, repeat testing of all controls and all associated samples.

Quality control and validity of results on the **cobas**® 6800/8800 Systems

- One Normal Human Plasma Negative Control [(-) C] and two positive controls [HIV-1 M/HIV-2 (+)C and HIV-1 O (+)C] are processed with each batch
- In the **cobas**® 6800/8800 software and/or report, check for flags and their associated results to ensure the batch validity.
- The batch is valid if no flags appear for all three controls.

Invalidation of results is performed automatically by the **cobas**® 6800/8800 software based on negative and positive control failures.

Control flags on cobas® 6800/8800 Systems

Table 15 Control flags for negative and positive controls

Negative Control	Flag	Result	Interpretation
(-) C	Q02 (Control batch failed)	Invalid	The entire batch is assigned invalid if the result for the (-) C is invalid.
Positive Control	Flag	Result	Interpretation
HIV-1 M/HIV-2 (+)C	Q02 (Control batch failed)	Invalid	The entire batch is assigned invalid if the result for the HIV-1 M/HIV-2 (+)C is invalid.
HIV-1 O (+)C	Q02 (Control batch failed)	Invalid	The entire batch is assigned invalid if the result for the HIV-1 O (+)C is invalid.

If the batch is invalid, repeat testing of the entire batch including samples and controls.

(-) C stands for NHP negative control, HIV-1 M/HIV-2 (+) C stands for cobas® HIV-1 M/HIV-2 positive control and HIV-1 O (+) C stands for cobas® HIV-1 O positive control in the cobas® 6800/8800 software.

Interpretation of results on the cobas® 5800 System

The results of the samples are shown in the cobas® 5800 software in the “Results” app.

For a valid control batch, check each individual sample for flags in the cobas® 5800 System softwares and/or report. The result interpretation should be as follows:

- A valid control batch may include both valid and invalid sample results.
- Samples associated with a valid control batch are shown as ‘Valid’ in the “Control result” column if all Control Target Results reported valid. Samples associated with a failed control batch are shown as ‘Invalid’ in the “Control result” column if all Control Target Results reported invalid.
- If the associated controls of a sample result are invalid, a specific flag will be added to the sample result as follows:
 - Q05D : Result validation failure because of an invalid positive control
 - Q06D : Result validation failure because of an invalid negative control
- The values in “Results” column for individual sample target result should be interpreted as shown in Table 16.
- If one or more sample targets are marked with “Invalid” the cobas® 5800 software shows a flag in the “Flags” column. More information on why the sample target(s) is reported invalid including flag information is shown in the detail view.

Results and their corresponding interpretation for detecting HIV-1 and HIV-2 are shown below in Table 16.

Table 16 Results for individual target result interpretation on the cobas® 5800 System

Result		Interpretation
HIV-1 Reactive	HIV-2 Reactive	All requested results were valid. Target signal detected for HIV-1 and HIV-2.
HIV-1 Reactive	HIV-2 Non-Reactive	All requested results were valid. Target signal detected for HIV-1. No target signal detected for HIV-2.
HIV-1 Non-Reactive	HIV-2 Reactive	All requested results were valid. No target signal detected for HIV-1. Target signal detected for HIV-2.
HIV-1 Non-Reactive	HIV-2 Non-Reactive	All requested results were valid. No target signal detected for HIV-1 or HIV-2.
Invalid	Invalid	Both HIV-1 and HIV-2 results are invalid. Original specimen should be re-tested to obtain valid HIV-1 and HIV-2 results. If the results are still invalid, a new specimen should be obtained.

Interpretation of results on the cobas® 6800/8800 Systems

For a valid batch, check each individual sample for flags in the cobas® 6800/8800 Systems software and/or report. The result interpretation should be as follows:

- A valid batch may include both valid and invalid sample results.
- Samples are marked with “Yes” in the column ‘Valid’ if all requested Target Results reported valid results. Samples marked with “No” in the column ‘Valid’ may require additional interpretation and action.
- The values in “Overall Result” column for individual samples should be interpreted as follows:
 - Reactive – All requested results are reactive or one of the requested results is reactive and the other non-reactive
 - Non-Reactive – All requested results are non-reactive
 - Invalid – At least one requested result is invalid
- Reported target results for individual samples are valid unless indicated otherwise.

Results and their corresponding interpretation for detecting HIV-1 and HIV-2 are shown in Table 17.

Table 17 Target results for individual target result interpretation on the cobas® 6800/8800 Systems

Valid	Overall Result	Target 1	Target 2	Interpretation
Yes	Reactive	HIV-1 Reactive	HIV-2 Reactive	All requested results were valid. Target signal detected for HIV-1 and HIV-2.
Yes	Reactive	HIV-1 Reactive	HIV-2 Non- Reactive	All requested results were valid. Target signal detected for HIV-1. No target signal detected for HIV-2.
Yes	Reactive	HIV-1 Non- Reactive	HIV-2 Reactive	All requested results were valid. No target signal detected for HIV-1. Target signal detected for HIV-2.
Yes	Non-Reactive	HIV-1 Non- Reactive	HIV-2 Non- Reactive	All requested results were valid. No target signal detected for HIV-1 or HIV-2.
No	Invalid	Invalid	Invalid	Both HIV-1 and HIV-2 results are invalid. Original specimen should be re-tested to obtain valid HIV-1 and HIV-2 results. If the results are still invalid, a new specimen should be obtained.

Procedural limitations

- **cobas**® HIV-1/HIV-2 Qualitative has been evaluated only for use in combination with the **cobas**® HIV-1/HIV-2 Qualitative Control Kit, **cobas**® NHP Negative Control Kit, **cobas omni** MGP Reagent, **cobas omni** Lysis Reagent, **cobas omni** Specimen Diluent, **cobas omni** Wash Reagent, and **cobas**® Specimen Pre-Extraction Reagent (used in dried blood spot workflow) for use on the **cobas**® 5800/6800/8800 Systems.
- Reliable results depend on proper sample type (EDTA plasma or serum) and sample collection, as well as storage and handling procedures. Use of the assay with other types of specimens may not yield accurate results.
- Detection of HIV-1 and HIV-2 nucleic acid is dependent on the number of virus particles present in the sample and may be affected by sample collection, storage and handling, patient factors (i.e., age, presence of symptoms), and/or stage of infection.
- Though rare, mutations within the highly conserved regions of a viral genome covered by **cobas**® HIV-1/HIV-2 Qualitative may affect primers and/or probe binding resulting in the failure to detect the presence of virus.
- Due to inherent differences between technologies, it is recommended that, prior to switching from one technology to the next, users perform method correlation studies in their laboratory to qualify technology differences. Users should follow their own specific policies/procedures.
- **cobas**® HIV-1/HIV-2 Qualitative is not intended for use as a screening test for the presence of HIV-1/HIV-2 in donated blood or blood products.

Non-clinical performance evaluation

Key performance characteristics performed on the cobas® 6800/8800 Systems

Limit of Detection (LoD)

WHO International Standards/Roche Primary Standards

The limit of detection of the cobas® HIV-1/HIV-2 Qualitative was determined by using the following standards:

- WHO 3rd International Standard for HIV-1 group M RNA (NIBSC code 10/152) for EDTA plasma, serum and DBS samples
- WHO International Standard for HIV-2 RNA (NIBSC code 08/150) for plasma and serum samples
- Roche Primary Standards for HIV-2 RNA for DBS samples
- Roche Primary Standards for HIV-1 group O RNA for EDTA plasma and serum samples

No international standard is currently available for HIV-1 group O RNA. The Roche HIV-1 group O RNA Standard is traceable to the CBER HIV-1 Subtype RNA Reference Panel #1 Lot 01. The Roche Primary Standards for HIV-1 group O RNA are derived from commercially available cultured virus stocks, P/N 2420 (Cat. No. 500493, SeraCare Life Sciences). The Roche HIV-2 RNA Standard is traceable to the WHO International Standard for HIV-2 RNA (NIBSC code 08/150). The Roche Primary Standards for HIV-2 RNA are derived from commercially available cultured virus stocks, P/N HIV-2 NIH-Z (Cat. No. 10-27-000, Applied Biotechnologies, Inc.). One copy of HIV-1 RNA is equivalent to 1.7 International Unit (IU) and one copy of HIV-2 RNA is equivalent to 0.2 IU.

Serial dilutions of the standards in HIV-negative human EDTA plasma, serum or whole blood for DBS were prepared. Panels of five or six concentration levels plus a negative were tested over three lots of cobas® HIV-1/HIV-2 Qualitative reagents, multiple runs, days, operators, and instruments.

For each virus, 95% PROBIT analysis on the data combined across dilution series and reagent lots was used to estimate the LoD, along with the lower and upper limit of the 95% confidence interval (Table 18). The reactivity rates observed in the LoD studies for each virus are summarized in Table 19 to Table 21.

Table 18 Results of 95% PROBIT analysis on LoD data collected with viral standards in EDTA plasma, serum and DBS

Matrices	Analyte	Measuring units	LoD	Lower 95% confidence limit	Upper 95% confidence limit
EDTA plasma	HIV-1 group M	copies/mL	12.6	10.9	15.2
	HIV-1 group O	copies/mL	14.8	12.8	17.7
	HIV-2	copies/mL	27.9	22.9	36.6
Serum	HIV-1 group M	copies/mL	12.1	10.5	14.5
	HIV-1 group O	copies/mL	12.6	10.9	15.2
	HIV-2	copies/mL	23.4	19.6	29.7
DBS	HIV-1 group M	copies/mL	255	224	299
	HIV-2	copies/mL	984	856	1169

Table 19 Reactivity rates summary for HIV-1 group M in EDTA plasma, serum and DBS

Matrices	HIV-1 group M RNA concentration (cp/mL)	Number of reactive	Number of valid replicates	% Reactive
EDTA plasma	40	189	189	100%
	30	189	189	100%
	20	187	189	99%
	10	174	189	92%
	5	124	189	66%
	2.5	91	189	48%
	0	0	189	0%
Serum	40	189	189	100%
	30	189	189	100%
	20	187	189	99%
	10	176	189	93%
	5	126	189	67%
	2.5	86	189	46%
	0	0	189	0%
DBS	750	252	252	100%
	600	252	252	100%
	360	246	250	98%
	180	220	249	88%
	90	163	252	65%
	45	109	250	44%
	0	0	107	0%

Table 20 Reactivity rates summary for HIV-1 group O in EDTA plasma and serum

Matrices	HIV-1 group O RNA concentration (cp/mL)	Number of reactive	Number of valid replicates	% Reactive
EDTA plasma	40	189	189	100%
	30	189	189	100%
	20	185	188	98%
	10	163	189	86%
	5	117	189	62%
	2.5	78	189	41%
	0	0	189	0%
Serum	40	189	189	100%
	30	189	189	100%
	20	186	189	98%
	10	173	189	92%
	5	132	189	70%
	2.5	91	189	48%
	0	0	189	0%

Table 21 Reactivity rates summary for HIV-2 in EDTA plasma, serum and DBS

Matrices	HIV-2 RNA concentration (cp/mL)	Number of reactive	Number of valid replicates	% Reactive
EDTA plasma	80	126	126	100%
	40	124	126	98%
	20	115	126	91%
	10	81	126	64%
	5	61	126	48%
	0	0	189	0%
Serum	80	126	126	100%
	40	125	126	99%
	20	114	126	90%
	10	96	126	76%
	5	49	126	39%
	0	0	189	0%
DBS	3000	252	252	100%
	1450	241	247	98%
	725	226	246	92%
	362	167	248	67%
	181	103	250	41%
	0	0	108	0%

Precision – within laboratory

The precision of cobas® HIV-1/HIV-2 Qualitative was determined using the following standards:

- Roche Secondary Standard for HIV-1 group M
- Roche Primary Standard for HIV-2

Two panels of individually formulated HIV-1 group M and HIV-2 target, each comprising 3 panel members at concentrations of approximately 0.6 x, 1 x and 3 x of the LoD of cobas® HIV-1/HIV-2 Qualitative were tested in this study. Testing was performed for the following variability components:

- day-to-day variability over 4 days
- lot-to-lot variability using 3 different reagent lots of cobas® HIV-1/HIV-2 Qualitative
- instrument-to-instrument variability using 3 different cobas® 6800/8800 Systems

Approximately 84 replicates were tested with each of the 3 panel members for each reagent lot for a total of 252 replicates over all reagent lots per target. Precision results were evaluated by calculating the percentage of reactive test results at each concentration level for each of the variability components analyzed.

The limits of two-sided 95% confidence intervals for each reactive rate were calculated for each of the three levels of HIV-1 group M and HIV-2 tested across 4 days, 3 reagent lots, and 3 cobas® 6800/8800 Systems. cobas® HIV-1/HIV-2 Qualitative is reproducible over multiple days, reagent lots and multiple instruments. The results from reagent lot-to-lot variability are summarized in Table 22 and Table 23.

Table 22 cobas® HIV-1/HIV-2 Qualitative reagent lot-to-lot precision summary (EDTA plasma)

Analyte	Concentration	Reagent lot	% Reactive (reactive/valid replicates)	Lower limit of 95% confidence interval	Upper limit of 95% confidence interval
HIV-1 group M	~3 x LoD	1	100% (84/84)	95.7%	100%
		2	100% (84/84)	95.7%	100%
		3	100% (84/84)	95.7%	100%
	~1 x LoD	1	98.8% (83/84)	93.5%	100%
		2	98.8% (83/84)	93.5%	100%
		3	100% (84/84)	95.7%	100%
	~0.6 x LoD	1	77.4% (65/84)	67.0%	85.8%
		2	76.2% (64/84)	65.7%	84.8%
		3	82.1% (69/84)	72.3%	89.6%
HIV-2	~3 x LoD	1	98.8% (83/84)	93.5%	100%
		2	96.4% (81/84)	89.9%	99.3%
		3	100% (84/84)	95.7%	100%
	~1 x LoD	1	98.8% (83/84)	93.5%	100%
		2	98.8% (83/84)	93.5%	100%
		3	97.6% (82/84)	91.7%	99.7%
	~0.6 x LoD	1	66.7% (56/84)	55.5%	76.6%
		2	69.0% (58/84)	58.0%	78.7%
		3	69.0% (58/84)	58.0%	78.7%

Table 23 cobas® HIV-1/HIV-2 Qualitative reagent lot-to-lot precision summary (DBS)

Analyte	Concentration	Reagent lot	% Reactive (reactive/valid replicates)	Lower limit of 95% confidence interval	Upper limit of 95% confidence interval
HIV-1 group M	~3 x LoD	1	100% (84/84)	95.7%	100%
		2	100% (83/83)	95.7%	100%
		3	97.6% (82/84)	91.7%	99.7%
	~1 x LoD	1	96.4% (81/84)	89.9%	99.3%
		2	96.4% (81/84)	89.9%	99.3%
		3	95.2% (80/84)	88.3%	98.7%
	~0.6 x LoD	1	88.0% (73/83)	79.0%	94.1%
		2	83.3% (70/84)	73.6%	90.6%
		3	88.1% (74/84)	79.2%	94.1%
HIV-2	~3 x LoD	1	100% (83/83)	95.7%	100%
		2	100% (84/84)	95.7%	100%
		3	100% (83/83)	95.7%	100%
	~1 x LoD	1	97.6% (82/84)	91.7%	99.7%
		2	97.6% (82/84)	91.7%	99.7%
		3	98.8% (83/84)	93.5%	100%
	~0.6 x LoD	1	88.1% (74/84)	79.2%	94.1%
		2	91.7% (77/84)	83.6%	96.6%
		3	85.7% (72/84)	76.4%	92.4%

Group/subtype verification and inclusivity

The performance of cobas® HIV-1/HIV-2 Qualitative on HIV-1 group M subtypes, group O, group N and HIV-2 group B was evaluated by:

- Verification of the limit of detection for HIV-1 group M subtypes, group O (verified by dilution in whole blood for DBS), group N and HIV-2 group B
- Verification of the inclusivity for HIV-1 group M subtypes, group O, group N and HIV-2 group A and group B

Verification of limit of detection for HIV-1 group M subtypes, group O, group N and HIV-2 group B

Clinical or cultured HIV samples for HIV-1 group M (A, C, D, F, G, H) and circulating recombinant forms (CRF01_AE, CRF02_AG), HIV-1 group N and HIV-2 group B were diluted in EDTA plasma, serum or whole blood for DBS, and additionally HIV-1 group O in whole blood for DBS, to the LoD concentration of the predominant group/subtype (HIV-1 group M subtype B or HIV-2 group A) based on the LoD determined with 95% PROBIT analysis over all lots combined. The reactive rate determination was performed with 42 replicates. Testing was conducted with 1 lot of cobas® HIV-1/HIV-2 Qualitative reagents. The results from HIV-1 are shown in Table 24 and the results from HIV-2 are shown in Table 25. These results verify that cobas® HIV-1/HIV-2 Qualitative detected HIV for HIV-1 group M (A, C, D, F, G, H, CRF01_AE,

CRF02_AG), HIV-1 group O, HIV-1 group N and HIV-2 group B at the claimed concentration for each matrix or below with an upper 95% confidence interval being equal to or greater to the expected reactivity rate of 95%.

Table 24 LoD verification of HIV-1 group M subtypes, group O, and group N in EDTA plasma, serum or whole blood for DBS

Group	Subtype	Plasma: 12.6 cp/mL			Serum: 12.1 cp/mL			DBS: 255 cp/mL		
		Number of valid replicates	Number of reactive	% Reactive (95% CI*)	Number of valid replicates	Number of reactive	% Reactive (95% CI*)	Number of valid replicates	Number of reactive	% Reactive (95% CI*)
M	A	42	40	95% (99%)	42	40	95% (99%)	41	37	90% (97%)
	C	42	41	98% (100%)	42	42	100% (99.4%)	42	42	100% (100%)
	D	42	37	88% (96%)	42	37	88% (96%)	42	39	93% (99%)
	F	42	38	90% (97%)	42	38	90% (97%)	42	40	95% (99%)
	G	42	40	95% (99%)	42	39	93% (99%)	42	42	100% (100%)
	H	42	38	90% (97%)	42	41	98% (100%)	42	41	98% (100%)
	CRF01_AE	42	38	90% (97%)	42	38	90% (97%)	42	41	98% (100%)
	CRF02_AG	42	36	86% (95%)	42	39	93% (99%)	42	42	100% (100%)
O		N/A	N/A	N/A	N/A	N/A	N/A	41	39	95% (99%)
N		42	39	93% (99%)	42	37	88% (96%)	41	40	98% (100%)

* Upper 95% confidence interval

Table 25 LoD verification of HIV-2 group B in EDTA plasma, serum or whole blood for DBS

Group	Plasma: 27.9 cp/mL			Serum: 23.4 cp/mL			DBS: 984 cp/mL		
	Number of valid replicates	Number of positives	% Reactive (95% CI*)	Number of valid replicates	Number of positives	% Reactive (95% CI*)	Number of valid replicates	Number of positives	% Reactive (95% CI*)
B	42	42	100% (100%)	42	42	100% (100%)	42	42	100% (100%)

* Upper 95% confidence interval

Verification of inclusivity for HIV-1 group M subtypes, group O, group N and HIV-2 group A and group B

The performance of cobas® HIV-1/HIV-2 Qualitative to detect subtypes of HIV-1 group M (A, C, D, F, G, H, J, K) and circulating recombinant forms (CRF01_AE, CRF02_AG, CRF12_BF, CRF14_BG), HIV-1 group O, HIV-1 group N, HIV-2 group A and HIV-2 group B was determined by testing unique clinical samples and/or culture isolated for each group or subtype in EDTA plasma or serum.

HIV-1 group M

A total of 105 unique HIV-1 group M clinical samples with known HIV-1 subtype were tested neat (undiluted) and after dilution to $\sim 5 \times \text{LoD}$ of cobas® HIV-1/HIV-2 Qualitative. All 105 clinical samples with known subtypes were detected neat and at $\sim 5 \times \text{LoD}$ (Table 26).

In addition, four HIV-1 group M subtype CRF12_BF and one HIV-1 group M subtype CRF14_BG clinical sample was tested after dilution series were prepared. One replicate of each of the neat samples and one of each dilution from 1:1.0E+01 to 1:5.0E+02 (2-4 dilutions per sample) for HIV-1 group M subtype CRF12_BF and from 1:2.0E+01 to 1:1.2E+02 (4 dilutions) for HIV-1 group M subtype CRF14_BG was tested, all yielding in reactive results. All of the tested clinical samples were detected at $\leq 5 \times \text{LoD}$.

Table 26 HIV-1 group M clinical samples

Subtype / circulating recombinant forms	% Reactive (reactive/samples tested) neat	% Reactive (reactive/samples tested) diluted to $\sim 5 \times \text{LoD}$
A	100% (10/10)	100% (10/10)
C	100% (10/10)	100% (10/10)
D	100% (10/10)	100% (10/10)
F	100% (10/10)	100% (10/10)
G	100% (10/10)	100% (10/10)
H	100% (10/10)	100% (10/10)
J	100% (5/5)	100% (5/5)
K	100% (9/9)	100% (9/9)
CRF01_AE	100% (10/10)	100% (10/10)
CRF02_AG	100% (10/10)	100% (10/10)
CRF12_BF	100% (2/2)	100% (2/2)
CRF14_BG	100% (9/9)	100% (9/9)

HIV-1 group O and HIV-1 group N

A total of 10 HIV-1 group O and one HIV-1 group N clinical or cultured sample was tested after dilution series were prepared. Two replicates of each of the neat samples and four of each dilution from 1:1.0E+01 to 1:4.8E+05 (3-5 dilutions per sample) for HIV-1 group O were tested, all yielding in reactive results. Two replicates of neat sample and four of each dilution from 1:1.0E+04 to 1:1.4E+05 (5 dilutions) for HIV-1 group N were tested. The neat sample and the dilutions from 1:1.0E+04 to 1:4.5E+04 yielded in 100% reactive results, while dilution 1:1.4E+05 yielded in a 50% reactive result. All of the tested samples were detected at $\leq 3 \times \text{LoD}$.

HIV-2

A total of 16 unique HIV-2 group A and group B clinical or cultured samples were tested neat (undiluted) and after dilution to ~5 x LoD of cobas® HIV-1/HIV-2 Qualitative. All 16 HIV-2 samples were detected neat and at ~5 x LoD (Table 27).

In addition, six HIV-2 group A and four HIV-2 group B clinical samples were tested after dilution series were prepared. One replicate of each of the neat samples and one of each dilution from 1:1.0E+01 to 1:9.0E+02 (2-5 dilutions per sample) for HIV-2 group A and from 1:2.0E+01 to 1:6.0E+01 (2-4 dilutions) for HIV-2 group B was tested, all yielding in reactive results. All of the tested clinical samples were detected at ≤ 3 x LoD.

Table 27 HIV-2 clinical or cultured samples

Subtype	% Reactive (reactive/samples tested) neat	% Reactive (reactive/samples tested) diluted to ~5 x LoD
A	100% (4/4)	100% (4/4)
B	100% (6/6)	100% (6/6)

Specificity

The specificity of cobas® HIV-1/HIV-2 Qualitative was determined by analyzing HIV negative EDTA plasma, HIV negative serum and HIV negative DBS samples from individual blood donors. A total of 613 individual EDTA plasma and 607 individual serum samples were tested with two lots of cobas® HIV-1/HIV-2 Qualitative reagents. In addition, 604 individual DBS samples were tested with three lots of cobas® HIV-1/HIV-2 Qualitative reagents. All samples tested were found non-reactive for HIV-1 and HIV-2. In each of the EDTA plasma, serum and DBS samples, the specificity of cobas® HIV-1/HIV-2 Qualitative was 100% (95% confidence limit: $\geq 99.5\%$).

Seroconversion panels

The performance of cobas® HIV-1/HIV-2 Qualitative was evaluated using commercially available seroconversion panels for HIV-1 group M.

HIV-1 group M Seroconversion panels

Twenty five commercially available seroconversion panels were used. Each panel member was tested neat with cobas® HIV-1/HIV-2 Qualitative and the results were compared to the results obtained with an FDA licensed 4th generation HIV Ag/Ab serology test and an FDA licensed comparator nucleic acid test tested neat. The overall performance results are shown in Table 28.

Table 28 Performance of cobas® HIV-1/HIV-2 Qualitative on HIV Seroconversion panels

HIV Seroconversion panel	Number of Panel Members tested	Number of Panel Members with reactive result			Days to first reactive result			Days earlier detection with cobas® HIV-1/HIV-2 Qualitative	
		cobas® HIV-1/HIV-2 Qualitative	Comparator NAT	HIV Ag/Ab Assay	cobas® HIV-1/HIV-2 Qualitative	Comparator NAT	HIV Ag/Ab Assay	Comparator NAT	HIV Ag/Ab Assay
HIV6243	10	6	6	4	18	18	25	0	7
HIV9011	11	3	3	2	30	30	38	0	8
HIV9012	8	5	5	3	9	7	16	-2	7
HIV9013	7	3	2	2	18	23	23	5	5
HIV9018	10	5	5	3	21	21	28	0	7
HIV9020	21	5	5	3	83	83	90	0	7
HIV9022	9	3	4	2	23	17	25	-6	2
HIV9030	16	6	5	3	40	40	47	0	7
HIV9031	19	8	6	4	120	131	146	11	26
HIV9034	13	4	4	3	41	41	46	0	5
HIV9076	9	3	3	3	66	66	66	0	0
HIV9089	6	5	5	3	7	7	16	0	9
HIV12008	13	7	7	5	21	21	28	0	7
PRB954	7	5	5	2	7	7	17	0	10
PRB956	5	4	4	2	40	40	47	0	7
PRB958	6	6	6	4	0	0	7	0	7
PRB961	9	4	4	2	19	19	27	0	8
PRB962	6	4	4	2	7	7	14	0	7
PRB963	7	4	5	2	9	7	17	-2	8
PRB967	6	5	5	3	3	3	17	0	14
PRB968	10	6	6	4	15	15	26	0	11
PRB969	10	7	6	3	53	53	70	0	17
PRB973	4	4	4	2	0	0	7	0	7
PRB976	4	4	4	2	0	0	7	0	7
PRB977	4	4	2	2	0	-*	13	-*	13
Total	230	120	115	70					

* Invalid result for the comparator NAT for the first panel member of panel PRB977, where a reactive NAT result was expected. The panel (PRB977) was therefore not included in the evaluation of the comparator NAT results.

Analytical specificity

The analytical specificity of cobas® HIV-1/HIV-2 Qualitative was evaluated for cross-reactivity with a panel of microorganisms at 10^5 or 10^6 particles, copies, or PFU/mL, for viral isolates and bacterial strains/yeast isolates, respectively (Table 29). The microorganisms were added to HIV negative human EDTA plasma and tested with and without HIV-1 and HIV-2 virus added to a concentration of approximately 3 x LoD of cobas® HIV-1/HIV-2 Qualitative for each virus. Non-reactive results were obtained with cobas® HIV-1/HIV-2 Qualitative for all microorganism samples without HIV-1 and HIV-2 target and reactive results were obtained for all of the microorganism samples with HIV-1 and HIV-2 targets. The tested microorganisms do not cross-react or interfere with cobas® HIV-1/HIV-2 Qualitative.

Table 29 Microorganisms tested for cross-reactivity

Viruses		Bacteria	Yeast
Adenovirus type 5	Varicella-Zoster Virus	Propionibacterium acnes	Candida albicans
Cytomegalovirus	West Nile Virus	Staphylococcus aureus	
Epstein-Barr Virus	St. Louis encephalitis Virus		
Hepatitis A Virus	Murray Valley encephalitis Virus		
Hepatitis B Virus	Dengue virus types 1, 2, 3, and 4		
Hepatitis C Virus	TBE Virus (strain HYPR)		
Hepatitis D Virus	Influenza A Virus		
Human T-Cell Lymphotropic Virus types 1 and 2	Zika Virus		
Human Herpes Virus Type-6	Human Papillomavirus		
Herpes Simplex Virus Type 1 and 2	Yellow Fever Virus		

EDTA plasma samples from each of the disease states (one from Adenovirus type 5 and ten from each of the other disease states) listed in Table 30 were tested with and without HIV-1 and HIV-2 added to a concentration of approximately 3 x LoD of cobas® HIV-1/HIV-2 Qualitative for each virus. These disease states do not cross-react or interfere with cobas® HIV-1/HIV-2 Qualitative.

Table 30 Disease states samples tested for analytical specificity

Disease state		
Adenovirus type 5	Hepatitis B Virus	Herpes Simplex Virus type 2
Cytomegalovirus	Hepatitis C Virus	Human T-cell lymphotropic Virus type I
Dengue Virus	Hepatitis E Virus	Human T-cell lymphotropic Virus type II
Epstein-Barr Virus	Herpes Simplex Virus type 1	West Nile Virus

Analytical specificity – interfering substances

Elevated levels of triglycerides (33 g/L), conjugated bilirubin (0.2 g/L), unconjugated bilirubin (0.2 g/L), albumin (60 g/L), hemoglobin (2 g/L) and human DNA (2 mg/L) in samples as well as the presence of autoimmune diseases such as systemic lupus erythematosus (SLE), rheumatoid arthritis (RA), and antinuclear antibody (ANA) have been tested in the presence and absence of HIV-1 and HIV-2 RNA.

In addition, drug compounds listed in Table 31 were tested at three times the C_{max} in the presence and absence of HIV-1 and HIV-2 RNA.

All potentially interfering substances show no interference with the test performance with the exception of triglycerides and human DNA. At concentrations higher than 25 g/L triglycerides and 1.5 mg/L human DNA invalid results could occur due to interference. Non-reactive results were obtained with cobas® HIV-1/HIV-2 Qualitative for all samples without HIV target and reactive results were obtained on all of the samples with HIV-1 and HIV-2 targets.

Table 31 Drug compounds tested for interference with the cobas® HIV-1/HIV-2 Qualitative

Class of drug	Generic drug name	
Immune Modulators	Peginterferon α-2a	Ribavirin
	Peginterferon α-2b	
HCV Inhibitors	Simeprevir	Sofosbuvir
Reverse Transcriptase or DNA Polymerase Inhibitors	Emtricitabine	Tenofovir
	Entecavir	Adefovir dipivoxil
	Foscarnet	Telbivudine
	Cidofovir	Aciclovir
	Lamivudine	Valganciclovir
	Ganciclovir	
Compounds for Treatment of Opportunistic Infections	Azithromycin	Pyrazinamide
	Clarithromycin	Rifabutin
	Ethambutol	Rifampicin
	Fluconazole	Sulfamethoxazole
	Isoniazid	Trimethoprim
Statin	Atorvastatin	
Selective Serotonin Reuptake Inhibitor	Fluoxetine	Paroxetine
	Sertraline	
Antihistamine	Loratadine	
Beta-blocker	Nadolol	
Decongestant	Phenylephrine HCl	
Nonsteroidal Anti-inflammatory drug	Naproxen	Ibuprofen
Pain reliever	Acetaminophen	Acetylsalicylic Acid
Vitamins	Ascorbic Acid	

Method Correlation

EDTA plasma and serum

The performance of cobas® HIV-1/HIV-2 Qualitative was compared to the COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0 (HIV-1 EDTA plasma samples) and to a CE-marked HIV-1/HIV-2 antibody differentiation test (HIV-1 and HIV-2 EDTA plasma and serum samples).

For the correlation to COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0, clinical EDTA plasma samples were analyzed at one external site. For HIV-1 positive and HIV-1 negative clinical EDTA plasma samples, a total percent agreement of 100% between the two tests was shown, demonstrating that the performance of cobas® HIV-1/HIV-2 Qualitative and COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0 is equivalent (Table 32).

Table 32 Summary of results for method correlation for HIV-1 EDTA plasma samples

Method Correlation HIV-1 EDTA plasma samples		COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0	
		Reactive	Non-reactive
cobas® HIV-1/HIV-2 Qualitative	Reactive	68	0
	Non-reactive	0	80

For the comparison to a CE-marked HIV-1/HIV-2 antibody differentiation test, clinical EDTA plasma or serum samples were analyzed at one external site. For HIV-1 positive and HIV-1 negative EDTA plasma or serum samples, a total percent agreement of 100% between the two tests was shown. For HIV-2 positive and HIV-2 negative EDTA plasma or serum samples, a total percent agreement of 99.7% between the two tests was shown. This demonstrates that the performance of cobas® HIV-1/HIV-2 Qualitative and a CE-marked HIV-1/HIV-2 antibody differentiation test is equivalent (Table 33 and Table 34).

Table 33 Summary of results for method correlation for HIV-1 EDTA plasma and serum samples

Method Correlation HIV-1 EDTA plasma and serum samples		CE-marked HIV-1/HIV-2 antibody differentiation test		
		Reactive	Non-reactive	Indeterminate
cobas® HIV-1/HIV-2 Qualitative	Reactive	138	0	0
	Non-reactive	0	164	1*

* The sample which showed a non-reactive result with cobas® HIV-1/HIV-2 Qualitative and an indeterminate result with the CE-marked HIV-1/HIV-2 antibody differentiation test was confirmed to be negative with an alternative CE-marked 4th generation HIV Ag/Ab serological test.

Table 34 Summary of results for Method Correlation for HIV-2 EDTA plasma and serum samples

Method Correlation HIV-2 EDTA plasma and serum samples		CE-marked HIV-1/HIV-2 antibody differentiation test		
		Reactive	Non-reactive	Indeterminate
cobas® HIV-1/HIV-2 Qualitative	Reactive	14	0	0
	Non-reactive	1	287	1*

* The sample which showed a non-reactive result with cobas® HIV-1/HIV-2 Qualitative and an indeterminate result with the CE-marked HIV-1/HIV-2 antibody differentiation test was confirmed to be negative with an alternative CE-marked 4th generation HIV Ag/Ab serological test.

DBS

The performance of cobas® HIV-1/HIV-2 Qualitative was compared to the COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0 by analysis of clinical early infant DBS samples at one external site. For HIV-1 positive and HIV-1 negative DBS samples, a total percent agreement of 99.6% between the two tests was shown, demonstrating that the performance of cobas® HIV-1/HIV-2 Qualitative and COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0 is equivalent (Table 35).

Table 35 Summary of results for Method Correlation for HIV-1 DBS samples

Method Correlation HIV-1 DBS samples		COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0	
		Reactive	Non-reactive
cobas® HIV-1/HIV-2 Qualitative	Reactive	127	1*
	Non-reactive	0	151

* The sample which showed a reactive result with cobas® HIV-1/HIV-2 Qualitative and a non-reactive result with the COBAS® AmpliPrep/COBAS® TaqMan® HIV-1 Qualitative Test, v2.0 was confirmed to be HIV-1 positive with nested PCR.

Whole system failure

The Whole System Failure rate for cobas® HIV-1/HIV-2 Qualitative was determined by testing 100 replicates of EDTA plasma and 100 replicates of whole blood for DBS spiked with both HIV-1 group M subtype B and HIV-2. These samples were tested at a target concentration of approximately 3 x LoD.

The results of this study determined that all replicates were valid and positive for the HIV-1 and HIV-2 targets, resulting in a Whole System Failure rate of 0% for EDTA plasma and DBS. The two-sided 95% exact confidence interval was 0% for the lower bound and 3.6% for the upper bound for each target and matrix.

Cross contamination

The cross-contamination rate for cobas® HIV-1/HIV-2 Qualitative was determined by testing 240 replicates of an HIV negative DBS sample and 225 replicates of a high titer HIV-1 DBS sample at 2.0E+07 cp/mL. The study was performed following the DBS sample preparation workflow. In total, five runs were performed with positive and negative samples in a checkerboard configuration.

All 240 replicates of the negative sample were non-reactive, resulting in a cross-contamination rate of 0%. The 95% confidence interval was 0% for the lower bound and 1.5% for the upper bound.

Clinical performance evaluation

Reproducibility

Reproducibility of cobas® HIV-1/HIV-2 Qualitative was evaluated in EDTA plasma across reagent lot, test site/instrument system, operator, days, batch, and within batch. Reproducibility testing was performed in three sites using three reagent lots, two cobas® 6800 Systems and one cobas® 8800 System, two operators over 6 days; three replicates of each panel member were performed for each batch. Each panel consisted of one negative panel member and six positive panel members. The negative percent agreement was estimated as 100%, with a corresponding 95% exact CI of (98.9%, 100.0%) and the positive percent agreement was 100% for each panel member for both HIV-1 and HIV-2.

For HIV-1 positive panel members, the coefficient of variation (CV(%)) for all panel members was $\leq 1.9\%$, demonstrating very low variability of cobas® HIV-1/HIV-2 Qualitative results across reagent lots, sites/instruments, days, operators, and batches.

Table 36 Attributable percentage of total variance, total precision Standard Deviation and CV(%) of cycle threshold Values from HIV-1 Reactive results with the cobas® HIV-1/HIV-2 Qualitative by positive HIV-1 panel member

Panel Member	N ^a	Lot ^b	Site ^b	Operator ^b	Day ^b	Batch ^b	Within-Batch ^b	Total Precision Standard Deviation ^c	Total Precision CV(%) ^d
~3 × LoD (3.78E1) HIV-1, Negative HIV-2	324	0.0% (0.0%)	0.0% (0.0%)	2.2% (0.3%)	6.6% (0.5%)	0.0% (0.0%)	91.1% (1.8%)	0.69	1.9%
>3 × LoD (1.00E5) HIV-1, Negative HIV-2	322	0.0% (0.0%)	15.5% (0.4%)	0.0% (0.0%)	33.1% (0.5%)	4.5% (0.2%)	46.9% (0.6%)	0.24	0.9%
>3 × LoD (1.00E5) HIV-1, ~3 × LoD (8.37E1) HIV-2	323	0.0% (0.0%)	7.8% (0.3%)	0.0% (0.0%)	45.0% (0.6%)	10.9% (0.3%)	36.3% (0.6%)	0.25	1.0%
~3 × LoD (3.78E1) HIV-1, >3 × LoD (1.00E5) HIV-2	323	1.5% (0.2%)	0.0% (0.0%)	0.0% (0.0%)	0.0% (0.0%)	0.0% (0.0%)	98.5% (1.8%)	0.67	1.8%

Note: The table only includes results with detectable analyte.

^a Number of valid tests with detectable analyte.

^b Cells within the columns for the lot, site, operator, day, batch, and within-batch components display the percentage of total variance (%) and within parentheses the percent coefficient of variation (CV%) of the component.

^c Calculated using the total variability from the SAS MIXED procedure.

^d CV(%) = (standard deviation / mean) * 100%.

For HIV-2 positive panel members, the coefficient of variation (CV(%)) for all panel members was $\leq 1.7\%$, demonstrating very low variability of cobas® HIV-1/HIV-2 Qualitative results across reagent lots, sites/instruments, days, operators, and batches.

Table 37 Attributable percentage of total variance, total precision Standard Deviation, and CV(%) of cycle threshold values from HIV-2 reactive results with the cobas® HIV-1/HIV-2 Qualitative by positive HIV-2 panel member

Panel Member	N ^a	Lot ^b	Site ^b	Operator ^b	Day ^b	Batch ^b	Within-Batch ^b	Total Precision Standard Deviation ^c	Total Precision CV(%) ^d
Negative HIV-1, ~3 × LoD (8.37E1) HIV-2	324	60.0% (1.3%)	3.3% (0.3%)	0.0% (0.0%)	14.6% (0.7%)	4.5% (0.0%)	17.6% (0.7%)	0.59	1.7%
Negative HIV-1, >3 × LoD (1.00E5) HIV-2	324	31.9% (0.9%)	10.0% (0.5%)	0.0% (0.0%)	32.9% (1.0%)	0.0% (0.0%)	25.1% (0.8%)	0.42	1.7%
>3 × LoD (1.00E5) HIV-1, ~3 × LoD (8.37E1) HIV-2	323	26.0% (0.7%)	4.0% (0.3%)	0.0% (0.0%)	9.9% (0.4%)	4.2% (0.3%)	55.9% (1.0%)	0.46	1.3%
~3 × LoD (3.78E1) HIV-1, >3 × LoD (1.00E5) HIV-2	323	38.2% (0.9%)	8.2% (0.4%)	0.0% (0.0%)	24.7% (0.8%)	0.0% (0.0%)	28.9% (0.8%)	0.39	1.5%

Note: The table only includes results with detectable analyte.

^a Number of valid tests with detectable analyte.

^b Cells within the columns for the lot, site, operator, day, batch, and within-batch components display the percentage of total variance (%) and within parentheses the percent coefficient of variation (CV) of the component.

^c Calculated using the total variability from the SAS MIXED procedure.

^d CV(%) = (standard deviation / mean) * 100%.

Clinical method comparison

HIV-1 and HIV-2 clinical sensitivity

The performance of cobas® HIV-1/HIV-2 Qualitative on the cobas® 6800/8800 Systems was compared to an alternative HIV-1 qualitative NAT or HIV-2 NAT in subjects known to be infected with Human Immunodeficiency Virus type 1 (HIV-1) or Human Immunodeficiency Virus type 2 (HIV-2).

Testing of specimens from HIV-1 infected individuals

Overall 1030 specimens were tested from subjects known to be HIV-1 positive with HIV-1 viral loads ≥ 100 copies/mL. Of 1030 specimens evaluable for statistical analysis, 537 (52.1%) were from female subjects, 752 (73.0%) were from African/African American subjects, and the median age of subjects was 37 years (range: 18-81 years). There were 736 HIV-1 B subtype specimens and 294 HIV-1 Non-B subtype specimens.

The HIV-1 sensitivity of cobas® HIV-1/HIV-2 Qualitative was 100% (1030/1030, 95% CI: 99.6% to 100%). The sensitivity is for samples with viral RNA concentrations equal to or greater than 100 copies/mL (Table 38). Similar performance was observed between plasma and serum specimens.

Table 38 HIV-1 Sensitivity of cobas® HIV-1/HIV-2 Qualitative for the HIV-1 known positive population

Population, Specimen Type	Total Known Positive Specimens	Number Reactive by Test	HIV-1 Sensitivity Estimate	95% Exact CI
HIV-1, Overall	1030	1030	100%	(99.6%, 100%)
HIV-1, Plasma	712	712	100%	(99.5%, 100%)
HIV-1, Serum	318	318	100%	(98.8%, 100%)

Testing of specimens from HIV-2 infected individuals

Overall, 183 specimens from subjects known to be HIV-2 positive with HIV-2 viral loads ≥ 100 copies/mL by HIV-2 Plasma RNA Quantitative Assay were tested. Of the 183 evaluable specimens, 92 (50.3%) were from male subjects, all were African subjects, and the median age of subjects was 50 years (range: 23-77 years).

The HIV-2 sensitivity of cobas® HIV-1/HIV-2 Qualitative was 99.5% (182/183, 95% CI: 97.0% to 99.99%) (Table 39). The sensitivity is for samples with viral RNA concentrations equal to or greater than 100 copies/mL. Similar performance was observed between plasma and serum specimens.

Table 39 HIV-2 Sensitivity of cobas® HIV-1/HIV-2 Qualitative for the HIV-2 known positive population

Population, Specimen Type	Total Known Positive Specimens	Number Reactive by Test	HIV-2 Sensitivity Estimate	95% Exact CI
HIV-2, Overall	183	182	99.5%	(97.0%, 99.99%)
HIV-2, Plasma	115	115	100.0%	(96.8%, 100%)
HIV-2, Serum	68	67	98.5%	(92.1%, 99.96%)

Clinical specificity

The clinical specificity of cobas® HIV-1/HIV-2 Qualitative on the cobas® 6800/8800 Systems was determined in comparison to a clinical algorithm using two immunoassays followed by NAT testing to resolve indeterminate results in an HIV low-risk population (US Centers For Disease Control and Prevention HIV Laboratory Testing Algorithm). The HIV low-risk population consisted of 1988 healthy blood donors and 3929 low-risk subjects from an area with less than 1% HIV prevalence. Of 5917 specimens evaluable for statistical analysis, 3301 (55.8%) were from female subjects, 3942 (66.6%) were from white/Caucasian subjects, and the median age of subjects was 36 (range: 17-92 years).

The overall HIV-1 specificity and HIV-2 specificity of cobas® HIV-1/HIV-2 Qualitative was 100%, with no difference between plasma and serum specimens (Table 40).

Table 40 Specificity of cobas® HIV-1/HIV-2 Qualitative by target analyte in the HIV low-risk population

Target Analyte, Specimen Type	Total cobas® HIV-1/HIV-2 Qualitative Nonreactive Subjects	Status-Negative Subjects by CDC HIV Testing Algorithm	Specificity Estimate	95% Exact CI
HIV-1, Overall ^a	5902	5902	100%	(99.94%, 100%)
HIV-1, Plasma ^a	3608	3608	100%	(99.90%, 100%)
HIV-1, Serum ^a	2294	2294	100%	(99.84%, 100%)
HIV-2, Overall ^b	5914	5914	100%	(99.94%, 100%)
HIV-2, Plasma ^b	3618	3618	100%	(99.90%, 100%)
HIV-2, Serum ^b	2296	2296	100%	(99.84%, 100%)

^a Fifteen specimens that were HIV-1 positive by the CDC HIV testing algorithm were not included in the specificity analysis for the HIV-1 target (10 were plasma and 5 were serum specimens).

^b Three serum specimens were not included because the CDC HIV testing algorithm result was not obtained due to insufficient specimen volume for HIV-2 NAT testing.

Prospective study of individuals at high risk for HIV-1 infection

The performance of cobas® HIV-1/HIV-2 Qualitative was compared to an alternative HIV-1 qualitative NAT in subjects at high risk for HIV-1 infection. All 519 specimens from subjects at high risk for HIV-1 infection were from subjects with at least 1 risk factor for HIV infection (injection drug user (current or ever), unprotected sex with an HIV-infected person (current or ever), diagnosed with sexually transmitted disease within the last year, multiple sex partners (more than 1 partner in the last 12 months), MSM (current or ever), unprotected sex with a person diagnosed with a sexually transmitted disease within the last year). Of the 519 evaluable specimens, 345 (66.5%) were from male subjects, 289 (55.7%) were from African/African American subjects, and the median age of subjects was 39 years (range: 18-80 years).

The HIV-1 PPA of cobas® HIV-1/HIV-2 Qualitative was 100% (5/5), and the HIV-1 NPA was 100% (514/514, 95% CI: 99.3% to 100%). Similar performance was observed for plasma and serum specimens.

Prospective study of individuals at high risk for HIV-2 infection

The performance of cobas® HIV-1/HIV-2 Qualitative was compared to a clinical algorithm using two immunoassays followed by NAT testing to resolve indeterminate results in subjects at high risk for HIV-2 infection (U.S. Centers For Disease Control and Prevention HIV Laboratory Testing Algorithm).

All specimens from subjects at high risk for HIV-2 infection were from an HIV-2 endemic area of West Africa (i.e., Guinea-Bissau, Cameroon, Cote d'Ivoire). Of 499 evaluable specimens, 366 (73.3%) were from male subjects, all subjects were African, and the median age of subjects was 28 years (range: 19-66 years).

The PPA of cobas® HIV-1/HIV-2 Qualitative for HIV-1 in the HIV-2 High-risk population was 79.0% (79/100; 95% CI: 69.7%-86.5%), and the NPA was 99.5% (395/397; 95% CI: 98.2%-99.9%). The PPA of cobas® HIV-1/HIV-2 Qualitative for HIV-2 in the HIV-2 High-risk population was 44.4% (4/9; 95% CI: 13.7%-78.8%), and the NPA was 100% (490/490; 95% CI: 99.2%-100%). Similar performance was observed for plasma and serum specimens.

System equivalency / system comparison

System equivalency of the cobas® 5800, cobas® 6800 and cobas® 8800 Systems was demonstrated via performance studies. The results presented in the Instructions for Use support equivalent performance for all systems.

Additional information



















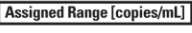



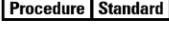




























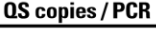
Key test features

Sample type	EDTA plasma, serum and dried blood spot (DBS)		
Minimum amount of sample required	650 µL for EDTA plasma and serum samples or one DBS sample (70 µL dried blood per spot) in 1150 µL cobas® Specimen Pre-Extraction Reagent (SPER)		
Sample process volume	500 µL for EDTA plasma and serum samples or 850 µL for DBS samples		
Analytical sensitivity		<u>HIV-1M</u>	<u>HIV-2</u>
	EDTA plasma	12.6 cp/mL	27.9 cp/mL
	Serum	12.1 cp/mL	23.4 cp/mL
	DBS	255 cp/mL	984 cp/mL
Specificity	100% (one-sided 95% confidence interval: 99.5%) (EDTA plasma/serum) 100% (one-sided 95% confidence interval: 99.5%) (DBS)		
Groups/subtypes – inclusivity	HIV-1 M (A–D, F–H, J, K, CRF01_AE, CRF02_AG, CRF12_BF, CRF14_BG), HIV-1 O, HIV-1 N, HIV-2 (A and B)		

Symbols

The following symbols are used in labeling for Roche PCR diagnostic products.

Table 41 Symbols used in labeling for Roche PCR diagnostics products

 Age/DOB	Age or Date of Birth		Device not for near-patient testing		QS IU per PCR reaction, use the QS International Units (IU) per PCR reaction in calculation of the results.
	Ancillary Software		Device not for self-testing		Serial number
	Assigned Range (copies/mL)		Distributor <i>(Note: The applicable country/region may be designated beneath the symbol)</i>		Site
	Assigned Range (IU/mL)		Do not re-use		Standard Procedure
	Authorized representative in the European Community		Female		Sterilized using ethylene oxide
	Barcode Data Sheet		For IVD performance evaluation only		Store in dark
	Batch code		Global Trade Item Number		Temperature limit
	Biological risks		Importer		Test Definition File
	Catalogue number		In vitro diagnostic medical device		This way up
	CE marking of conformity; this device is in conformity with the applicable requirements for CE marking of an in vitro diagnostic medical device		Lower Limit of Assigned Range		Ultrasensitive Procedure
	Collect date		Male		Unique Device Identifier
	Consult instructions for use		Manufacturer		Upper Limit of Assigned Range
	Contains sufficient for <n> tests		Negative control		Urine Fill Line
	Content of kit		Non-sterile		US Only: Federal law restricts this device to sale by or on the order of a physician.
	Control		Patient Name		Use-by date
	Date of manufacture		Patient number		
	Device for near-patient testing		Peel here		
	Device for self-testing		Positive control		
			QS copies per PCR reaction, use the QS copies per PCR reaction in calculation of the results.		

Technical Support

For technical support (assistance), please reach out to your local affiliate:
https://www.roche.com/about/business/roche_worldwide.htm

Manufacturer and importer

Table 42 Manufacturer and importer



Roche Molecular Systems, Inc.
1080 US Highway 202 South
Branchburg, NJ 08876 USA
www.roche.com

Made in USA



Roche Diagnostics GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany

Trademarks and patents

See <https://diagnostics.roche.com/us/en/about-us/patents>

Copyright

©2022 Roche Molecular Systems, Inc.



Roche Diagnostics GmbH
Sandhofer Str. 116
68305 Mannheim
Germany



References

1. American Academy of HIV Medicine. *Fundamentals of HIV Medicine for the HIV Specialist*. Washington, D.C.: American Academy of HIV Medicine, 2007.
2. UNAIDS. Global AIDS Update. Available at: https://www.unaids.org/sites/default/files/media_asset/global-AIDS-update-2016_en.pdf. Accessed December 6, 2020.
3. Hoover DR, Saah AJ, Bacellar H, et al. Clinical manifestations of AIDS in the era of pneumocystis prophylaxis. Multicenter AIDS Cohort Study. *N Engl J Med*. 1993;329:1922-6. PMID: 7902536.
4. Campbell-Yesufu OT, Gandhi RT. Update on human immunodeficiency virus (HIV)-2 infection. *Clin Infect Dis*. 2011;52:780-7. PMID: 21367732.
5. Centers for Disease Control and Prevention. Laboratory Testing for the Diagnosis of HIV Infection. Available at: <https://stacks.cdc.gov/view/cdc/23447>. Accessed December 6, 2020.
6. Gökengin D, Geretti AM, Begovac J, et al. 2014 European Guideline on HIV testing. *Int J STD AIDS*. 2014;25:695-704. PMID: 24759563.
7. Masciotra S, McDougal JS, Feldman J, et al. Evaluation of an alternative HIV diagnostic algorithm using specimens from seroconversion panels and persons with established HIV infections. *J Clin Virol*. 2011;52 Suppl 1:S17-22. PMID: 21981983.
8. O'Brien M, Markowitz M. Should we treat acute HIV infection? *Curr HIV/AIDS Rep*. 2012;9:101-10. PMID: 22415472.
9. Branson BM, Mermin J. Establishing the diagnosis of HIV infection: new tests and a new algorithm for the United States. *J Clin Virol*. 2011;52 Suppl 1:S3-4. PMID: 21993308.
10. Cohen MS, Shaw GM, McMichael AJ, Haynes BF. Acute HIV-1 Infection. *N Engl J Med*. 2011;364:1943-54. PMID: 21591946.
11. World Health Organization. WHO Recommendations on the Diagnosis of HIV Infection in Infants and Children. Available at: <https://www.who.int/hiv/pub/paediatric/diagnosis/en/>. Accessed December 6, 2020.
12. Wittek M, Stürmer M, Doerr HW, Berger A. Molecular assays for monitoring HIV infection and antiretroviral therapy. *Expert Rev Mol Diagn*. 2007;7:237-46. PMID: 17489731.
13. Longo MC, Berninger MS, Hartley JL. Use of uracil DNA glycosylase to control carry-over contamination in polymerase chain reactions. *Gene*. 1990;93:125-8. PMID: 2227421.
14. Savva R, McAuley-Hecht K, Brown T, Pearl L. The structural basis of specific base-excision repair by uracil-DNA glycosylase. *Nature*. 1995;373:487-93. PMID: 7845459.
15. Mol CD, Arvai AS, Slupphaug G, et al. Crystal structure and mutational analysis of human uracil-DNA glycosylase: structural basis for specificity and catalysis. *Cell*. 1995;80:869-78. PMID: 7697717.
16. Higuchi R, Dollinger G, Walsh PS, Griffith R. Simultaneous amplification and detection of specific DNA sequences. *Biotechnology (N Y)*. 1992;10:413-7. PMID: 1368485.
17. Heid CA, Stevens J, Livak KJ, Williams PM. Real time quantitative PCR. *Genome Res*. 1996;6:986-94. PMID: 8908518.

18. Centers for Disease Control and Prevention. Biosafety in Microbiological and Biomedical Laboratories, 5th ed. <https://www.cdc.gov/labs/pdf/CDC-BiosafetyMicrobiologicalBiomedicalLaboratories-2009-P.PDF>. Accessed December 2, 2020.
19. Clinical and Laboratory Standards Institute. Protection of laboratory workers from occupationally acquired infections. Approved Guideline-Fourth Edition. https://clsi.org/media/1459/m29a4_sample.pdf. Accessed December 2, 2020.

Document revision

Document Revision Information	
Doc Rev. 3.0 09/2022	Updated front page and Table 2 and 3 with additional P/N for the control kits. Updated Trademarks and patents section, including the link. Please contact your local Roche Representative if you have any questions.