WHO Prequalification of In Vitro Diagnostics PUBLIC ASSESSMENT REPORT

Product: STANDARD Q Malaria P.f/Pan Ag Test
WHO reference number: PQDx 0347-117-00

STANDARD Q Malaria P.f/Pan Ag Test with product code 09MAL30D, manufactured by SD Biosensor, Inc., CE-Mark regulatory version, was accepted for the WHO list of prequalified in vitro diagnostics and was listed on 6 March 2020.

Summary of WHO prequalification assessment for STANDARD Q Malaria P.f/Pan Ag Test

	Date	Outcome
Prequalification listing	6 March 2020	listed
Dossier assessment	31 January 2019	MR
Product performance evaluation	2020	MR

MR: Meets Requirements

Report amendments and/or product changes

This public report has since been amended. Amendments may have arisen because of changes to the prequalified product for which the WHO has been notified and has undertaken a review. Amendments to the report are summarized in the following table, and details of each amendment are provided below.

Public report amendment	Summary of amendment	Date of report amendment
2.0	1. Change of IFU from IFU in English to multilanguage IFU (EN, ES, FR, PT). 2. Change of outer package.	18 May 2021.
3.0	1. Addition of manufacturer site for Cutting, Assembly, Buffer Preparation & dispensing, sealing, printing, packing and shipping process. 2. Addition of suppliers for raw materials. (1) Upper device, (2) Lower device, (3) Aluminum Pouch, (4) Inverted cup, (5) Lancet, (6) Alcohol swab". 3. Updated lancet label.	19 February 2024.
4.0	The manufacturing site is located in the public report of Plot no. 38, Sector 4, IMT Manesar, Gurugram, Haryana 122052, India.	23 April 2024
5.0	Updated labels of inverted cup, sterile lancet and alcohol swab.	5 August 2025

6.0	<ol style="list-style-type: none"> 1. Updated the PKG label, adding the "M" mark for better identification. 2. Established the PKG label, including the print area for the procurement. 3. Labelling for the sterile lancet has been updated (manufacturer's address, latest European Representative symbol, KGMP symbol and Korean language addition) (PQC-IVD-2026-0030). 	23 April 2026
-----	--	---------------

Intended use

According to the claim of intended use from SD Biosensor, Inc, "*STANDARD Q Malaria P.f/Pan Ag Test is a rapid and membrane based immunochromatography for the qualitative detection of Plasmodium falciparum (P. falciparum) specific Histidine Rich Protein 2 (HRP-2) and Plasmodium species (P. falciparum, vivax, ovale and malariae) specific Plasmodium lactate dehydrogenase (pLDH) in human capillary and venous whole blood specimens of patients suspected of having malaria. STANDARD Q Malaria P.f/Pan Ag Test is intended to be used by trained healthcare or laboratory professionals or other health care workers who have received appropriate training. This product can be used by trained lay providers operating in point-of-care settings in resource-limited lower- and middle-income countries. This product is not intended for self-testing.*"

Test kit contents

Component	25 tests (product code 09MAL30D)
Test device (individually in a foil pouch with desiccant)	25
Buffer Bottle	1 x 4 mL
Inverted cup (5µL)	Pack of 25
Sterile lancet	Pack of 25
Alcohol swab	Pack of 25
Instructions for use	1

Items required but not provided

- Anti-coagulant tube containing heparin, EDTA or sodium citrate for collection of venous whole blood
- Micropipette and tip
- Timer
- PPE (Personal Protective Equipment)
- Pen/pencil
- Extra lancets and alcohol swabs
- Biosafety sharps container
- Biohazard container

- Sterile gauze

Storage

The test kit must be stored between 2 and 40°C.

Shelf-life upon manufacture¹

24 months.

Dossier assessment

SD Biosensor, Inc. submitted a product dossier for STANDARD Q Malaria P.f/Pan Ag Test as per the *“Instructions for compilation of a product dossier”* (PQDx_018 version 3). The information (data and documentation) submitted in the product dossier was reviewed by WHO staff and external technical experts (assessors) appointed by WHO.

The manufacturer's responses to the discrepancies found during dossier screening and assessment findings were accepted on 31 January 2019.

Based on the product dossier screening and assessment findings, the product dossier for the STANDARD Q Malaria P.f/Pan Ag Test meets WHO prequalification requirements.

Manufacturing site inspection

The inspection of the manufacturing site(s) was conducted to assess whether the manufacturer's quality management system (QMS) and manufacturing practices are in alignment with:

- applicable international standards, such as ISO 13485 (Medical devices – Quality management systems – Requirements for regulatory purposes);
- the manufacturer's own documented procedures and quality requirements; and
- other relevant international standards and guidelines applicable to in vitro diagnostic (IVD) medical devices. The WHO's Public Inspection Reports are accessible at:

<https://extranet.who.int/pqweb/vitro-diagnostics/who-public-inspection-reports>

Based on the site inspection and corrective action plan review, the quality management system for the STANDARD Q Malaria P.f/Pan Ag Test meets WHO prequalification requirements.

¹ The assigned device shelf-life is based on stability data generated from the date of manufacture. The finished goods shelf-life, calculated from the date of packaging completion, may be shorter depending on the time elapsed between manufacture and final packaging of the device.

Product performance evaluation

STANDARD Q Malaria P.f/Pan Ag Test² was included in the eighth round of WHO product testing for RDTs for malaria antigen detection, completed in 2018. The product was evaluated against a *Plasmodium falciparum* cultured line panel, *P. falciparum* wild-type parasite panel, *P. vivax* wild-type parasite panel and a negative panel. Thermal stability was assessed after 2 months of storage at elevated temperature and humidity, and a descriptive ease-of-use assessment was recorded.

Based on the demonstrated *P. falciparum* panel detection score (88.0% at 200 parasites/ μ L), *P. vivax* panel detection score (100% at 200 parasites/ μ L), false-positive rates (0% for clean negatives, 0% for *P. falciparum* at 200 parasites/ μ L, 0% for *P. vivax* at 200 parasites/ μ L, 0.5% for *P. falciparum* at 2000 parasites/ μ L, 0% for *P. vivax* at 2000 / μ L) and invalid rate (0%), STANDARD Q Malaria P.f/Pan Ag Test meets the current laboratory evaluation requirements for prequalification.

Summary performance characteristics	Panel detection score		False positive rate (%)			Invalid rate (%)
	200 parasites/ μ L		200 parasites/ μ L	Clean negatives		
	Pf	Pv	Pf	Pv		
STANDARD Q Malaria P.f/Pan Ag Test	88.0	100	0	0	0	0

Based on these results, the STANDARD Q Malaria P.f/Pan Ag Test meets the current performance evaluation requirements for prequalification.

Labelling review

The labelling submitted for the STANDARD Q Malaria P.f/Pan Ag Test assay was reviewed by WHO staff and external technical experts appointed by WHO. The review evaluated the labelling for clarity and consistency with the information submitted in the product dossier, alignment with international guidance and standards, and suitability for the intended users and settings in WHO Member States, including low- and middle-income countries.

The table below provides traceability of the labelling documents reviewed during the assessment, including document titles, version numbers, approval dates, and control identifiers.

² The product code of the product included in the WHO product testing of malaria RDTs was 09MAL30B, which corresponded to the same product, with a different kit configuration (i.e. number of tests) <https://www.who.int/malaria/publications/atoz/9789241514965/en/>

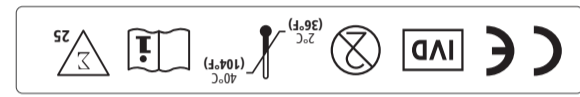
Controlled Labelling References

Document Type	Document Title	Version / Revision	Date Approved	Controlled Document No.
Outer box artwork	Q Malaria P.f./Pan Ag_PKG	R5	2025.10	B25MAL3ML4R5-WHO
	Q Malaria P.f./Pan Ag_PKG	R2	2025.10	INO25MAL3ML4R2-WHO
	Q Malaria P.f./Pan Ag_PKG	R1	2025.10	INXB25MAL3ML4R1-WHO
	Q Malaria P.f./Pan Ag_PKG	R0	2025.10	PB25MAL3ML4R0-WHO
	Q Malaria P.f./Pan Ag_PKG	R0	2025.10	PINO25MAL3ML4R0-WHO
	Q Malaria P.f./Pan Ag_PKG	R0	2025.10	PINXB25MAL3ML4R0-WHO
Pouch / Device label	Device pouch	R3	2022.05	ML25RT7MLR3
	Device pouch	R0	2022.04	ML25RT13ENR0
Reagent bottle labels	Buffer bottle_Korea	R1	2017.10	L36RT1ENR1
	Buffer bottle_India	R0	2023.12	L35RT0INR0
Accessory labelling	Inverted cup_Korea	R0	N/A	P000902502REV.0
	Inverted cup_India	R2	2020.02	L15IC1ENR2
	Sterile Lancet_Tianjin Huahong	R0	2020.07	B05SL2ENR0
	Sterile Lancet_RMS	R4	2025.06	B06LCT2ENR4
	Sterile Lancet_Phoenix	N/A	2023.09	N/A
	Sterile Lancet_iCare	N/A	2023.09.15	IFU-02-BL-01-NO.256
Instructions for Use (IFU)	IFU	R4	2023.10	L23MAL3ML4R4-WHO
	IFU	R5	2023.10	L23MAL3INR5-WHO
	IFU	R0	2023.11	L23MMAL3INR0-WHO

Labels

SD BIOSENSOR

www.sdbiosensor.com

- CONTENTS
- 1) Test device (individually in a foil pouch with desiccant) x 25
 - 2) Inverted cup (5 µl) x 25
 - 3) Buffer bottle x 1
 - 4) Sterile lancet x 25
 - 5) Alcohol swab x 25
 - 6) Instructions for use x 1

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STANDARD Q Malaria P.f/Pan Ag Test

Manufactured by
SD Biosensor, Inc.

Head office
C-4thA5th, 16, Deogyong-daero 155beon-gil, Yeongtong-gu,
Seonsi, Gyeonggi-do, 16994, REPUBLIC OF KOREA

Manufacturing site
74, Osonggongnyong 4-ro, Osong-eup, Heungdeok-gu,
Cheongju-si, Chungcheongbuk-do, 28161, REPUBLIC OF KOREA

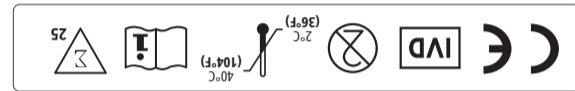
Authorized Representative

EC REP
MT Promed Consulting GmbH
Ernst-Heckel-Strasse 7 66386 St. Ingbert Germany
Phone : +49 6894 581020, Fax : +49 6894 581021

STANDARD

SD BIOSENSOR

www.sdbiosensor.com

CONTENTS
1) Test device (individually in a foil pouch with desiccant) x 25
2) Inverted cup (5 µl) x 25
3) Buffer bottle x 1
4) Sterile lancet x 25
5) Alcohol swab x 25
6) Instructions for use x 1

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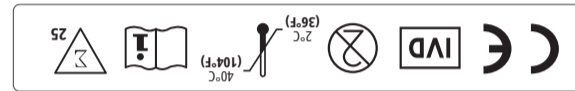
Manufacturing site
Plot No.38, Sector 4, IMT Manesar, Gurgaon, Haryana 122052, INDIA
Manufacturing License No.: MF/GMD/2018/00005

EC REP MT Promedix Consulting GmbH
Ernst-Heckel-Strasse 7 66386 St. Ingbert Germany
Phone : +49 6894 581020, Fax : +49 6894 581021

STANDARD

SD BIOSENSOR

www.sdbiosensor.com

CONTENTS
1) Test device (individually in a foil pouch with desiccant) x 25
2) Inverted cup (5 µl) x 25
3) Buffer bottle x 1
4) Sterile lancet x 25
5) Alcohol swab x 25
6) Instructions for use x 1

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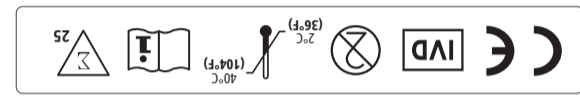
STANDARD Q Malaria P.f/Pan Ag Test

Manufactured by
SD Biosensor, Inc.

Head office
C-4thASh, 16, Deogyong-daero 155beon-gil, Yeongtong-gu,
Seonsi, Gyeonggi-do, 16994, REPUBLIC OF KOREA

Manufacturing site
74, Osongsaenggyeong 4-ro, Osong-eup, Heungdeok-gu,
Cheongju-si, Chungcheongbuk-do, 28161, REPUBLIC OF KOREA

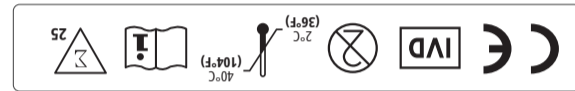
Authorized Representative

EC REP
MT Promed Consulting GmbH
Ernst-Heckel-Strasse 7 66386 St. Ingbert Germany
Phone : +49 6894 581020, Fax : +49 6894 581021

STANDARD

SD BIOSENSOR

www.sdbiosensor.com

- 1) Test device (individually in a foil pouch with desiccant) x 25
- 2) Inverted cup (5 µl) x 25
- 3) Buffer bottle x 1
- 4) Sterile lancet x 25
- 5) Alcohol swab x 25
- 6) Instructions for use x 1

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25 PCS
28G

INTENDED USE

To obtain a capillary blood specimen from the fingertip.

INSTRUCTIONS FOR USE

To use, twist-off the protective cap.

CAUTION

The lancet is guaranteed sterile while protective cap is sealed to the base.

Do not use if the seal has been damaged or broken.

B05SL2ENR0

Issue Date : 2020.07

Manufactured by

Tianjin Huahong Technology Co., Ltd.
A01, Plant B No. 278, Hangkong Road,
Tianjin Pilot Free Trade Zone
(Air Port Industrial Park),
300308 Tianjin, China

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To use, twist-off the protective cap.

CAUTION

The lancet is guaranteed sterile while protective cap is sealed to the based.
Do not use if the seal has been damaged or broken.

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 Manufactured by
Beijing Ruicheng Medical Supplies Co., Ltd.
Building 5, No.8 Yanqi West Road, Yanqi
Economic Development Zone, Huiairou
District 101407 Beijing P.R. China

Authorized Representative
Lotus Nl B.V.
Koningin Julianaplein 10, 1e Verd,
2595AK The Hague, Netherlands.
Tel: +31644168999
Email: info@lotusnl.com

BR01CT28N04
Issue date: 2023.06

SURFACE JOB

110MM

220MM

110MM

100MM

Disposable Sterile Lancets

Product code : SM.01

 LOT No. : XXXXXXXX

 MFG Date : YYYY-MM

 EXP Date : YYYY-MM

25PCS
28G

INTENDED USE

To obtain a capillary blood specimen from the fingertip.

INSTRUCTIONS FOR USE

To use, twist-off the protective cap.

Issue Date : 2023.09

CAUTION

The lancet is guaranteed sterile while protective cap is sealed to the base.

Do not use if the seal has been damaged or broken.

 Manufactured by

 Authorized Representative

Phoenix Innovative Healthcare Manufacturers Pvt. Ltd.
EL-209, Shil Mahape Road, Electronic Zone, MIDC,
TTC Industrial Area, Mahape, Navi Mumbai - 400 710,
Maharashtra, India

Med Path Gm6H
Miles-van-der-Rohe-Strasse 8,80807
Munich, Germany

CE 0197

STERILE

25 Pcs.

size: 95mm x 100mm No.256

95mm

Blood Lancet

 LOT No. : 240311

Product code : BL-28G

 MFG Date : 2024-03-11

25 PCS

 EXP Date : 2029-02-11

28G

INTENDED USE

To obtain the capillary blood specimen for test.

INSTRUCTIONS FOR USE

To use, twist-off the protective cap.

IFU-02-BL-01-NO.256

Issue Date: 2023-09-15

CAUTION

The lancet is guaranteed sterile while protective cap is sealed to the base.

Do not use if the seal has been damaged or broken.

Manufactured by

Guangzhou iCare Medical Technology Co., Ltd.
First floor A No.8, Lianhua Port Industrial Zone,
Lotus Mountain Bonded Area, Shilou Town,
Panyu District , Guangzhou,
People's Republic of China.

Tel: +86-020-31169913 Fax: +86-020-84881433

Authorized Representative

Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany

Tel: +49-40-2513175 Fax: +49-40-2513175

100mm

0123

STERILE R

STANDARD Q Malaria P.f/Pan Ag Test

Product Code: 510BA03


Buffer bottle

LOT XXXXXX

YYYY.MM.DD.

Vol : 4ml

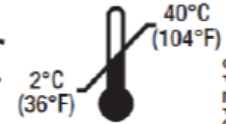YYYY.MM.DD.

L36RT1ENR1
Issue date : 2017.10


STANDARD Q Malaria P.f/Pan Ag Test

Product Code: R/MALAG

Buffer bottle

LOT XXXXXXXXXXXX

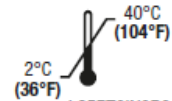YYYY.MM.DD.

YYYY.MM.DD.

L35RTQINORO
Issue date : 2023.12

Instructions for use³

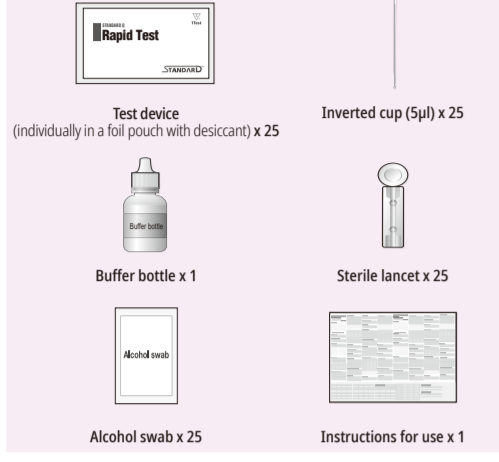
³ English version of the IFU was the one that was assessed by WHO. It is the responsibility of the manufacturer to ensure correct translation into other languages.

STANDARD Q Malaria P.f/Pan Ag
STANDARD™ Q Malaria P.f/Pan Ag Test

PLEASE READ INSTRUCTIONS CAREFULLY BEFORE YOU PERFORM THE TEST **SD BIOSENSOR**

KIT CONTENTS

MATERIALS REQUIRED BUT NOT PROVIDED

- Anti-coagulant tube containing heparin, EDTA or sodium citrate for collection of venous whole blood
- Micropipette and tip
- Timer
- PPE (Personal Protective Equipment)
- Pen/pencil
- Extra lancets and alcohol swabs
- Biosafety sharps container
- Biohazard container
- Sterile gauze

SPECIMEN COLLECTION AND PREPARATION

Capillary whole blood

- Capillary whole blood should be collected aseptically by fingerstick.
- Select the finger that is not calloused and clean the fingertip by wiping with an alcohol swab.
- Squeeze the end of the fingertip and pierce with a sterile lancet.
- Wipe the first drop with a sterile gauze.
- Collect capillary whole blood and it must be tested immediately after collection.

Venous whole blood

- Collect venous whole blood into a commercially available anti-coagulant tube such as heparin, EDTA, or sodium citrate by venipuncture.
- If venous whole blood in an anti-coagulant tube is stored in a refrigerator at 2-8°C/36-46°F, the specimen can be used for testing within 72 hours after collection.
- Do not use hemolyzed blood specimen.

CAUTION

Anticoagulants such as heparin, EDTA, or sodium citrate do not affect the test result.

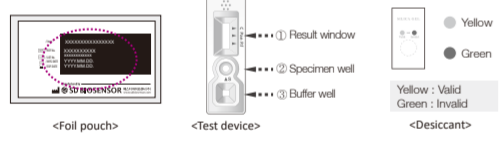
As known relevant interference, hemolytic specimen, rheumatoid factors-containing specimen and lipemic, icteric specimen may lead to inaccurate results.

Use separate inverted cup for each specimen in order to avoid cross-contamination of either specimens which can cause erroneous results.

PREPARATION AND TEST PROCEDURE

Preparation

- Carefully read the instructions for using the STANDARD Q Malaria P.f/Pan Ag Test.
- Look at the expiry date at the back of the foil pouch. Use another lot, if expiry date has passed.
- Open the foil pouch, and check the test device and the desiccant inside the foil pouch.

Test Procedure

Capillary and venous whole blood can be used for the STANDARD Q Malaria P.f/Pan Ag Test.

1. Test with a sterile lancet

- Select the finger that is not calloused and clean the fingertip by wiping with an alcohol swab.
- Dry and pierce the wiped fingertip with a sterile lancet to bleed.

- Wipe away the first drop of blood with sterile gauze.
- Take the inverted cup(5µl) provided and collect 5µl of the capillary whole blood specimen by dipping the circular end of the inverted cup into the specimen.

- To apply the collected capillary whole blood specimen to the specimen well(circle) of the test device, place the circular end of the inverted cup on the specimen well pad and then press down lightly.

1-1. Test by venipuncture

- Gently invert 4-5 times for proper mixing.

- Using a micropipette, draw 5µl of venous whole blood specimen.

- Apply the collected venous whole blood specimen to the specimen well(circle) of the test device.

2. Dropping of Buffer

- Hold the buffer bottle at 90° (perpendicular) to the test device and not touching the specimen well to avoid contamination. Add 3 drops (90µl) of the buffer into the buffer well(square) of the test device.

3. Reading Time

- Read the test result in 15 minutes. The test can be read up to 30 minutes.

CAUTION

- Do not read test results after 30 minutes. It may give false results.
- If the T line does not appear until 15 minutes, make sure to observe the appearance of the T line until 30minutes.

INTERPRETATION OF TEST RESULT

* "C" Control Line | "Pan" (P.f., P.v., P.o. and P.m.) Line | "P.f." P.f Line

Negative

The presence of only one colored band ("C" Control line) within the result window indicates a negative result.

Positive

P.f. Positive or mixed positive of P.v., P.o., and/or P.m. with P.f.

P.f. Positive

P.f., P.v., P.o., or P.m. positive or mixed positive of these

CAUTION

The presence of any line, no matter how faint the result is considered positive.

Invalid

If the control band ("C" Control line) is not visible within the result window, the result is considered invalid. The directions may not have been followed correctly or the test device may have deteriorated. Re-test with a new specimen and a new test device.

*NOTE • P.f.: *Plasmodium falciparum* • P.v.: *Plasmodium vivax*, • P.o.: *Plasmodium ovale*, • P.m.: *Plasmodium malariae*

- A colored band will appear in the top section of the result window to show that the test is working properly. This band is the control line (C).
 - Colored bands will appear in the middle and lower section of the result window. These bands are the *Plasmodium falciparum*, *vivax*, *ovale* and *malariae* line (Pan) and the *Plasmodium falciparum* line (P.f.) respectively.
- *This test only indicates the presence of Histidine Rich Protein-2 (HRP-2) of *Plasmodium falciparum* and *Plasmodium lactate dehydrogenase (pLDH) of Plasmodium species (Plasmodium falciparum, vivax, ovale, and malariae)* in human capillary and venous whole blood specimen and should not be used as the sole criteria for the diagnosis of malaria. As other diagnostic tests, all test results should be considered with other clinical history available.**

EXPLANATION AND SUMMARY

Introduction

Malaria remains an important cause of illness and death in children and adults in countries in which it is endemic. Globally, an estimated 3.2 billion people in 97 countries and territories are at risk of being infected with malaria and developing disease, and 1.2 billion are at high risk (>1 in 1,000 chance of getting malaria in a year). According to the World Malaria Report 2015, there were 210 million cases of malaria globally in 2015 (uncertainty range 149-303 million) and 438,000 malaria deaths (range 236,000-635,000), representing a decrease in malaria cases and deaths of 37% and 60% since 2000, respectively. The protocol parasites that cause malaria are from *Plasmodium falciparum*, *vivax*, *ovale* and *malariae* with the first two species causing the most infections worldwide. Classic symptoms of malaria include fever, headache, chills, vomiting, shivering and convulsions. In some rare forms of *P. falciparum*, the patient may present with delirium or coma. Severe anemia is often attributed to the cause of death from malaria. Accurate and prompt diagnosis of malaria is of utmost importance due to the morbidity associated with the other malarial forms. Rapid diagnostic test is an ideal diagnostic tool for malaria diagnosis in that it can provide a rapid determination if the patient is infected with malaria allowing for accurate treatment and improved outcomes. STANDARD Q Malaria P.f/Pan Ag Test, a reliable and sensitive screening test, would enhance the accuracy of the diagnosis of malaria infection and thus make clinical treatment decision effectively.

NOTE

P.f.: *Plasmodium falciparum*
P.v.: *Plasmodium vivax*
P.o.: *Plasmodium ovale*
P.m.: *Plasmodium malariae*

Intended use

STANDARD Q Malaria P.f/Pan Ag Test is a rapid and membrane based immunochromatography for the qualitative detection of *Plasmodium falciparum* (*P. falciparum*) specific Histidine Rich Protein 2 (HRP-2) and *Plasmodium* species (*P. falciparum*, *vivax*, *ovale* and *malariae*) specific *Plasmodium lactate dehydrogenase (pLDH)* in human capillary and venous whole blood specimens of patients suspected of having malaria. STANDARD Q Malaria P.f/Pan Ag Test is intended to be used by trained healthcare or laboratory professional or other health care workers who have received appropriate training. This product can be used by trained lay providers operating in point-of-care settings in resource-limited lower- and middle-income countries. This product is not intended for self-testing.

Test principle

STANDARD Q Malaria P.f/Pan Ag Test contains two pre-coated lines, "P.f." (*P. falciparum*), "Pan" (*Plasmodium* species: *P. falciparum*, *vivax*, *ovale* and *malariae*) as test lines and "C" as control line on the surface of the nitrocellulose membrane. The test lines and control line in the result window of the test device are not visible before applying any specimens. Monoclonal anti-*P. falciparum* HRP-2 is coated on the P.f test line, monoclonal anti-Malaria pLDH is coated on the Pan test line, and monoclonal anti-chicken IgY is coated on the control line region. During the test, the *P. falciparum* specific HRP-2 antigen and/or *Plasmodium* species specific pLDH in the specimen react to the gold-conjugated monoclonal anti-Malaria HRP-2 and/or gold-conjugated monoclonal anti-Malaria pLDH, and then bind to the test lines. Any *P. falciparum* specific HRP-2 antigen-antibody gold particle complex and/or *Plasmodium* species specific pLDH antigen-antibody gold particle complex also migrate with the buffer and are immobilized by monoclonal anti-*P. falciparum* HRP-2 and/or monoclonal anti-Malaria pLDH at the two individual test lines to formation of violet test colored band(s) which confirms a positive result. Absence of this violet colored band indicates a negative result. The control line is used for procedural control, and should always appear if the test procedure is performed properly and the test reagents of the control line are working.

KIT STORAGE AND STABILITY

- Store the sealed pouch and the buffer provided in the kit at 2-40°C/36-104°F out of the direct sunlight for the duration of its shelf life.
- Do not open the foil pouch until you are ready to perform a test.
- Close the buffer cap tightly after using, and then store it at 2-40°C/36-104°F out of the direct sunlight. It is stable until the expiry date of the kit and the buffer label after opening its cap, if it is tightly closed.

WARNINGS AND PRECAUTIONS

- Do not freeze.
- Do not use beyond the expiration date.
- Do not re-use the test kit.
- Do not use the test kit if the pouch is damaged or the seal is broken.
- Do not use buffer of another lot.
- Do not smoke, drink or eat while handling specimen.
- Wear personal protective equipment, such as gloves and lab coats when handling kit reagents. Wash hands thoroughly afterwards.
- Clean up spills thoroughly using an appropriate disinfectant.
- Handle all specimens as if they contain infectious agents.
- Observe established precautions against microbiological hazards throughout testing procedures.
- Dispose of all specimens and materials used to perform the test as bio-hazard waste. Laboratory chemical and bio-hazard wastes must be handled and discarded in accordance with all local, state, and national regulations.
- In the pouch, there is a desiccant containing a humidity indicator, the humidity indicator should be yellow. If the humidity indicator is green, throw away the test device and take another test device. Throw away the desiccant in the nonsharps (non-infectious) disposal.
- Before testing, check that the lot number of buffer on the bottle matches the lot number of buffer on the package and the pouch you are using.
- The buffer contains 0.095% sodium azide as a preservative which may be toxic if ingested. When disposed of through a sink, flush with large quantities of water.
- For *in vitro* diagnostic use only.

Potential Cross-reacting Substances

STANDARD Q Malaria P.f/Pan Ag Test had no cross-reactance with the following microorganisms such as other hepatitis(Hepatitis A virus, Hepatitis B virus, Hepatitis C virus), Respiratory infectious disease (Chikungunya virus, Zika virus, Dengue virus, Yellow fever virus), Intestinal infectious disease (Rotavirus, E. coli(O159), *Salmonella typhi*), *Streptococcus pneumoniae*, *Streptococcus pyogenes*, *Legionella pneumophila*, *Treponema pallidum*. Also, this product had no cross-reactance with HIV Serconversion panel, HCV positive plasma, HCV core Ag positive plasma, Hbs Ag positive plasma, HAV IgM positive plasma, Dengue IgM positive plasma, Tick borne encephalitis IgM positive plasma, Chikungunya positive plasma, Zika virus positive plasma, *Salmonella typhi* IgM positive plasma, *Leishmania* positive plasma, Japanese Encephalitis positive plasma, *Brucella* IgM positive plasma and West Nile Virus positive plasma.

Diagnostic Sensitivity & Specificity

1. Venous whole blood

1618 venous whole blood specimen were tested using the STANDARD Q Malaria P.f/Pan Ag Test to determine its clinical sensitivity and specificity.

- [Clinical sensitivity]**
- P.f. confirmed specimen on P.f line: 99.58% (476/478, 95% CI: 98.50-99.95%)
 - P.f. confirmed specimen on Pan line: 56.69% (271/478, 95% CI: 52.12-61.19%)
 - P.v., P.m. and P.o. confirmed specimen on Pan line: 100% (129/129, 95% CI: 97.18-100%)
 - P.f. and P.m. co-infection confirmed specimen on P.f line: 100% (11/11, 95% CI: 71.51-100%)
 - P.f. and P.m. co-infection confirmed specimen on Pan line: 72.73% (8/11, 95% CI: 39.03-93.98%)
- [Clinical specificity]** 100% (1000/1000, 95% CI: 99.63-100%)

LIMITATION OF TEST

- The test procedure, precautions, and interpretation of results for this test must be followed strictly when testing.
 - STANDARD Q Malaria P.f/Pan Ag Test is designed for use only on human capillary and venous whole blood specimens.
 - Other clinically available tests are required if questionable results are obtained. As with all diagnostic tests, a definitive clinical diagnosis should not be based on the results of as a single test, but should only be made by the physician after all clinical and laboratory findings have been evaluated.
- QUALITY CONTROL**
- A colored line appearing in the control line is an internal reagent and procedural control. It will appear if the test has been performed correctly and the reagent is reactive.
 - Control materials are not supplied with this test kit. However, it is recommended that positive and negative controls be tested as a good laboratory practice to confirm the test procedure and to verify proper test performance.
 - If there is a problem with the result such as invalid result, retest with a new kit and/or specimen. If the problem is repeated, contact SD Biosensor through your local distributor.

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SYMBOL

REF	Reference number	Caution	Use by	LOT	Batch code	Consult Instructions for Use	Do not re-use
IVD	<i>In vitro</i> Diagnostics	Note	Manufacturer	Date of manufacture	Full fill the requirements of Directive 98/79/EC on <i>in vitro</i> diagnostic medical devices	Contains Sufficient for <= 2 Tests	Keep away from sunlight
	To indicate the temperature limitations in which the transport package has to be kept and handled.		Do not use if packaging is damaged		Authorized Representative	MT Promed Consulting GmbH Ernst-Heckel-Straße 7 66386 St. Ingbert Germany Phone : +49 6894 581020, Fax : +49 6894 581021	

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BIBLIOGRAPHY

- World Malaria Report 2015, World Health Organization, p. 8-12, 2015.
- Confronting Plasmodium vivax malaria, World Health Organization, p. 1-3, 2015.
- Bioassay in Microbiological and Biomedical Laboratories, 5th Edition, U.S. Department of Health and Human Services, CDC, NIH, Washington DC, P.182-184, 2007.
- Guidelines for the treatment of malaria, World Health Organization, p.5-6, 2006.

- Recommended selection criteria for procurement of malaria rapid diagnostic tests, World Health Organization, p.1-4, 2015.
- C. Beattie, G.W. Long, M.C. Elogy, Hoffman, G.W. Long, W.R. Weiss, S.M. Maret, A.J. Oloo, Diagnosis of malaria by detection of Plasmodium falciparum HRP-2 antigen with a rapid dipstick antigen-capture assay, TheLancet.Vol. 343, Issue: 3897, 1994, p.564-568.
- A. Mooney, Rapid diagnostic tests for malaria parasites. Clin Microbiol Rev, Vol. 15, 2002, p. 66-78.
- LB Ochola, P. Vounatsou, T. Smith, M. Mabaso, C. Newton. The reliability of

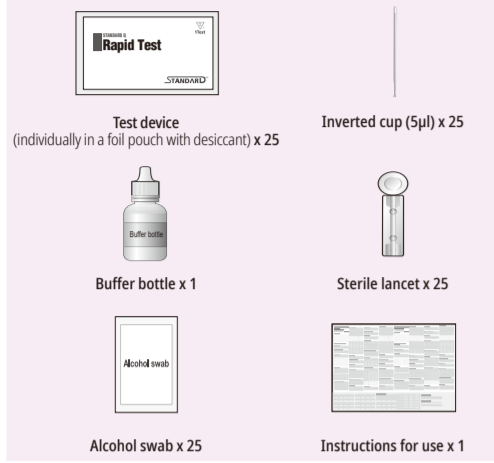
- diagnostic techniques in the diagnosis and management of malaria in the absence of a gold standard, Lancet Infect Dis, Vol. 6, Issue 9, 2006, p.582-588.
- R. K. Abba, J.J. Deeks, P. Ollaro, C.M. Naing, S.M. Jackson et al. Rapid diagnostic tests for diagnosing uncomplicated P. falciparum malaria in endemic countries. Cochrane Database Syst Rev, Vol. 18, Issue 12, 2011.
- M. Ahmed, M. Hossain, A.S. Suzzamen, M. Alam, A. Khan et al. Rapid Diagnosis of malaria by Antigen Detection. Bangladesh Journal of Medical Microbiology, Bangladesh J Med Microbiol, Vol. 3, Issue 1, 2009, p. 14-16.

STANDARD Q Malaria P.f/Pan Ag
STANDARD™ Q Malaria P.f/Pan Ag Test

PLEASE READ INSTRUCTIONS CAREFULLY BEFORE YOU PERFORM THE TEST

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- 1) **Capillary whole blood**
 - 1. Capillary whole blood should be collected aseptically by fingerstick.
 - 2. Select the finger that is not calloused and clean the fingertip by wiping with an alcohol swab.
 - 3. Squeeze the end of the fingertip and pierce with a sterile lancet.
 - 4. Wipe the first drop with a sterile gauze.
 - 5. Collect capillary whole blood and it must be tested immediately after collection.

Venous whole blood

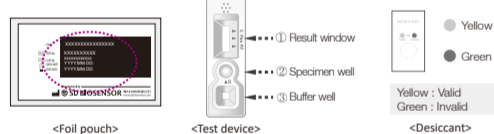
- 1. Collect venous whole blood into a commercially available anti-coagulant tube such as heparin, EDTA, or sodium citrate by venipuncture.
- 2. If venous whole blood in an anti-coagulant tube is stored in a refrigerator at 2-8°C/36-46°F, the specimen can be used for testing within 72 hours after collection.
- 3. Do not use hemolyzed blood specimen.

CAUTION

Anticoagulants such as heparin, EDTA, or sodium citrate do not affect the test result. As known relevant interference, hemolytic specimen, rheumatoid factors-contained specimen and lipemic, icteric specimen may lead to inaccurate results. Use separate inverted cup for each specimen in order to avoid cross-contamination of either specimens which can cause erroneous results.

PREPARATION AND TEST PROCEDURE

- 1. Carefully read the instructions for using the STANDARD Q Malaria P.f/Pan Ag Test.
- 2. Look at the expiry date at the back of the foil pouch. Use another lot, if expiry date has passed.
- 3. Open the foil pouch, and check the test device and the desiccant inside the foil pouch.

CAUTION Bring the kit contents and the specimens to room temperature before testing.

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- 1. Test with a sterile lancet**
 - 1. Select the finger that is not calloused and clean the fingertip by wiping with an alcohol swab.
 - 2. Dry and pierce the wiped fingertip with a sterile lancet to bleed.

3. Wipe away the first drop of blood with sterile gauze.
4. Take the inverted cup(5µl) provided and collect 5µl of the capillary whole blood specimen by dipping the circular end of the inverted cup into the specimen.
5. To apply the collected capillary whole blood specimen to the specimen well(circle) of the test device, place the circular end of the inverted cup on the specimen well pad and then press down lightly.

1-1. Test by venipuncture

1. Gently invert 4-5 times for proper mixing.
2. Using a micropipette, draw 5µl of venous whole blood specimen.
3. Apply the collected venous whole blood specimen to the specimen well(circle) of the test device.

2. Dropping of Buffer

1. Hold the buffer bottle at 90° (perpendicular) to the test device and not touching the specimen well to avoid contamination. Add 3 drops (90µl) of the buffer into the buffer well(square) of the test device.

3. Reading Time

1. Read the test result in 15 minutes. The test can be read up to 30 minutes.

CAUTION

- Do not read test results after 30 minutes. It may give false results.
- If the T line does not appear until 15 minutes, make sure to observe the appearance of the T line until 30 minutes.

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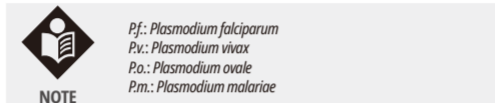
1. A colored band will appear in the top section of the result window to show that the test is working properly. This band is the control line (C).
 2. Colored bands will appear in the middle and lower section of the result window. These bands are the *Plasmodium falciparum*, *vivax*, *ovale* and *malariae* line (Pan) and the *Plasmodium falciparum* line (P.f.) respectively.
- *This test only indicates the presence of Histidine Rich Protein-2 (HRP2) of *Plasmodium falciparum* and *Plasmodium lactate dehydrogenase (pLDH)* of *Plasmodium species (Plasmodium falciparum, vivax, ovale, and malariae)* in human capillary and venous whole blood specimen and should not be used as the sole criteria for the diagnosis of malaria. As other diagnostic tests, all test results should be considered with other clinical history available.

EXPLANATION AND SUMMARY

Introduction

Malaria remains an important cause of illness and death in children and adults in countries in which it is endemic. Globally, an estimated 3.2 billion people in 97 countries and territories are at risk of being infected with malaria and developing disease, and 1.2 billion are at high risk (>1 in 1,000 chance of getting malaria in a year). According to the World Malaria Report 2015, there were 210 million cases of malaria globally in 2015 (uncertainty range 149-303 million) and 438,000 malaria deaths (range 236,000-635,000), representing a decrease in malaria cases and deaths of 37% and 60% since 2000, respectively. The protozoal parasites that cause malaria are from *Plasmodium falciparum*, *vivax*, *ovale* and *malariae* with the first two species causing the most infections worldwide. Classic symptoms of malaria include fever, headache, chills, vomiting, shivering and convulsions. In some rare forms of *P. falciparum*, the patient may present with delirium or coma. Severe anemia is often attributed to the cause of death from malaria. Accurate and prompt diagnosis of malaria is of utmost importance due to the morbidity associated with the other malarial forms. Rapid diagnostic test is an ideal diagnostic tool for malaria diagnosis in that it can provide a rapid determination if the patient is infected with malaria allowing for accurate treatment and improved outcomes. STANDARD Q Malaria P.f/Pan Ag Test, a reliable and sensitive screening test, would enhance the accuracy of the diagnosis of malaria infection and thus make clinical treatment decision effectively.

Intended use

STANDARD Q Malaria P.f/Pan Ag Test is a rapid and membrane based immunochromatography for the qualitative detection of *Plasmodium falciparum (P. falciparum)* specific Histidine Rich Protein 2 (HRP-2) and *Plasmodium species (P. falciparum, vivax, ovale and malariae)* specific *Plasmodium lactate dehydrogenase (pLDH)* in human capillary and venous whole blood specimens of patients suspected of having malaria. STANDARD Q Malaria P.f/Pan Ag Test is intended to be used by trained healthcare or laboratory professional or other health care workers who have received appropriate training. This product can be used by trained lay providers operating in point-of-care settings in resource-limited lower- and middle-income countries. This product is not intended for self-testing.

Test principle

STANDARD Q Malaria P.f/Pan Ag Test contains two pre-coated lines, "P.f." (*P. falciparum*), "Pan" (*Plasmodium species: P. falciparum, vivax, ovale and malariae*) as test lines and "C" as control line on the surface of the nitrocellulose membrane. The test lines and control line in the result window of the test device are not visible before applying any specimens. Monoclonal anti-*P. falciparum* HRP-2 is coated on the P.f. test line, monoclonal anti-Malaria pLDH is coated on the Pan test line, and monoclonal anti-chicken IgY is coated on the control line region. During the test, the *P. falciparum* specific HRP-2 antigen and/or *Plasmodium species* specific pLDH in the specimen react to the gold-conjugated monoclonal anti-Malaria HRP-2 and/or gold-conjugated monoclonal anti-Malaria pLDH, and then bind to them respectively. Any *P. falciparum* specific HRP-2 antigen-antibody gold particle complex and/or *Plasmodium species* specific pLDH antigen-antibody gold particle complex also migrate with the buffer and are immobilized by monoclonal anti-*P. falciparum* HRP-2 and/or monoclonal anti-Malaria pLDH at the two individual test lines to formation of violet test colored band(s) which confirms a positive result. Absence of this violet colored band indicates a negative result. The control line is used for procedural control, and should always appear if the test procedure is performed properly and the test reagents of the control line are working.

KIT STORAGE AND STABILITY

1. Store the sealed pouch and the buffer provided in the kit at 2-40°C/36-104°F out of the direct sunlight for the duration of its shelf life.
2. Do not open the foil pouch until you are ready to perform a test.
3. Close the buffer cap tightly after using, and then store it at 2-40°C/36-104°F out of the direct sunlight. It is stable until the expiry date of the kit and the buffer label after opening its cap, if it is tightly closed.

WARNINGS AND PRECAUTIONS

1. Do not freeze.
2. Do not use beyond the expiration date.
3. Do not re-use the test kit.
4. Do not use the test kit if the pouch is damaged or the seal is broken.
5. Do not use buffer of another lot.
6. Do not smoke, drink or eat while handling specimen.
7. Wear personal protective equipment, such as gloves and lab coats when handling kit reagents. Wash hands thoroughly afterwards.
8. Clean up spills thoroughly using an appropriate disinfectant.
9. Handle all specimens as if they contain infectious agents.
10. Observe established precautions against microbiological hazards throughout testing procedures.
11. Dispose of all specimens and materials used to perform the test as bio-hazard waste. Laboratory chemical and bio-hazard wastes must be handled and discarded in accordance with all local, state, and national regulations.
12. In the pouch, there is a desiccant containing a humidity indicator, the humidity indicator should be yellow. If the humidity indicator is green, throw away the test device and take another test device. Throw away the desiccant in the nonsharps (non-infectious) disposal.
13. Before testing, check that the lot number of buffer on the bottle matches the lot number of buffer on the package and the pouch you are using.
14. The buffer contains 0.095% sodium azide as a preservative which may be toxic if ingested. When disposed of through a sink, flush with large quantities of water.
15. For *in vitro* diagnostic use only.

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STANDARD Q Malaria P.f/Pan Ag Test had no cross-reaction with the following microorganisms such as other hepatitis(Hepatitis A virus, Hepatitis B virus, Hepatitis C virus), Respiratory infectious disease(Influenza A, B), Vector-borne infectious disease (Chikungunya virus, Zika virus, Dengue virus, Yellow fever virus), Intestinal infectious disease (Rotavirus, E. coli(O159), *Salmonella typhi*), *Streptococcus pneumoniae*, *Streptococcus pyogenes*, *Legionella pneumophila*, *Treponema pallidum*. Also, this product had no cross-reaction with HIV Serconversion panel, HCV positive plasma, HCV core Ag positive plasma, Hbs Ag positive plasma, HAV IgM positive plasma, Dengue IgM positive plasma, Tick borne encephalitis IgM positive plasma, Chikungunya positive plasma, Zika virus positive plasma, *Salmonella typhi* IgM positive plasma, *Leishmania* positive plasma, Japanese Encephalitis positive plasma, *Brucella* IgM positive plasma and West Nile Virus positive plasma.

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1. **Venous whole blood**
 - 1618 venous whole blood specimen were tested using the STANDARD Q Malaria P.f/Pan Ag Test to determine its clinical sensitivity and specificity.
 - [Clinical sensitivity]
 - P.f. confirmed specimen on P.f line: 99.58% (476/478, 95% CI: 98.50-99.95%)
 - P.f. confirmed specimen on Pan line: 56.69% (271/478, 95% CI: 52.12-61.19%)
 - P.v., P.m. and P.o. confirmed specimen on Pan line: 100% (129/129, 95% CI: 97.18-100%)
 - P.f. and P.m. co-infection confirmed specimen on P.f line: 100% (11/11, 95% CI: 71.51-100%)
 - P.f. and P.m. co-infection confirmed specimen on Pan line: 72.73% (8/11, 95% CI: 39.03-93.98%)
 - [Clinical specificity] 100% (1000/1000, 95% CI: 99.63-100%)

LIMITATION OF TEST

1. The test procedure, precautions, and interpretation of results for this test must be followed strictly when testing.
2. STANDARD Q Malaria P.f/Pan Ag Test is designed for use only on human capillary and venous whole blood specimens.
3. Other clinically available tests are required if questionable results are obtained. As with all diagnostic tests, a definitive clinical diagnosis should not be based on the results of a single test, but should only be made by the physician after all clinical and laboratory findings have been evaluated.

QUALITY CONTROL

1. A colored line appearing in the control line is an internal reagent and procedural control. It will appear if the test has been performed correctly and the reagent is reactive.
2. Control materials are not supplied with this test kit. However, it is recommended that positive and negative controls be tested as a good laboratory practice to confirm the test procedure and to verify proper test performance.
3. If there is a problem with the result such as invalid result, retest with a new kit and/or specimen. If the problem is repeated, contact SD Biosensor through your local distributor.

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

1. Test with a sterile lancet

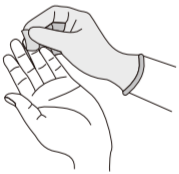
- Select the finger that is not calloused and clean the fingertip by wiping with an alcohol swab.

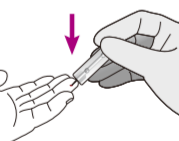
- Dry and pierce the wiped fingertip with a sterile lancet to bleed.

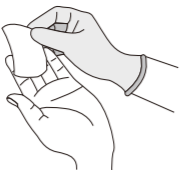
- Wipe away the first drop of blood with sterile gauze.

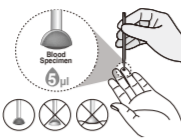- Take the inverted cup(5µl) provided and collect 5µl of the capillary whole blood specimen by dipping the circular end of the inverted cup into the specimen.

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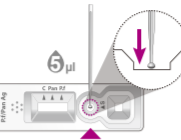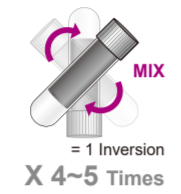
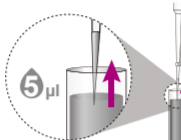
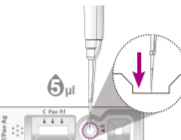
Positive

P.f. Positive or mixed positive of P.v., P.o., and/or P.m. with P.f. | P.f. Positive | P.f., P.v., P.o., or P.m. positive or mixed positive of these

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- A colored band will appear in the top section of the result window to show that the test is working properly. This band is the control line (C).
- Colored bands will appear in the middle and lower section of the result window. These bands are the *Plasmodium falciparum*, *vivax*, *ovale* and *malariae* line (Pan) and the *Plasmodium falciparum* line (P.f.) respectively.

***This test only indicates the presence of Histidine Rich Protein-2 (HRP-2) of *Plasmodium falciparum* and *Plasmodium lactate dehydrogenase (pLDH) of Plasmodium species (Plasmodium falciparum, vivax, ovale, and malariae)* in human capillary and venous whole blood specimen and should not be used as the sole criteria for the diagnosis of malaria. As other diagnostic tests, all test results should be considered with other clinical history available.**

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Intended use

STANDARD Q Malaria P.f/Pan Ag Test is a rapid and membrane based immunochromatography for the qualitative detection of *Plasmodium falciparum* (*P. falciparum*) specific Histidine Rich Protein 2 (HRP-2) and *Plasmodium* species (*P. falciparum*, *vivax*, *ovale* and *malariae*) specific *Plasmodium lactate dehydrogenase* (pLDH) in human capillary and venous whole blood specimens of patients suspected of having malaria. STANDARD Q Malaria P.f/Pan Ag Test is intended to be used by trained healthcare or laboratory professional or other health care workers who have received appropriate training. This product can be used by trained lay providers operating in point-of-care settings in resource-limited lower- and middle-income countries. This product is not intended for self-testing.

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KIT STORAGE AND STABILITY

- Store the sealed pouch and the buffer provided in the kit at 2-40°C/36-104°F out of the direct sunlight for the duration of its shelf life.
- Do not open the foil pouch until you are ready to perform a test.
- Close the buffer cap tightly after using, and Pan line: It is at 2-40°C/36-104°F out of the direct sunlight. It is stable until the expiry date of the kit and the buffer label after opening its cap, if it is tightly closed.

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STANDARD Q Malaria P.f/Pan Ag Test had no cross-reactance with the following microorganisms such as other hepatitis(Hepatitis A virus, Hepatitis B virus, Hepatitis C virus), Respiratory infectious disease (Chikungunya virus, Zika virus, Dengue virus, Yellow fever virus), Intestinal infectious disease (Rotavirus, E. coli(O159), *Salmonella typhi*), *Streptococcus pneumoniae*, *Streptococcus pyogenes*, *Legionella pneumophila*, *Treponema pallidum*. Also, this product had no cross-reactance with HIV Serconversion panel, HCV positive plasma, HCV core Ag positive plasma, Hbs Ag positive plasma, HAV IgM positive plasma, Dengue IgM positive plasma, Tick borne encephalitis IgM positive plasma, Chikungunya positive plasma, Zika virus positive plasma, *Salmonella typhi* IgM positive plasma, *Leishmania* positive plasma, Japanese Encephalitis positive plasma, *Brucella* IgM positive plasma and West Nile virus positive plasma.

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(Clinical sensitivity)

- P.f. confirmed specimen on P.f line: 99.58% (476/478, 95% CI: 98.50-99.95%)
- P.f. confirmed specimen on Pan line: 56.69% (271/478, 95% CI: 52.12-61.19%)
- P.v., P.m. and P.o. confirmed specimen on Pan line: 100% (129/129, 95% CI: 97.18-100%)
- P.f. and P.m. co-infection confirmed specimen on P.f line: 100% (11/11, 95% CI: 71.51-100%)
- P.f. and P.m. co-infection confirmed specimen on Pan line: 72.73% (8/11, 95% CI: 39.03-93.98%)

(Clinical specificity) 100% (1000/1000, 95% CI: 99.63-100%)

LIMITATION OF TEST

- The test procedure, precautions, and interpretation of results for this test must be followed strictly when testing.
- STANDARD Q Malaria P.f/Pan Ag Test is designed for use only on human capillary and venous whole blood specimens.
- Other clinically available tests are required if questionable results are obtained. As with all diagnostic tests, a definitive clinical diagnosis should not be based on the results of as a single test, but should only be made by the physician after all clinical and laboratory findings have been evaluated.

QUALITY CONTROL

- A colored line appearing in the control line is an internal reagent and procedural control. It will appear if the test has been performed correctly and the reagent is reactive.
- Control materials are not supplied with this test kit. However, it is recommended that positive and negative controls be tested as a good laboratory practice to confirm the test procedure and to verify proper test performance.
- If there is a problem with the result such as invalid result, retest with a new kit and/or specimen. If the problem is repeated, contact SD Biosensor through your local distributor.

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BIBLIOGRAPHY

- World Malaria Report 2015, World Health Organization, p.8-12, 2015.
- Confronting Plasmodium vivax malaria, World Health Organization, p.1-3, 2015.
- Bioassay in Microbiological and Biomedical Laboratories, 5thEdition, U.S. Department of Health and human Services, CDC, NIH, Washington DC, F182-184, 2007.
- Guidelines for the treatment of malaria, World Health Organization, p.5-6, 2006.
- Recommended selection criteria for procurement of malaria rapid diagnostic tests, World Health Organization, p.1-4, 2015.
- C. Beattie, G.W. Long, Mc Elroy, Hoffman, G.W. Long, W.R. Weiss, S.M. Maret, A.J. Oloo, Diagnosis of malaria by detection of Plasmodium falciparum HRP-2 antigen with a rapid dipstick antigen-capture assay, TheLancet.Vol. 343, Issue: 8897, 1994, p.564-568.
- A. Moody, Rapid diagnostic tests for malaria parasites. Clin Microbiol Rev, Vol. 15, 2002, p. 66-78.
- LB Ochoła, P. Vounatsou, T. Smith, M. Mabaso, C. Newton. The reliability of diagnostic techniques in the diagnosis and management of malaria in the absence of a gold standard, Lancet Infect Dis, Vol. 6, Issue 9, 2006, p.582-588.
- K. Abba, J.J. Deeks, P. Ollaro, C.M. Naing, S.M. Jackson et al. Rapid diagnostic tests for diagnosing uncomplicated P. falciparum malaria in endemic countries. Cochrane Database Syst Rev, Vol. 18, Issue 12, 2011.
- M. Ahmed, M. Hossain, A.S. Suzzamen, M. Alam, A. Khan et al. Rapid Diagnosis of malaria by Antigen Detection. Bangladesh Journal of Medical Microbiology, Bangladesh J Med Microbiol, Vol. 3, Issue 1, 2009, p. 14-16.

REF	Reference number	Caution	Use by	LOT	Batch code	Consult Instructions for Use	Do not re-use
IVD	In vitro Diagnostics	Note	Manufacturer	Date of manufacture	Full fill the requirements of Directive 98/79/EC on in vitro diagnostic medical devices	Contains Sufficient for <=2 Tests	Keep away from sunlight
	Indicate that you should keep the product dry		To indicate the temperature limitations in which the transport package has to be kept and handled.				Do not use if packaging is damaged

SYMBOL

REF	Reference number	Caution	Use by	LOT	Batch code	Consult Instructions for Use	Do not re-use
IVD	In vitro Diagnostics	Note	Manufacturer	Date of manufacture	Full fill the requirements of Directive 98/79/EC on in vitro diagnostic medical devices	Contains Sufficient for <=2 Tests	Keep away from sunlight
	Indicate that you should keep the product dry		To indicate the temperature limitations in which the transport package has to be kept and handled.				Do not use if packaging is damaged

Authorized Representative
MT Promed Consulting GmbH
Ernst-Heckel-Straße 7 66386 St. Ingbert Germany
Phone: +49 6894 581020, Fax: +49 6894 581021

SD Biosensor, Inc.

C-4th85th, 16, Deogyong-dae 1556beon-gil, Yeongtong-gu, Suwon-si, Gyeonggi-do, 16690, REPUBLIC OF KOREA

Manufacturing site

Plot no. 38, Sector 4, IMT Manesar, Gurugram, Haryana 122052, INDIA
Please contact us for any complaints/inquiries/suggestions via email (ts@sdbiosensor.com), phone (+82-80-970-9700) or website (www.sdbiosensor.com).